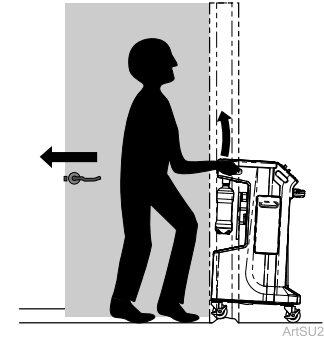
WARNING

Equipment not suitable for use in the presence of a flammable anesthetic mixture. Isoflurane and Sevoflurane are recommended for use in the presence of the Midmark Mobile Dental Delivery System.


WARNING

Exercise care when transporting over thresholds and other low-lying objects. Failure to do so can lead to injury.



[English](#)
[Español](#)
[Français](#)

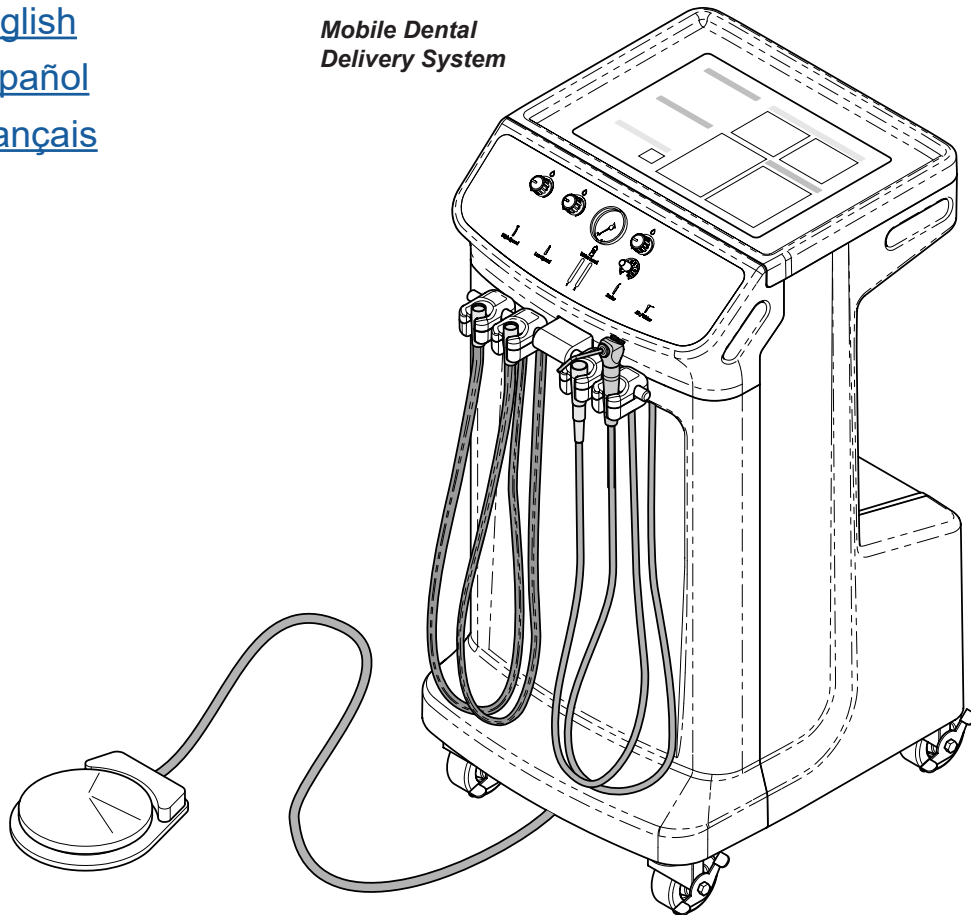Mobile Dental Delivery System

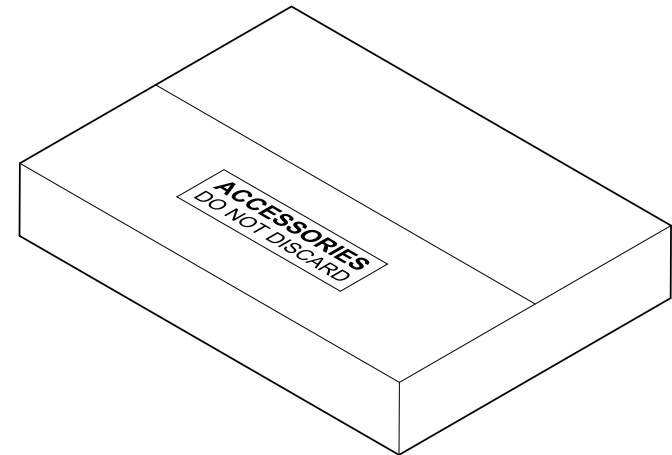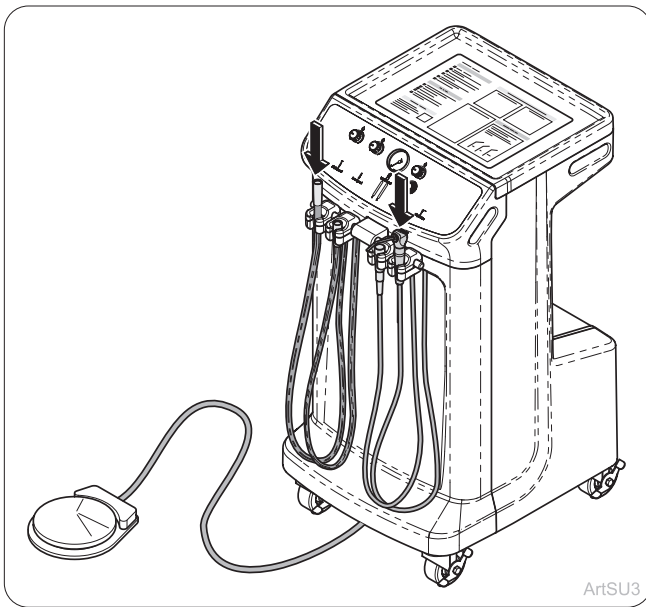Accessory Box
(Do not discard until all boxes inside are emptied and installed onto Delivery System)


ArtSU3

Step 1: Un-box

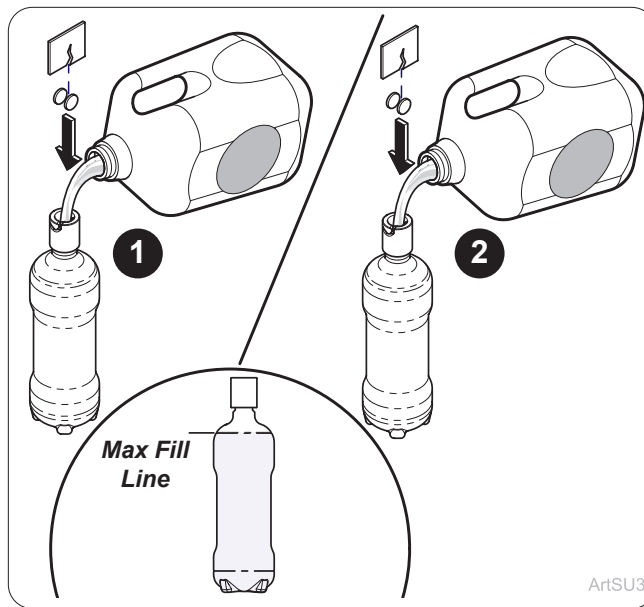
Remove syringe and tubing packaging and place in holders. Move into place and place foot control on floor.

ArtSU3

Step 2: Initial Waterline Shock Treatment

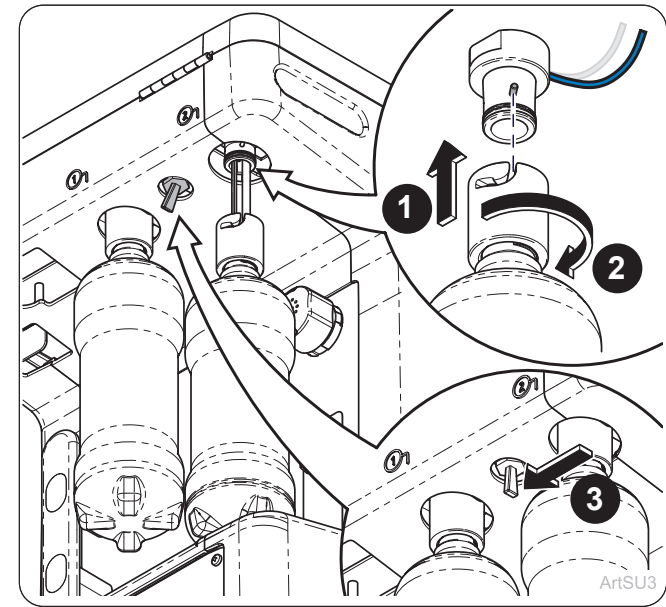
In U.S. only, use VistaTab™ Dental Waterline Cleaner Tablets directions for use. Outside U.S. use a comparable product to shock waterlines.

ArtSU3

Step 3: Initial Waterline Shock Treatment

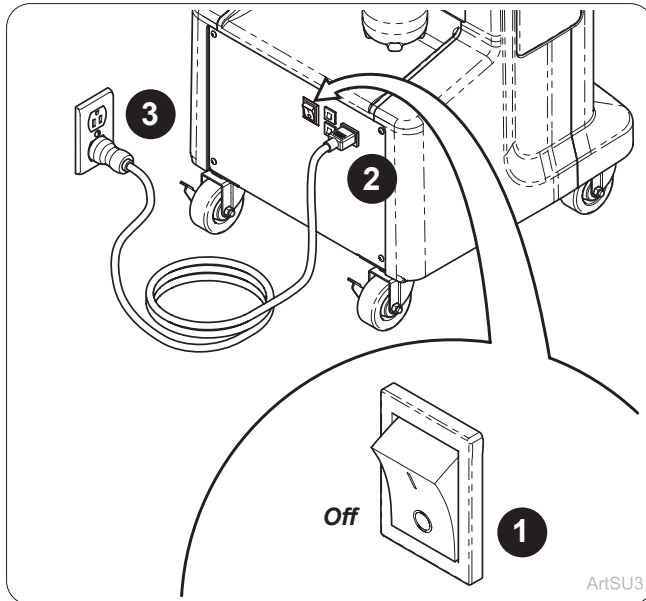
Allow tablets to dissolve. Install water bottles and toggle switch towards water bottle labeled 1.

ArtSU3

Step 4: Initial Waterline Shock Treatment

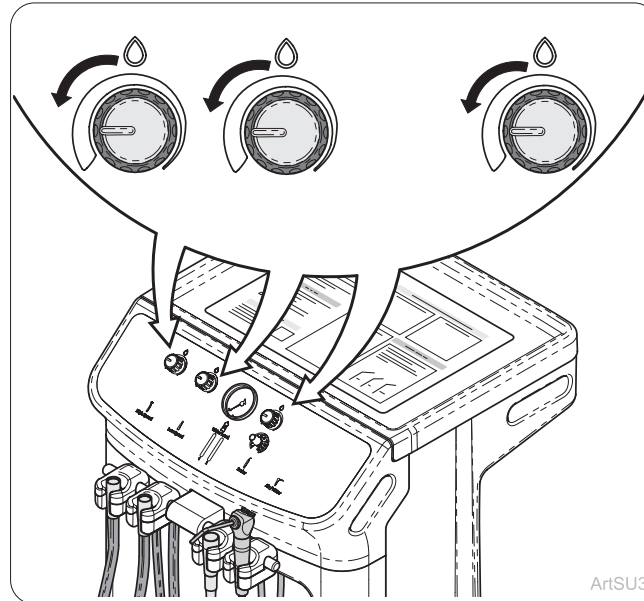
Make sure power switch is turned to off. Plug power cord into dental delivery. Plug into power outlet.

ArtSU3

Step 5: Initial Waterline Shock Treatment

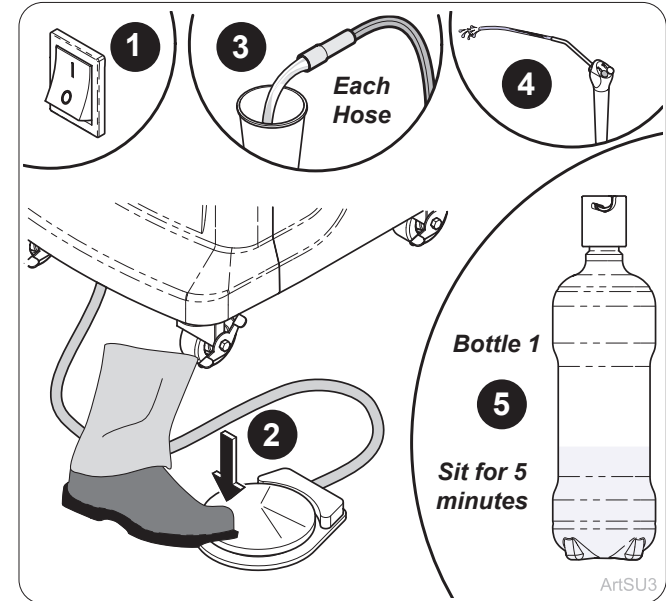
Turn water adjustment knobs open to ensure water flows through each handpiece hose.


ArtSU3

Step 6: Initial Waterline Shock Treatment

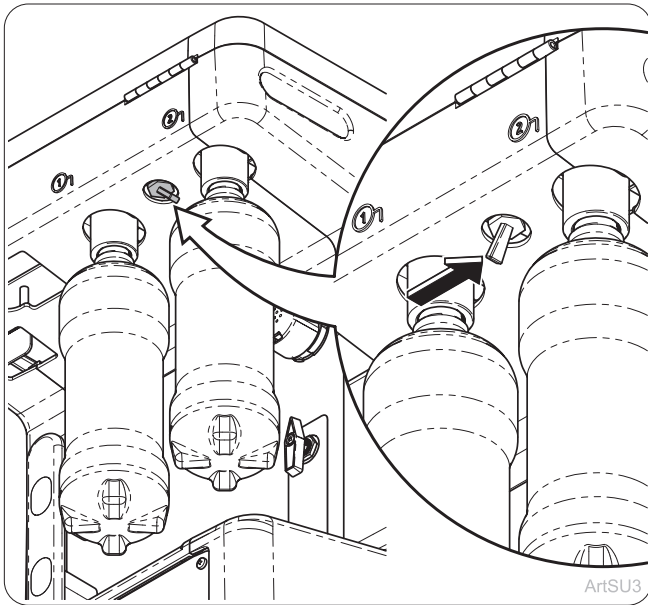
Turn power on and remove each hose separately for best flow. Step on foot control. Flow treated water through all hoses and syringe, then let sit for 5 minutes.

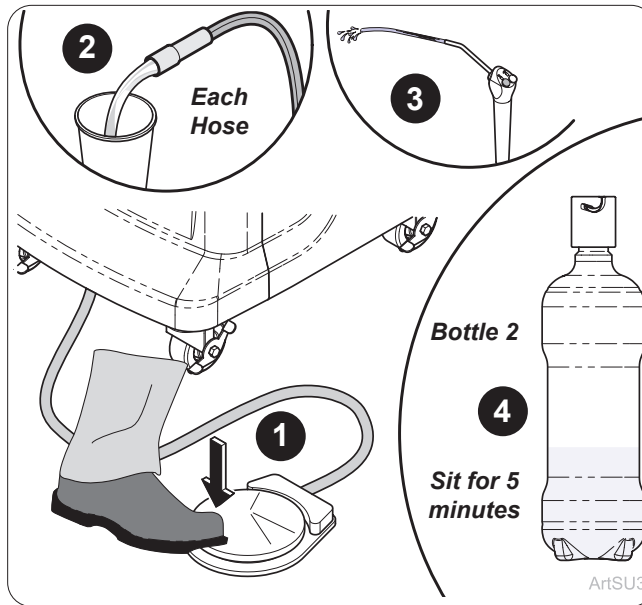
Step 7: Initial Waterline Shock Treatment
Toggle water bottle switch to bottle labeled 2.

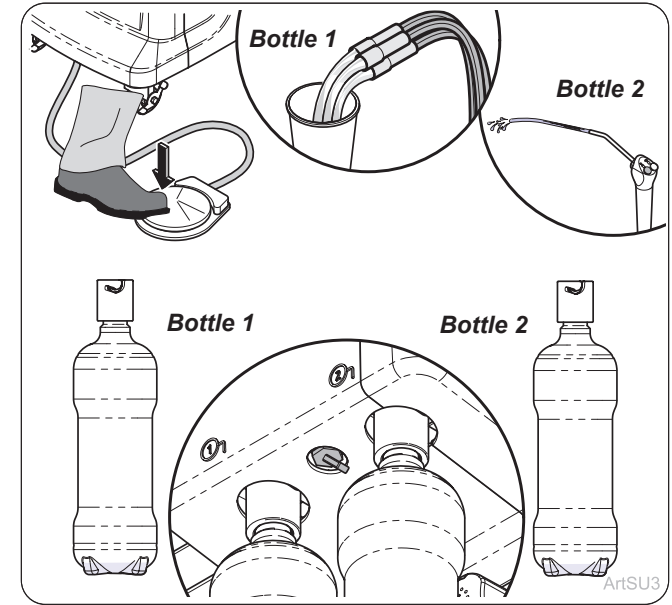
Step 8: Initial Waterline Shock Treatment
Remove each hose separately for best flow. Step on foot control. Flow treated water through all hoses and syringe, then let sit for 5 minutes.

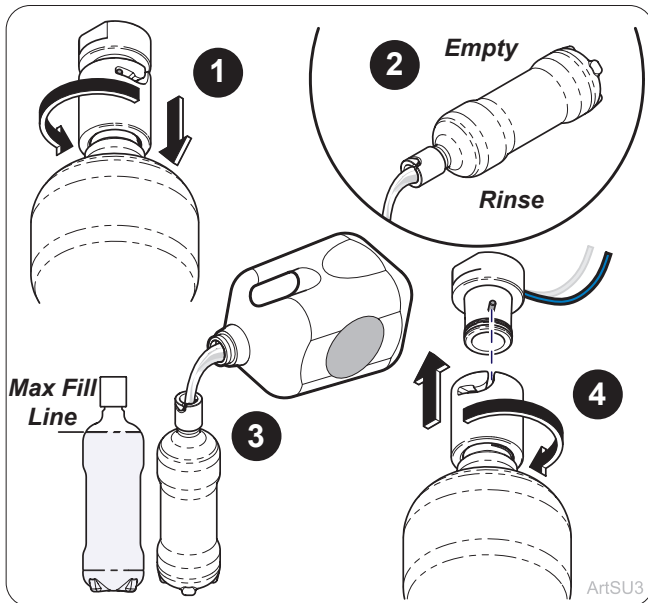Step 9: Initial Waterline Shock Treatment
After the treated water has been in lines for 5 minutes flush both bottles 1 and 2 through the system. Turn power off.

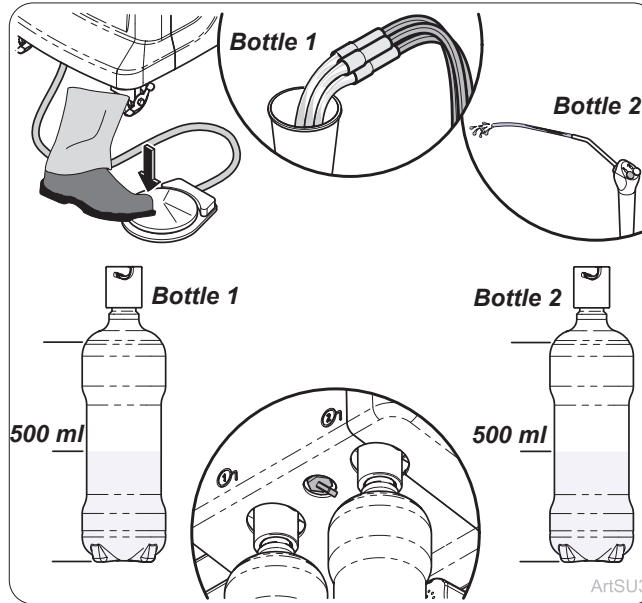Step 10: Initial Waterline Shock Treatment
Remove, empty and rinse both water bottles, fill with distilled water and reinstall. Turn power on.

Step 11: Initial Waterline Shock Treatment
Flush at least 500 ml through all hoses from both bottles to remove any residual chemicals before use. Refill with distilled water when needed.

Equipment Alert

Do not over tighten. Can strip threads.


ArtSU3

Step 12: Scaler Handpiece

Screw scaler tip onto scaler and tighten with scaler tool.

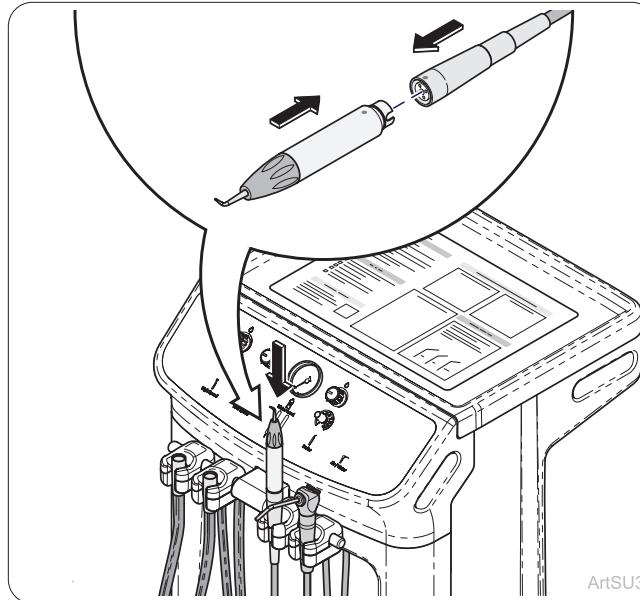
Note: Refer to Acteon instruction for use for detailed information.

ArtSU3

Step 13: Scaler Handpiece

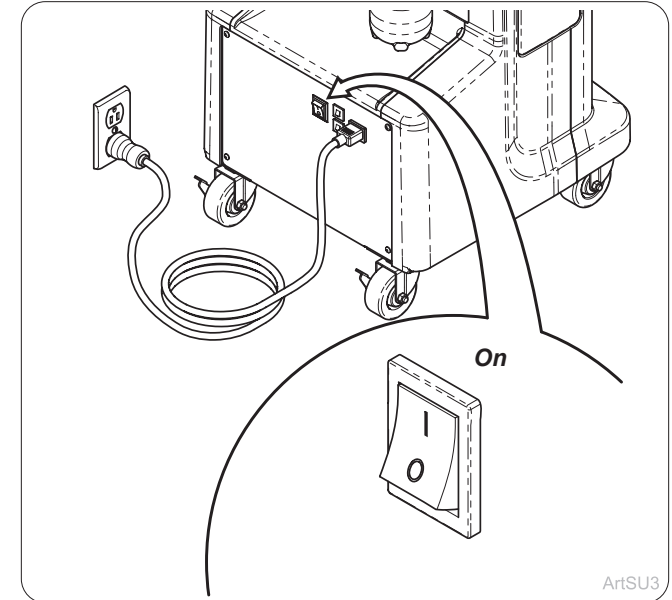
Plug scaler handpiece into hose connection by lining up the two dots, **do not twist**. Place in designated scaler handpiece holder.

ArtSU3

Step 14: Scaler Handpiece

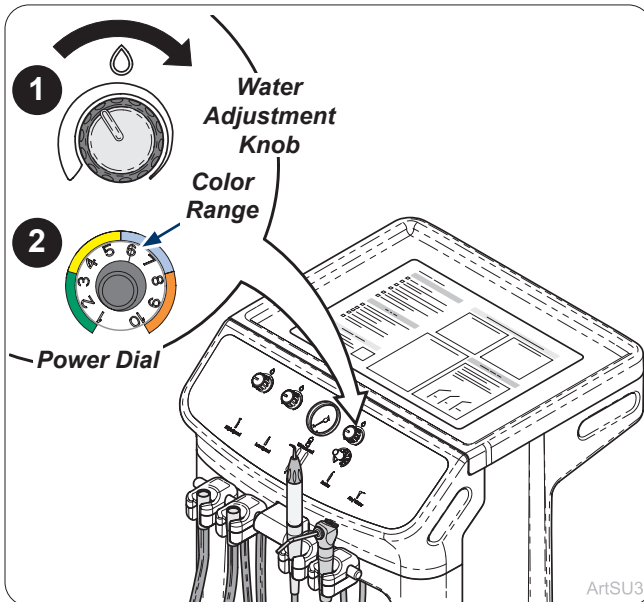
Turn on mobile dental delivery.

ArtSU3

Step 15: Scaler Handpiece

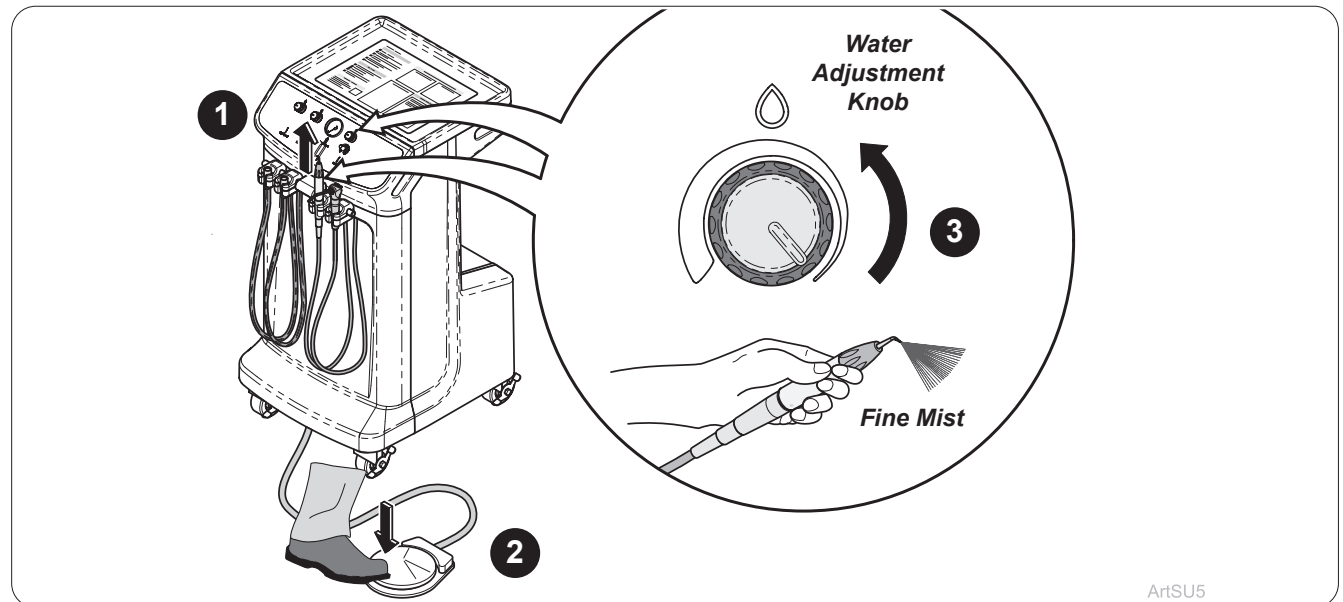
Rotate water adjustment knob to off. Set scaler power dial to lowest position in the blue section .

ArtSU5


Step 16: Scaler Handpiece

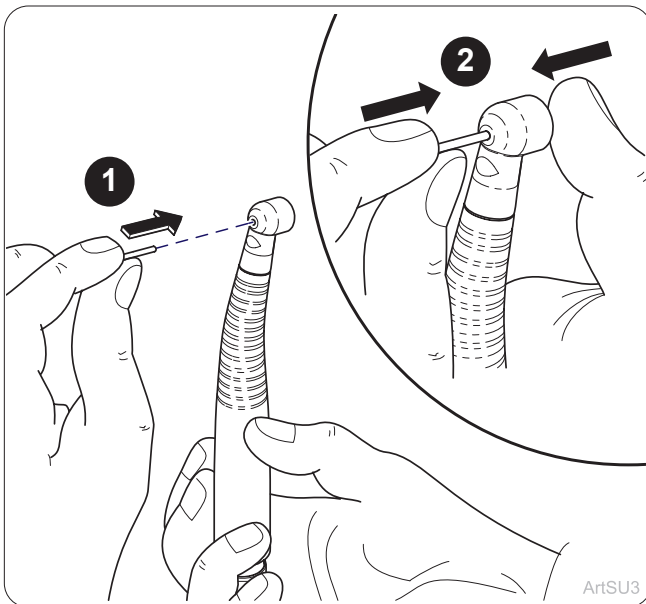Remove scaler from holder, depress foot pedal, and increase water adjustment knob until scaler water flow is a fine mist.


Step 17: High-Speed Handpiece

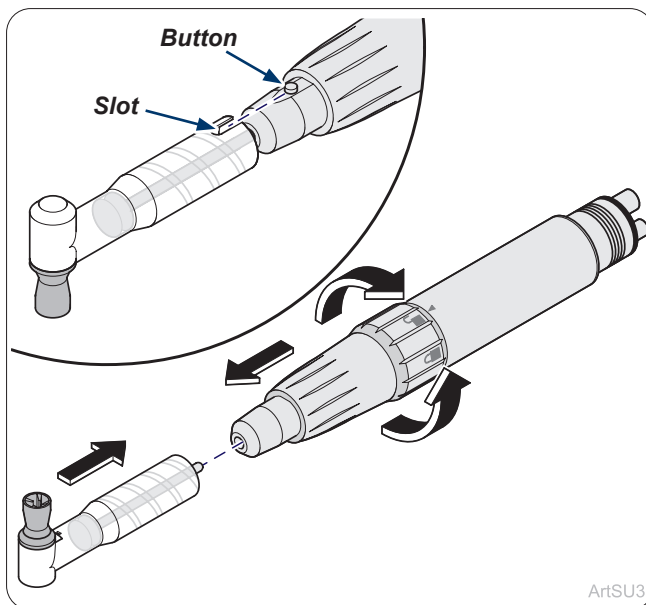To properly seat bur, insert it into chuck until it stops. Press your thumb on the button, then push bur until it is fully seated in the chuck.


Step 20: Low-Speed Handpiece

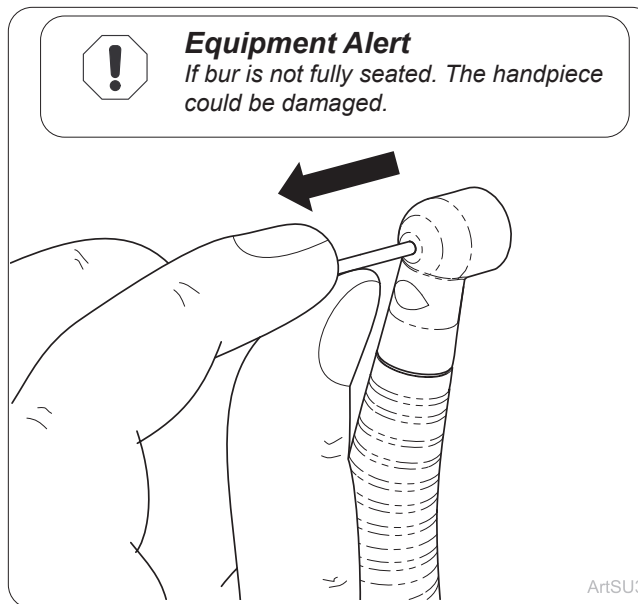Turn chuck release ring counter-clockwise to unlock. Install Prophyl angle. Turn chuck release ring clockwise to lock.


Step 18: High-Speed Handpiece

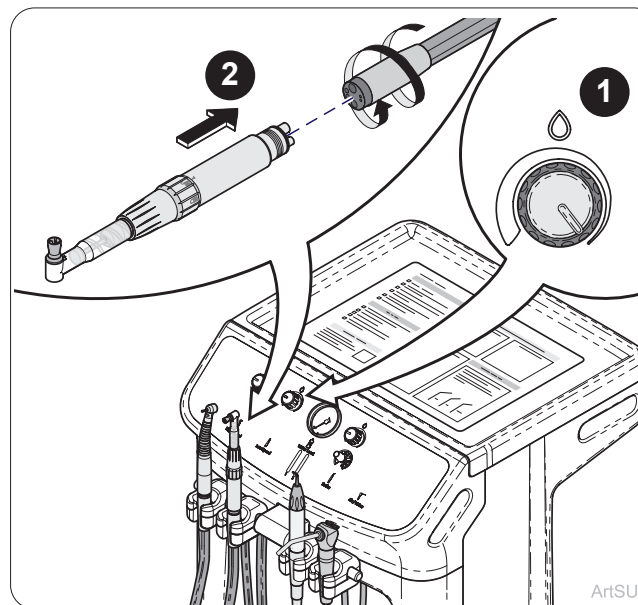Pull lightly back on bur to ensure it is properly seated.


Step 21: Low-Speed Handpiece

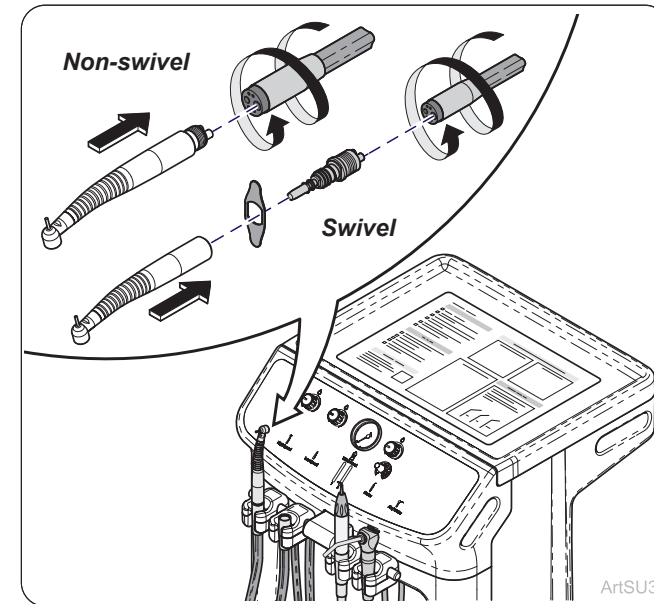Confirm corresponding water knob off. Line up components properly and connect.


Step 19: High-Speed Handpiece

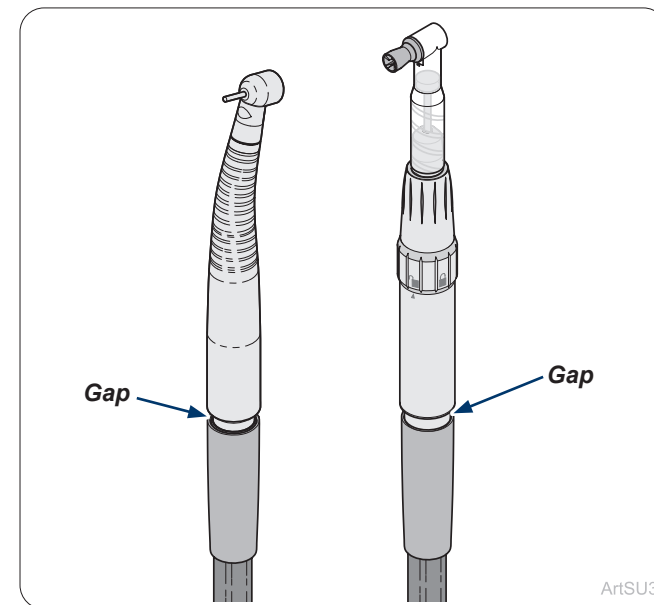Condition handpiece and attach to tubing. Refer to manufacturer's instruction within handpiece box for conditioning details.


Equipment Alert

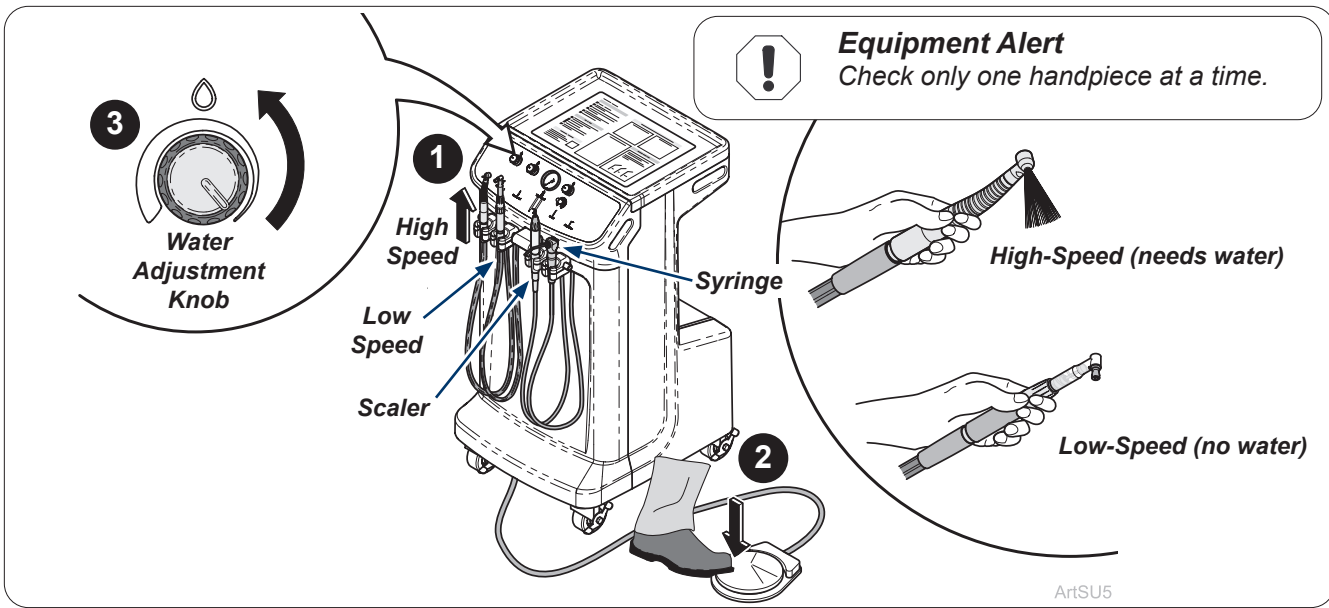Do not overtighten, a gap is expected between the nut and handpiece.


Step 22: Confirm Handpiece Pressure (PSI)

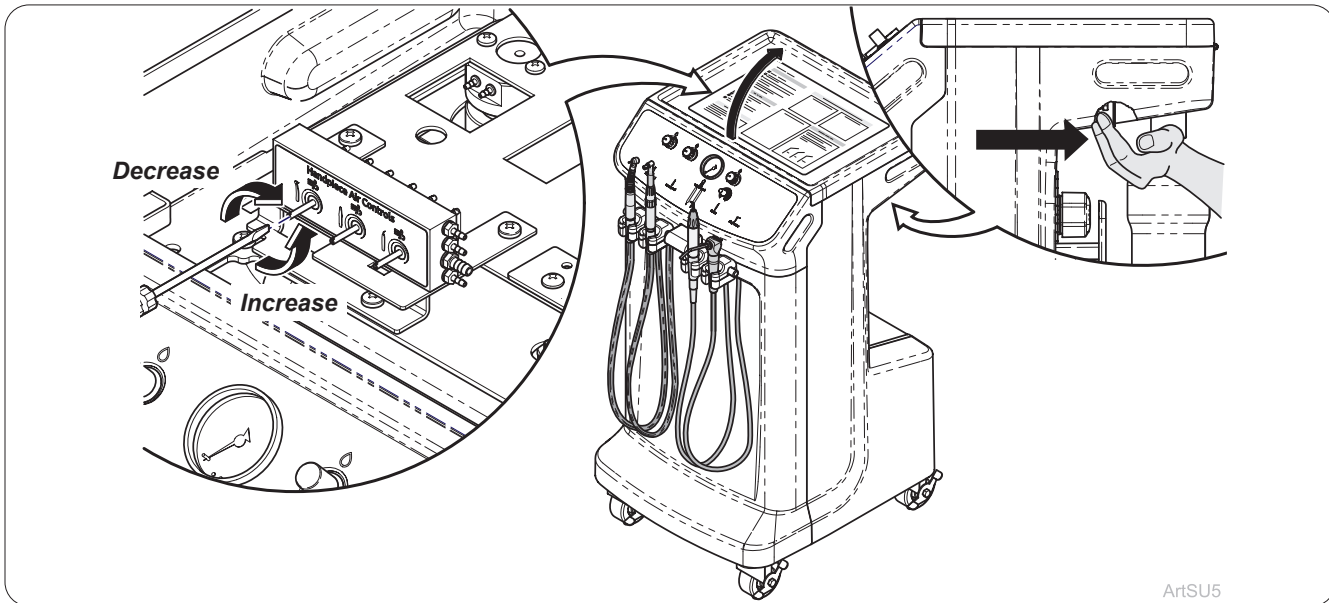Handpiece valves are factory set between 32-40 psi for high-speed and between 35-50 psi for low-speed.

Step 23: Handpiece Air Controls (if necessary)

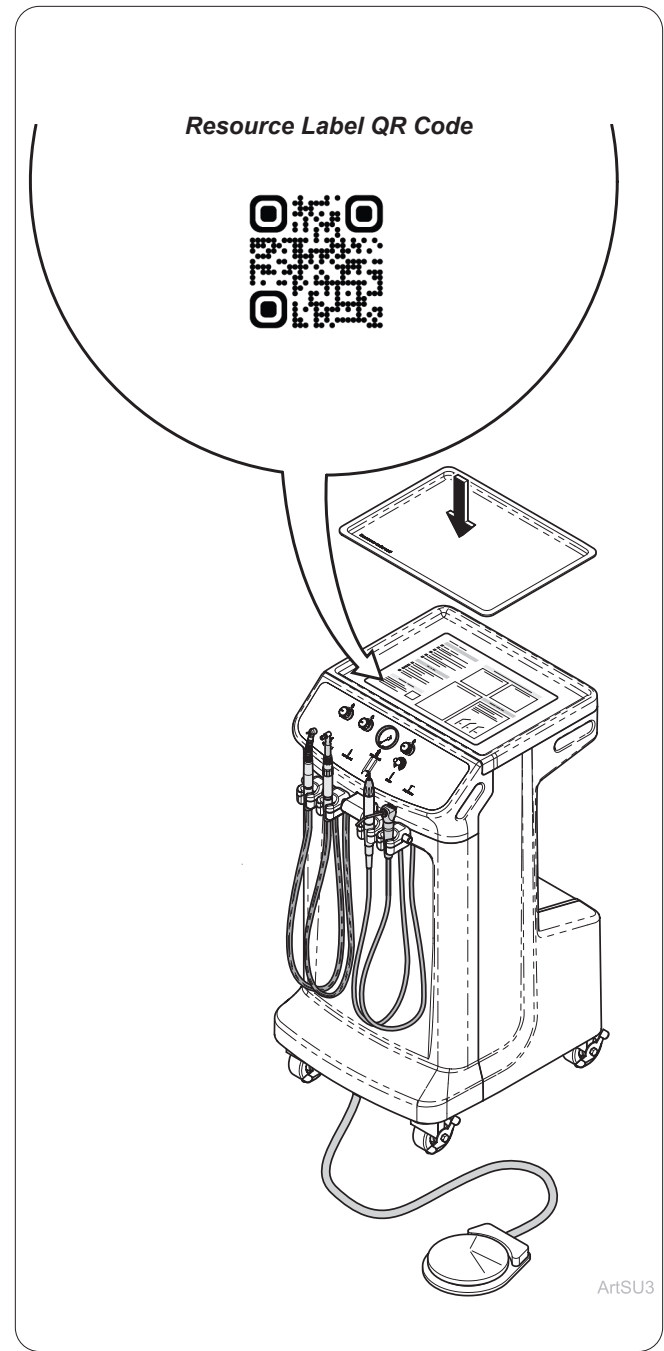Midmark handpiece pressure is factory set. Remove handpiece from holder and depress foot control to verify handpiece pressure (psi), adjust as needed under lid.

Step 24: Resource Label

Utilize resource label QR code to access product training videos, care and maintenance details, reorderable items and clinical training. Place mat onto lid after QR code access.

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