

Oil-Less Compressor Maintenance Kit

Models P52, P72 [002-10805-00, 002-10806-00]

Applies to Models:
P52 (all)
P72 (all)




WARNING

Disconnect compressor at electrical power. Drain storage tank of all air and liquid.

Components:

Coalescing Filter Maintenance kit
Intake Filters (2 per head)

Step 1: Disconnect power to compressor and drain storage tank of all air and liquid.

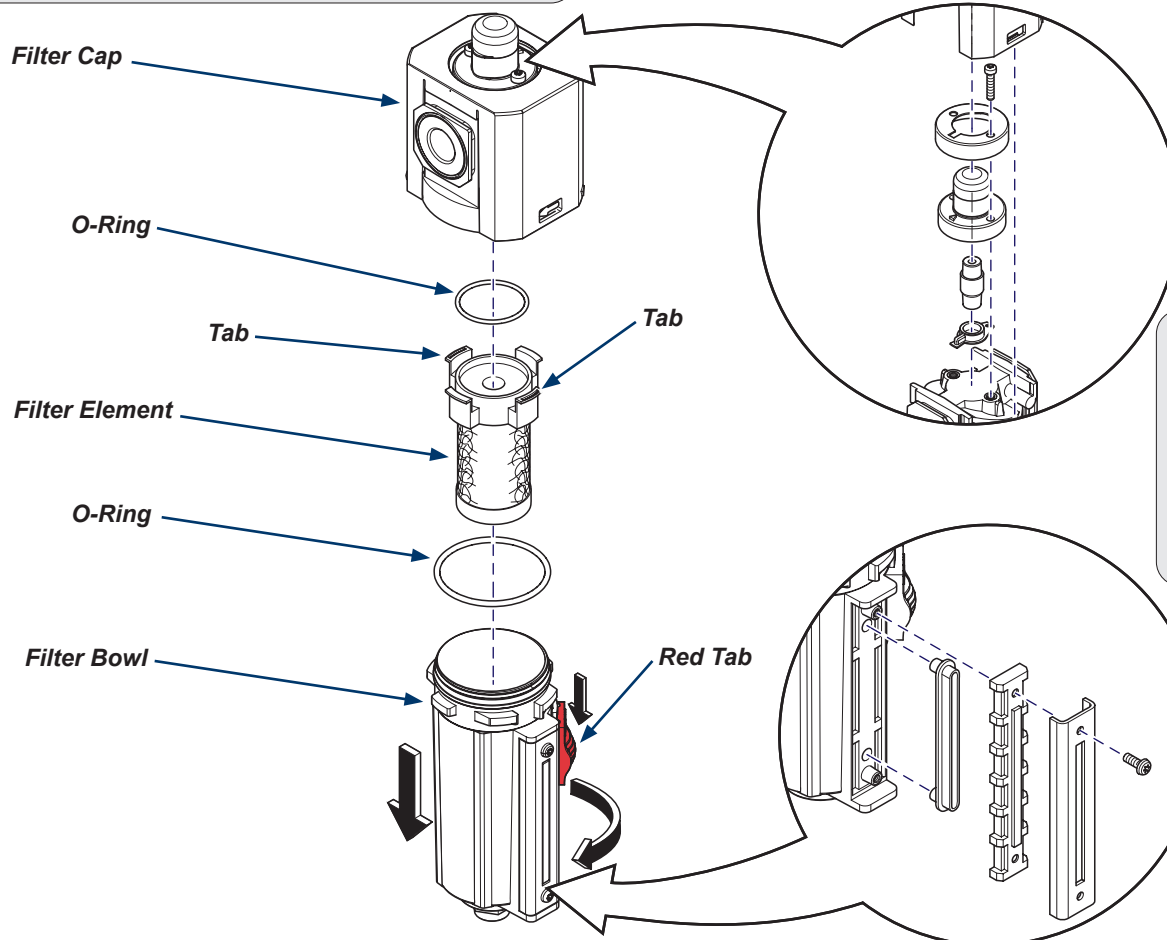

Note

If air flow indicator is malfunctioning or leaking use included parts to repair / replace.

Step 3: Replace filter element and o-rings.

- Remove o-ring from top of filter bowl and install new one.
- Remove o-ring from inside of filter cap and install new one.
- Remove old filter element and place new filter element in the bowl and clip tabs onto top of bowl.

Note

If moisture level indicator is malfunctioning or leaking use included parts to repair / replace.

Step 2: Remove Filter Bowl.

- Push down on red tab.
- Rotate filter bowl to the stop and pull down.


Step 4: Reinstall Filter Bowl.

- Line up and insert filter bowl up into filter cap and rotate opposite direction to lock in place.

ArtFull

Step 5: Replace both intake filter elements.

- A) Firmly grab rubber boot, twist, rock and pull outward.
- B) Remove intake filter from rubber boot.
- C) Insert new intake filter fully into rubber boot.
- D) Press into compressor head.
- E) Firmly press around rubber boot flange to secure in place.

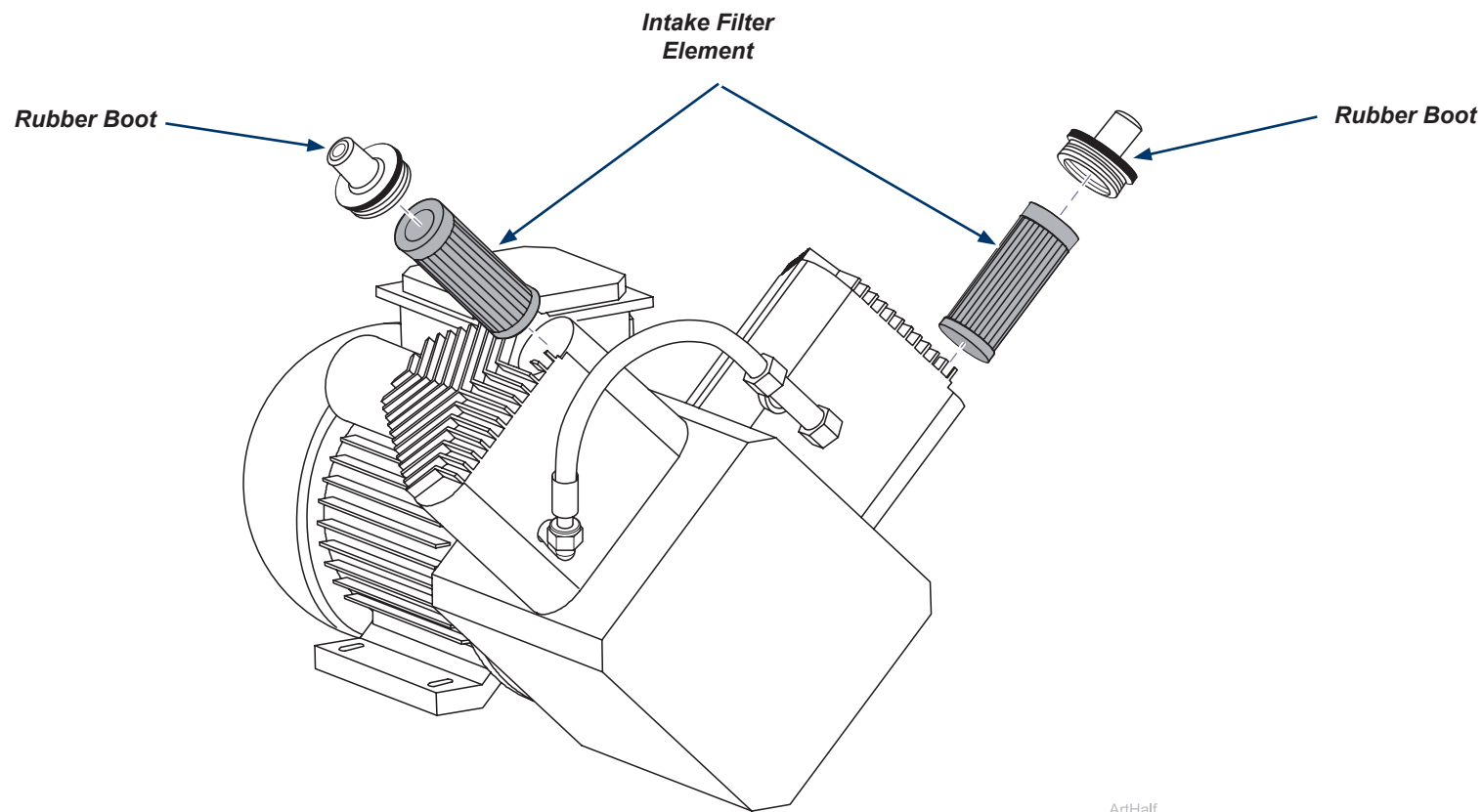

ArtHalf