WARNING

To prevent risk of shock always disconnect power before removing covers or performing any service procedure.


WARNING

Allow sterilizer to cool to ambient / room temperature prior to removing cover. Release pressure by pulling ring of pressure relief valve if necessary.


Step 1: Remove top cover access panel.

- Push a flat blade screwdriver firmly down into the notch in the access panel.
- Use screwdriver to pry up access cover and remove.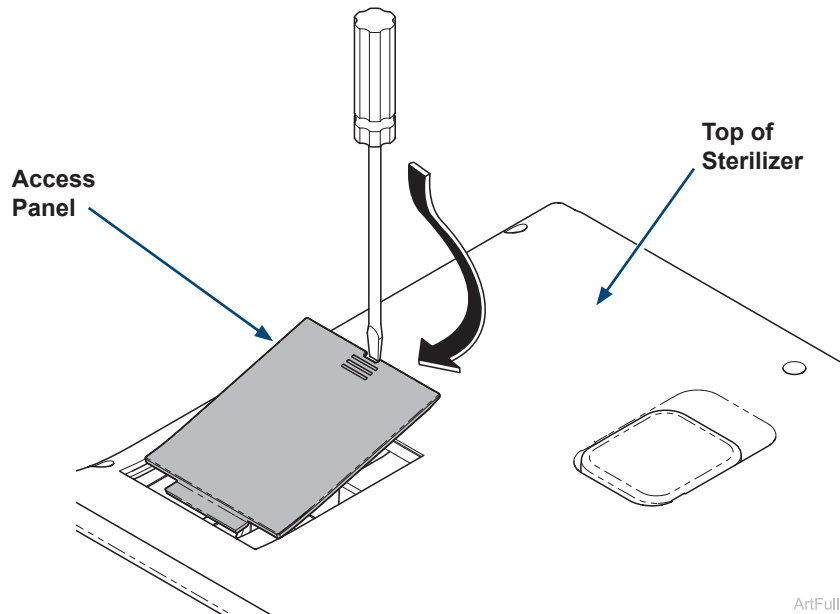


Step 2: Remove auto fill connection port plug.

- Locate connection port plug in top of the reservoir.
- Pull up to remove connection port plug.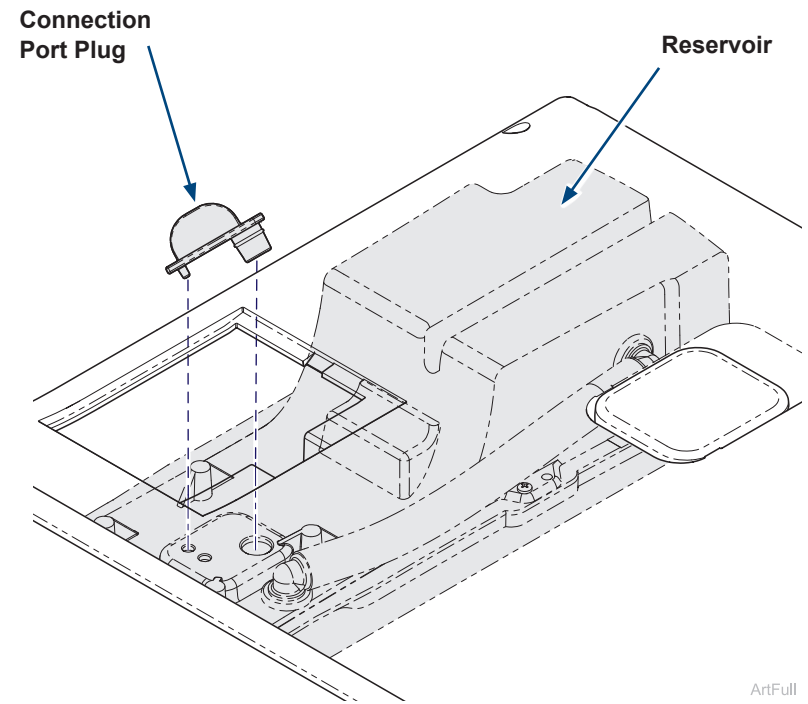


Step 3: Route tubing + attach to auto fill assembly.

- A) Push tubing into the hole in the top right corner of the back cover.
- B) Keep pushing until the tubing travels through the groove in the reservoir and out the access opening.
- C) Insert the tubing firmly into the quick connect fitting located in the back of the auto fill assembly.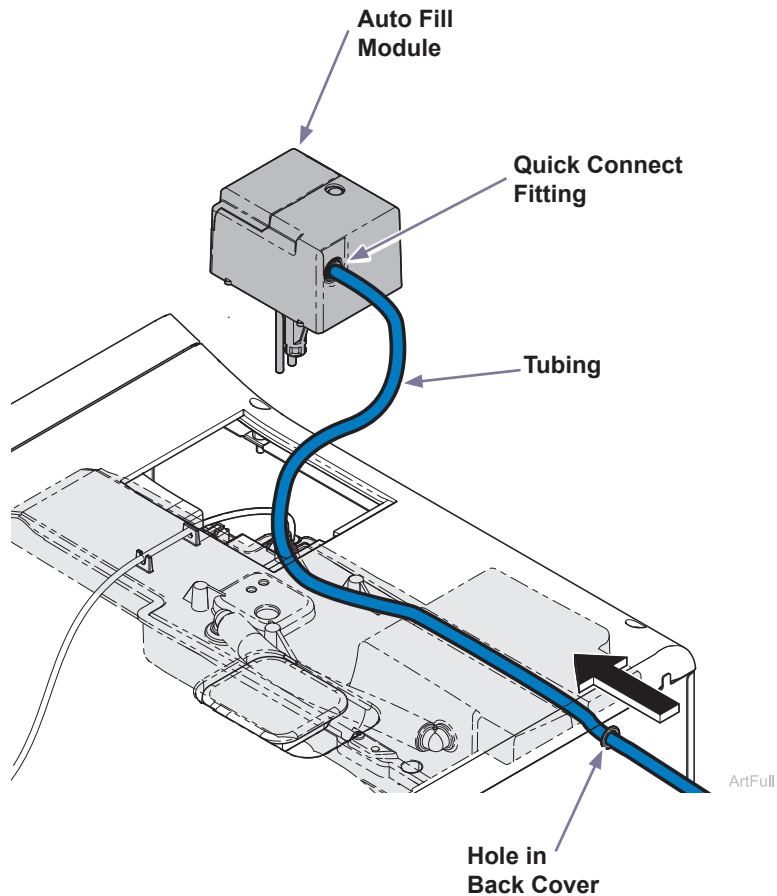


Step 4: Connect auto fill assembly wire harness.

- A) Locate sterilizer connector on the side of the access opening.
- B) Connect auto fill assembly wire harness to sterilizer connector.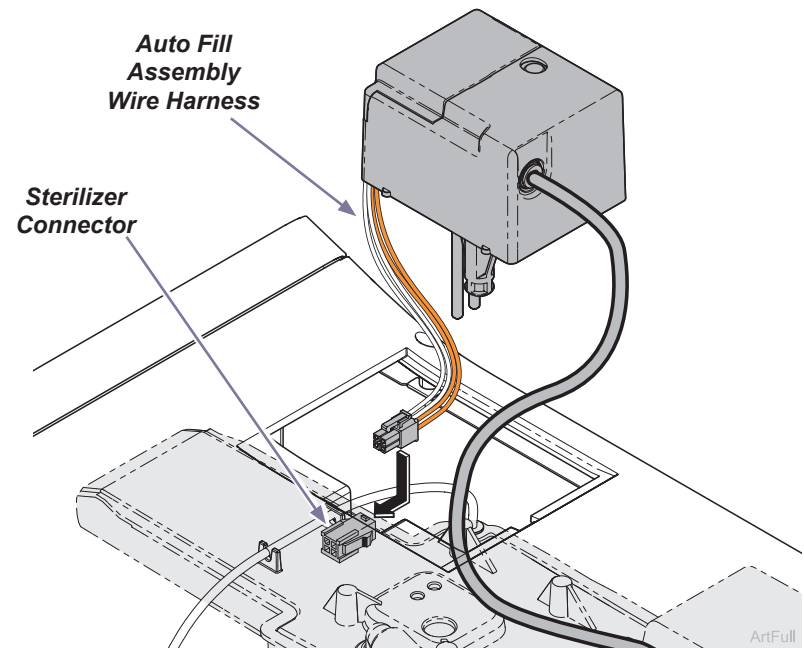

Step 5: Install auto fill assembly.

- A) Line up water level sensor bars and water valve in holes of the reservoir.
- B) Keep tube and wire harness from being damaged.
- C) Insert auto fill assembly into reservoir and push down until it clicks into place.

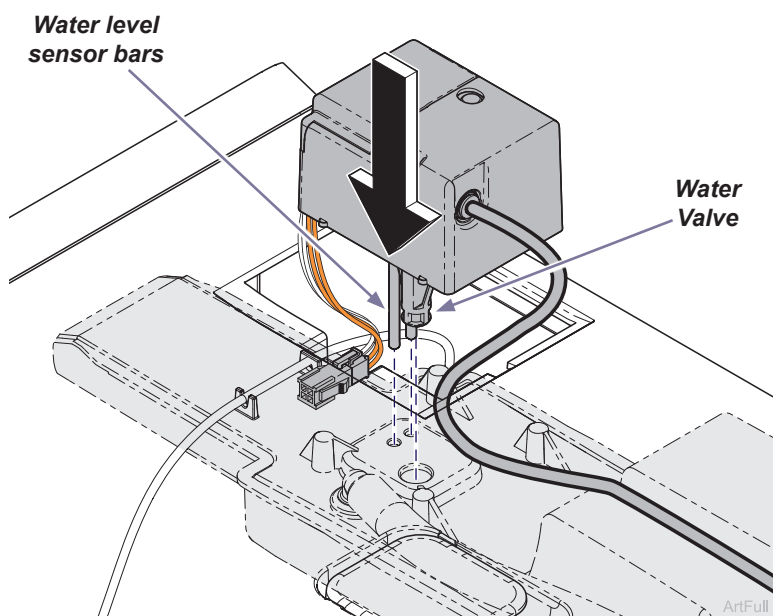Step 6: Attach tubing to water source.

- A) Attach tubing to water source that meets water purity specifications.

Note

Adapter fitting must be compatible with 6.35 mm (0.25 in) outer diameter x 4.04 mm (0.159 in) inner diameter polyurethane tubing, 90 Shore 'A' tubing hardness.

Note

Required water pressure is between 45 and 85 PSI.

Water Purity Specifications

Water Quality Specification Chart		
AAMI ST108 (ref.)		
	Units	Specification
Hardness	mg/L	< 1
Conductivity	µS/cm	< 10
pH @ 25C	pH	5 - 7.5
Chlorides	mg/L	< 1
Bacteria	CFU/mL	< 10
Endotoxin	EU/mL	< 10

Step 7: Enable the auto fill feature.

- A) Plug in sterilizer.
- B) From main screen press the menu button ☰.
- C) Select settings menu ⚙️.
- D) Select water menu 💧.
- E) Toggle auto fill to the on position.
- F) Confirm auto fill accessory is installed and save.
- G) Verify the auto fill is functioning. Drain some water out of the drain tube if needed to check operation.

Note: Water should fill reservoir to the water level sensor bars and shut off.

Caution

Check for leaks at both the connection to the Auto Fill unit and at the water source connection. Improper connections could result in facility water damage.

LAST CHANGED
05/22/2023


NEXT CHANGE DUE
05/25/2023

AUTO-FILL ⓘ
☐ Off

Save

Step 7: Install new access panel cover.

- A) Line up front edge of access panel and drop into place.
- B) Push down on back on access cover until it clicks into place.

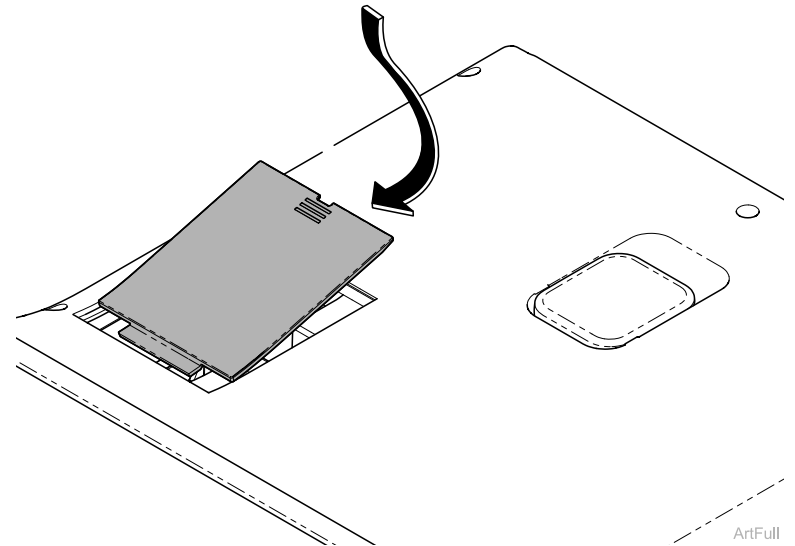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