WARNING

To prevent risk of shock always disconnect power before removing covers or performing any service procedure.

WARNING



Allow sterilizer to cool to ambient / room temperature prior to removing cover. Release pressure by pulling ring of pressure relief valve if necessary.

Step 2: Remove right side cover.

- Remove top two right side screws and loosen right rear bottom screw.
- Pull corner tab out of corner slot and rotate top of side cover down to free bottom tabs from base of sterilizer.


Note: Completely removing the right side cover is not required for installation of this kit.

Step 1: Disconnect power.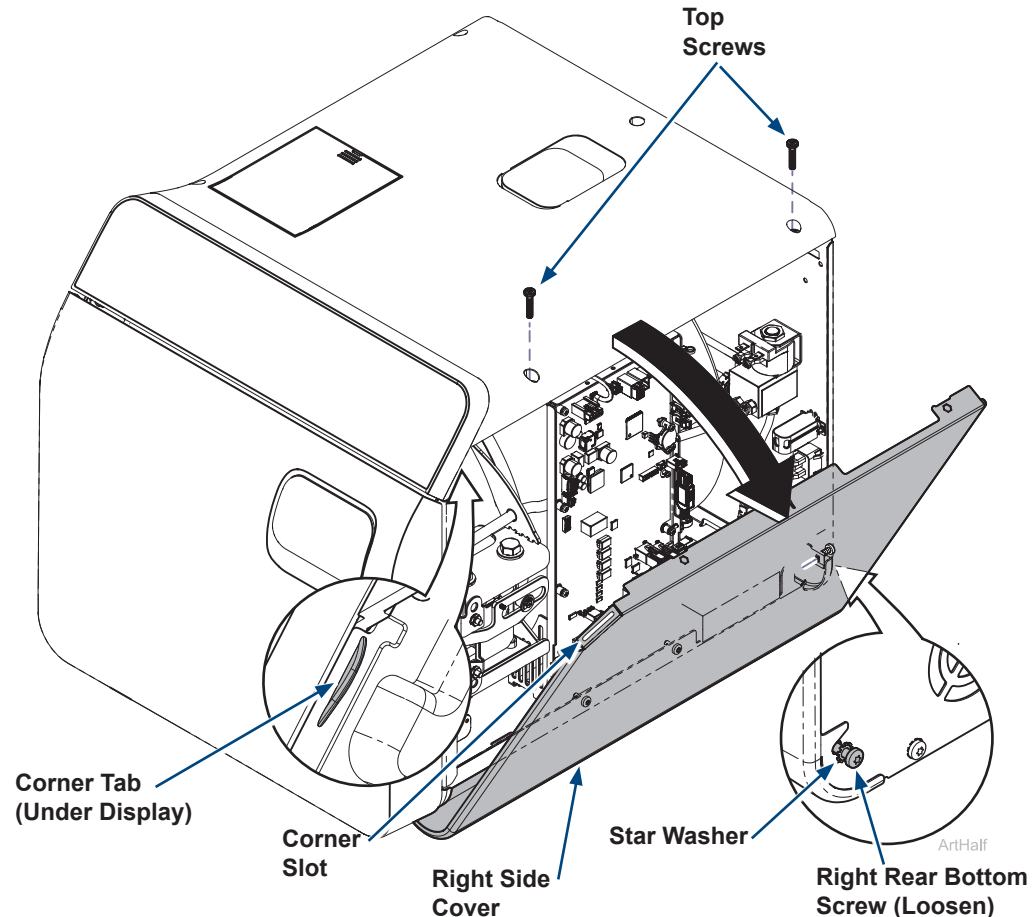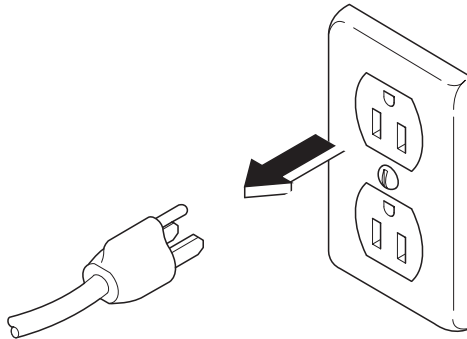


Step 3: Secure USB PCB board to back cover.

- A) Line up USB PC board to the three holes in the back cover.
- B) Secure USB PC board with three screws.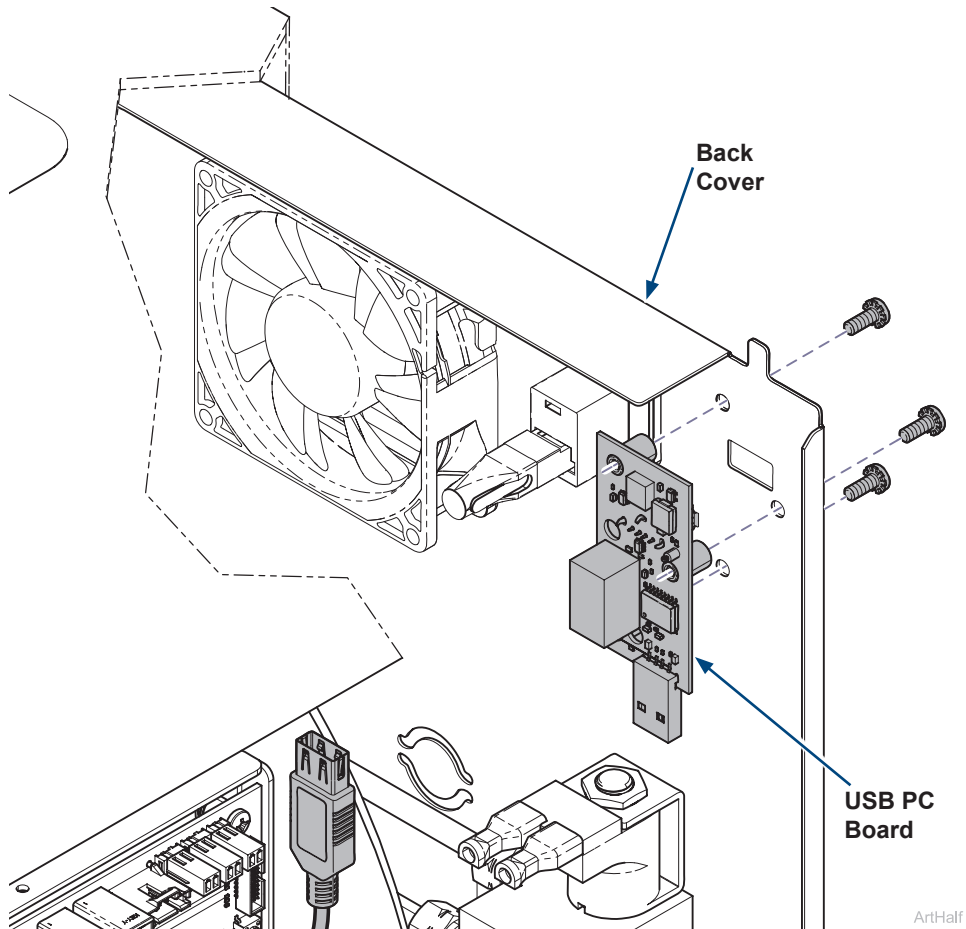

Step 4: Connect USB cable.

- A) Locate USB cable on right side of main PC board.
- B) Cut the cable tie holding the USB cable to the right side of the PC board.
- C) Attach USB cable to bottom of USB PC board.

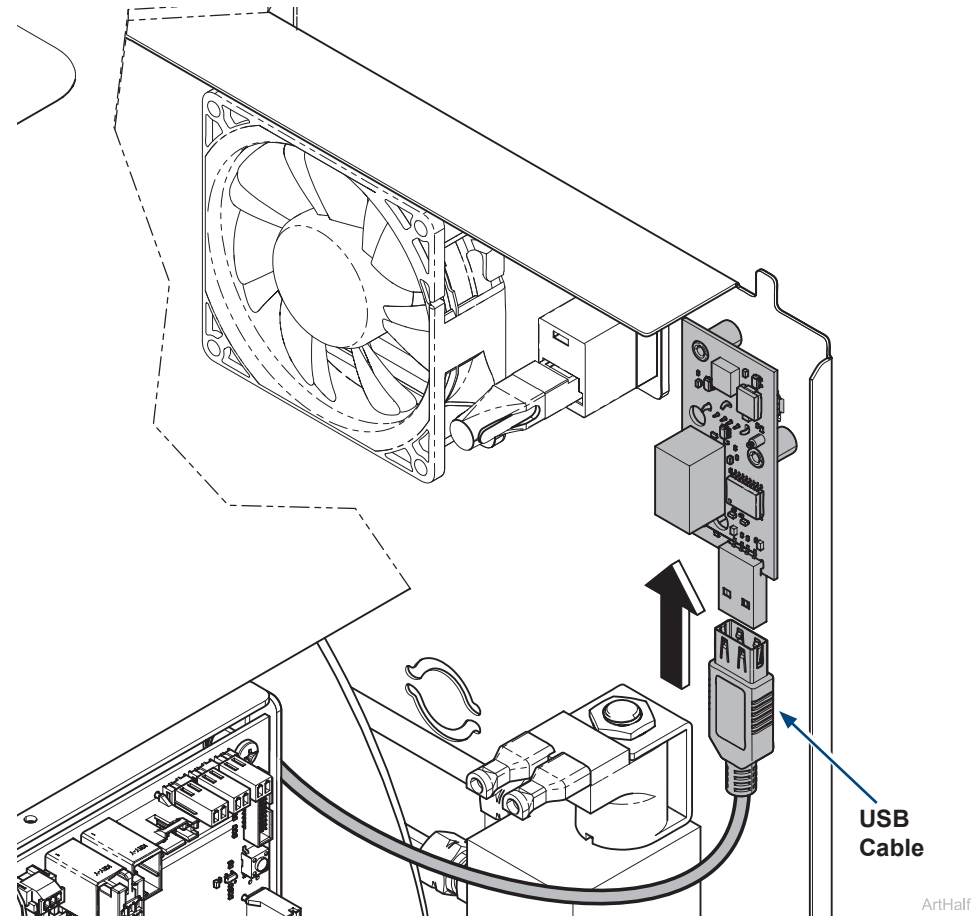
Step 5: Install right side cover.

- A) Insert bottom tabs of right side cover into base of sterilizer.
- B) Rotate top of side cover up into position and snap corner tab into corner slot.
- C) Install top two right side screws and tighten right rear bottom screw.
- D) Run a test cycle and check for leaks.


WARNING

For effective grounding make sure star washer is located between screw head and tab of side cover when tightening right rear bottom screw.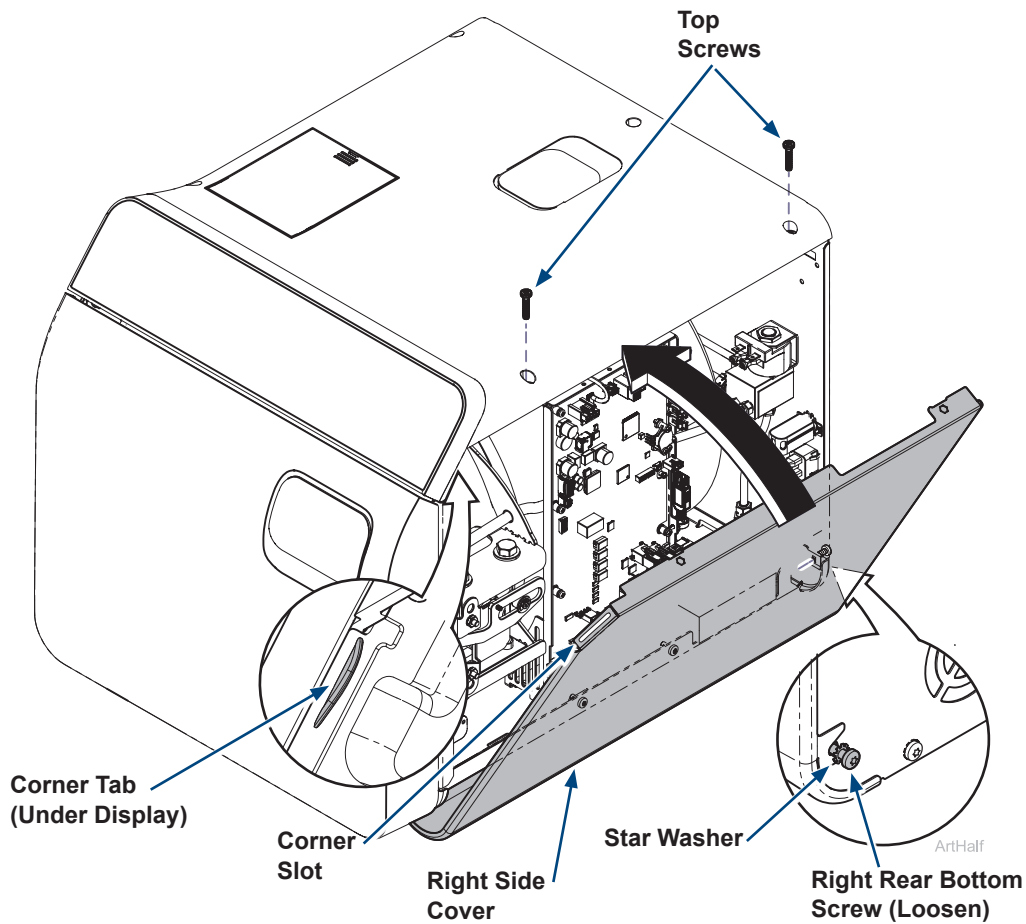

Step 6: Install label.


- A) Attach USB port label to the back of the sterilizer.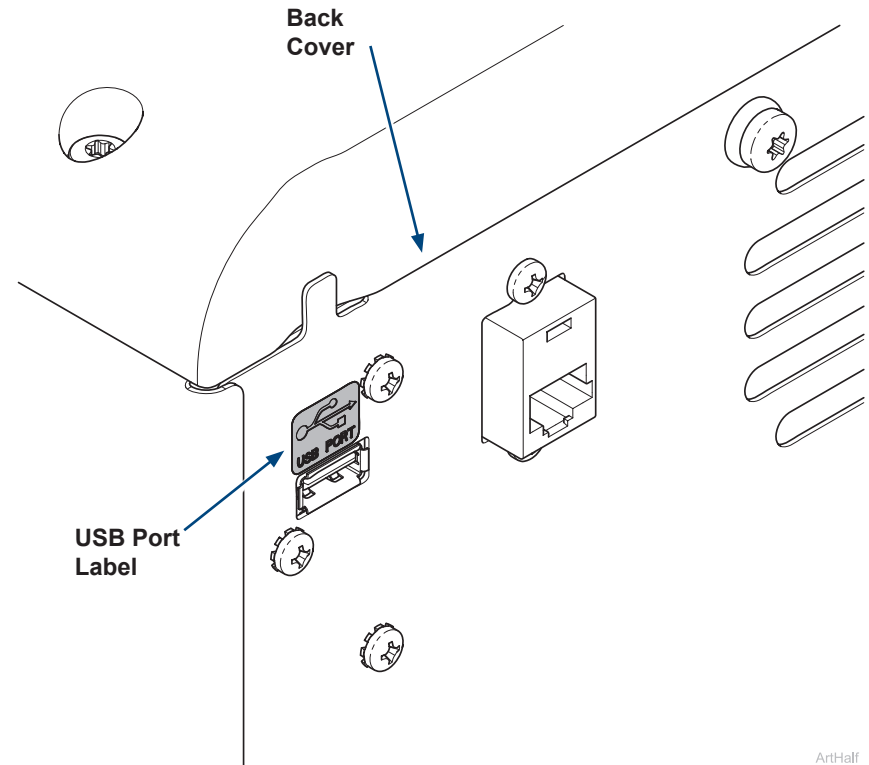

Step 7: Install external USB cable if desired.

A) Install external USB cable and route around the side of the sterilizer if desired.

Note: The USB drive can be inserted directly into the USB port on the back of the sterilizer or into the optional external USB cable to export records.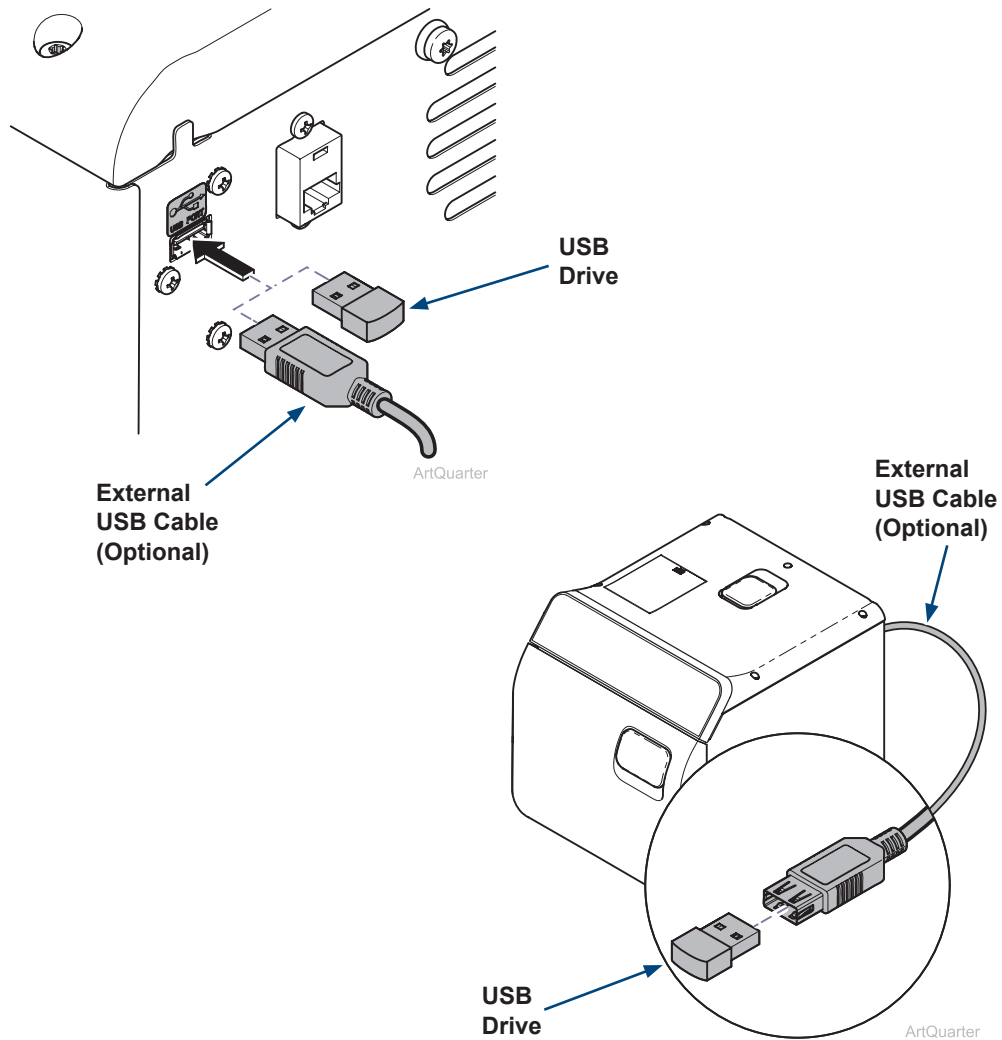

Step 8: Export records.

A) Plug in sterilizer.

B) From main screen press the menu button ≡.


C) Select cycle history menu 🕒.

D) Select download icon ⬇️.

E) Toggle on or off the records to be down loaded.

F) Insert the Midmark USB drive into the back USB port.

G) Press the start export button.

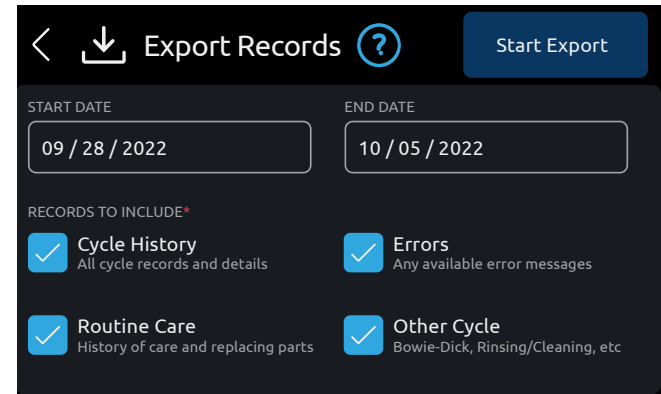USB Drive.

- To ensure compatibility only use USB drives provided by Midmark for the sterilizer.
- Midmark recommends backing up contents of the sterilizer to a PC daily, or at least weekly at a minimum.
- The USB drive is used to transfer data from sterilizer to PC for backup.

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