


Quick Reference Guide


Midmark® Digital ECG

- 01** In IQmanager®, **Select** a patient from the Patient List or **Select** New Patient and enter demographics and vital signs. (Patient's date of birth and sex must be correct.)
- 02** **Prepare** the patient for the test. **NOTE:** Proper patient preparation is essential for minimized artifact, a clean ECG and accurate results.
 - a. Shave** hair (if necessary) from electrode sites. (Refer to the opposite side of this card for proper placement.)
 - b. Clean** skin with a mild soap and water. Let air dry.
- 03** **Adhere** electrodes on the patient.
- 04** **Attach** the ECG clips from the patient cable to the electrodes according to color coding.
- 05** **Instruct** the patient to relax in a supine position.
- 06** **Select** the New Test button.
- 07** **Select** the ECG button. **Enter** Technician, Physician and Indication. **Choose** the appropriate ECG Lead Group. **Select** OK.
- 08** Wait for ECG tracings to pass twice (~20 seconds) and the patient's baselines are stabilized. **Select** the Analyze button. (The ECG will be saved.)
- 09** **Select** the Review button. **Select** Exit or **Select** Print and then Exit.

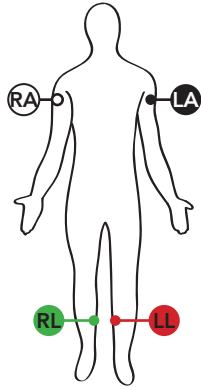

Limb Lead Placement

- RA (White)** – Right lateral side of the upper arm below the shoulder
- LA (Black)** – Left lateral side of the upper arm below the shoulder
- RL (Green)** – Inside calf, midway between knee and ankle
- LL (Red)** – Inside calf, midway between knee and ankle

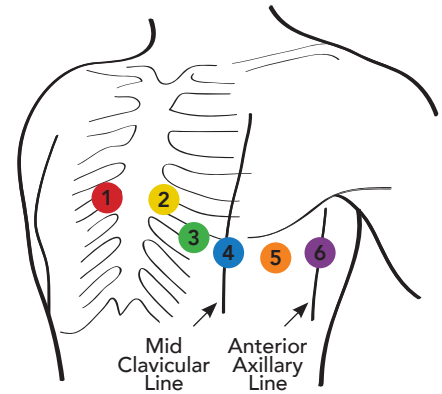

Standard 12-Lead Placement (Precordial)

- 1 V1 (Red)** – Fourth intercostal place at the right margin of the sternum
- 2 V2 (Yellow)** – Fourth intercostal place at the left margin of the sternum
- 3 V3 (Green)** – Midway between V2 and V4 (on top of the 5th rib)
- 4 V4 (Blue)** – Fifth intercostal place at the left midclavicular line
- 5 V5 (Orange)** – At the horizontal level of V4, at the left anterior line
- 6 V6 (Purple)** – At the horizontal level of V4, at the midaxillary line