




Specifications: Midmark M11® Steam Sterilizer

M11 (-050/-051/-053)	
Physical Dimensions:	
Overall Length with Plug	59.7 cm (23.5 in)
Overall Width	45.2 cm (17.8 in)
Overall Height	47.2 cm (18.6 in)
Height of Back	44.2 cm (17.4 in)
Support Surface (minimum)	55.4 cm x 67.3 cm (21.8 in x 26.5 in)
Chamber	27.9 cm dia. x 45.7 cm deep (11 in dia. x 18 in deep)
Standard Tray, Large (Usable tray area)	23 cm x 38.7 cm x 2.2 cm (9-1/8 in x 15-1/4 in x 7/8 in)
Standard Tray, Small (Usable tray area)	17 cm x 38.7 cm x 2.2 cm (6-3/4 in x 15-1/4 in x 7/8 in)
Weights:	
Weight with Empty Reservoir	44.9 kg (99 lb)
Shipping Weight	59.4 kg (131 lb)
Weight of Large Tray	431 grams (.95 lb)
Weight of Small Tray	349 grams (.77 lb)
Electrical Rating:	
<i>Note: A separate (dedicated) circuit is recommended for this sterilizer. Sterilizer should not be connected into an electrical circuit with other appliances or equipment unless circuit is rated for the additional load.</i>	
115 V models	115 VAC 12 Amp, 50/60 Hz
230 V models	230 VAC 6.4 Amp, 50/60 Hz
Heat Emission	5275 kJ (5000 BTU) / hr during operation
Water Reservoir Capacity	5.3 liters (1.4 gallons) to Full Mark
Pouch Rack	6 slot

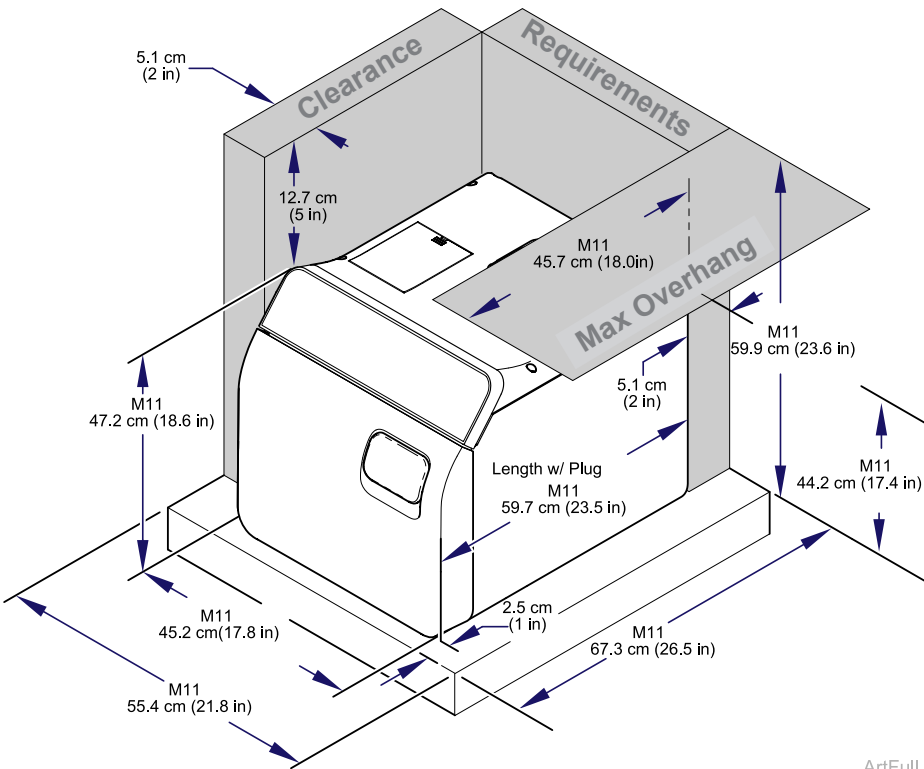

ArtFull

Support Surface:

- Surface should be water-resistant material (e.g. laminate, stainless steel, solid surface materials, stone).
- Surface must be level to ensure chamber fills with correct water level. Improper water level in the chamber could cause a sterilizer malfunction. Removing leveling feet shims can affect the height at the front of the sterilizer.

Clearance Requirements:

To ensure proper air circulation and to allow access to the reservoir fill port and drain coupling, adhere to the minimum clearance requirements listed below. If the sterilizer will frequently be operated without pauses between cycles, position unit where steam will not damage materials or equipment in the surrounding area. If the sterilizer will be used in an enclosed area and will frequently be operated without pauses between cycles, a direct to drain solution is recommended.

Back of Unit to Back Wall	5.1 cm (2 in)
Front Support Surface to Front Sterilizer	2.5 cm (1 in)
Sides of Unit to Side Wall	5.1 cm (2 in)
Distance above Unit	12.7 cm (5 in)
Maximum Upper Overhang (from back of sterilizer)	45.7 cm (18 in)
Minimum Height Under Cabinet or Shelf	59.9 cm (23.6 in)