

# M9® and M11® Right Side Cover Kit

002-10875-00 M9  
002-10876-00 M11

## Applies to Models:

M9 (-050/-053)

M11 (-050/-051/-053)

## Special Tools:

T25 Torx Screw Driver



### WARNING


To prevent risk of shock always disconnect power before removing covers or performing any service procedure.



### WARNING


Allow to cool to ambient / room temperature prior to removing cover. Release pressure by pulling ring of pressure relief valve if necessary.

## Step 1: Disconnect power.

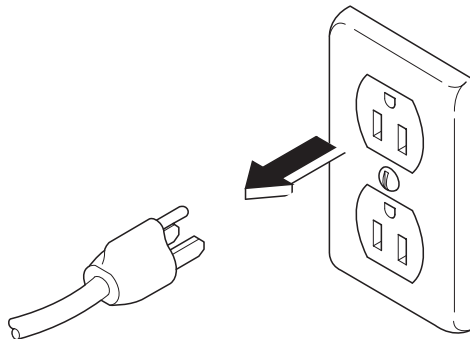

## Step 2: Remove right side cover.

- Remove top two right side screws and loosen right rear bottom screw.
- Pull corner tab out of corner slot and rotate top of side cover down to free bottom tabs from base of sterilizer.

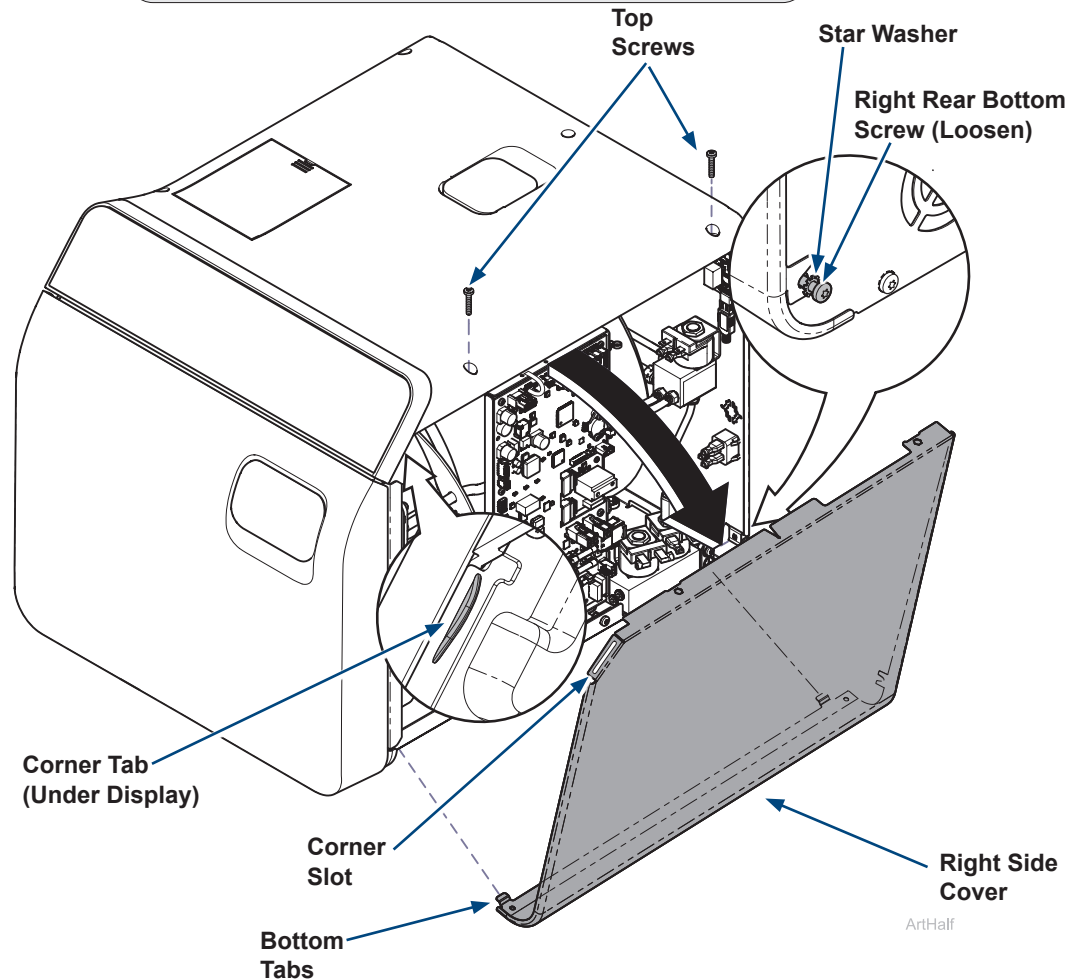


### Step 3: Install right side cover.

- A) Insert bottom tabs of right side cover into base of sterilizer.
- B) Rotate top of side cover up into position and snap corner tab into corner slot.
- C) Install top two right side screws and tighten right rear bottom screw.



#### WARNING

For effective grounding make sure star washer is located between screw head and tab of side cover when tightening right rear bottom screw.

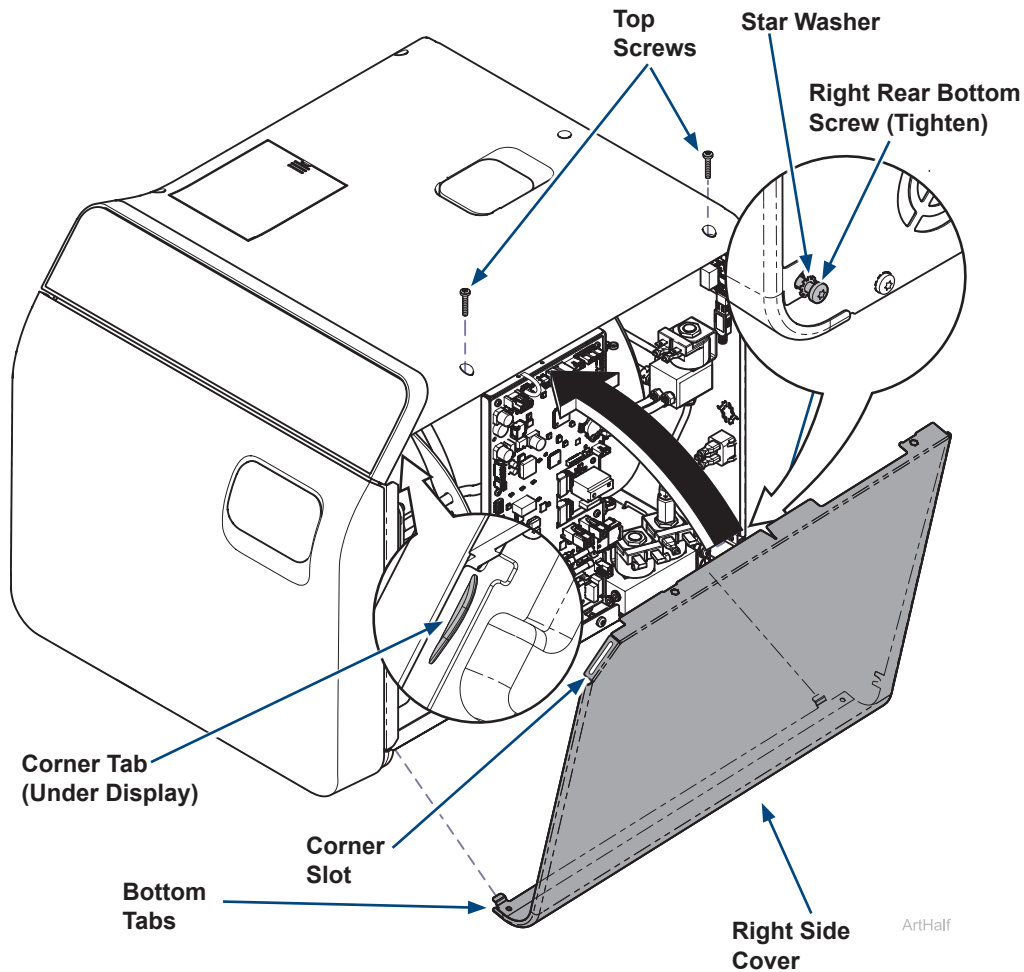

ArtHalf