

## M9® and M11® Door Gasket Kit

**M9** 002-10879-01

**M11** 002-10880-01

### Applies to Models:

M9 (-050/-053)

M11 (-050/-051/-053)

### Special Tools:

none




### **WARNING**

Allow to cool to ambient / room temperature prior to removing cover.  
Release pressure by pulling ring of pressure relief valve if necessary.



### **Step 1: Remove Door and Dam Gaskets.**

- Allow sterilizer to cool and open door.
- Remove dam gasket by pulling it off the door studs.
- Remove door gasket by pulling it out of the gasket housing.
- Clean the gasket housing and the face of the chamber.

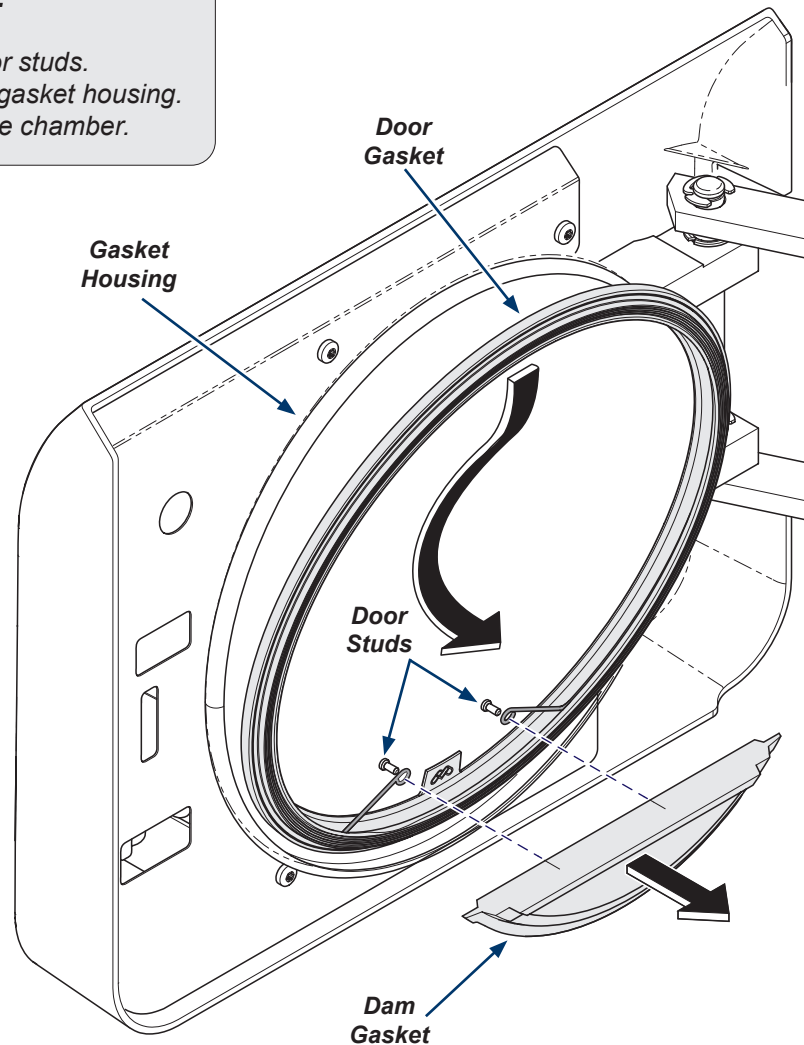

### Step 2: Install Door Gasket.

- A) Install door gasket into bottom of the gasket housing and center tab between door studs.
- B) Install left and right sides and finish at top.
- C) Install gasket ring eyelets onto door studs, work gasket ring into v-groove of door gasket. Start on left and right sides and finish at top.
- D) Make sure door gasket fits evenly and flush all the way around gasket housing.

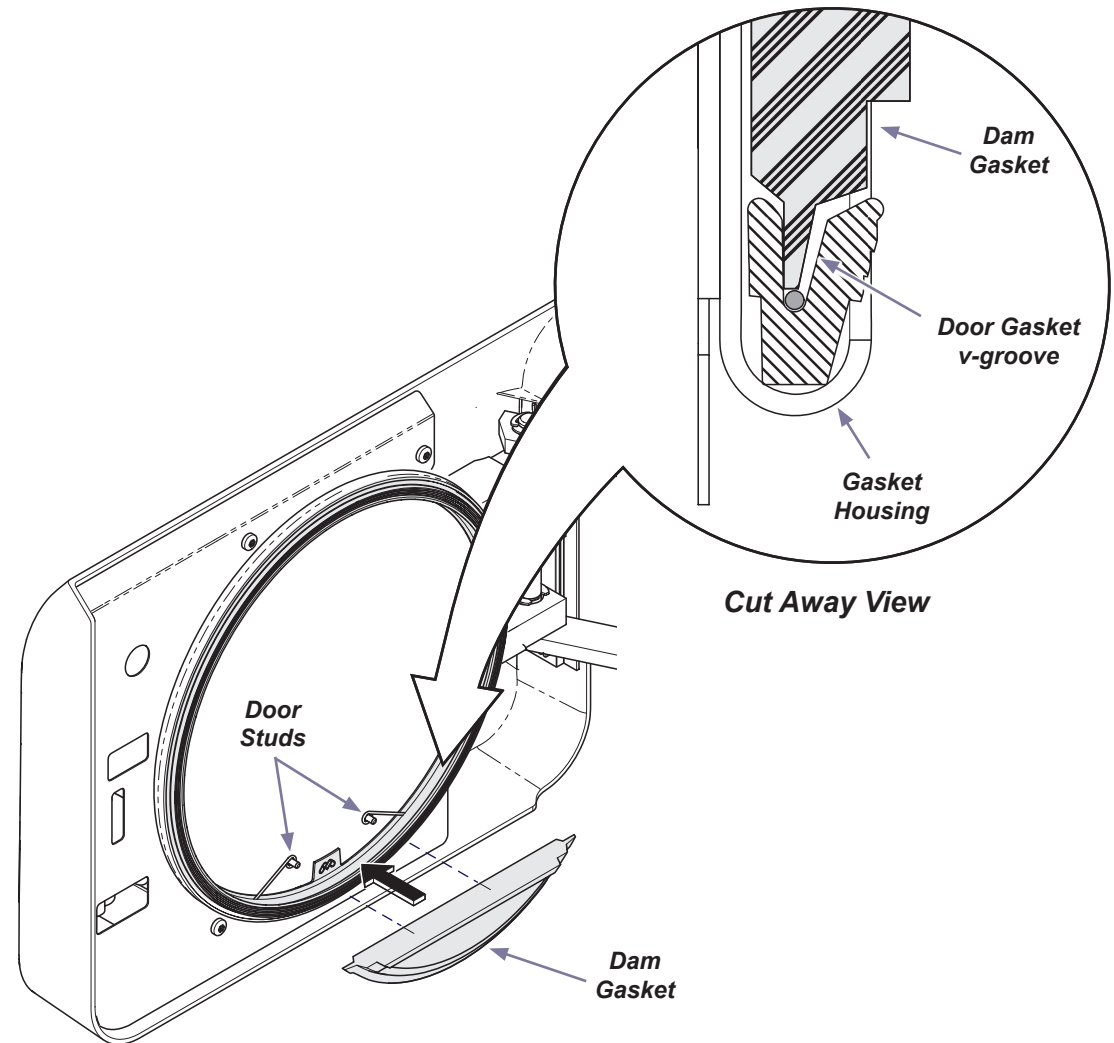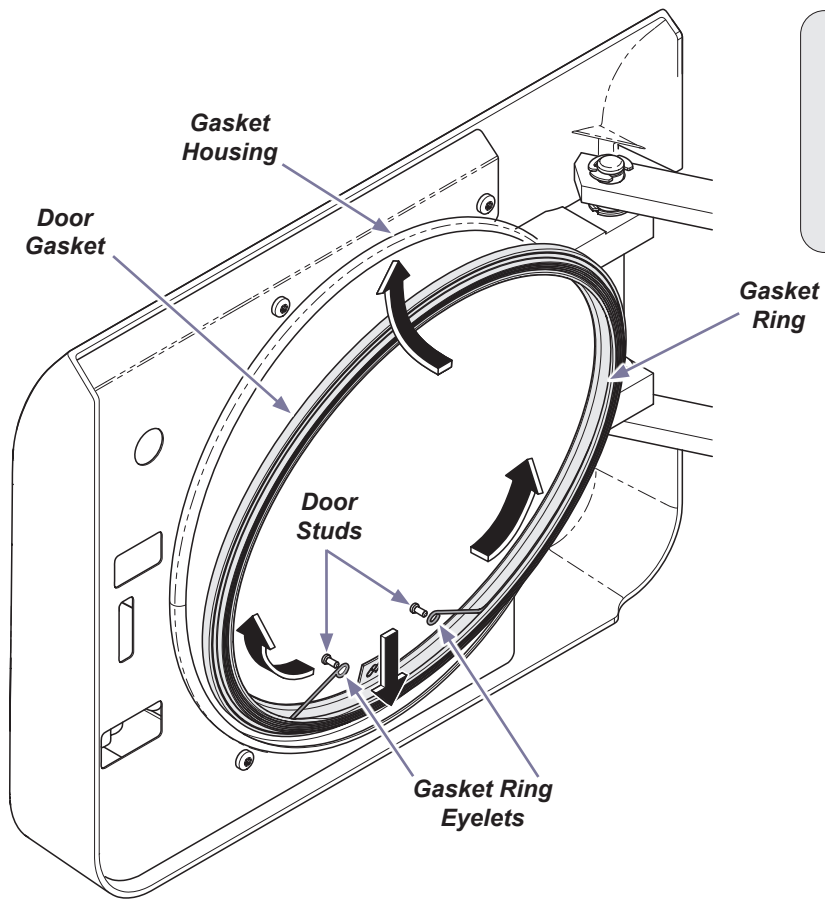

### Step 3 : Install Dam Gasket.

- A) Align dam gasket holes over the door studs.
- B) Tuck bottom edge of dam gasket into the center v-groove of door gasket.
- C) Push dam gasket on to studs until it lies flat against the door.