



Maintenance Kit

M9
M11

002-10877-00
002-10878-00

Applies to Models:
M9 (-050/-053)
M11 (-050/-051/-053)

PART	QTY	NOTES:
Door Gasket	1	Remove old gasket. Clean the gasket groove on the door and the chamber face. Install door gasket into bottom of the gasket housing and center tab between door studs. Install the left and right sides and finish at the top. Make sure that the gasket fits evenly and flush all the way around.
Door Gasket Ring	1	Install gasket ring eyelets onto door studs, work gasket ring into v-groove of door gasket. Start on left and right sides and finish at top.
Dam Gasket	1	Align dam gasket holes over the door studs. Tuck bottom edge of dam gasket into the center v-groove of door gasket. Push dam gasket on to studs until it lies flat against the door.
Filters	2	Remove trays and tray rack from chamber and clean. Clean chamber. Twist and pull to remove filters from holes located in bottom and back wall of chamber. Press firmly to seat new filters in holes.



WARNING
Allow sterilizer to cool completely
before installing Maintenance Kit.

