

PC Board Replacement [002-11054-00]

Components:
PC Board

Special Tools:
none



WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.



WARNING

To prevent risk of shock always disconnect power before removing covers or performing any service procedure.




Equipment Alert

Verify the serial label on the PC Board (Vxxxxxxx) matches serial number on the compressor.

To replace PC Board...

- A) Turn power off at main power supply box.
- B) Remove cover from electrical control box.
- C) Unplug wire harnesses from PC Board.
- D) Remove three pressure sensor tubes from PC Board.
- E) Raise clip on standoffs to release PC Board.
- F) Remove PC Board from electrical control box.
- G) Verify the serial label on the PC Board (Vxxxxxxx) matches serial number on the compressor.
- H) Install new PC Board into electrical control box.
- I) Connect wires and pressure sensor tubes.
Use extra pressure sensor hoses provided in kit if necessary.
- J) Install electrical cover and run a test cycle.
- K) To re-connect device, see connectivity instructions on the Midmark website.

Note: Refer to diagram on next page when reconnecting PC Board. Also refer to wire diagram on inside of electrical cover.

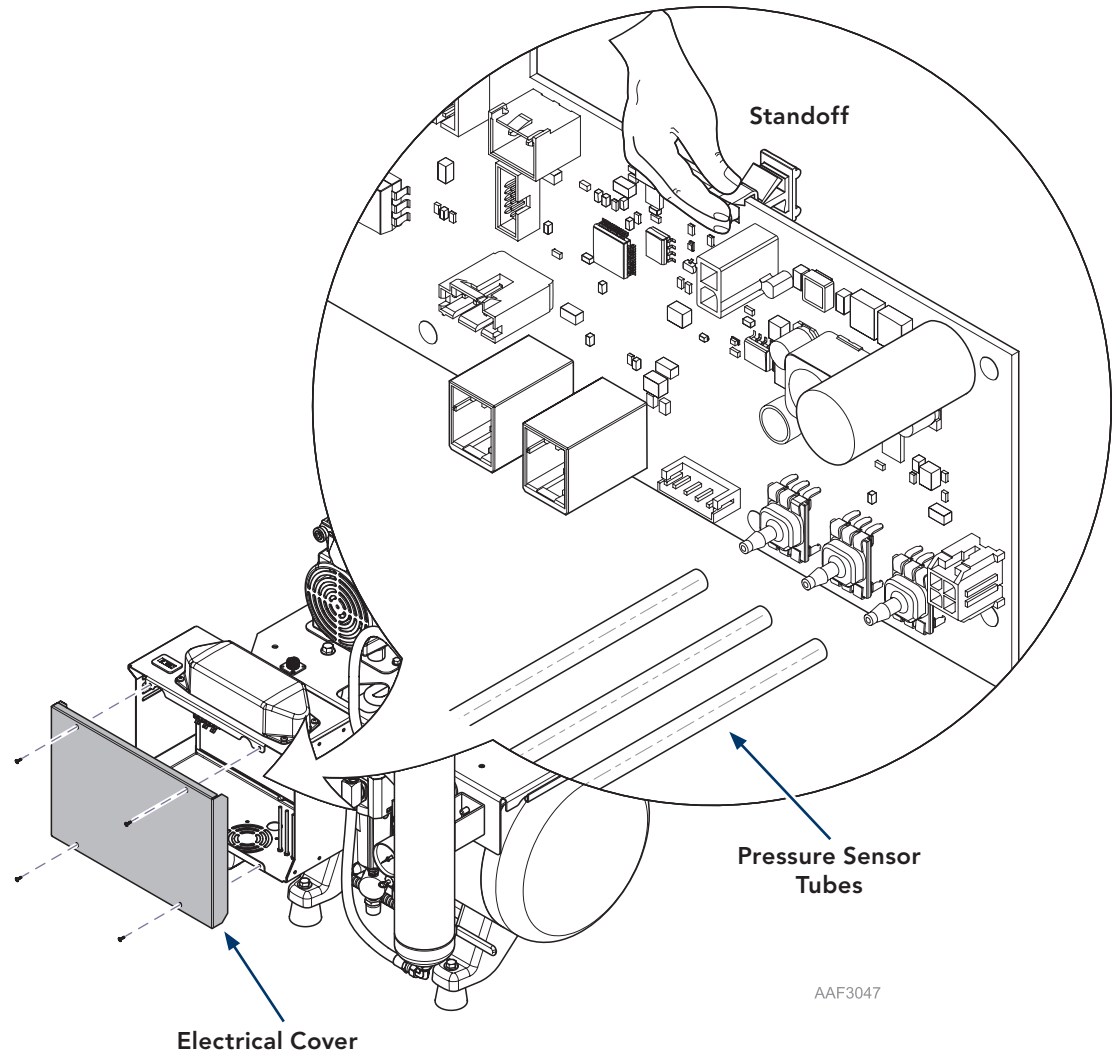

AAF3047

