

GA500 Drive Unit (VFD) [002-11053-00]

Applies to Models:

Single Vac Units

G3-050, G5-050, G7-050

Twin Vac Units

G6-050, G10-050, G14-050



WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.




WARNING

To prevent risk of shock always disconnect power before removing covers or performing any service procedure.

To replace VFD box...

- A) Turn power off at on/off switch and main power supply box.
- B) Remove electrical cover from base unit.
- C) Move Slide Lock on VFD cover to the Left. Push cover down and pull outward.
- D) Disconnect wires from VFD box.
- E) Remove VFD box from base unit.
- F) Install new VFD box into base unit.
- G) Connect wires and push cover back onto VFD box.
- H) Install electrical cover and run a test cycle.

Note: Refer to wire diagram on next page when reconnecting VFD box. Also refer to base unit wire diagram on inside of electrical cover.

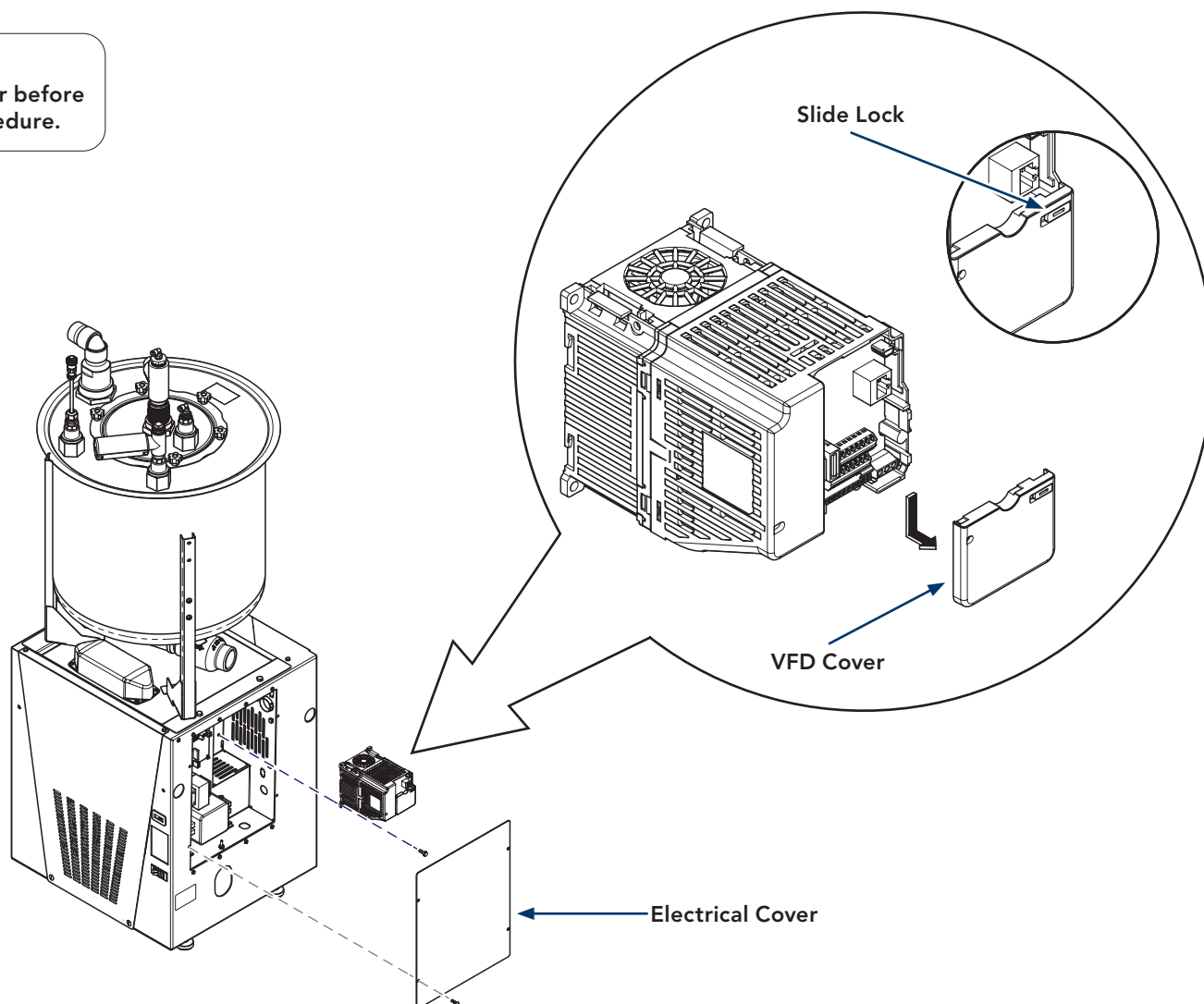

VFD Wiring Diagram

