

Provided on each trolley:

- (2) 1/4" set screw with lock patch.
- (2) 1/2" set screw with lock patch.
- (1) Locking pin.

Monitor Trolley

The Monitor Trolley uses (2) 1/4" long set screws. The Monitor Trolley does not have a stub extending to its bottom and utilizes a down tube with a larger outside diameter.

Note: If 1/2" long set screws are used, they will extend out of the trolley casting.
Indicator pin (2) 1/4" set screws

Light Trolley

The Light Trolley uses (2) 1/2" long set screws. The Light Trolley contains a stub extending to its bottom and utilizes a down tube with a smaller outside diameter.

Note: If 1/4" long set screws are used, they will fall into the trolley and not tighten the assembly.
Indicator / Lock Pin (2) 1/2" set screws



WARNING

Warning, The locking pin screw and set screws should only be installed after the down tube has been mounted and is hanging freely from the trolley. If the locking pin screw cannot be threaded into the trolley casting with the down tube mounted, it can indicate there is an error in the setup. Ensure that the correct set screws are used for each trolley. Failure to do so could result in damage to equipment or personal injury.

