


Quick Reference Guide
631-001

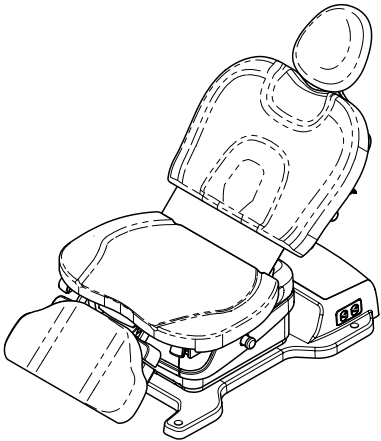

This guide outlines basic operation of the chair.
See the User Guide for additional instructions on
operation and care, as well as regulatory compliance
information.

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

- Clean upholstery with diluted bleach solution.
Ratio - 10:1 (water:bleach)
- Clean the chair weekly, wiping the painted metal
and plastic surfaces with a clean, soft cloth and mild
cleaner.

Calling for Service

If service is required, contact Midmark directly.


Medical US 844.856.1230 / Canada 937.526.8585
Monday - Thursday 8:00 AM to 6:00 PM ET
Friday 8:00 AM to 5:00 PM ET
midmark.com



Caution
Headrest must be securely locked in place
before performing any exam or procedure.


To adjust the headrest...

- A) Loosen locking knob.
- B) Pivot headrest to desired position.
- C) Tighten locking knob to lock in place.

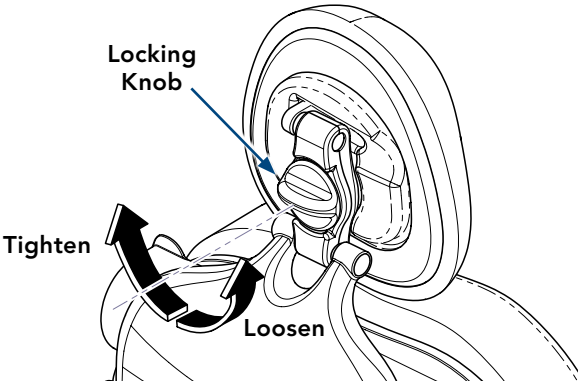


To extend / retract the foot section...

- A) Squeeze handle on either side of foot section.
- B) Move foot section in / out to desired position.
- C) Release handle to lock in position.

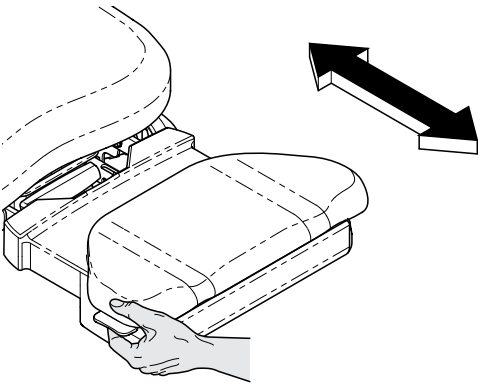

Note

The Foot Up and Down functions will be disabled when the debris tray is not in the
stowed position.
(You will hear a series of “beeps” if the debris tray is not in the stowed position when foot
Up / Down is activated.)

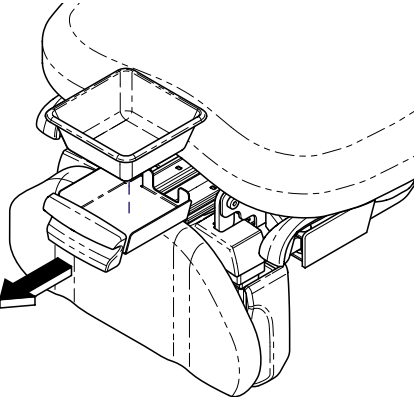

To access the debris tray...

- A) Ensure foot section is in the stowed position.
- B) Fully lower the manual foot extension.
- C) Pull the debris tray outward.



Equipment Alert

Operating more than two functions simultaneously in a maximum load condition can
cause the chair to exceed its current rating and may cause branch circuit overcurrent
protection (e.g. circuit breakers) to trip. Limit operation to no more than two functions
simultaneously under these conditions.

To operate the Base / Back / Tilt / Foot functions...

Press and hold the appropriate button.




003-10743-00 Rev AA2 (10/21/24)

Quick Chair™ Function
(w/ Foot Control only)

To activate the Quick Chair™ function...
Press and release the Quick Chair™ button.

Note: You will hear a single “beep” and the chair will move to the Quick Chair™ position.

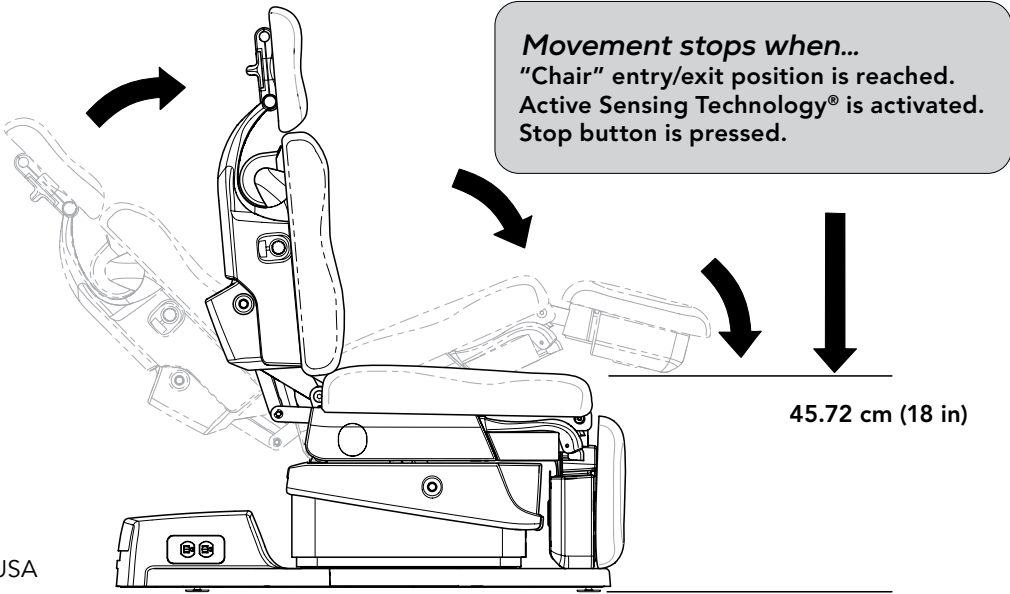

Home Function

Note
When using the home button that automates downward motion, chair will stop when a clearance from the chair back to the base enclosure reaches 12 cm (4.72 in).

To lower the base to 45.72 cm (18 in)...
Press and release the Home button.

Note: You will hear a single “beep” and the chair will move to the Home position.

