WARNING

This chair weighs approximately 229 kg (505 lb).
Use an assistant when lifting chair.


Equipment Alert

The minimum door clearance is 71.12 cm (28 in).
Use care in moving chair through doorways.

Step 1: Remove chair from skid...

- A) Remove four nuts and clamps.
- B) Carefully lift chair off of skid.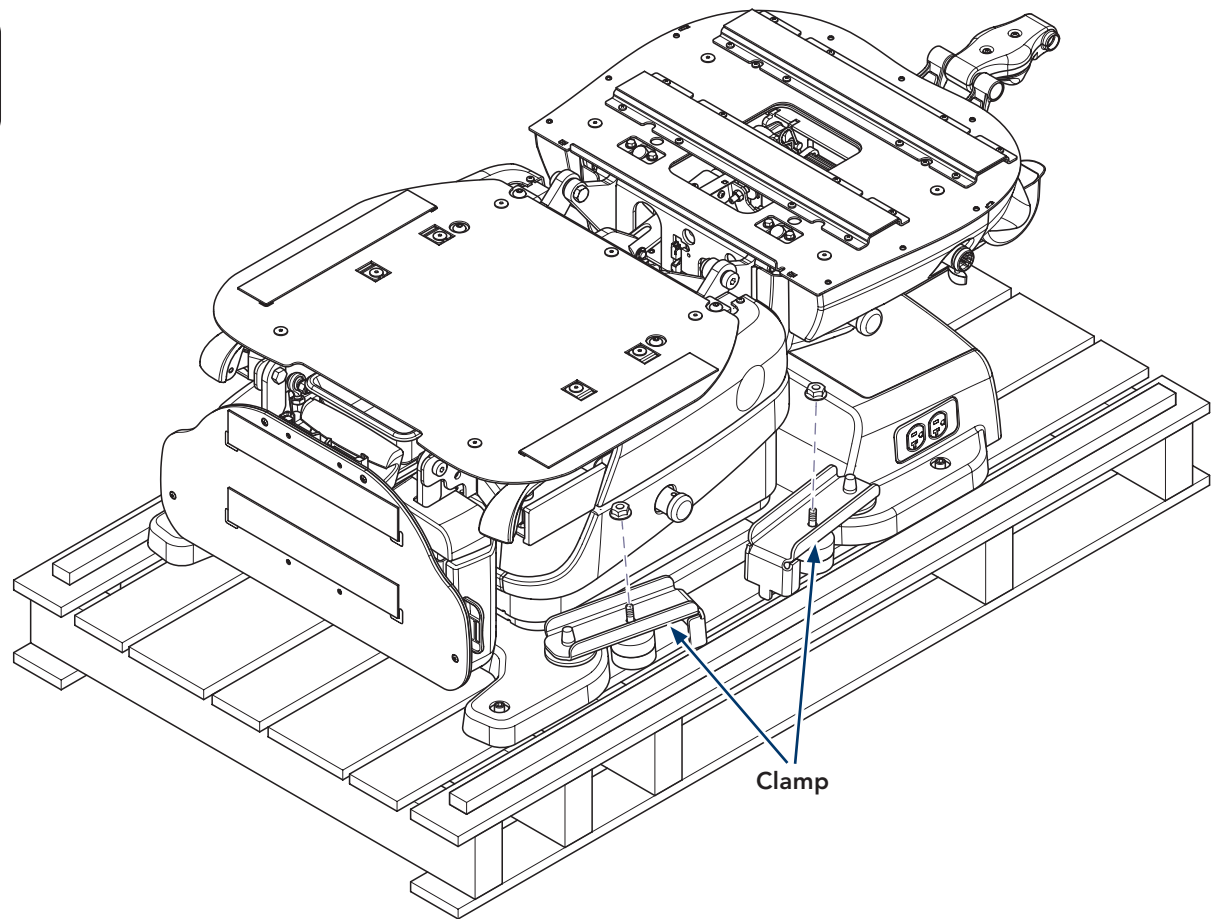



Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local authority having jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements.

Step 2: Install Hand Control Holster...

- A) Install two mounting pins and secure with thumb screws.
- B) Install holsters onto mounting pins.

Step 3: Connect Foot Control...

- A) Plug connector into modular jack at front of chair.

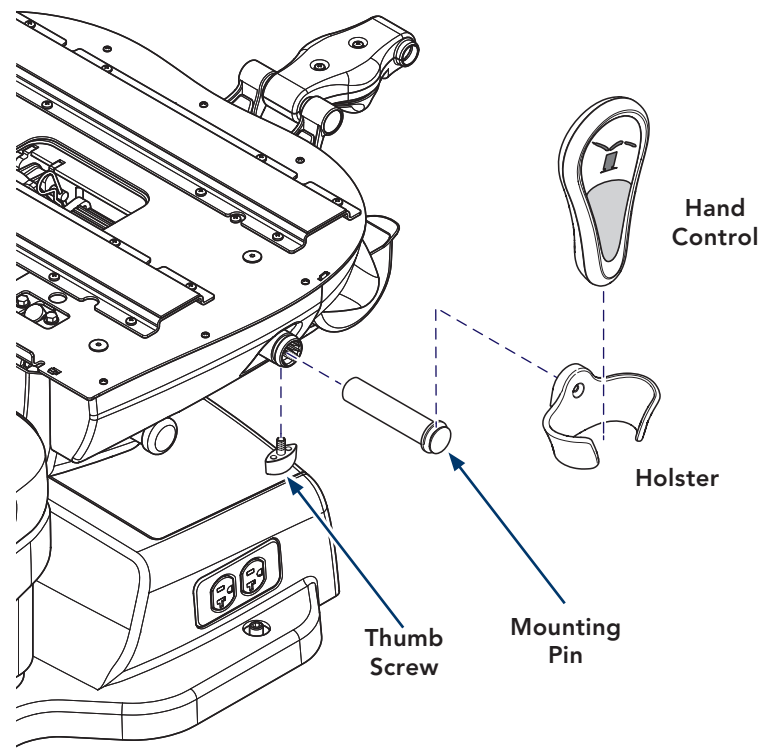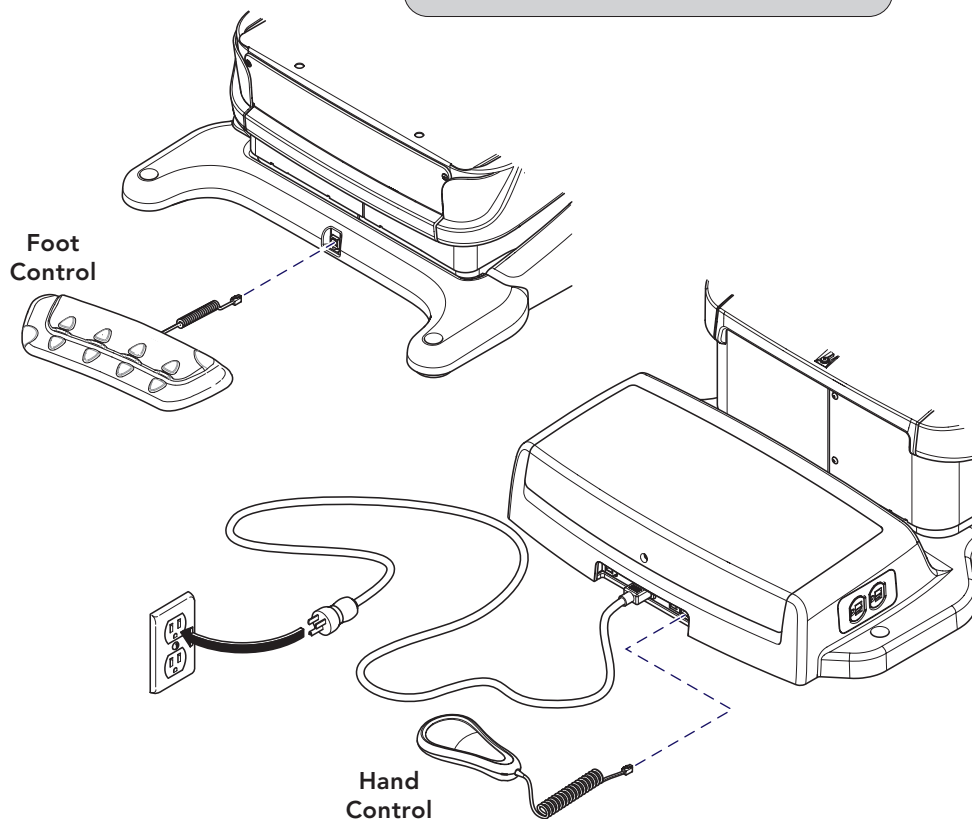Step 4: Connect Hand Control...

- A) Plug connector into modular jack on either side of AC inlet.

Note

Chair must be positioned in normal use such that the power cords can be easily accessed to unplug.

Step 5: Connect Power Cord...

- A) Plug square end into AC inlet.
- B) Plug three pronged plug into wall outlet.

Equipment Alert



The chair must be leveled to ensure proper operation.

Step 6: Level the chair...

- A) Using a level and Allen wrench, adjust leveling screws as necessary.
- B) Install four hole plugs.

Step 7: Remove foam supports...

- A) Remove two foam blocks from behind the footrest handles.
- B) Remove four base shroud supports.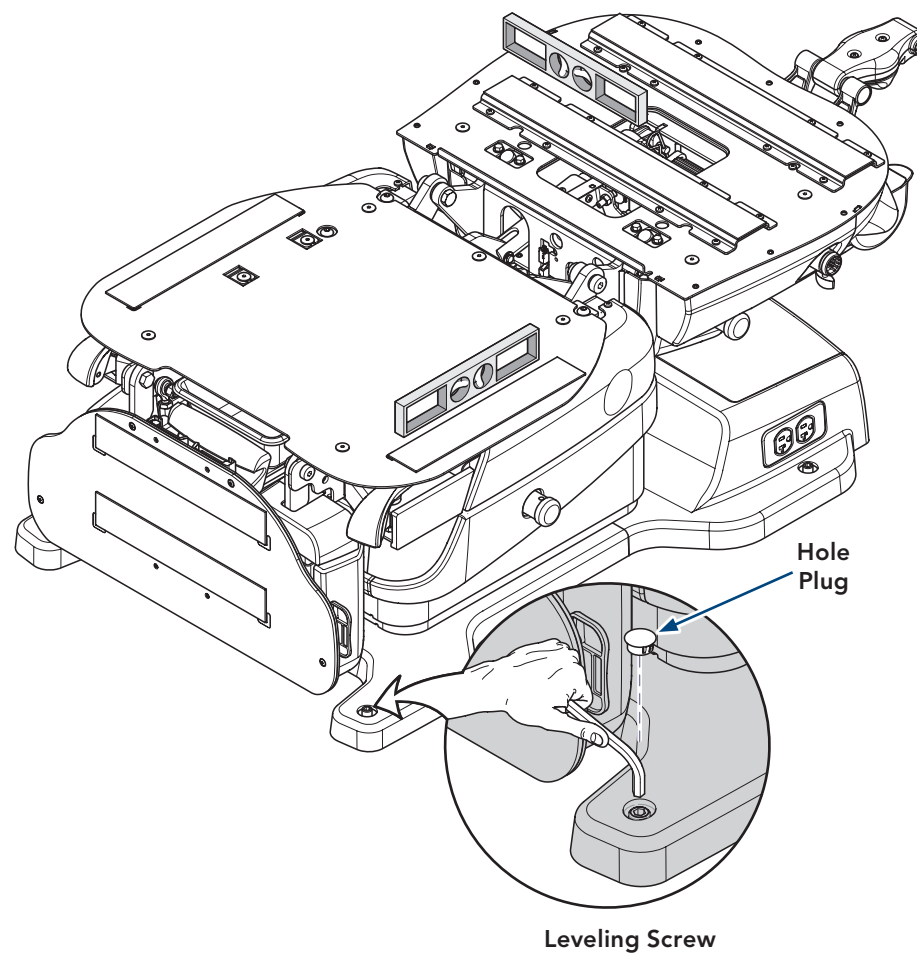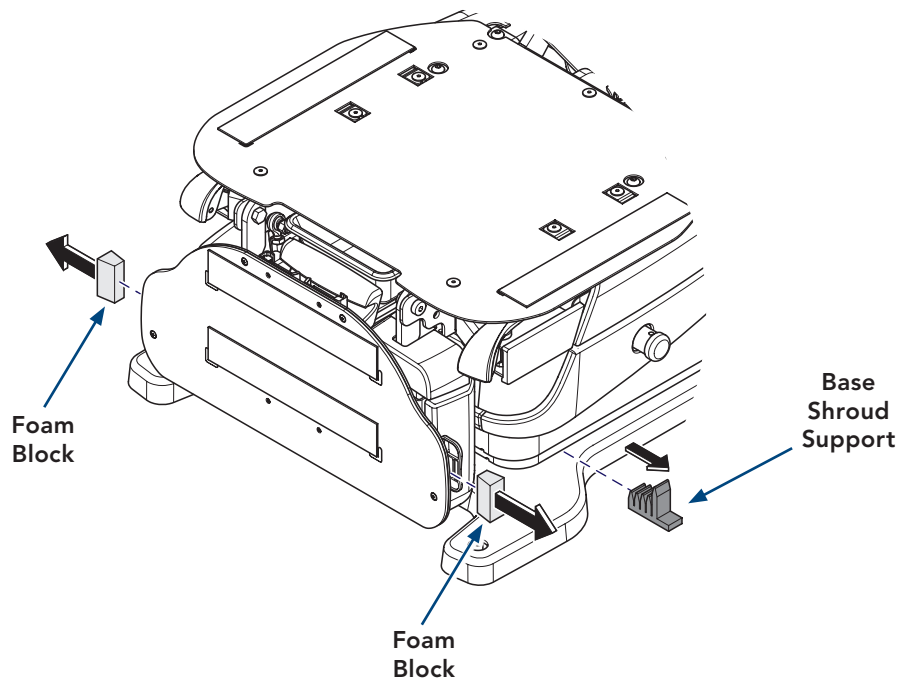

Step 8: Verify proper operation...

A) Use the foot control to activate the following functions:

Base UP / DOWN
Back UP / DOWN
Tilt UP / DOWN
Foot UP / DOWN
Quick Chair™
Home


B) Activate the HOME function again. While the chair is moving, press the STOP button. (All movement should stop).

C) Repeat A and B using the hand control.

Note: The Quick Chair™ feature is on the foot control only.

Note

If a malfunction is detected, contact Midmark Technical Service by calling: 1.800.MIDMARK

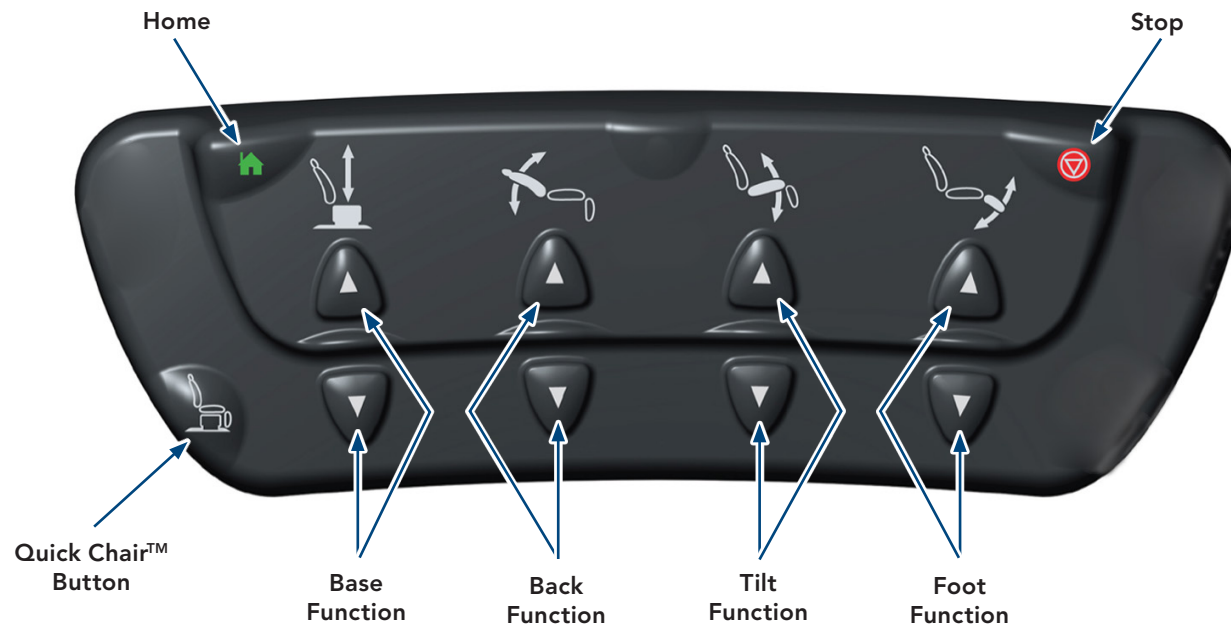Step 9: Install upholstered top (not shown)

Note: Set Up instructions are packaged with the upholstery.

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