Midmark Assistant's Delivery System Cabinet Mounted (ADS-1200-001) Setup Guide

Applies to Models:
ADS-1200-001

Special Tools:
none

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

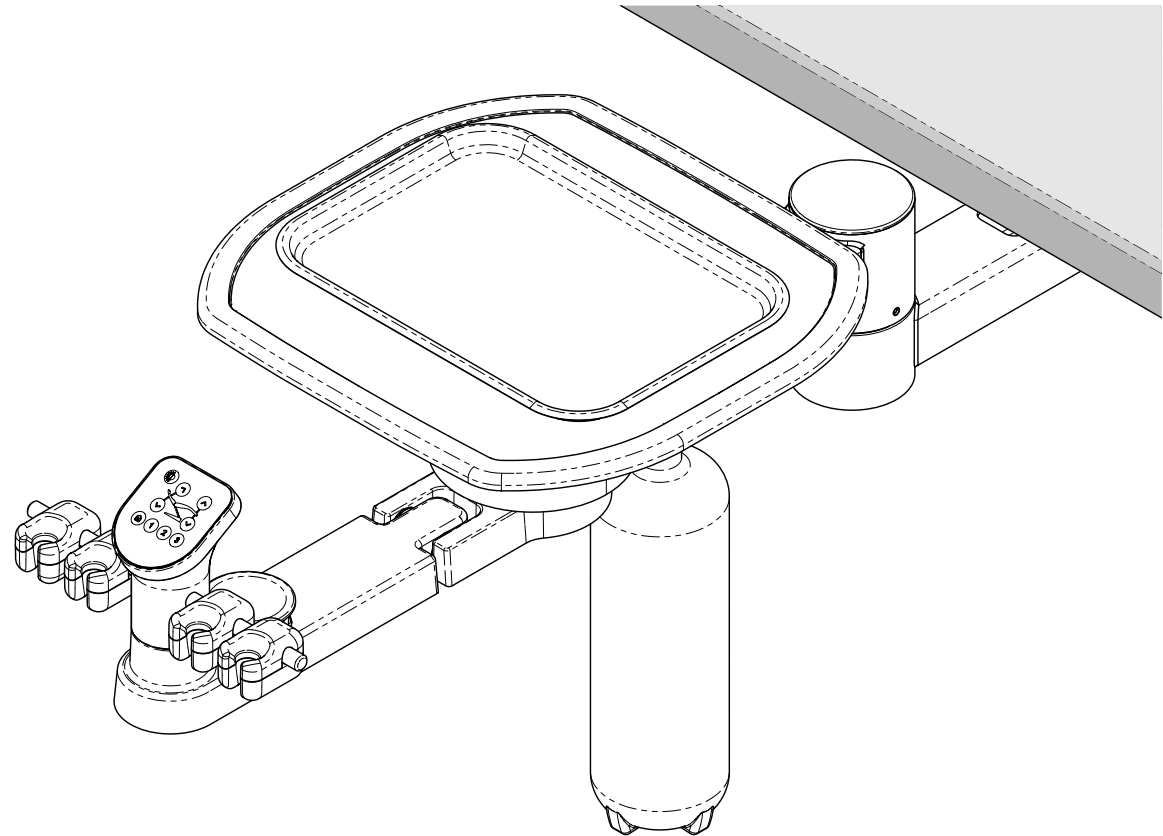
Equipment Alert

Before installing the Midmark Assistant's Delivery System, ensure the 12:00 cabinet is level and secure to the floor.



Note

Keeping Assistant's Delivery System packaging wrap on while mounting system will ensure an easier setup.

Hardware Kit

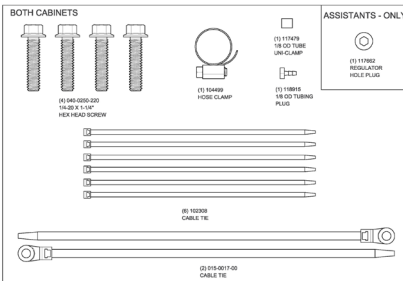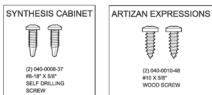THIS HARDWARE INSTALLATION KIT
(020-11817-01)
APPLIES TO MIDMARK
12:00 DELIVERY SYSTEM AND
12:00 ASSISTANTS
COUNTERTOP MOUNT


See installation manuals included in product for more detailed installation procedures or scan the QR code to download online.

003-10759-99 Rev AA5 (2/2/26)


ArtFull

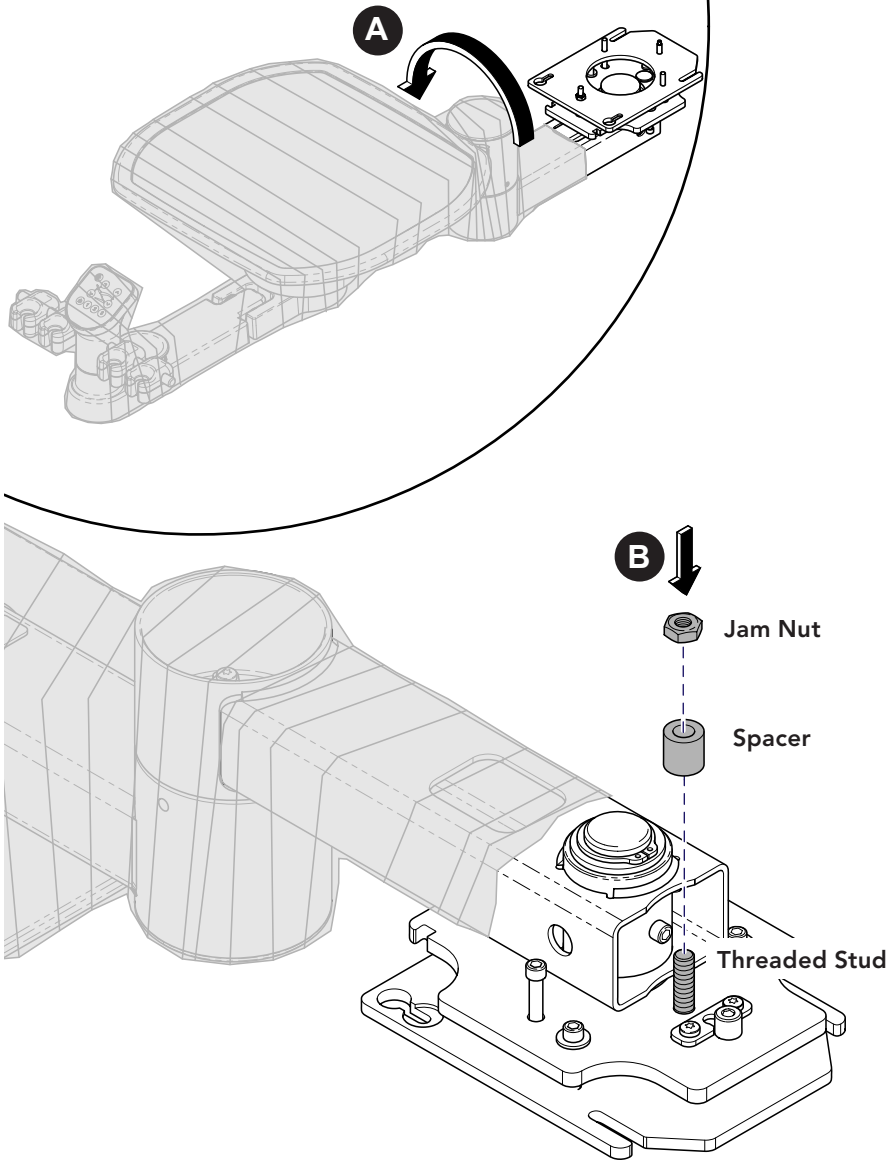Note
For all other cabinetry start at step 5.


Step 1: Artizan Expressions TS3680 and TS3690 Series - Only
A) Rotate assistant's delivery system over onto its top.
B) Install spacer and hex jam nut onto threaded stud.

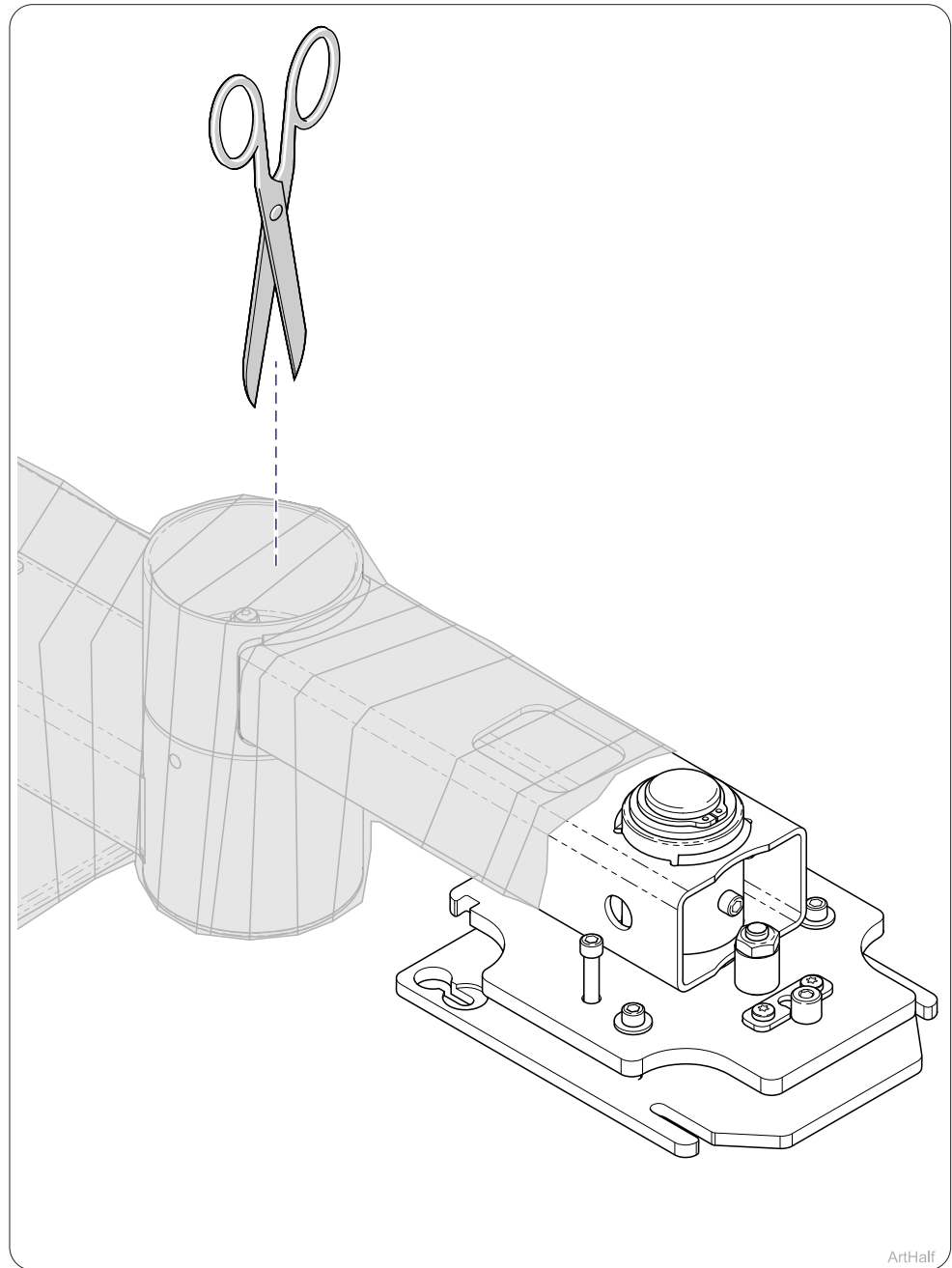ArtHalf

Step 2: Artizan Expressions TS3680 and TS3690 Series - Only
A) Make an opening in shipping wrap to access for stop key removal.

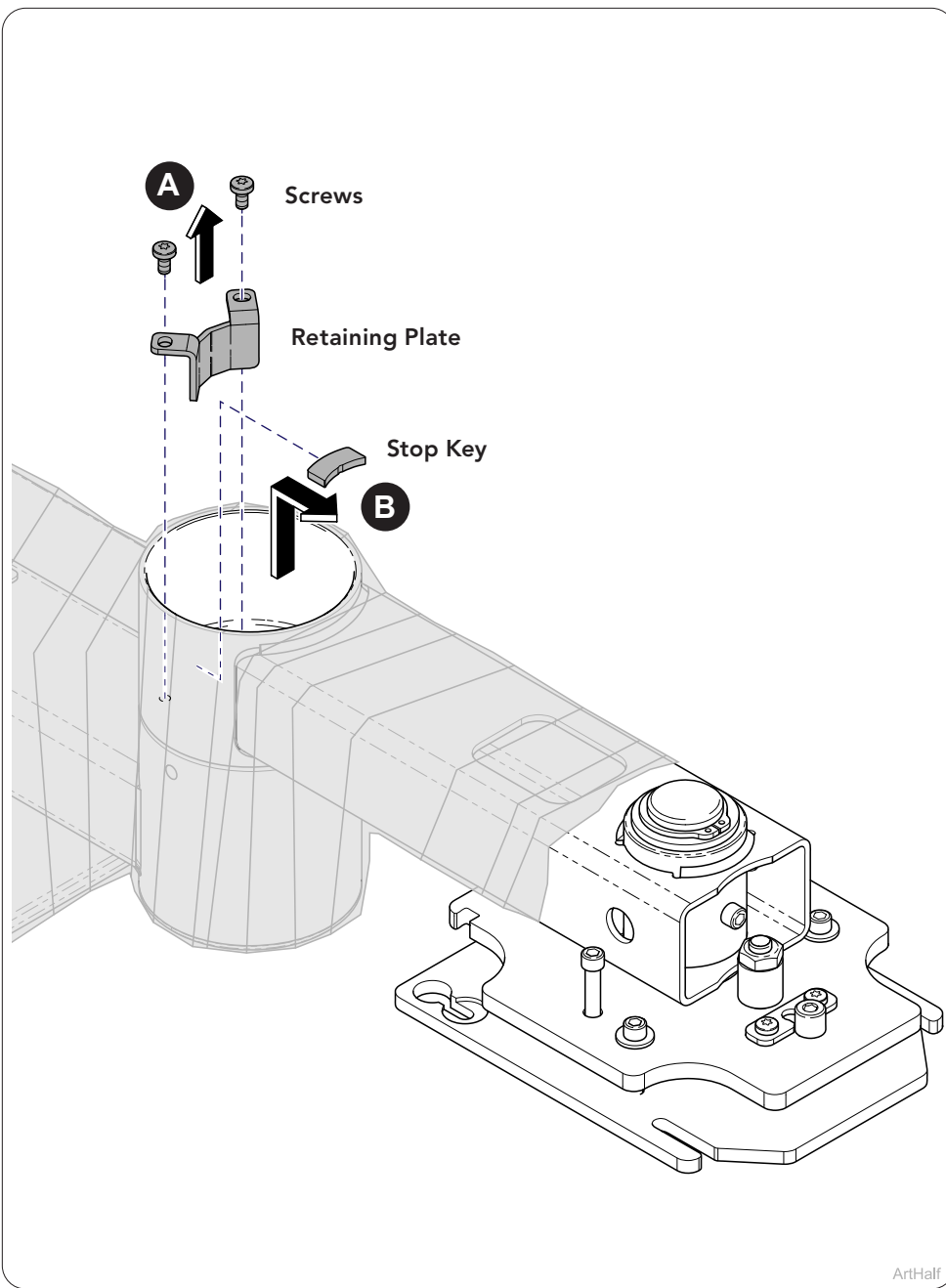
ArtHalf

Step 3: Artizan Expressions TS3680 and TS3690 Series - Only

- A) Remove screws (2) and retaining plate (1).
- B) Remove stop key from mounting arm slot.

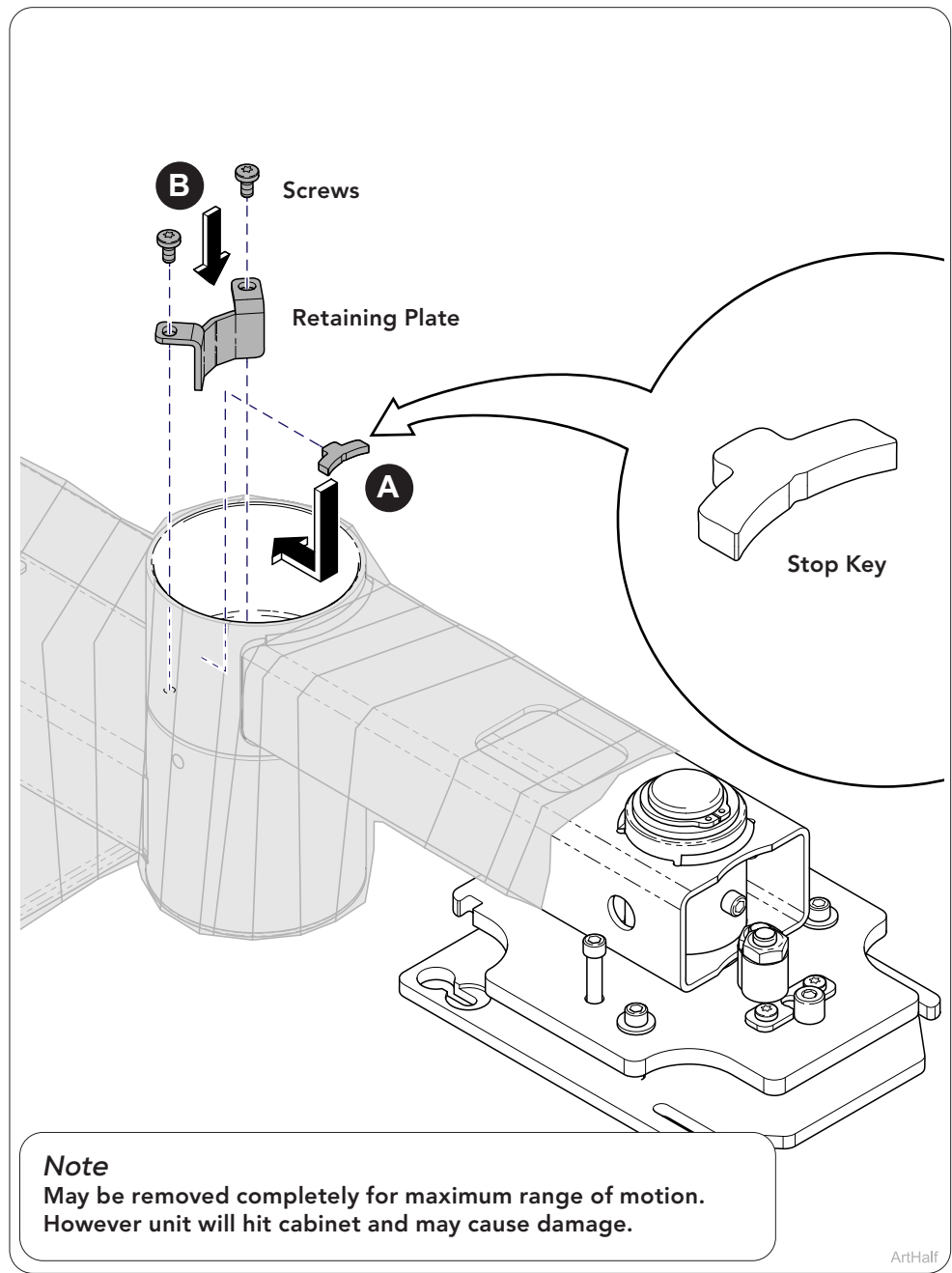ArtHalf


Note

May be removed completely for maximum range of motion. However unit will hit cabinet and may cause damage.


Step 4: Artizan Expressions TS3680 and TS3690 Series - Only

- A) Insert stop key into slot inside mounting arm.
- B) Install retaining plate and mounting screws (2).

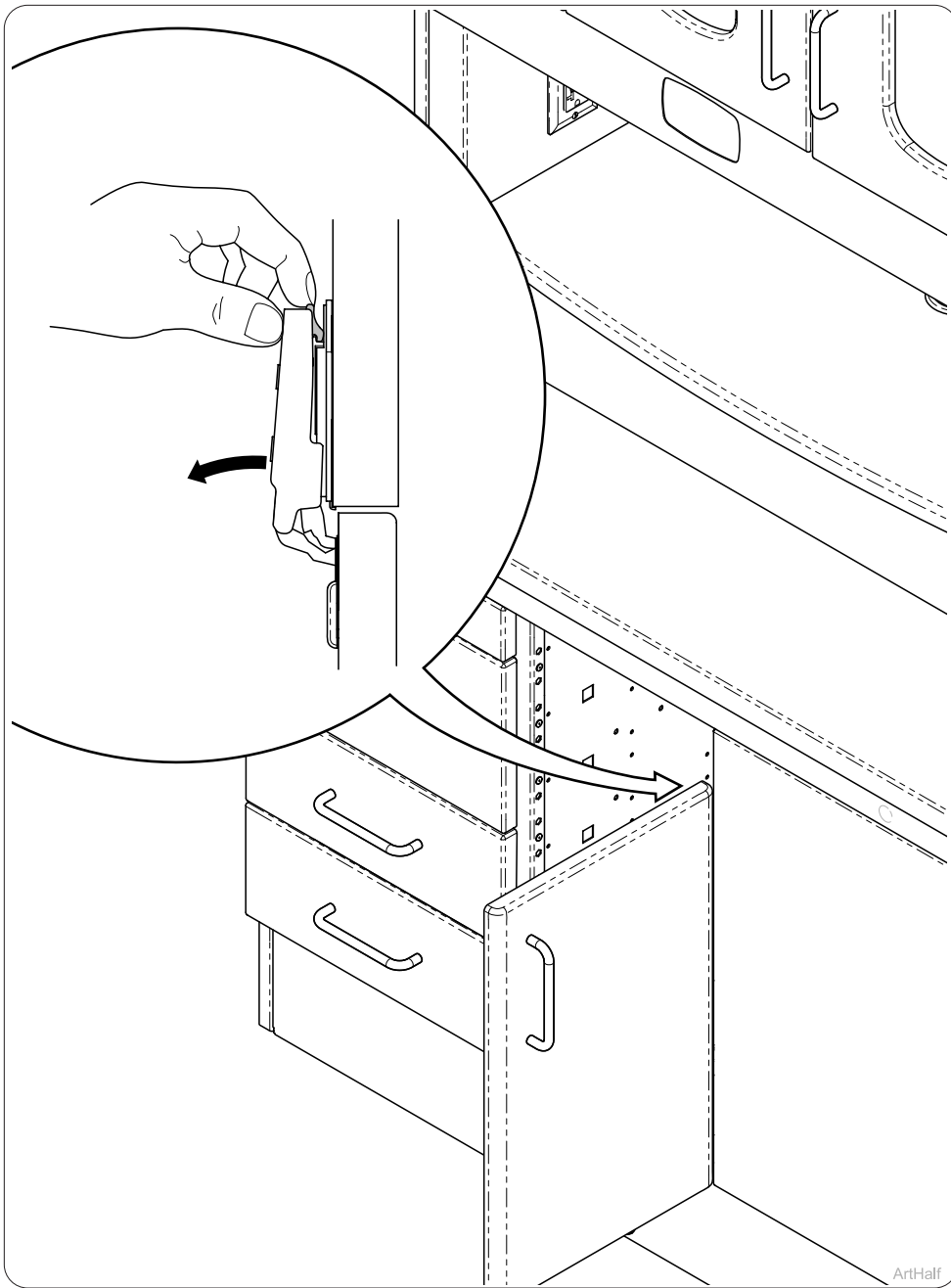
ArtHalf

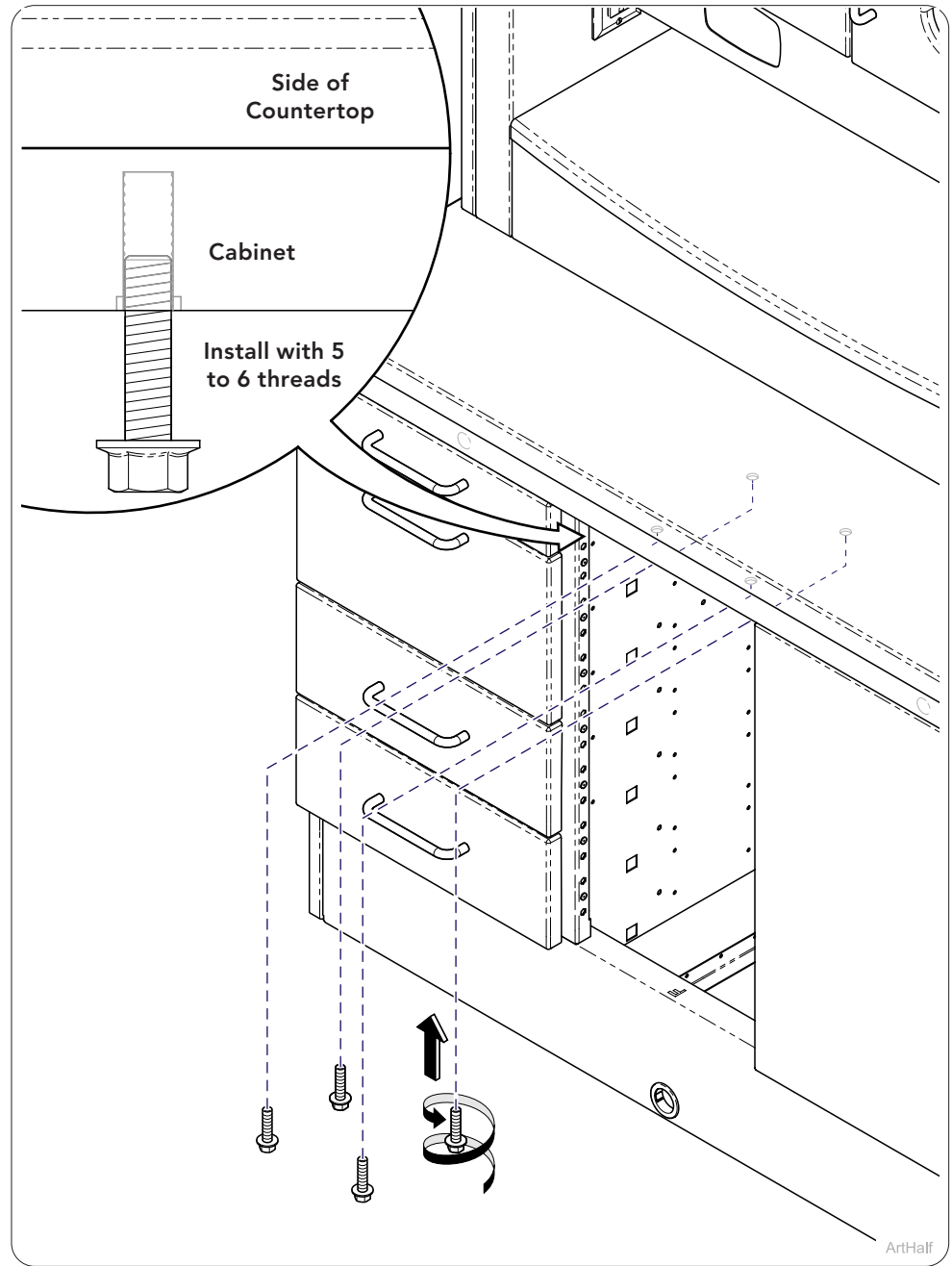Step 5:
A) Remove front cabinet door.

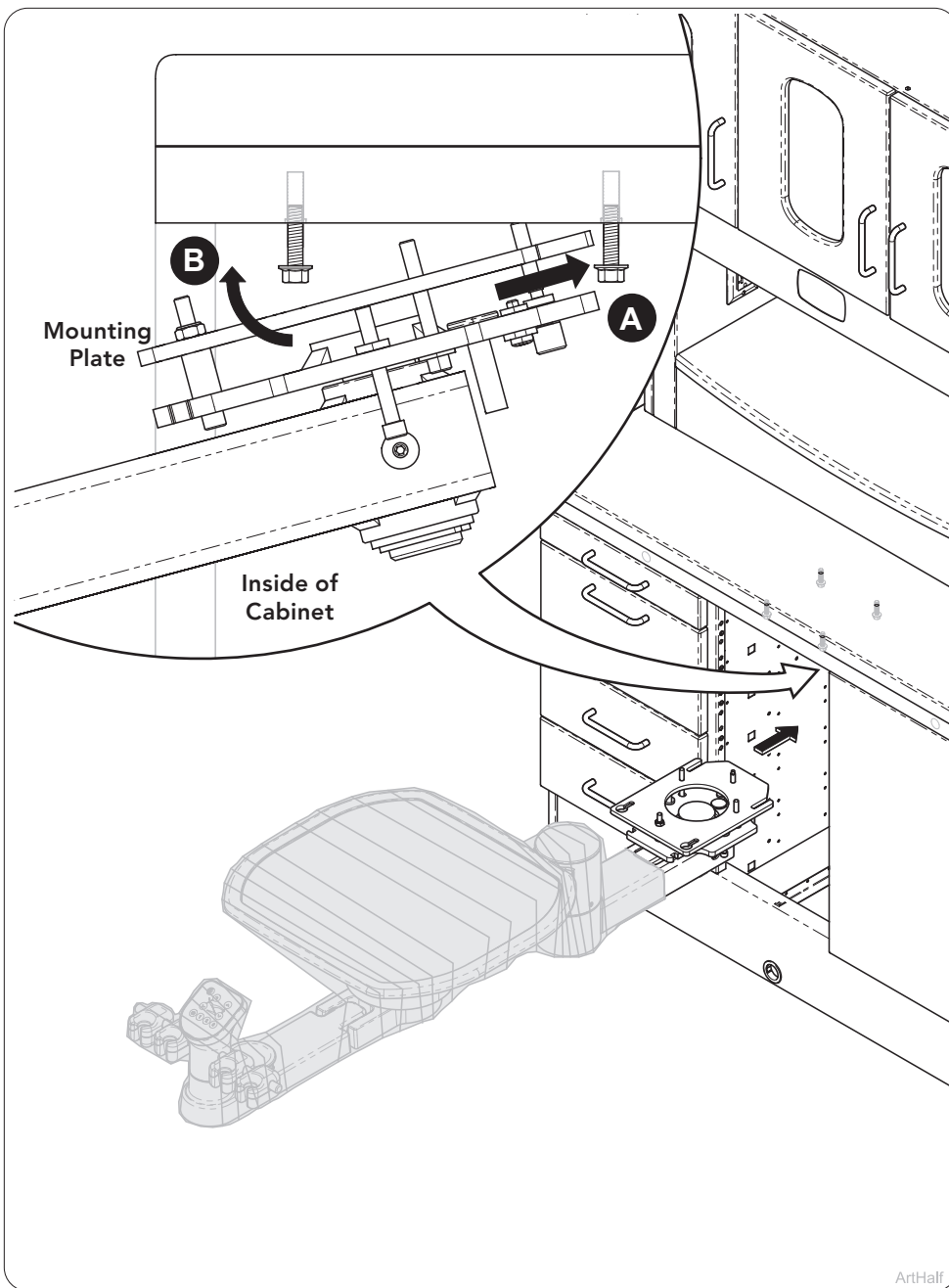
Step 6:
A) Install mounting bolts (4) inside cabinet.


Step 7: Mounting Assistant's Delivery System.

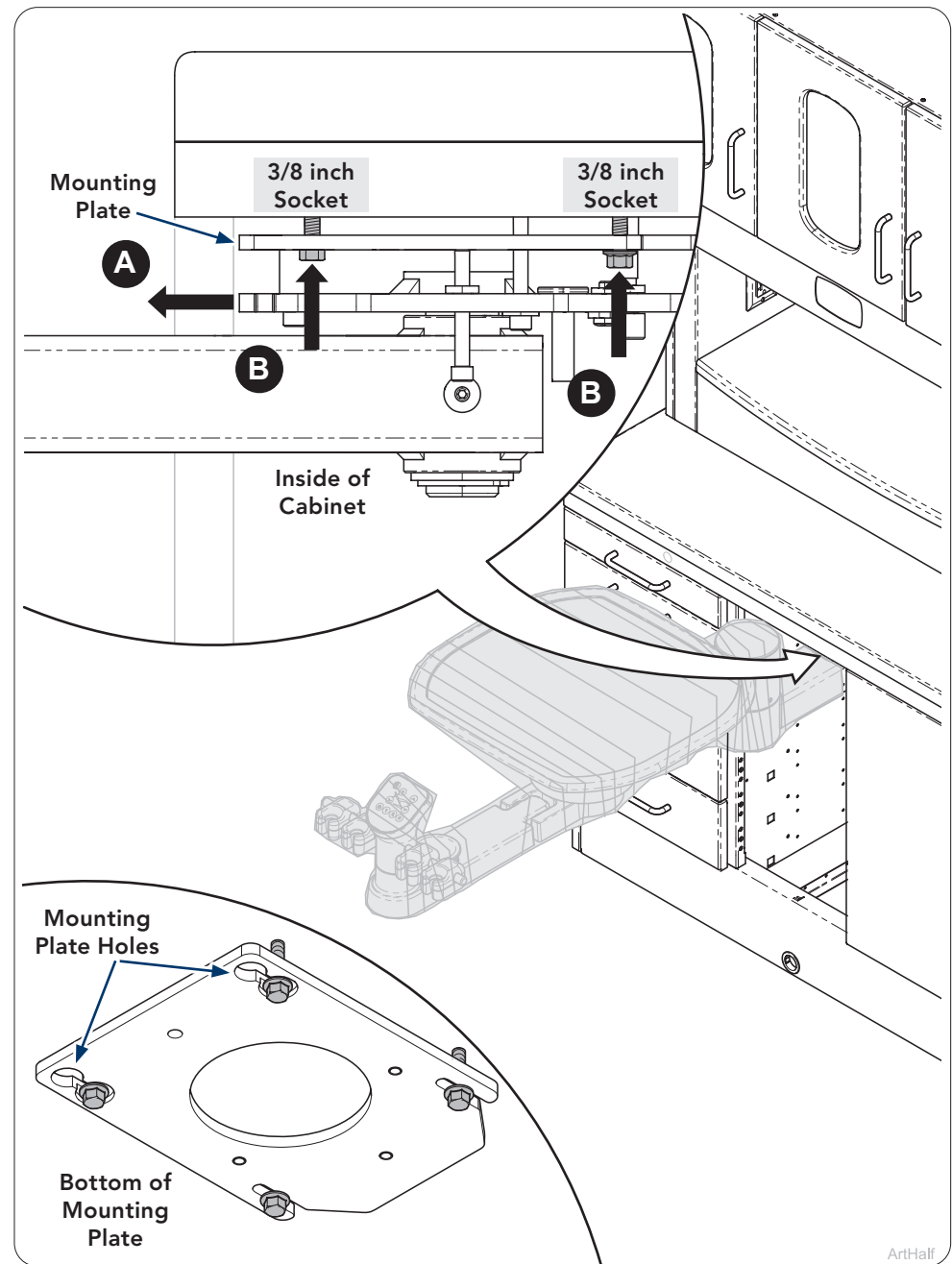- A) Install assistant's delivery system into cabinet sliding onto rear bolts (2).
- B) Lift up front of worksurface so bolts go through front of mounting plate holes.


Step 8: Mounting Assistant's Delivery System.

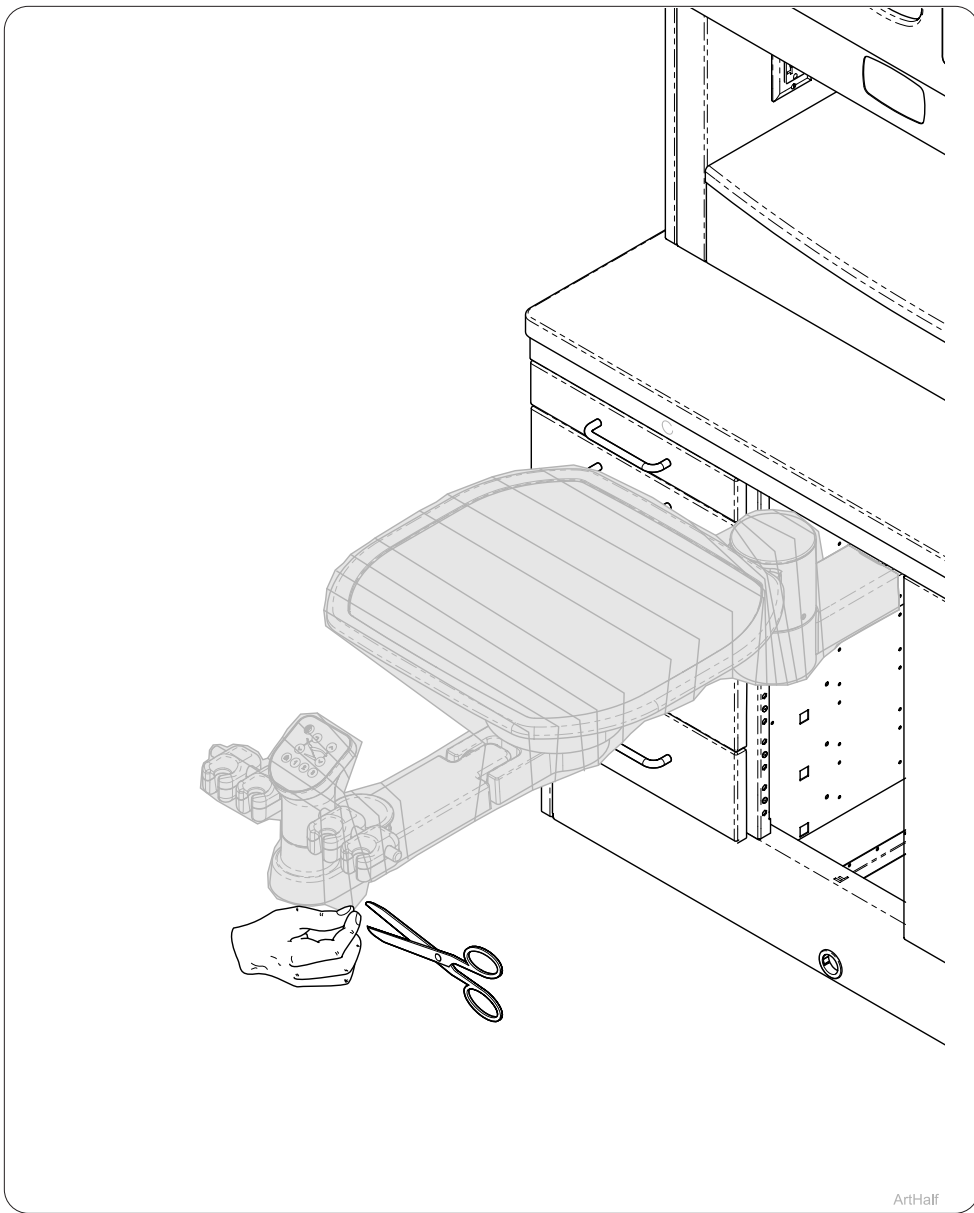- A) Slide assistant's delivery system forward so front bolts (2) insert into plate as shown.
- B) Tighten bolts (4).


ArtHalf

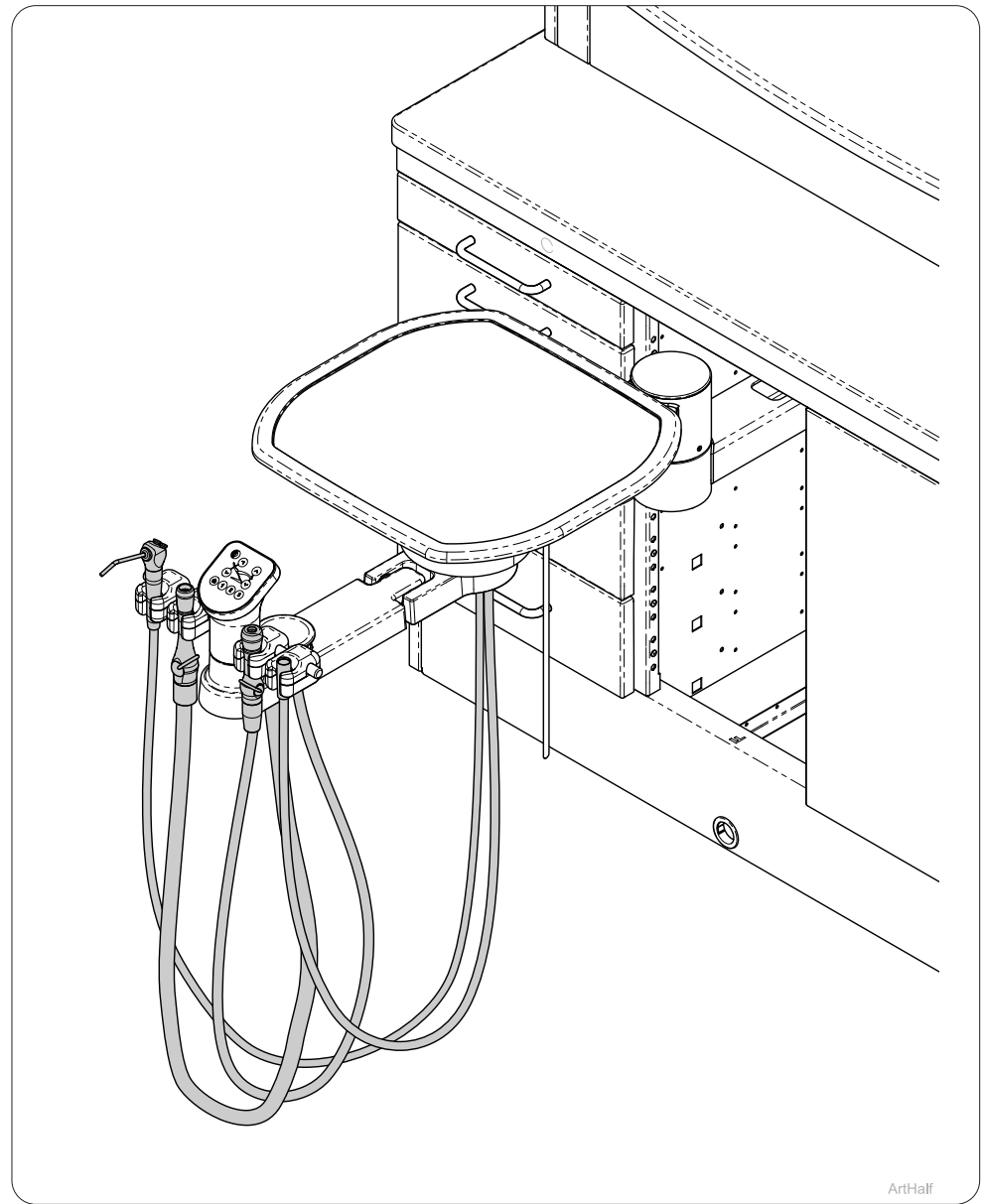Step 9:
A) Remove packaging from assistant's delivery system.

ArtHalf

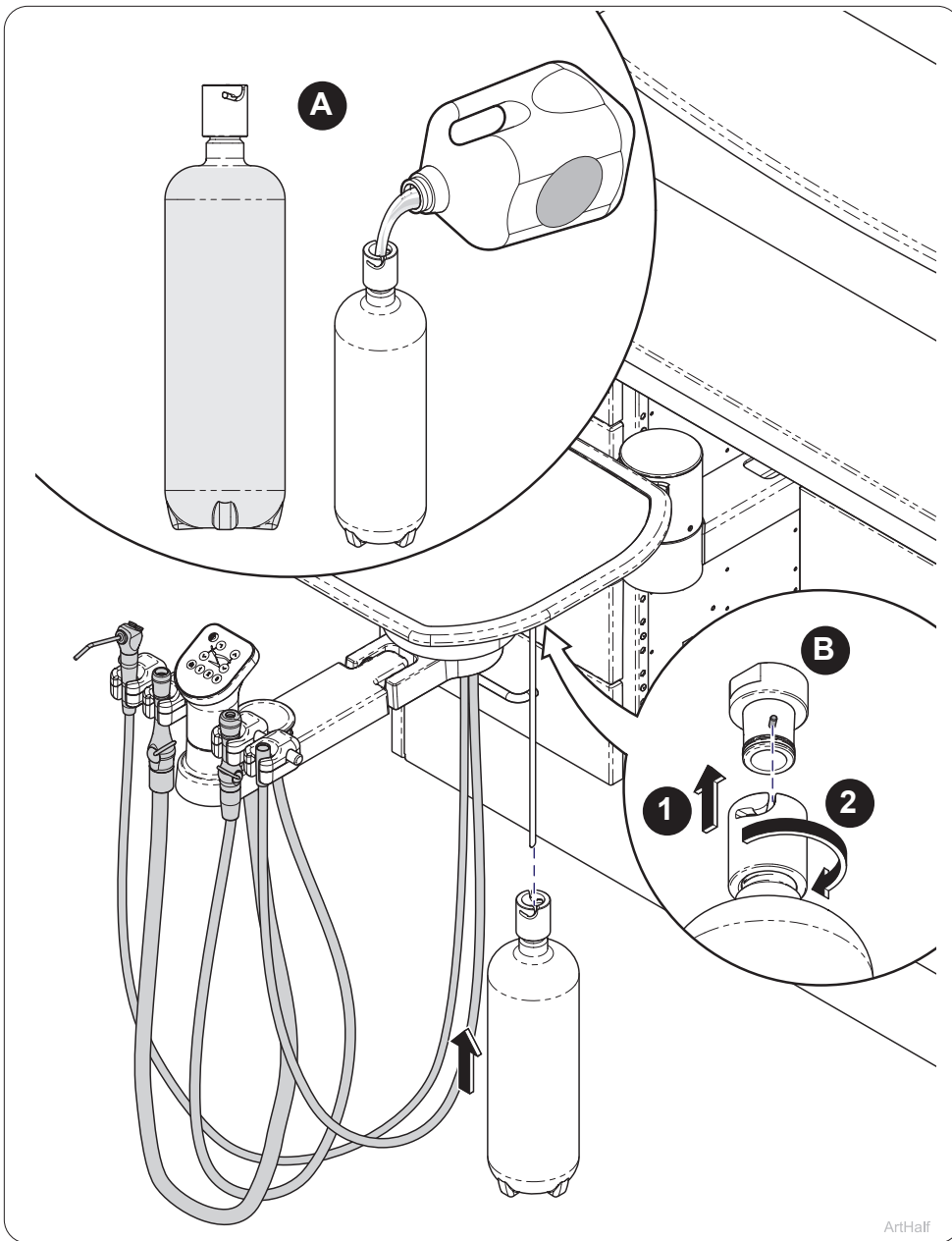
Step 10:
A) Place tubing and handpieces into handpiece holder.
B) The syringe is positioned in the first position location from left to right.

Step 11:

- A) Fill water bottle with water that meets local drinking codes.
- B) Install water bottle as shown.

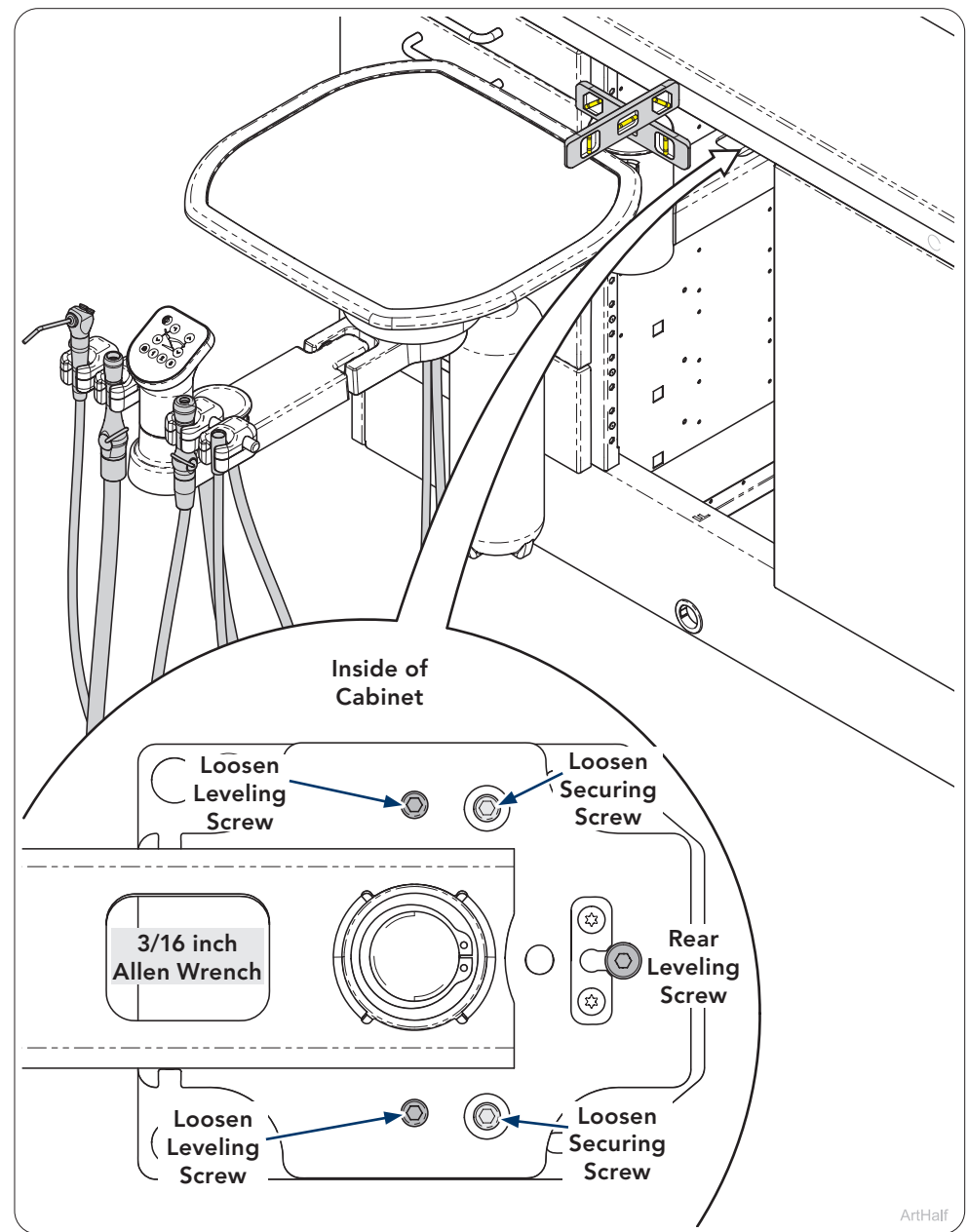
ArtHalf

Step 12: Leveling Mounting Arm.

- A) Loosen left and right securing and leveling screws prior to leveling.
- B) Level delivery system mounting arm front to back with rear screw (1).
- C) Level delivery system mounting arm using left and right leveling screws for side to side leveling.
- D) Retighten securing screw to keep adjustments.

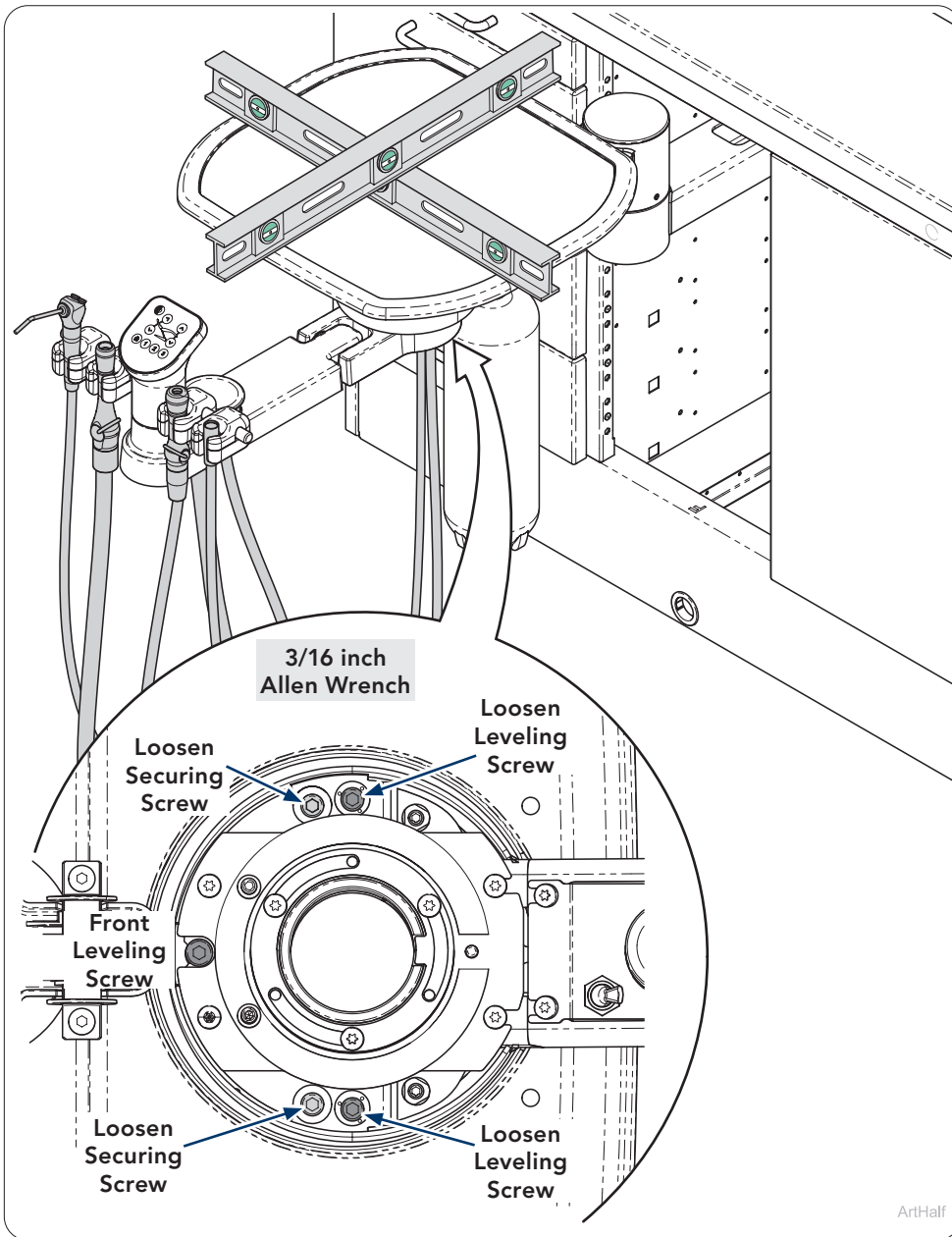
ArtHalf

ArtHalf

Step 13: Leveling Delivery System.

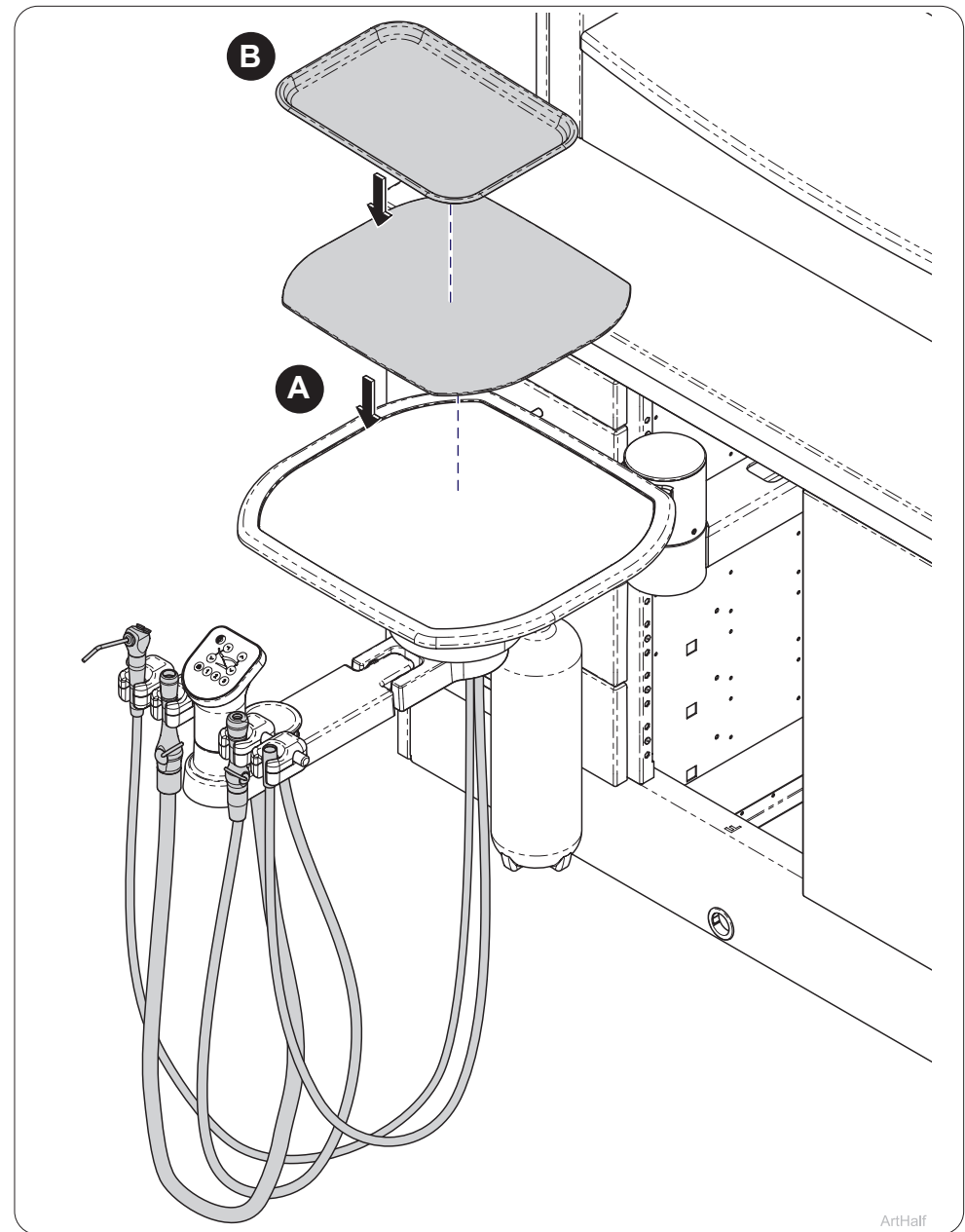
- A) Loosen left and right securing and leveling screws prior to leveling.
- B) Level delivery system front to back with front screw (1).
- C) Level delivery system using left and right leveling screws for side to side leveling.
- D) Retighten securing screw to keep adjustments.

ArtHalf

Step 14:

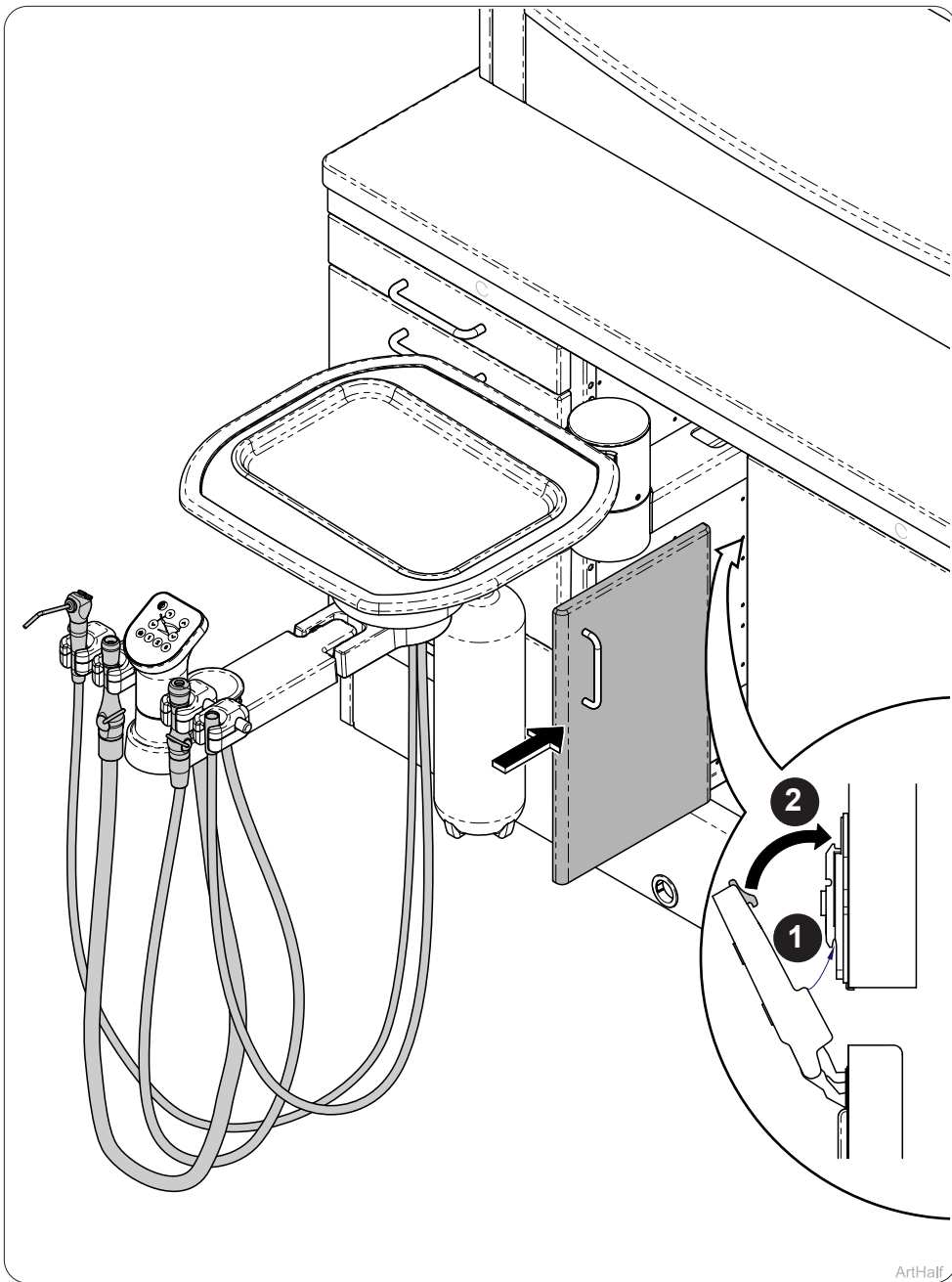- A) Place pad onto delivery system.
- B) Place tray onto pad.


Step 15:
A) Install door back onto cabinet as shown.

Regulator Installation

Regulator installed to shutoff valve.


- A) Insert regulator into compression fitting of shut-off valve and tighten compression fitting.
- B) Assistant ONLY, remove barb fitting and replace with provide plug. Use teflon tape or thread sealant not provided to secure plug.

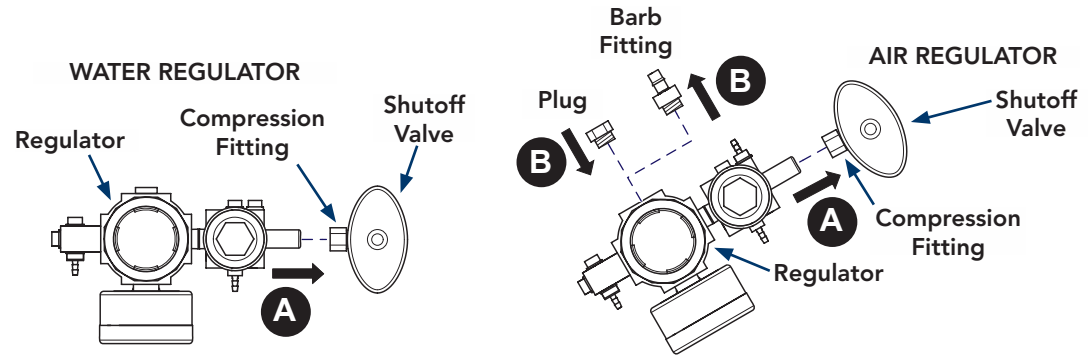Note: Delivery and Assistant's Unit Tubing Connections are located in cabinet base.

Tubing Connections (Cabinet)

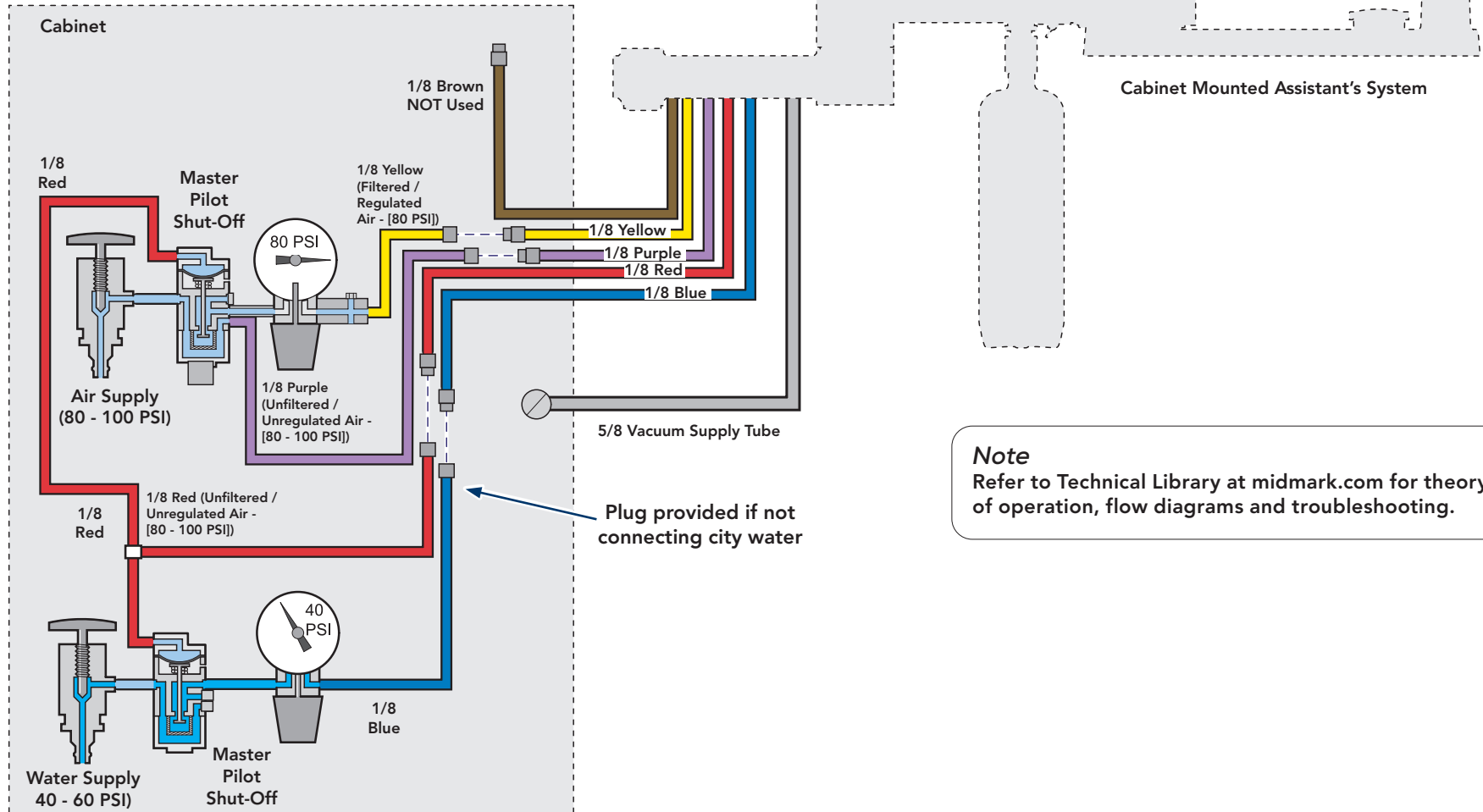
Step 16:

Note
Refer to Technical Library at midmark.com for theory of operation, flow diagrams and troubleshooting.


Electrical Connections

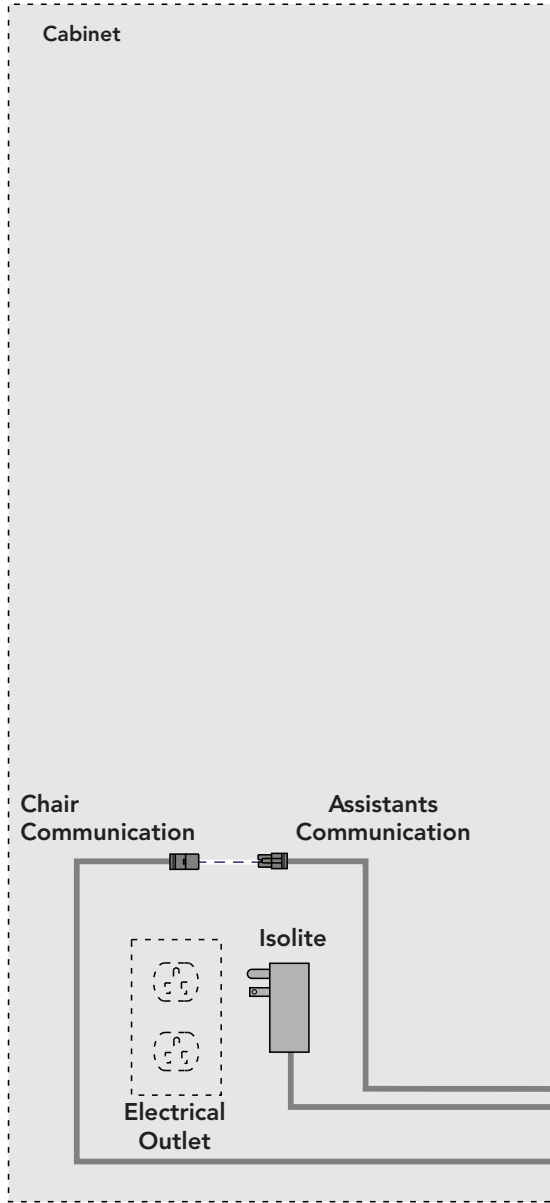Step 17:

Step 18: Covering Tubing and Wiring.

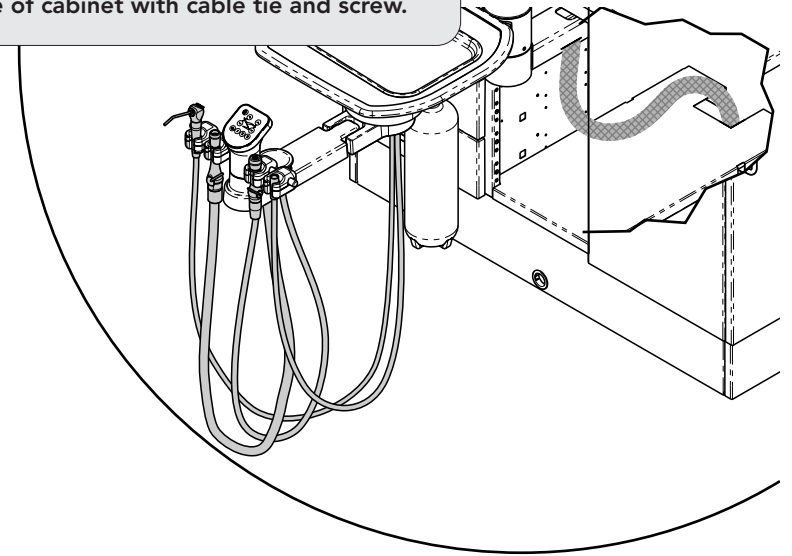- A) Use the hook and loop braided sleeve to wrap and secure the tubing and wiring within the cabinet.
- B) Secure braided sleeve to side of cabinet with cable tie and screw.

Cabinet Mounted Assistant's System

Assistant's Communication

Isolite

Chair

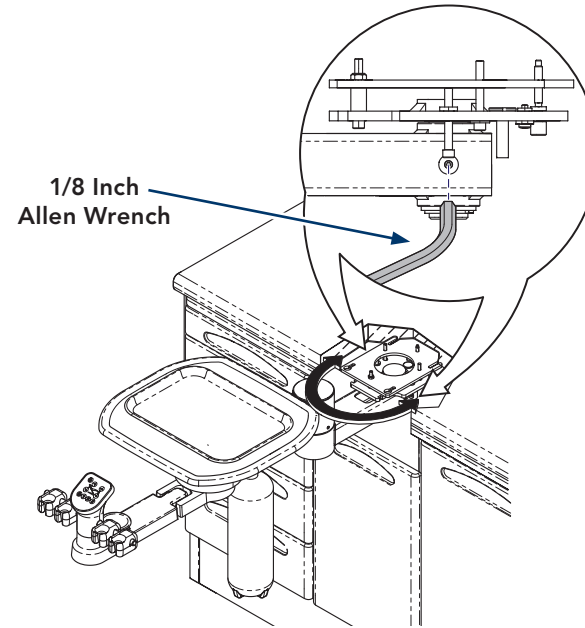
Tension Adjustments

Assistant's System Cabinet Mount Rotation Tension Adjustment

The rotation tension on the assistant's system cabinet mount can be adjusted to your desired level.

Adjusting Assistant's System cabinet mount rotation tension.

A) To adjust the tension, use a 1/8 inch allen wrench to turn the two set screws. Rotate them clockwise to increase the tension and counterclockwise to decrease it.


Assistant's System Arm Vertical Rotation Tension Adjustment

The vertical rotation tension on the assistant's system arm can be adjusted to your desired level.

Adjusting Assistant's System Arm Vertical Rotation Tension.

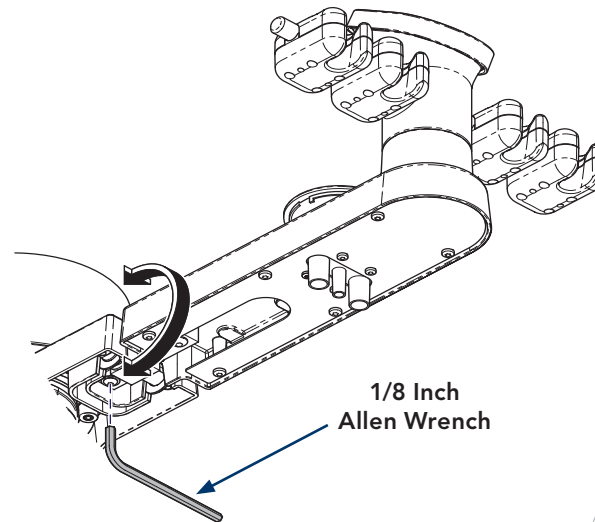A) To adjust the tension, use a 1/8 inch Allen wrench to turn the button head screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

ArtFull

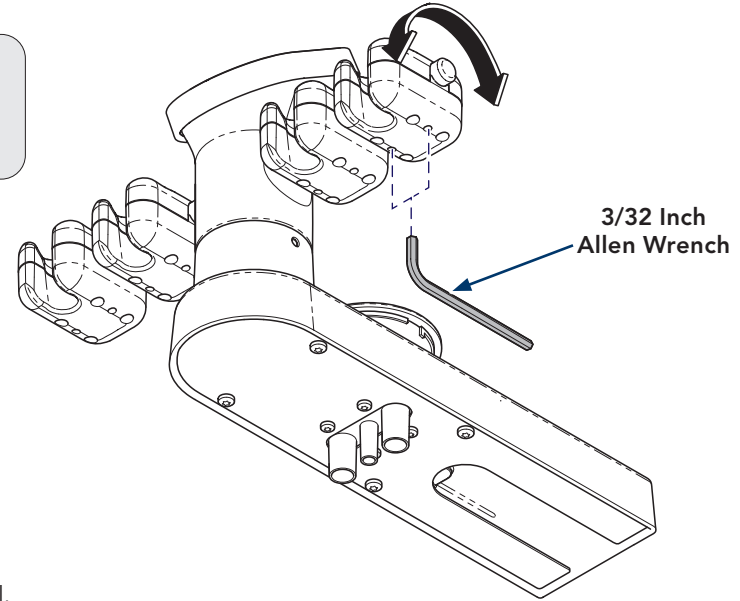
Tension Adjustments

Assistant's System Handpiece Holder Tension Adjustment

The rotation tension on the assistant's system handpiece holders can be adjusted individually to your desired level.

Adjusting tension on each handpiece holders rotation.

A) To adjust the tension, use a 3/32 inch allen wrench to turn the two set screws. Rotate them clockwise to increase the tension and counterclockwise to decrease it.


ArtFull

Assistant's System Head Rotation Tension Adjustment

The rotation tension on the assistant's system head rotation can be adjusted to your desired level.

Adjusting tension on the assistant's head rotation.

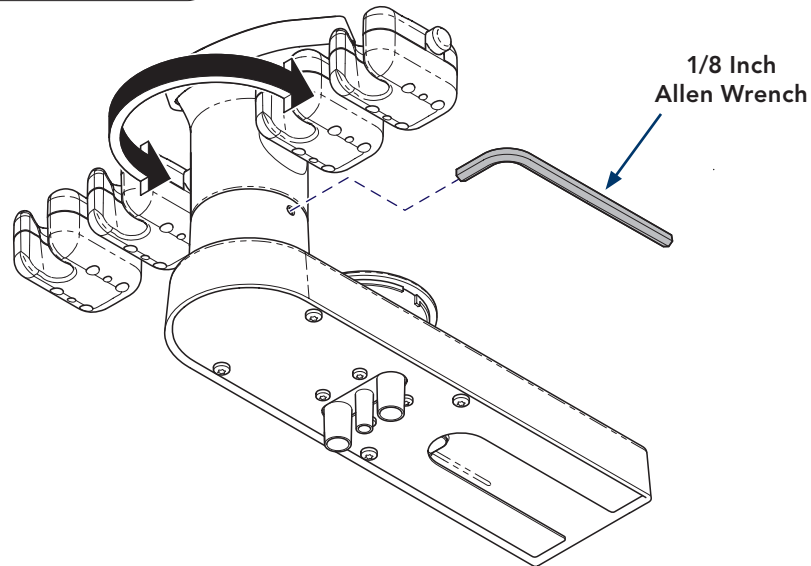A) To adjust the tension, use a 1/8 inch allen wrench to turn the set screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

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


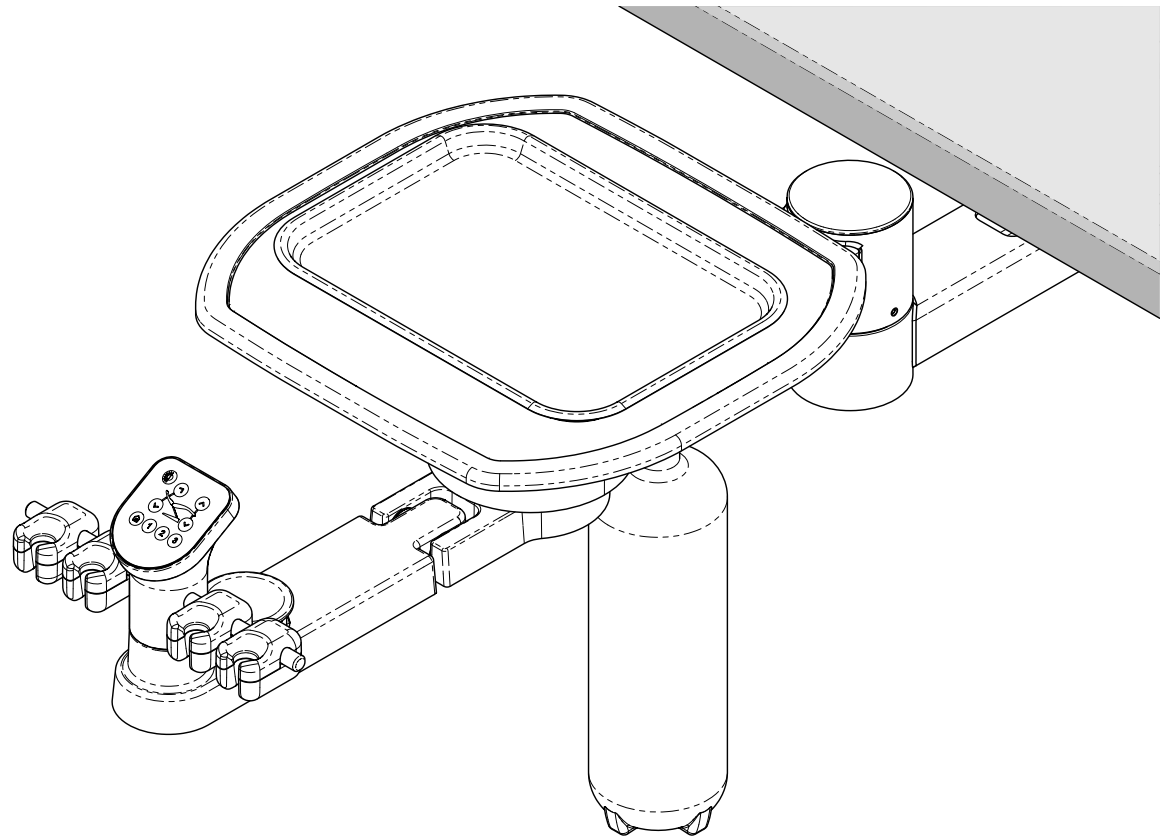
Kit de montaje

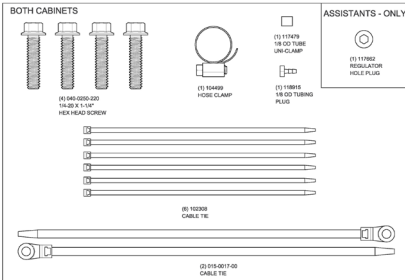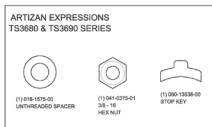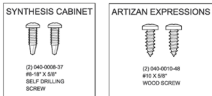THIS HARDWARE INSTALLATION KIT
(028-11817-01)
APPLIES TO MIDMARK
12-00 DELIVERY SYSTEM AND
12-00 ASSISTANTS
COUNTERTOP MOUNT

See installation manuals included in product for more detailed installation procedures or scan the QR code to download online.

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