# Midmark Power Supply Setup (029-11724-00)

**Applies to Models:**  
CH-1000 with CH-MOUNT  
DS-1200

**Special Tools:**  
none

**Note**


Power Supply units are available in 120V~ models. Make certain you are installing the correct one for the electrical mains supply power.

**Power Supply**

The Power Supply centralizes electrical supply power inputs and distributes that power as needed to existing operatory components.

**Note**

Service performed on fuses and / or power cords shall be performed by a Midmark recommended Service Technician.

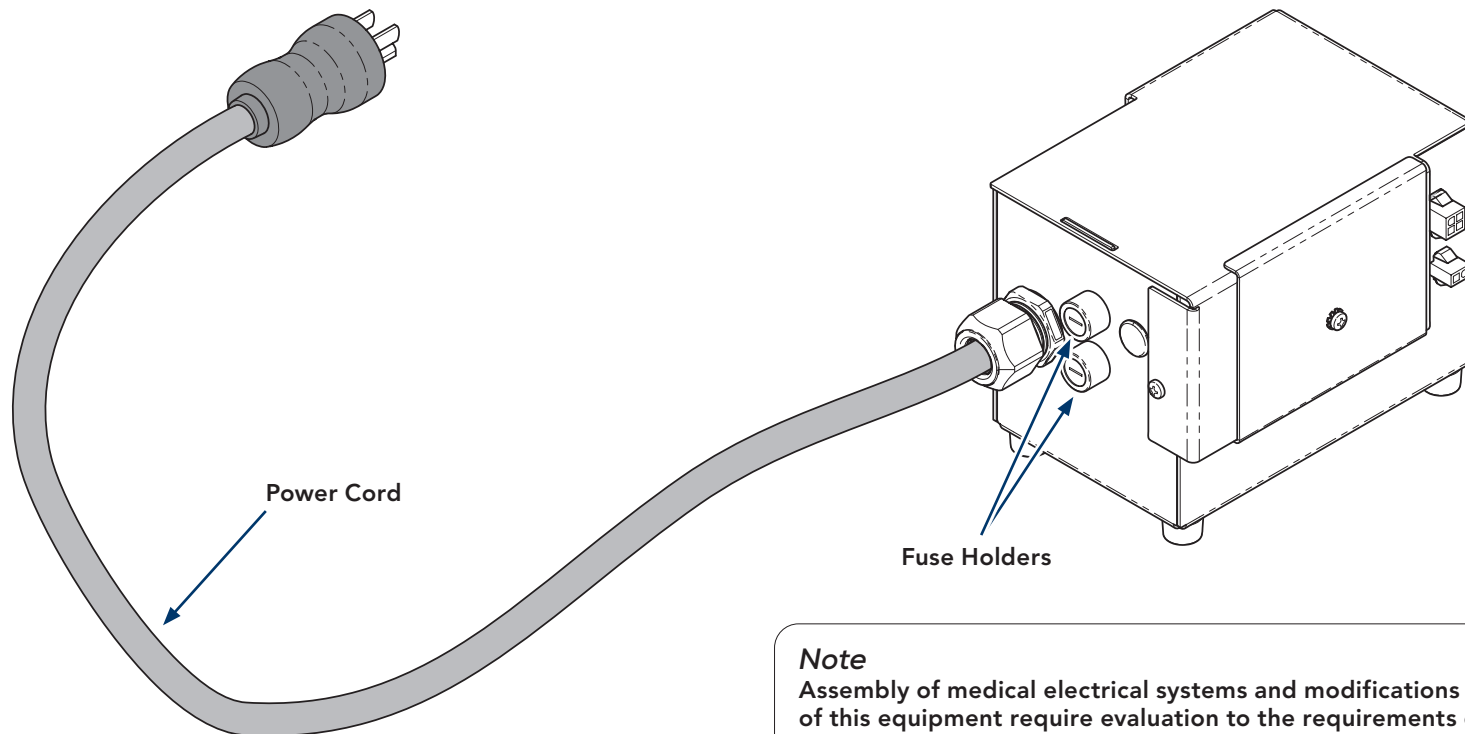**Note**

Assembly of medical electrical systems and modifications during the service life of this equipment require evaluation to the requirements of the 60601 series of medical electrical equipment safety standards.

ArtFull

## Symbols Glossary

### **WARNING**

Indicates a potentially hazardous situation which could result in serious injury.

### **Caution**

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

### **Equipment Alert**


Indicates a potentially hazardous situation which could result in equipment damage.

### **Note**

Amplifies a procedure, practice, or condition.

Follow instructions for use

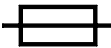Type T Fuse

Protective Earth Ground

TUV

## Intended Use

The Power Supply is intended to be a non-user-accessible part. It is designed to deliver electricity at predetermined voltage levels (DC) to a Midmark Dental Light.

## Electromagnetic Interference

Midmark dental operatory components are designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this operatory, remove the interfering device from room and / or plug product into an isolated circuit.

## Transportation and Storage Conditions

Transportation and Storage Temperature ..... -30°C to 60°C (-22°F to 140°F)  
Relative Humidity ..... 10% to 90% (non-condensing)  
Atmospheric Pressure ..... 50 kPa to 106 kPa (7.3 psig to 15.3 psig)

## Operating Conditions

Ambient Operating Temperature Range ..... 15°C to 35°C (59°F to 95°F)  
Relative Humidity ..... 10% to 90% (non-condensing)  
Atmospheric Pressure ..... 69 kPa to 106 kPa (10 psig to 15.3 psig)

## Disposal of Equipment

At the end of this product's life, the unit, accessories and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of this equipment and other consumable goods.

## Call for Service

Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

Model / serial number  
Date of purchase  
Symptom(s) of malfunction

### WARNING

Do not modify this equipment without authorization of the manufacturer.

### WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

### WARNING

Only use Midmark specified fuses, failure to do so could result in a fire or an electrical hazard.

### WARNING

Do not use multi-outlet adapters, outlet strips or extension cords with this power supply.

### WARNING

Only connect approved Midmark dental chair, delivery systems, lights, monitor and power supply when assembling the dental operatory (medical electrical system).  
Midmark approved products listed below:

CH-1000	Midmark Dental Chair
DS-1100-001	Midmark Left-Right Chair-Mounted Doctor's Dental Delivery System
DS-1200-001	Midmark 12 O'clock Column/Cabinet-Mounted Doctor's Dental Delivery System
ADS-1100-001	Midmark Chair Mounted Assistant's Dental Delivery System
ADS-1200-001	Midmark Cabinet Mounted Assistant's Dental Delivery System
LT-1100-001	Midmark Left-Right Chair-Mounted Dental LED Light
LT-1200-001	Midmark Cabinet-Mounted Dental LED Light
LT-1300-001	Midmark Wall-Mounted Dental LED Light
LT-1400-XXX	Midmark Ceiling-Mounted Dental LED Light
LT-1500-XXX	Midmark Track-Mounted Dental LED Light with Monitor
LT-1600-XXX	Midmark Track-Mounted Dental LED Light
9A738002	Midmark Left-Right Chair-Mounted Dental Monitor
029-11724-00	Midmark Dental Power Supply

Other combinations of equipment have not been evaluated for safety.


# Electrical Connections (for Cabinet)

## WARNING


To avoid the risk of electrical shock this equipment must be connected to a power outlet with a protective earth and be compliant with local codes. Install one that meets these requirements if one does not exist:

120 VAC Model: 120V~, 60Hz, 12A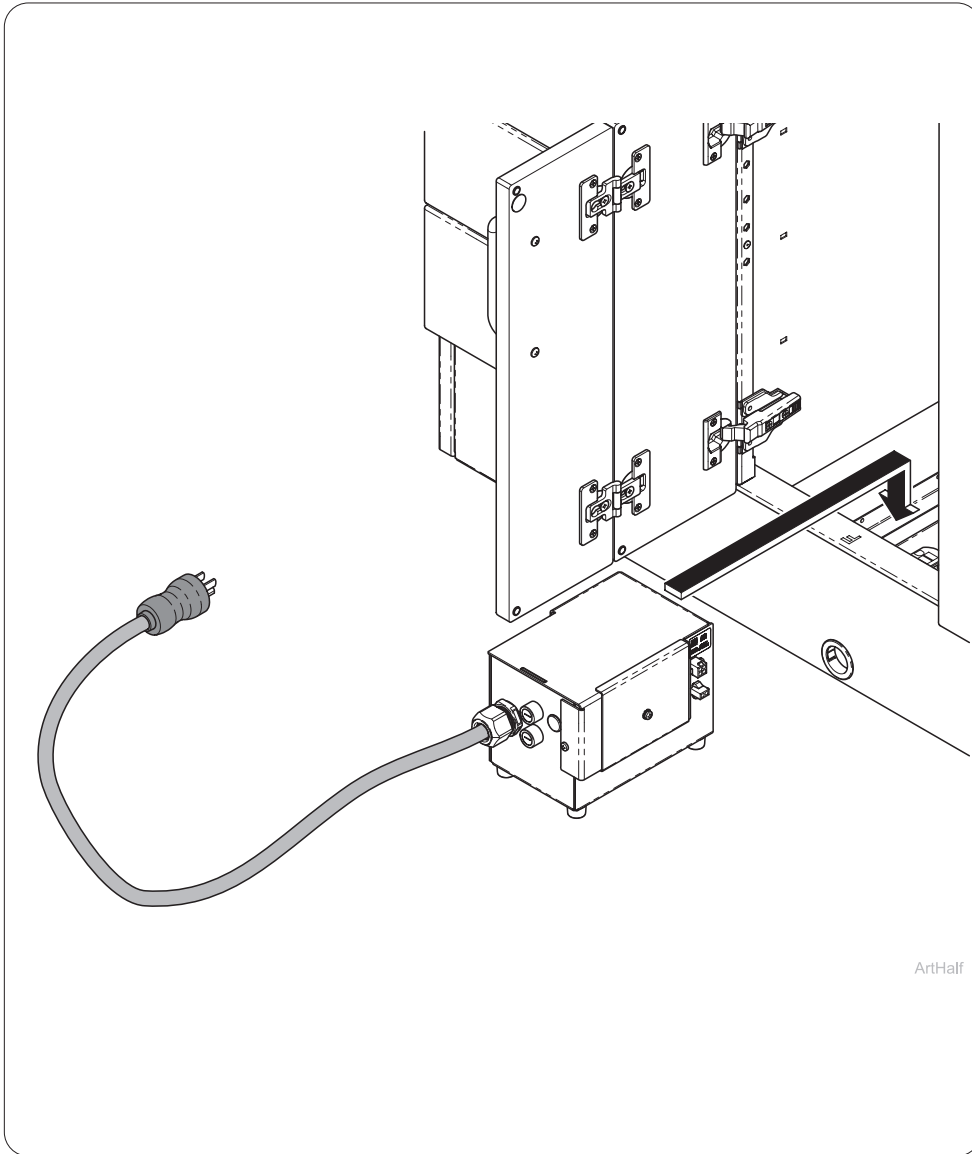

### Step 1:


Open center door of 12:00 cabinet base. Install power supply into cabinet.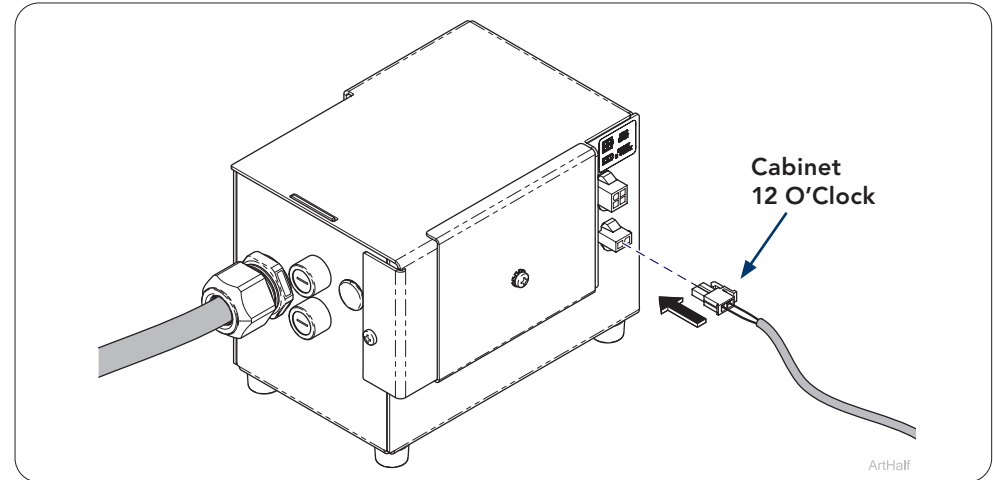

### Step 2:

Plug in 2 pin connector from unit into power supply.

**Note:** Refer to Midmark Unit setup manuals for installation on to wall for cabinet.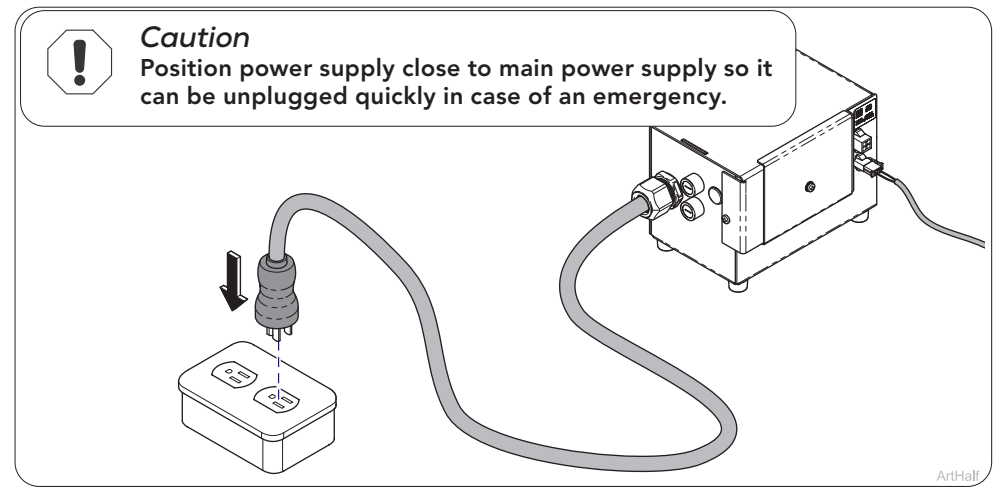


### Step 3:


Plug in power supply into main power outlet.

# Electrical Connections


(for Chair J-Box)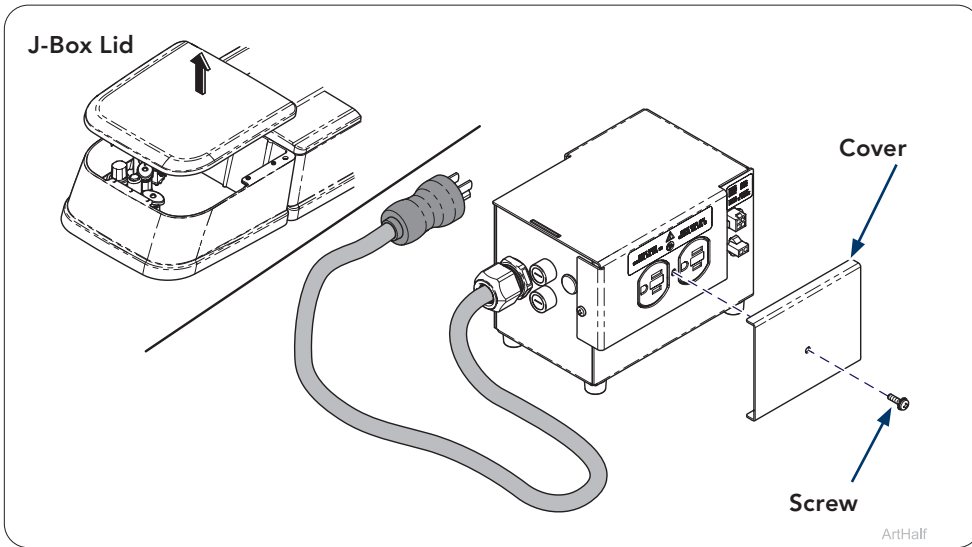


**Step 1:**  
Remove J-Box lid. Remove screw and cover from power supply.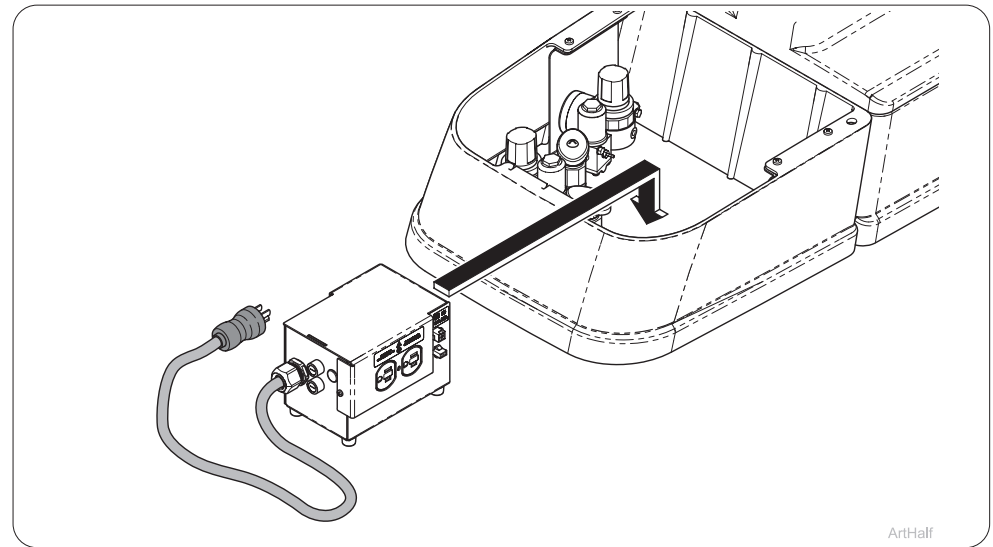


**Step 2:**  
Place power supply into J-Box in front of chair.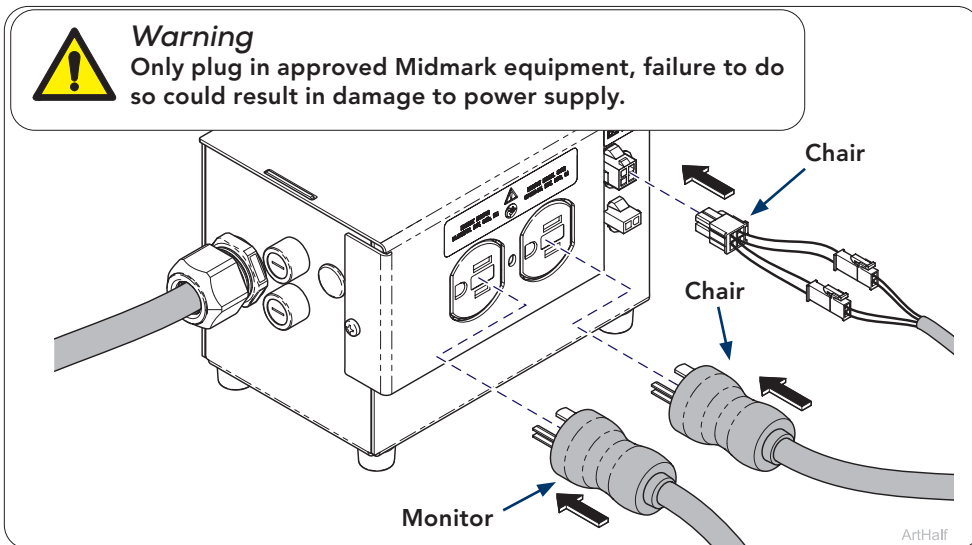

**Step 3:**  
Plug in 4 pin connector from Light into power supply.  
**Note: Refer to Midmark Light setup manuals for installing on to chair.**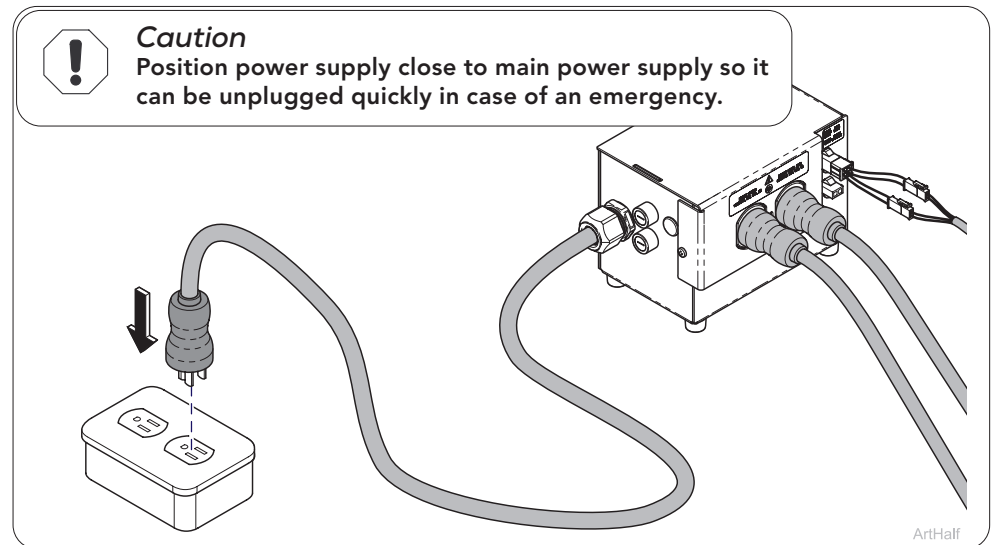

**Step 4:**  
Plug in power supply into outlet.
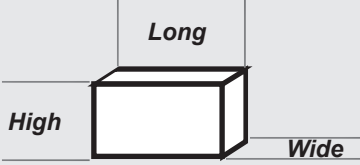
<b>Specifications</b>	
Electrical Requirements:	120 V~ , 60Hz, 12A
	Fuse F1, F2: T3.15A H 250 V
Classifications:	Protection Against Electric Shock: Class I Ingress Protection (IEC 60529): IPX0 Mode of Operation: Continuous Operation
Dimensions:  7.2 in. (18.3 cm.) Long 4.5 in. (11.4 cm.) Wide 4.7 in. (11.9 cm.) High	

**WARNING**

Only use Midmark specified fuses, failure to do so could result in a fire or an electrical hazard.

<b>Certification and Compliance</b>	
<b>Certifications:</b>	ANSI/AAMI ES60601-1:2005 + A1:2012 + A2:2021 CAN/CSA-C22.2 No. 60601-1:14 + A2:22
<b>Compliance:</b>	IEC 60601-1-2:2014+A1:2020 IEC TS 60601-4-2:2024 ISO 9680:2021 ANSI/AAMI/IEC 60601-1-2:2014 + A1:2021

## EMC - Manufacturer's Declaration and Guidance

### Guidance and manufacturer's declaration – electromagnetic emissions

The Midmark Power Supply is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Power Supply should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic environment – guidance
RF Emissions CISPR 11	Group 1	The Midmark Dental Delivery System uses RF energy only for its internal functions. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A	The Midmark Dental Delivery System is suitable for use in industrial areas and professional healthcare facilities. The Midmark Dental Delivery System is not intended to be used in a residential environment.
Harmonic emissions IEC 61000-3-2	NA	Not applicable do to the Midmark Dental Delivery System is 120 VAC 60 Hz only and does not exceed 220 VAC.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	NA	

### Recommended separation distances between portable and mobile RF communications equipment and the Midmark Dental Delivery System

The Midmark Dental Delivery System is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Midmark Dental Delivery System can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and the Midmark Dental Delivery System as recommended below, according to the maximum output power of the communications equipment.

Radiated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \times \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \times \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \times \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.34
10	3.69	3.69	7.38
100	11.67	11.67	23.34

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Guidance and manufacturer's declaration – electromagnetic immunity		
The Midmark Dental Delivery System is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Dental Delivery System should assure that it is used in such an environment.		
Immunity Test	Compliance Level	Electromagnetic environmental - guidance
Portable and mobile RF communications equipment should be used no closer to any part of the Midmark Dental Delivery System including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:		
Conducted RF IEC 61000-4-6 (2013)	3V	$d = 1.2 \times \sqrt{P}$
Radiated RF IEC 61000-4-3 (2010)	3V/m	$d = 1.2 \times \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \times \sqrt{P}$ 800 MHz to 2.5 GHz
Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation in meters (m).  Field strength from fixed RF transmitters, as determined by the electromagnetic site survey. a) Should be less than the compliance level in each frequency range. b) Interference may occur in the vicinity of equipment marked with the following symbol: 		
<b>NOTE 1:</b> At 80 MHz and 800 MHz, the higher frequency range applies		
<b>NOTE 2:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.		
<p><sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location is which the Midmark Dental Delivery System is used exceeds the applicable RF compliance level above, the Midmark Dental Delivery System should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Midmark Dental Delivery System.</p> <p><sup>b</sup> Over the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.</p>		

**Guidance and manufacturer's declaration – electromagnetic immunity**

The Midmark Dental Delivery System is intended for use in the electromagnetic environment specified below. The customer or the user of the Midmark Dental Delivery System should assure that it is used in such an environment.

<b>Immunity Test</b>	<b>Compliance Level</b>	<b>Electromagnetic environmental - guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2 (2008)	± 8 kV Contact ± 2kV, ± 4 kV, ± 8 kV, ± 15 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Proximity Fields from RF Wireless Communications Equipment IEC61000-4-3 (2010)	3 V/m (80 MHz -2.7 GHz) 80% AM at 1kHz	
Electrical fast Transient / burst IEC 61000-4-4 (2012)	± 2kV @ 100kHz Rep. (AC Mains) ± 1kV @ 100kHz Rep. (I/O Ports)	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5 (2017)	± 0.5 kV, ± 1 kV L-L ± 0.5 kV, ± 1 kV, & ± 2 kV L-PE	Mains power quality should be that of a typical commercial or hospital environment.
Conducted RF IEC61000-4-6 (2013)	3 V m) 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 (2009)	30 A/m, 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 (2017)	0% UT for 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°. 0% UT for 1 cycle 70% UT for 25/30 cycles 0% UT for 250/300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Midmark Dental Delivery System requires continued operation during power mains interruptions, it is recommended that the Midmark Dental Delivery System be powered from an uninterruptible power supply or a battery.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

## Warranty Information

Scan the following QR code for access to the Dental Chair warranty.

## Warranty Registration

Scan the following QR code for access to the Dental Chair warranty registration.

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