Midmark Rear Assistant's Delivery Chair Mounted (ADS-1100-001) Setup Guide

Applies to Models:
ADS-1100-001

Special Tools:
none

WARNING


Midmark requires installer to anchor dental chair to floor to ensure proper operation and optimal stability when installing chair with rear-mounted accessories.

Hardware Kit

THIS HARDWARE INSTALLATION KIT
(002-10994-00)
APPLIES TO MIDMARK CHAIR
REAR MOUNTED ASSISTANT'S

REAR ASSISTANT'S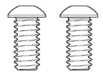

(2) 940-0312-52
5/16-18 X 3/4" HEX SCREW

(3) 040-0010-204
#10-24 X 1/2"
FLANGE HEAD TORX SCREW

(1) 104499
HOSE CLAMP


(2) 015-0013-03
CABLE TIE


See installation manuals included in product for more detailed installation procedures or scan the QR code to download online

066-10397-00 Rev. AA2 (5/12/2025)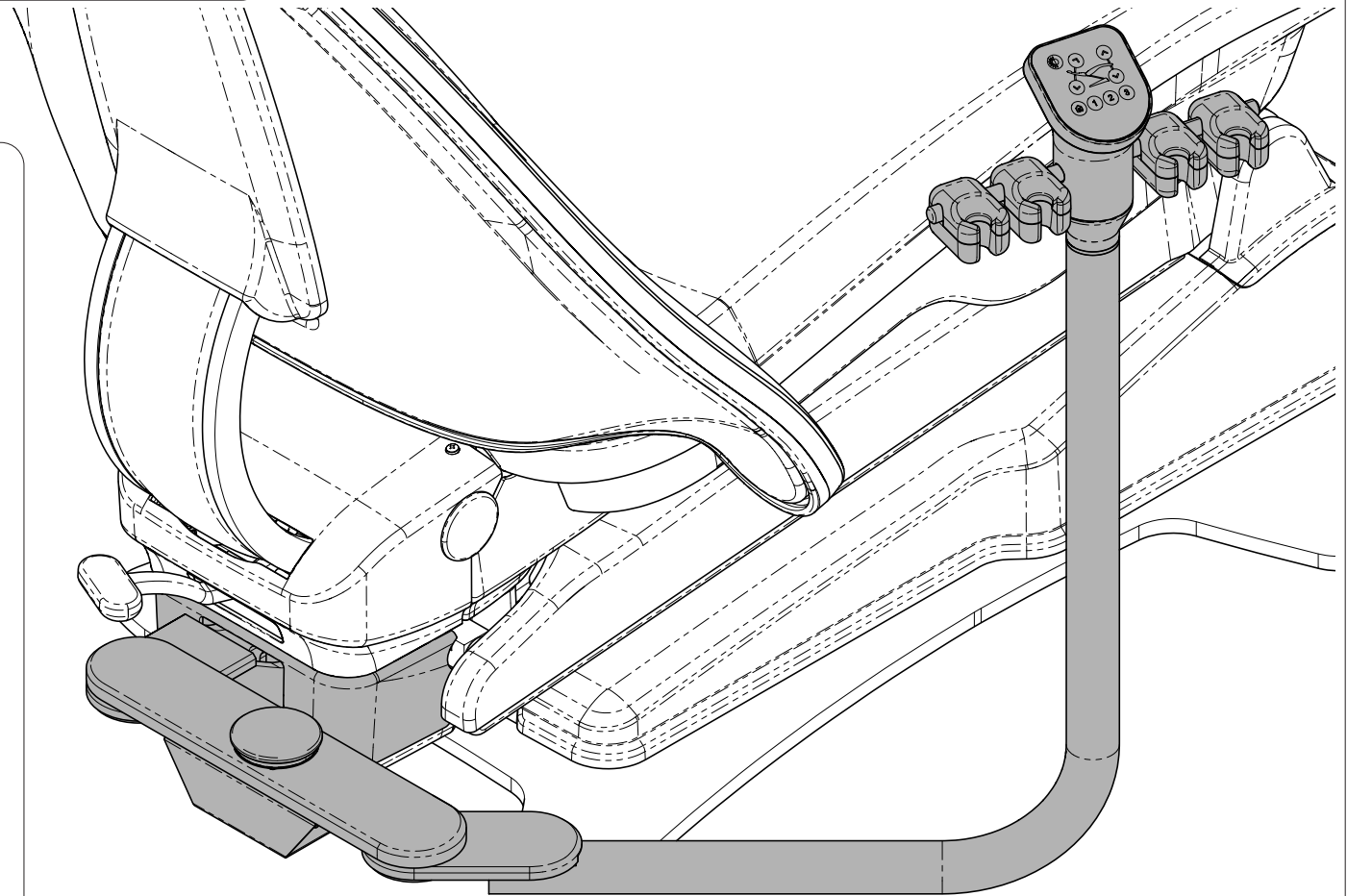


ArtFull

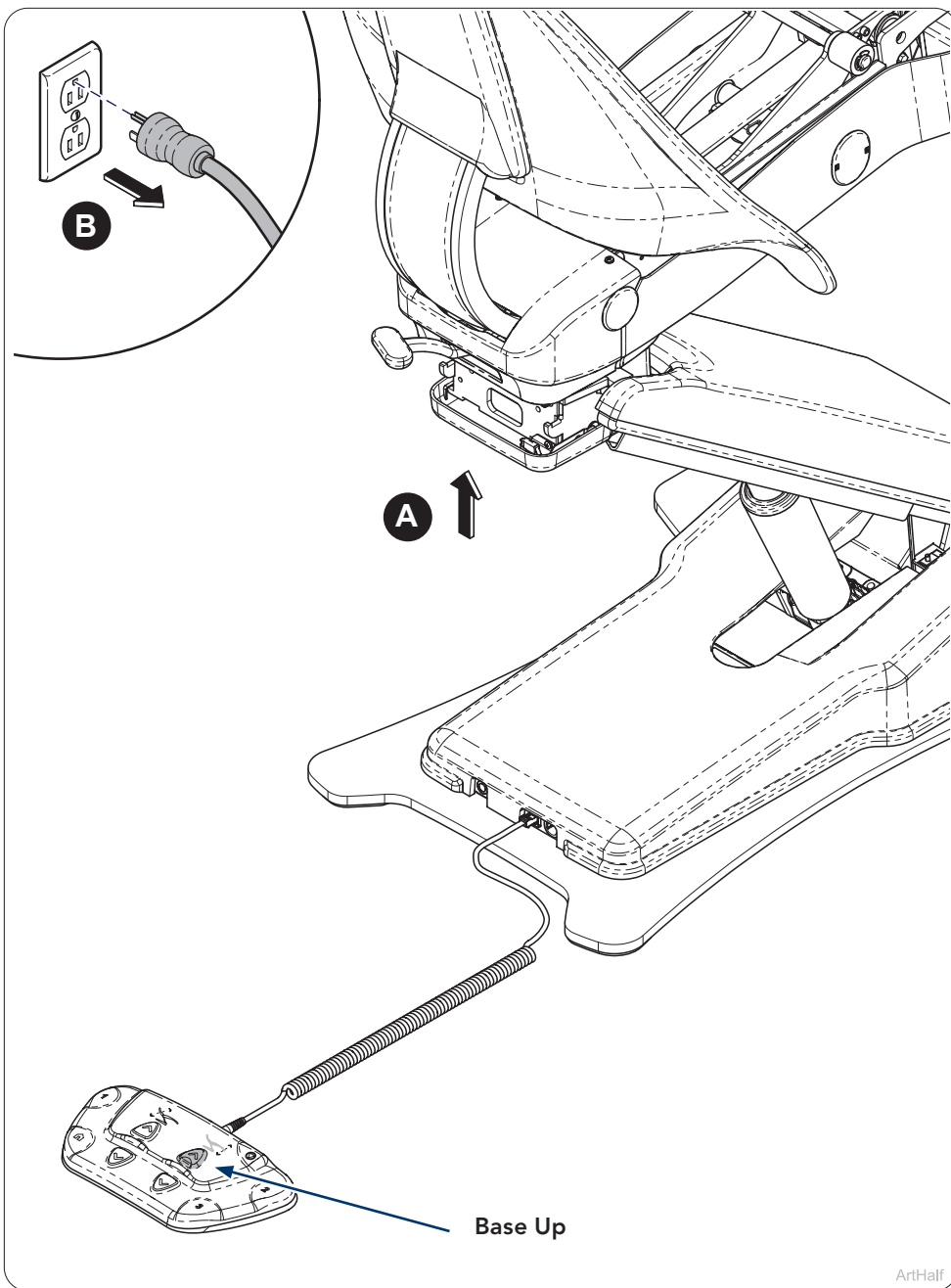
Step 1:

- A) Raise chair to highest point by press Base Up on foot control.
- B) Unplug power to chair.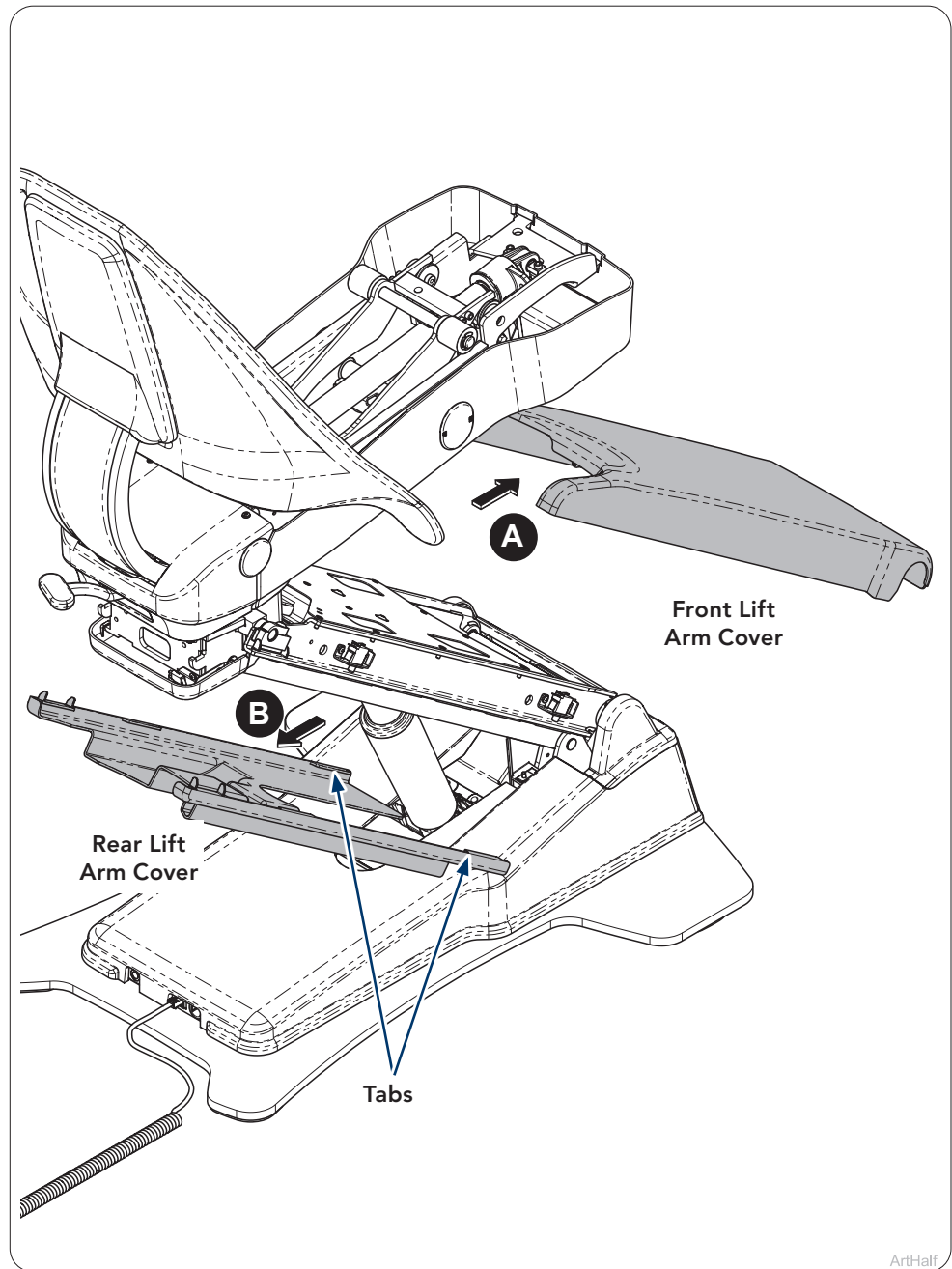

Step 2:

- A) Remove front lift arm cover from chair.
- B) Remove rear lift arm cover by pulling tabs (6) out away from frame.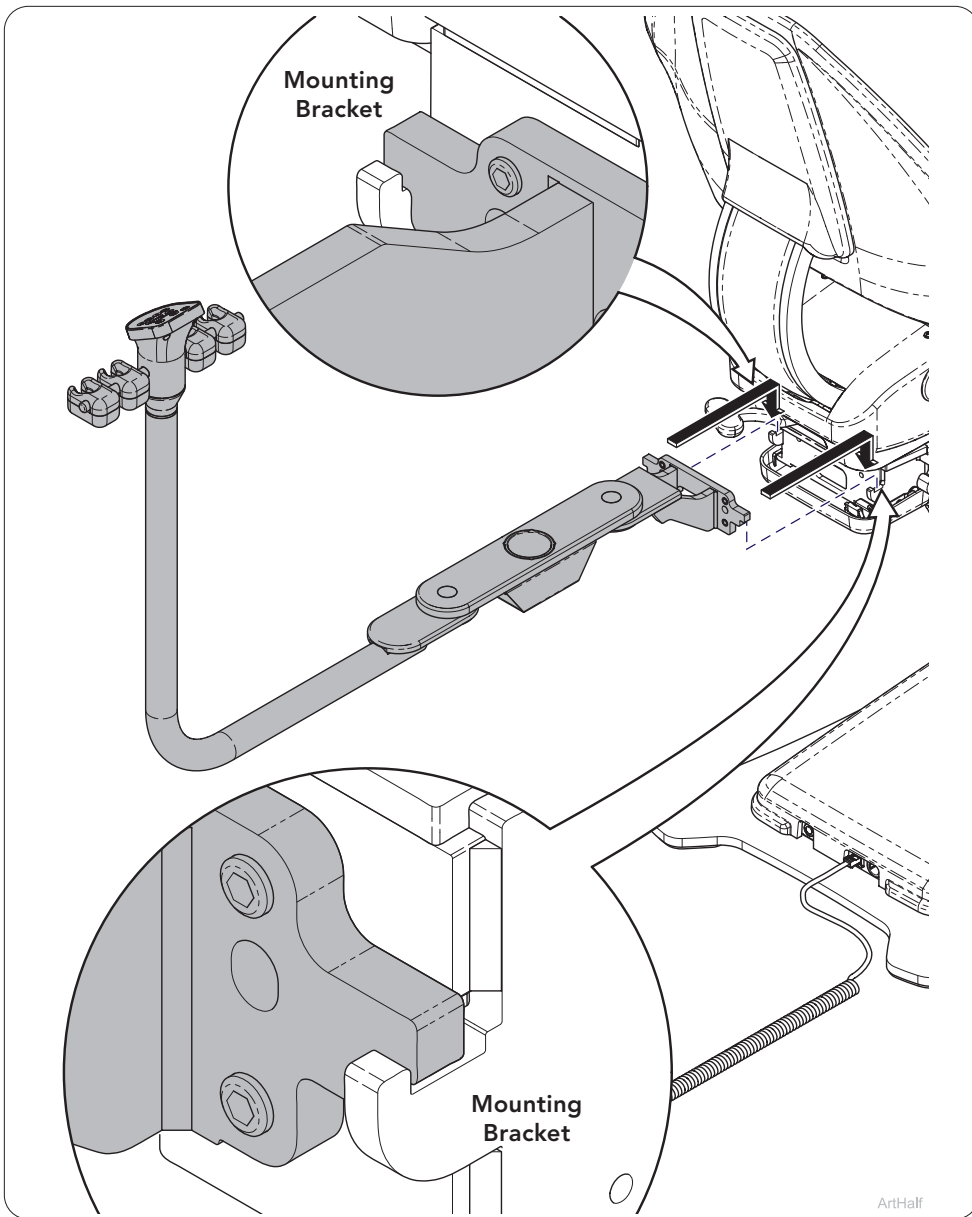


Warning

Rear assistant's must remain straight out from chair and SECURED before rotating, failure to do so will result in product to fall from chair.


Step 3:

A) Attach the rear assistant's to the back of the chair and ensure the unit is straight out and supported with mounting bracket as shown.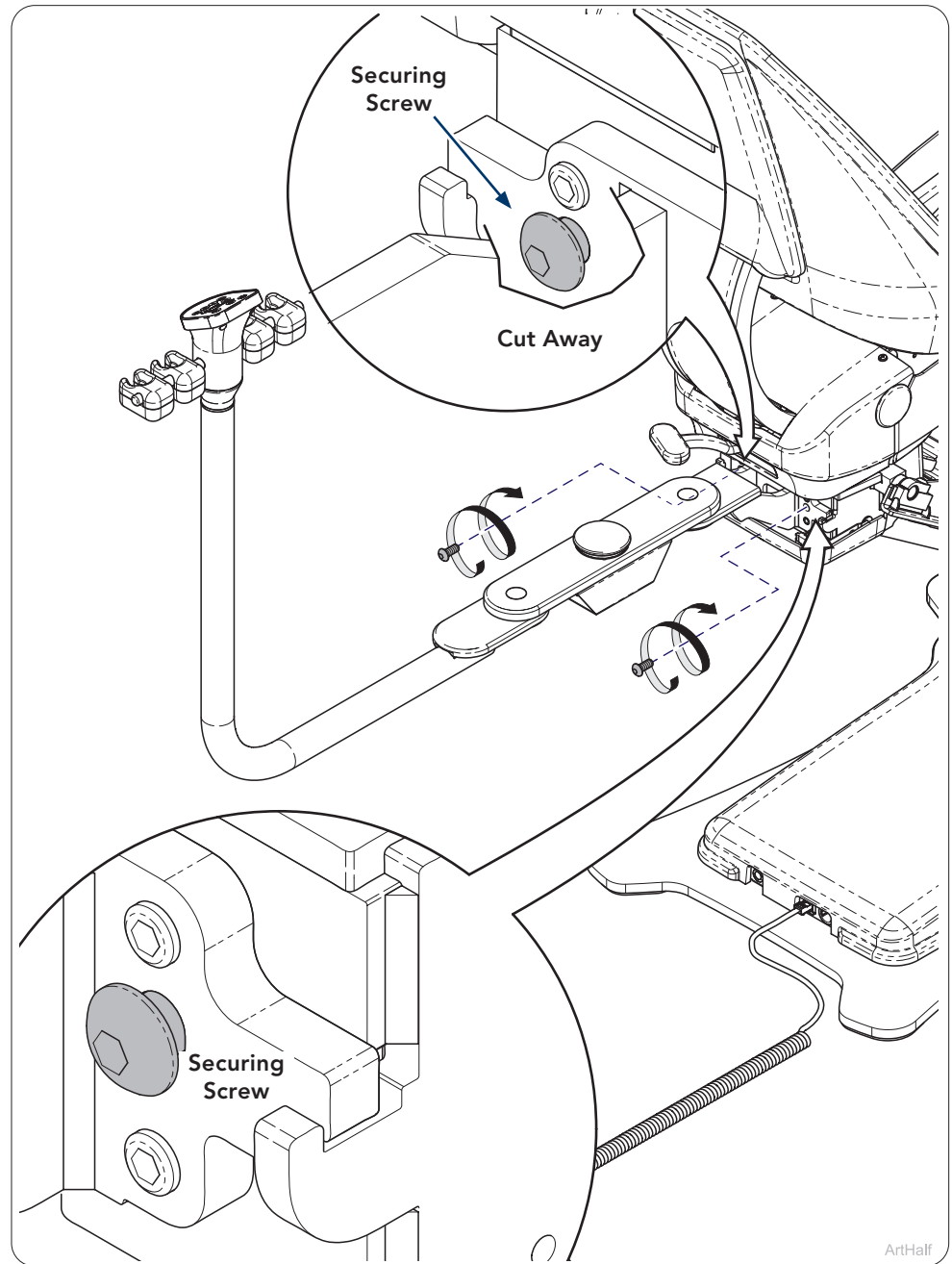


Step 4:


A) Loosely fasten the button head securing screws (2) as shown.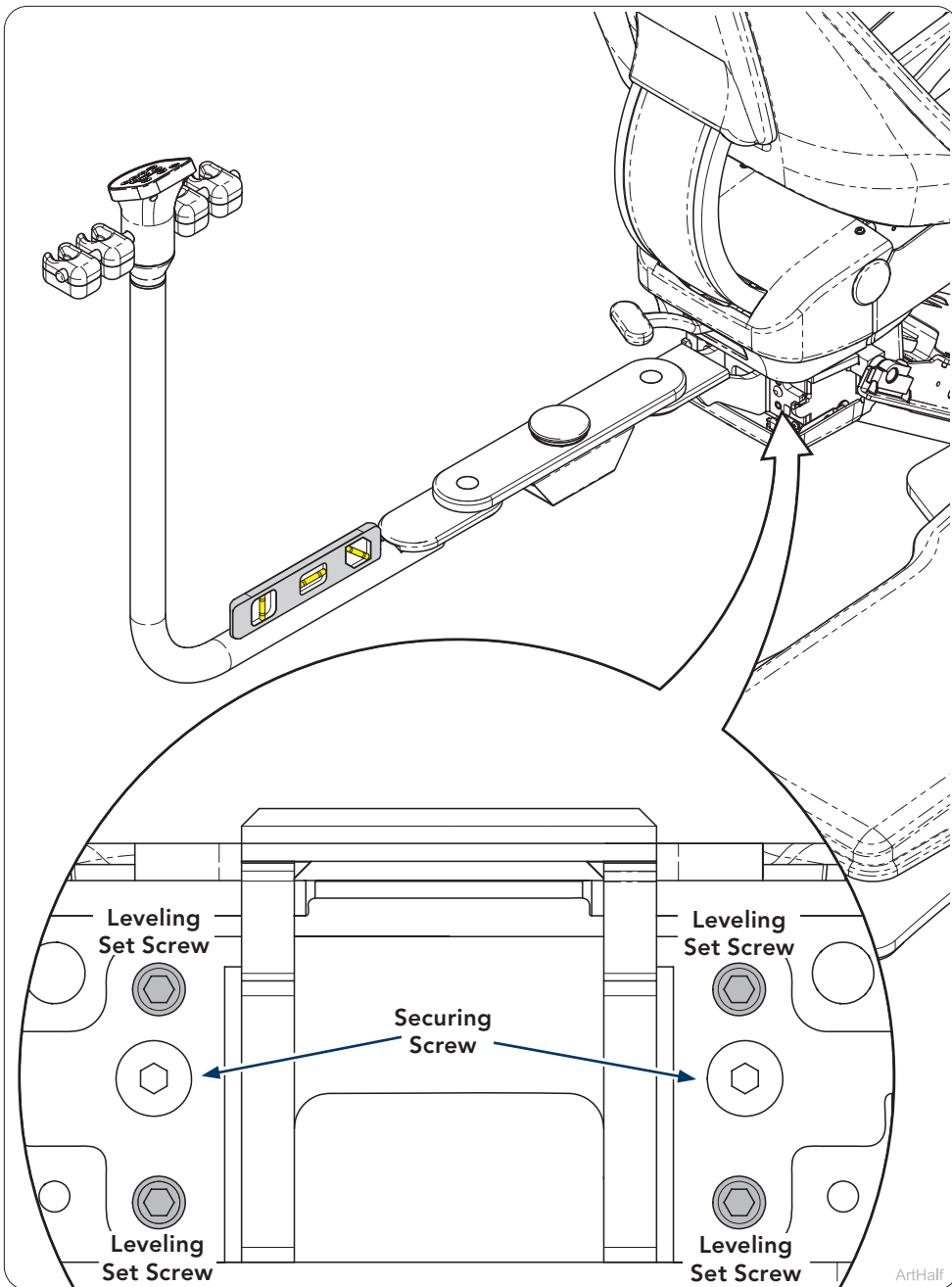

Step 5:
A) Level the rear assistant's with set screws (4).

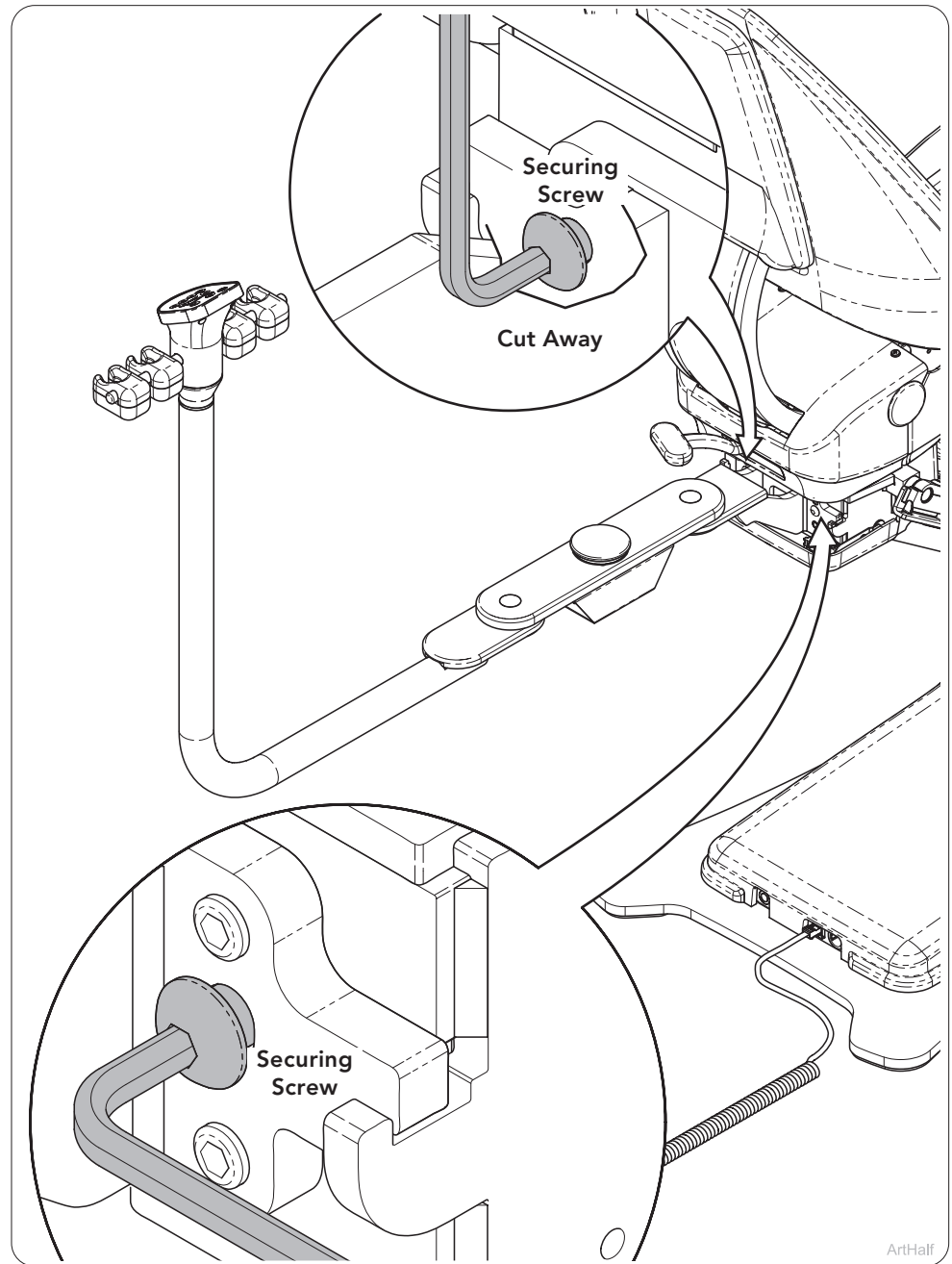
Step 6:
A) Tighten button head securing screws (2) to keep leveling adjustments.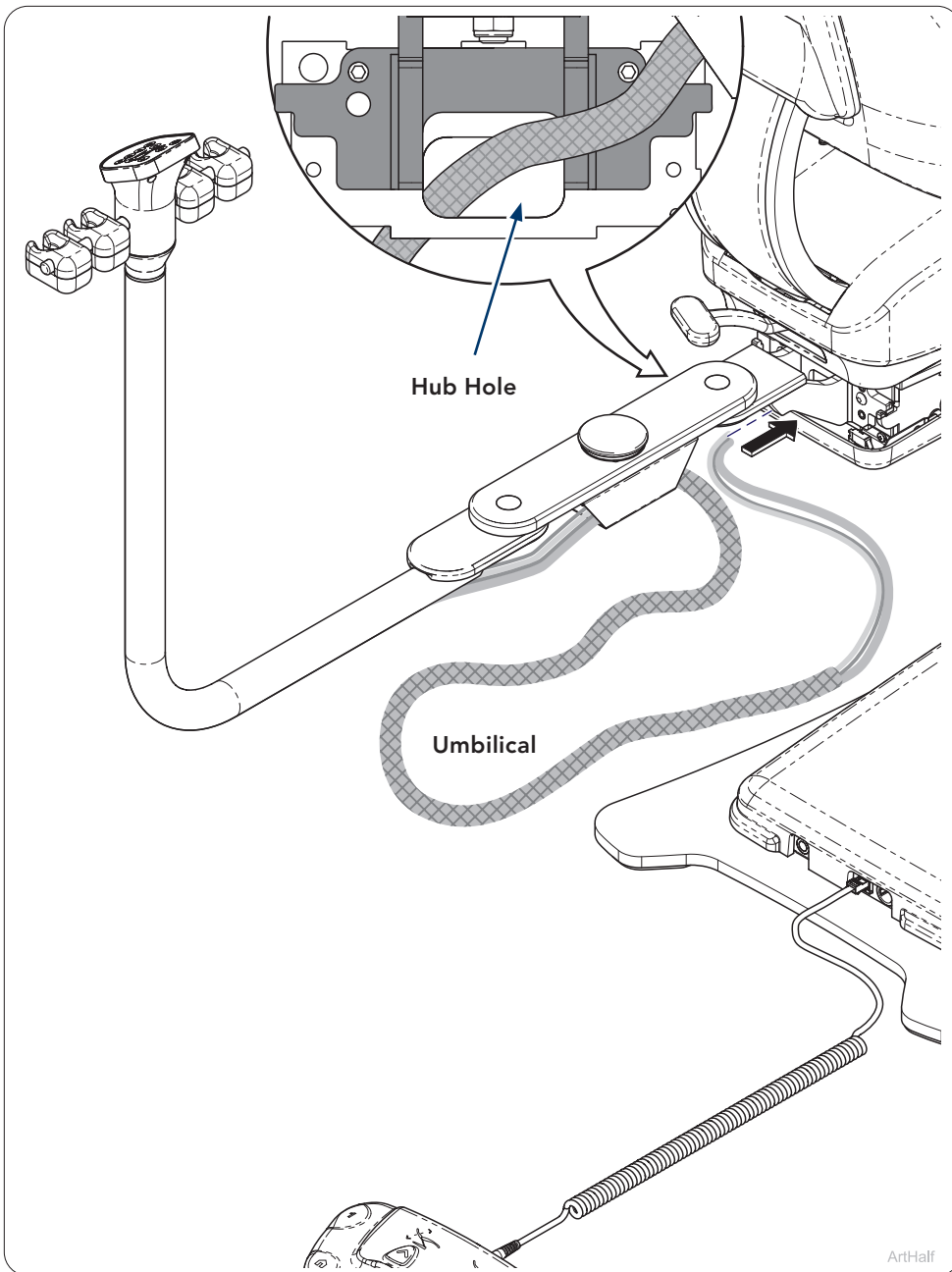


Step 7:

- A) Route rear assistant's umbilical through hub hole so it is under chair lift arm.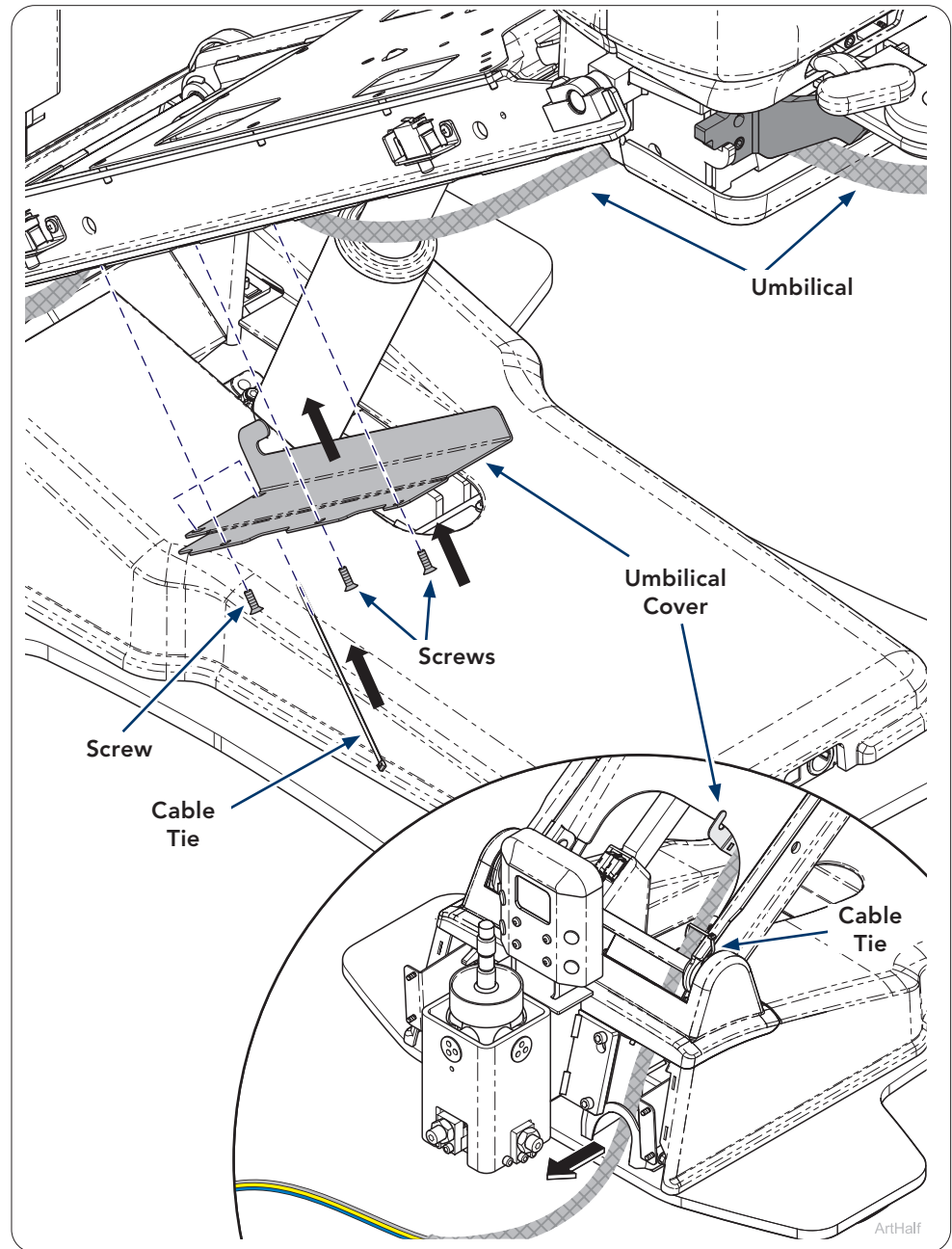


Step 8:


- A) Route the umbilical on the patient left through bottom of lift arm.
 B) Install umbilical cover, hooking onto lift arm, secure with screws (3).
 C) Install cable tie (2) to secure umbilical to cover and lift arm.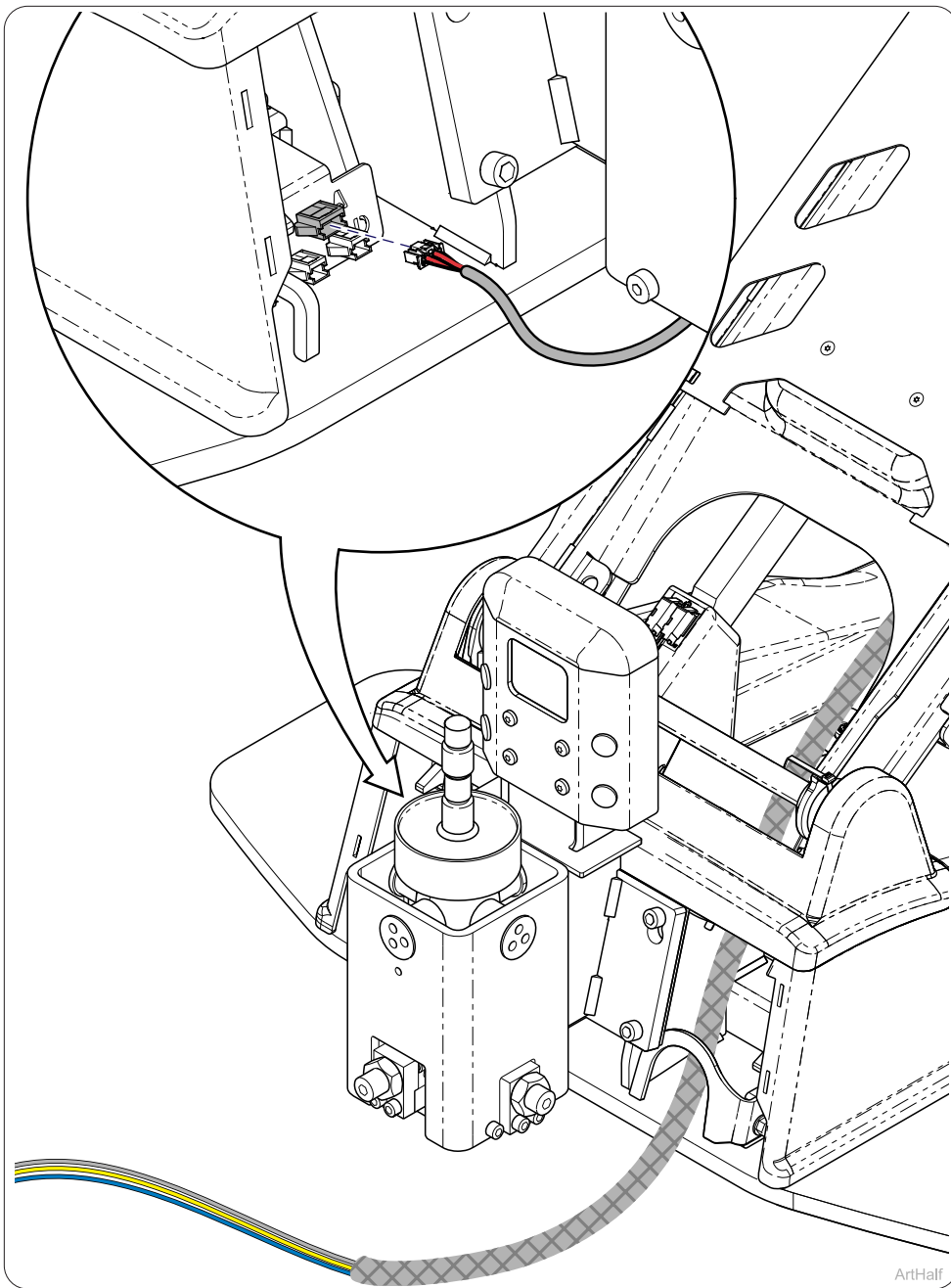


Step 9:
 A) Route the CAN Cable under Left-Right mount in front and connect the to the chair into connector labeled A.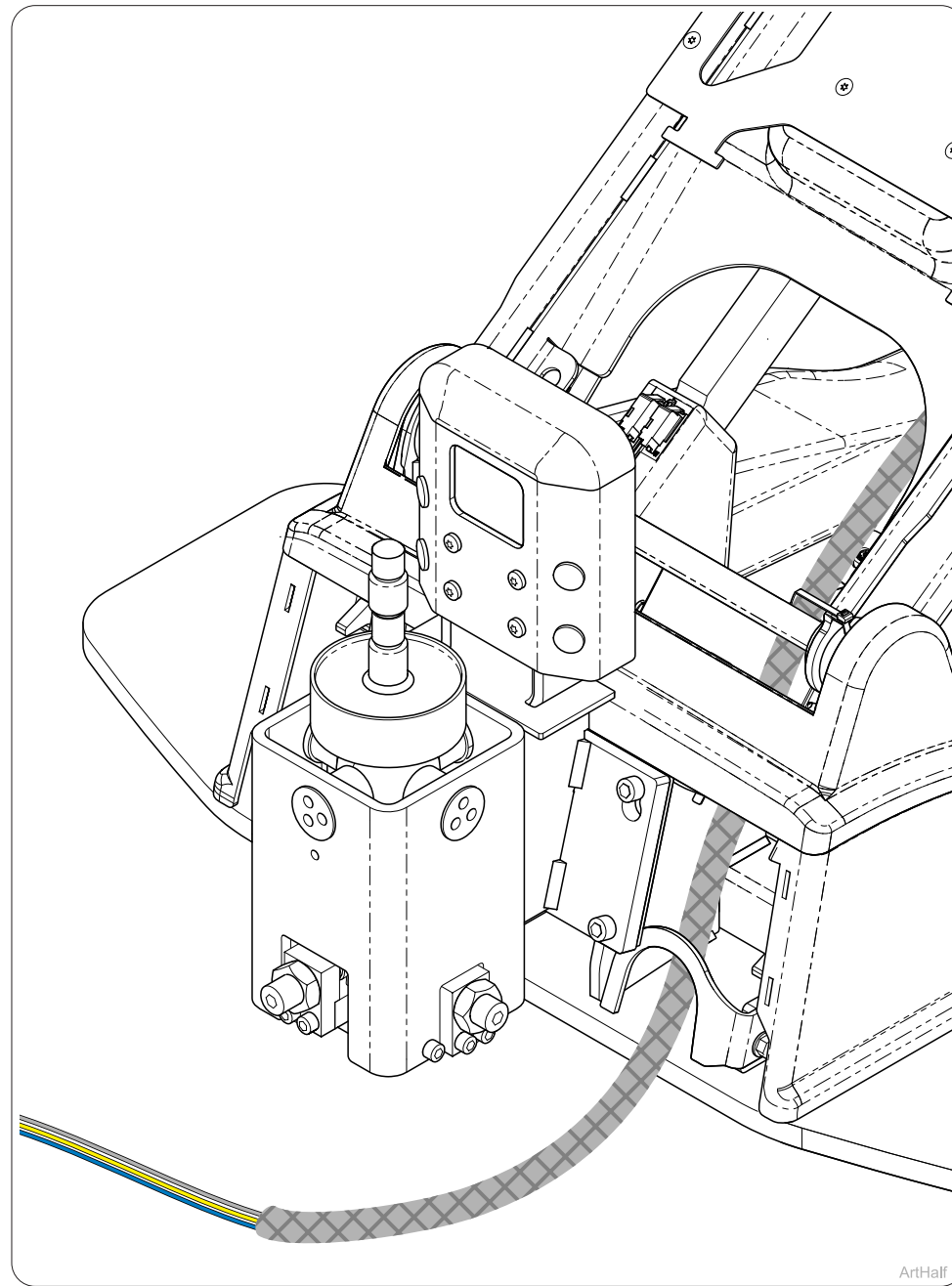


Step 10:
 A) Refer to last page for tubing hook ups in J-box.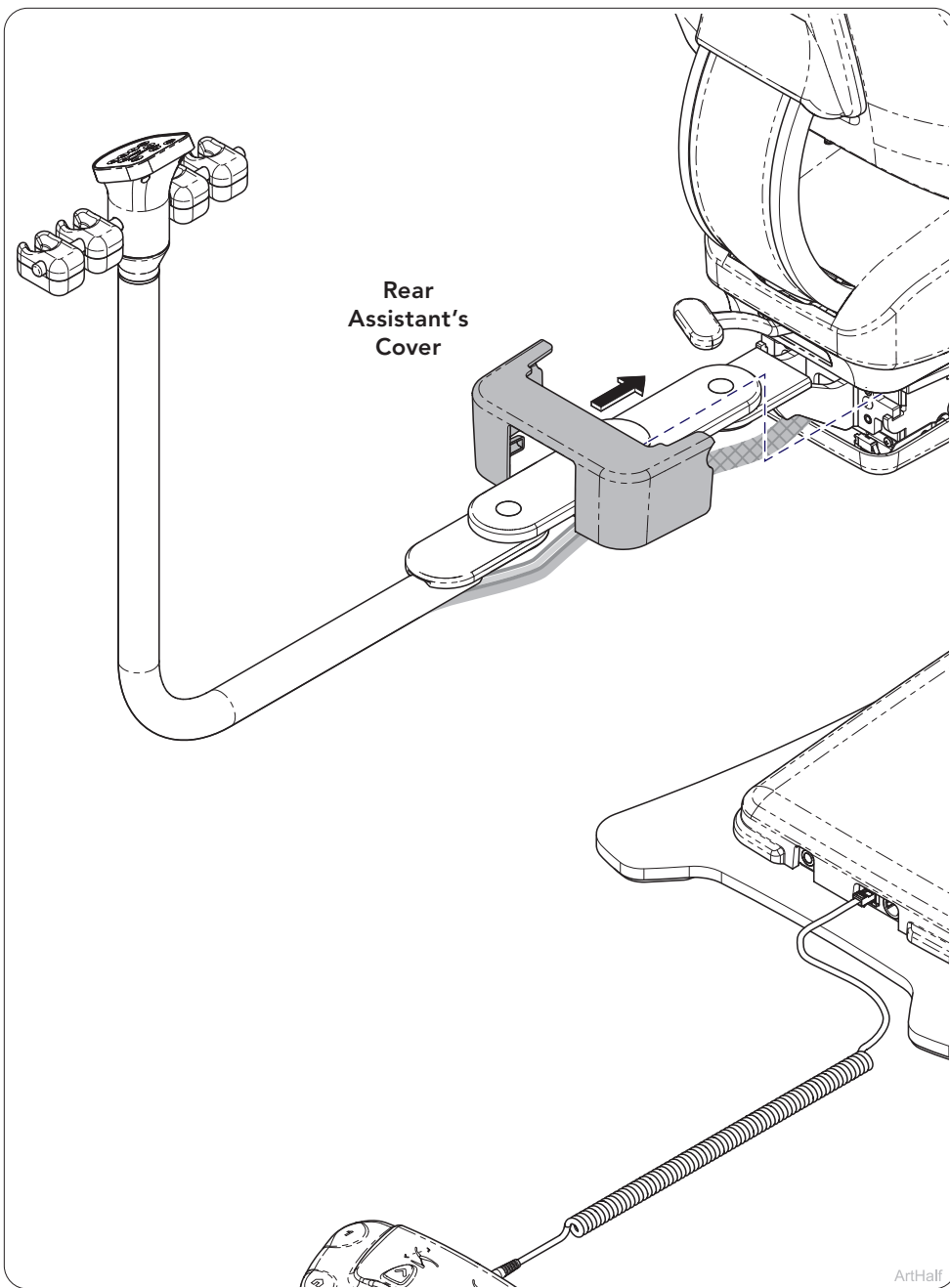

Step 11:
A) Install rear assistant's cover.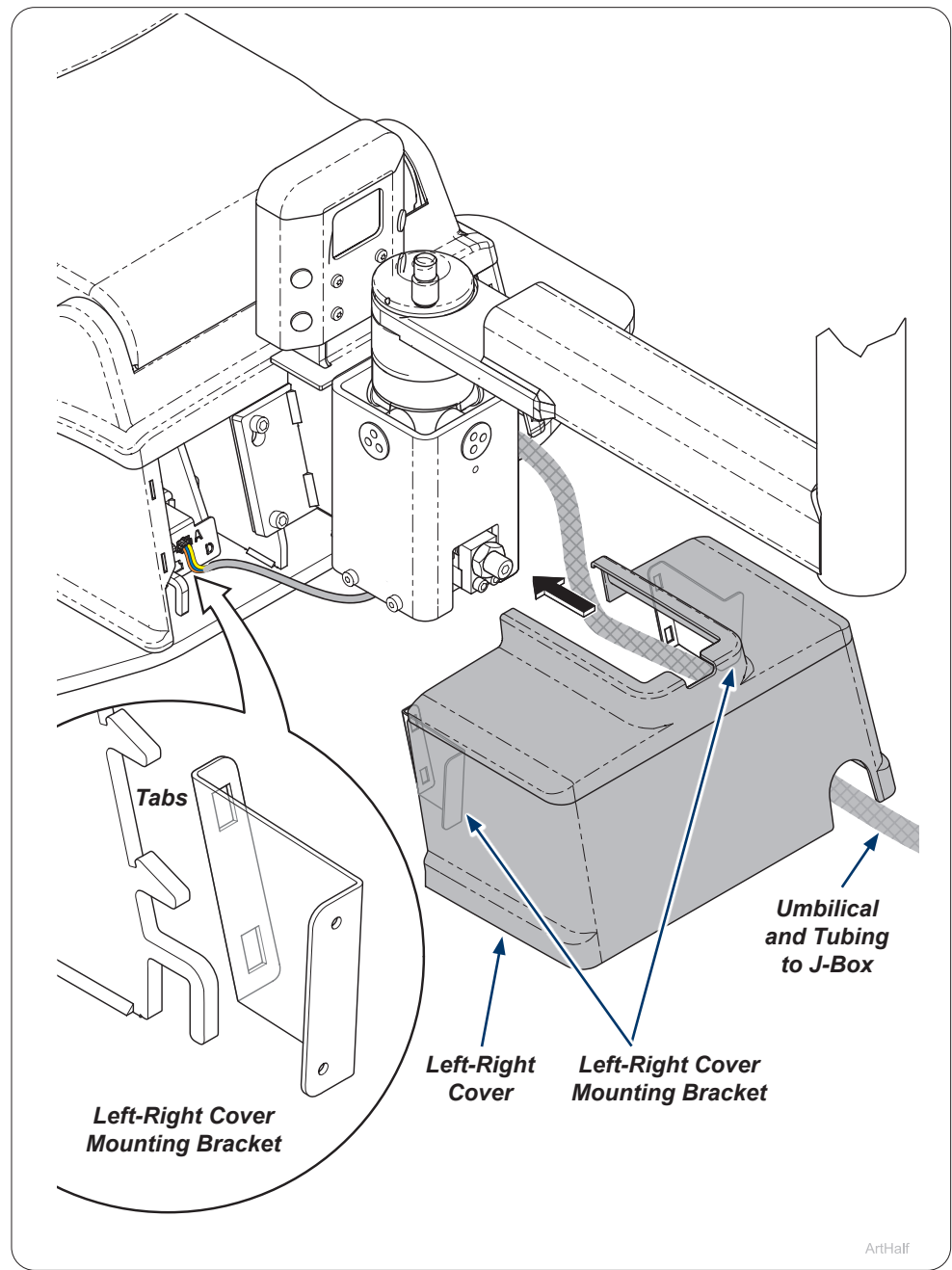


Step 12:
A) Push left-right cover on to chair until both left-right cover mounting brackets lock on to tabs.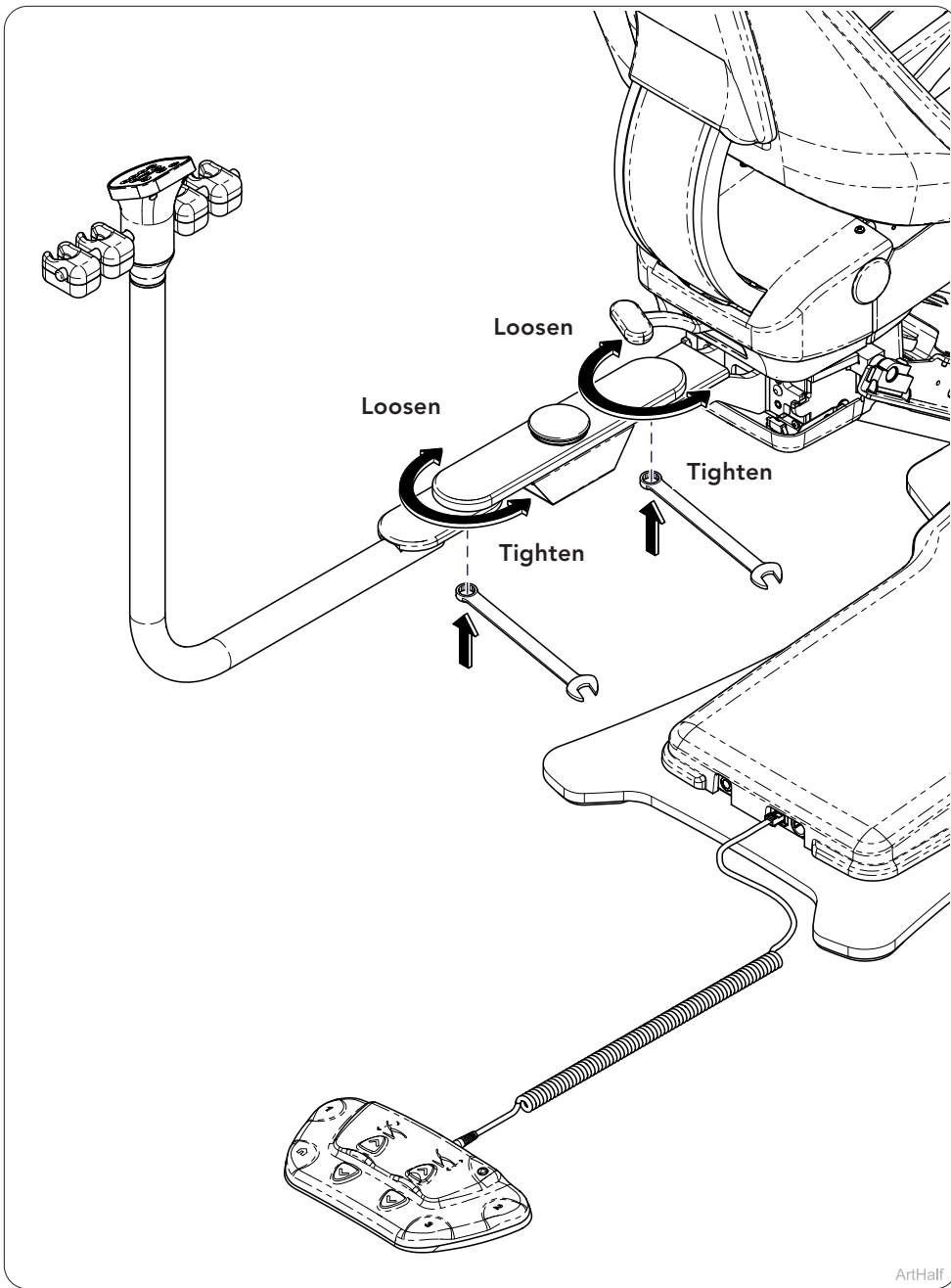

ArtHalf

Step 13:

A) Adjust mount bolts if rear assistant's drift as shown.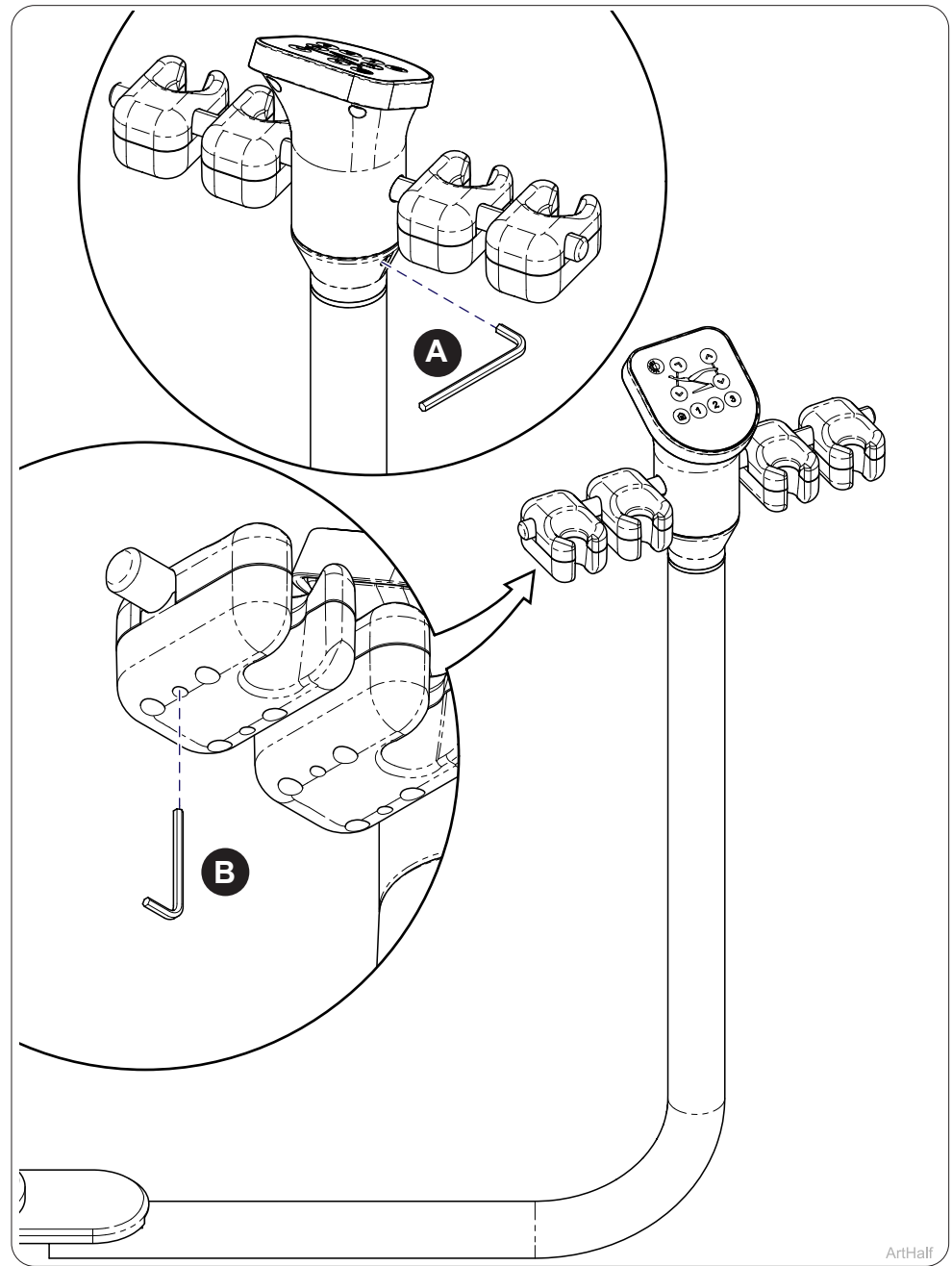

ArtHalf

Step 14:

A) Tension can be adjusted with Allen wrench for assistant's.
 B) Tension can be adjusted with Allen wrench for handpieces.




Midmark Dental Chair (CH1000) Setup Guide

Applies to Models:
CH1000

Note

All CH-1000 models use these instructions for setup.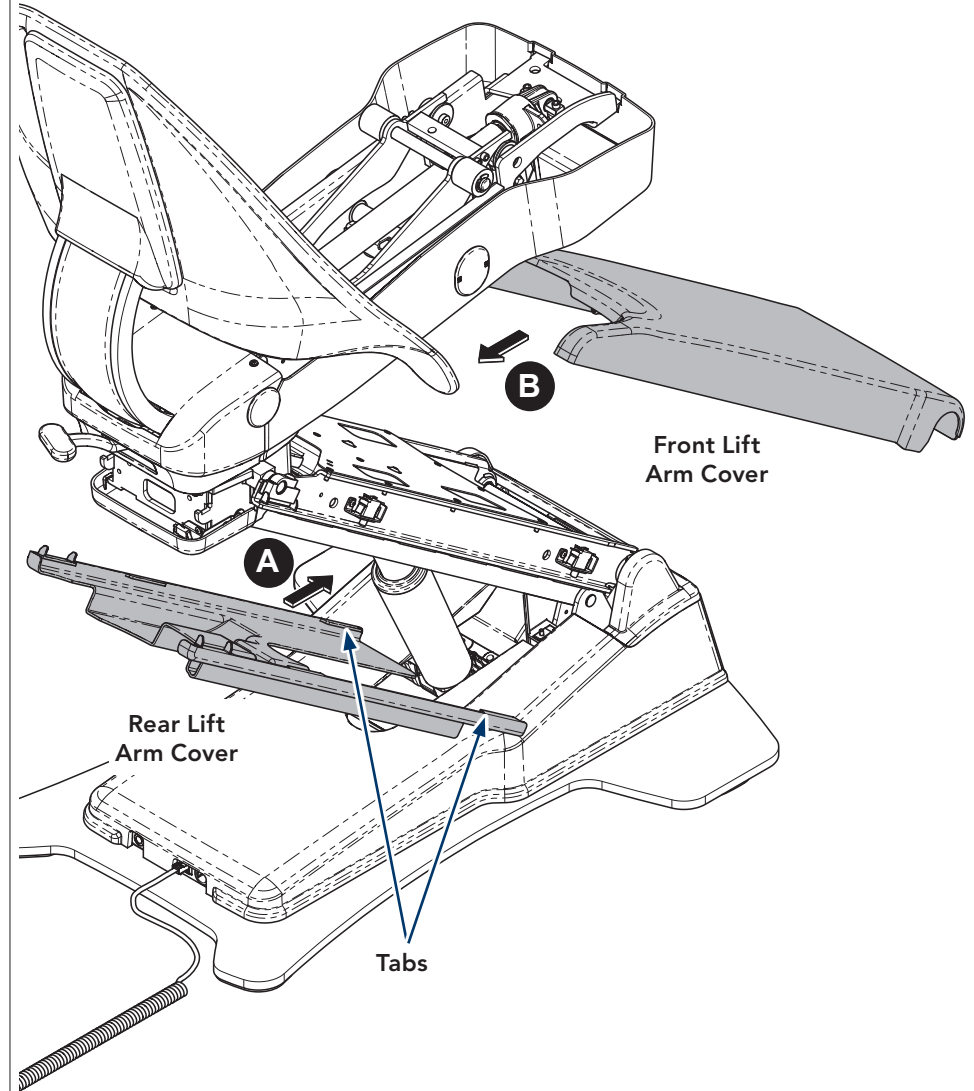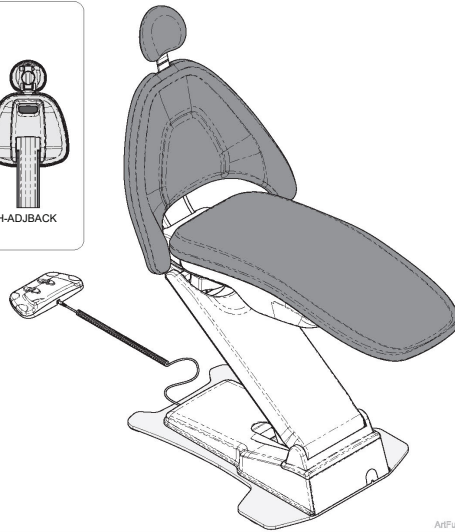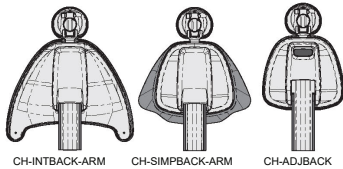

Step 15:

A) Refer back to chair setup to verify other Midmark equipment will not need to be setup.

Step 16: Install covers only if all equipment is complete.



A) Install rear lift arm cover by pulling tabs (6) out to clip onto frame.
B) Install front lift arm cover from chair.

Regulator Installation

Regulator installed to shutoff valve.

Insert regulator into compression fitting of shut-off valve and tighten compression fitting.


Note: Rear Assistant's Unit Tubing Connections are in junction box and L/R Cover Box.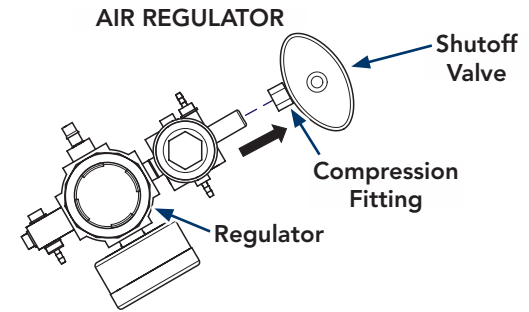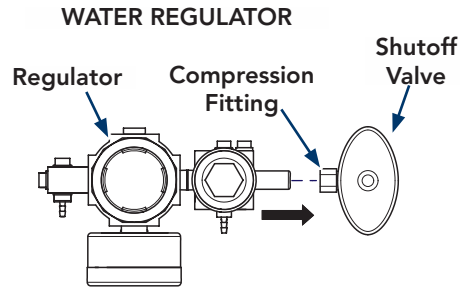

Tubing Connections

Note

Refer to Technical Library at midmark.com for theory of operation, flow diagrams and troubleshooting.

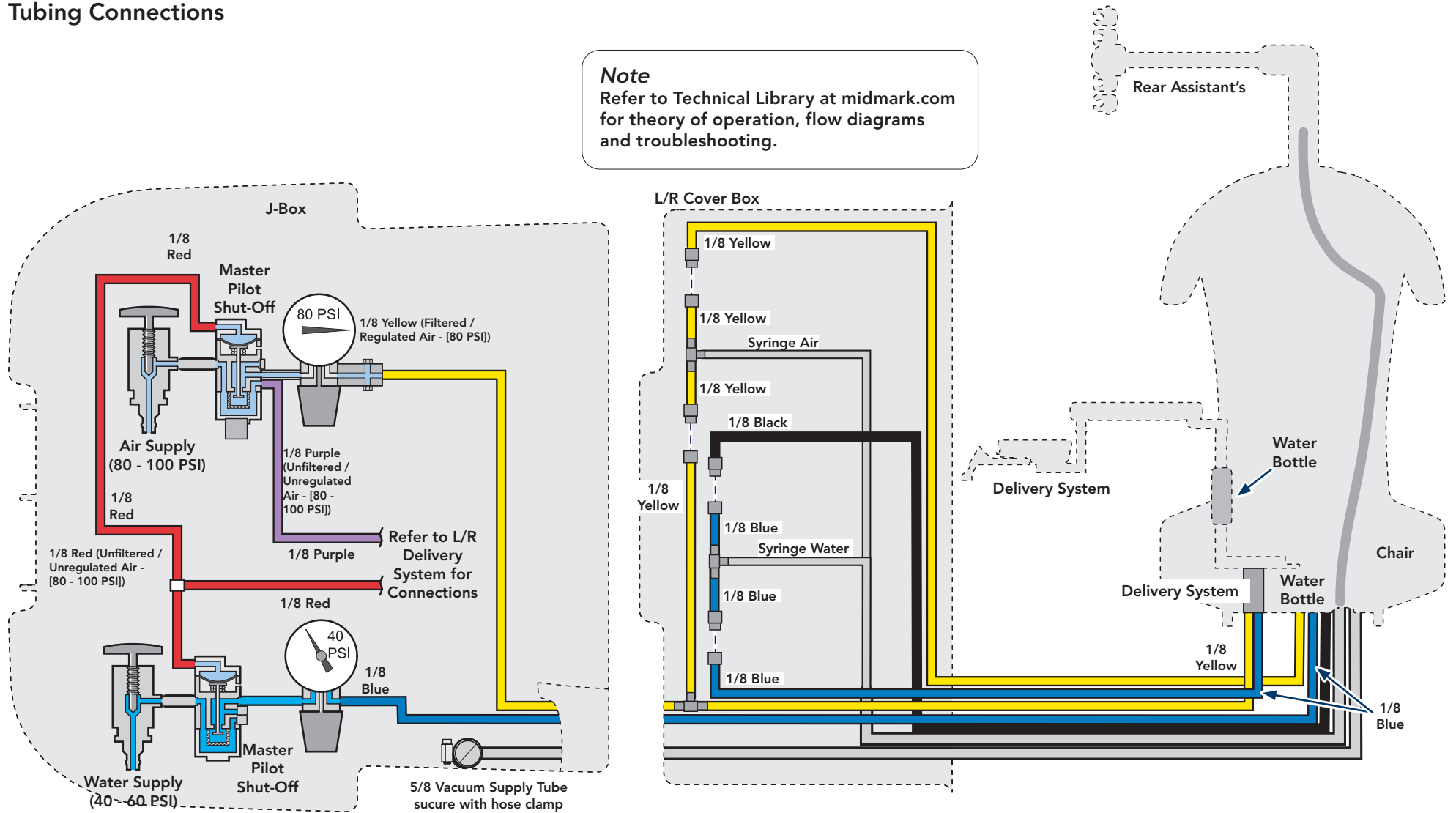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