Midmark Delivery System Column Mounted (DS-1200-001) Setup Guide

Applies to Models:
DS-1200-001

Special Tools:
none

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

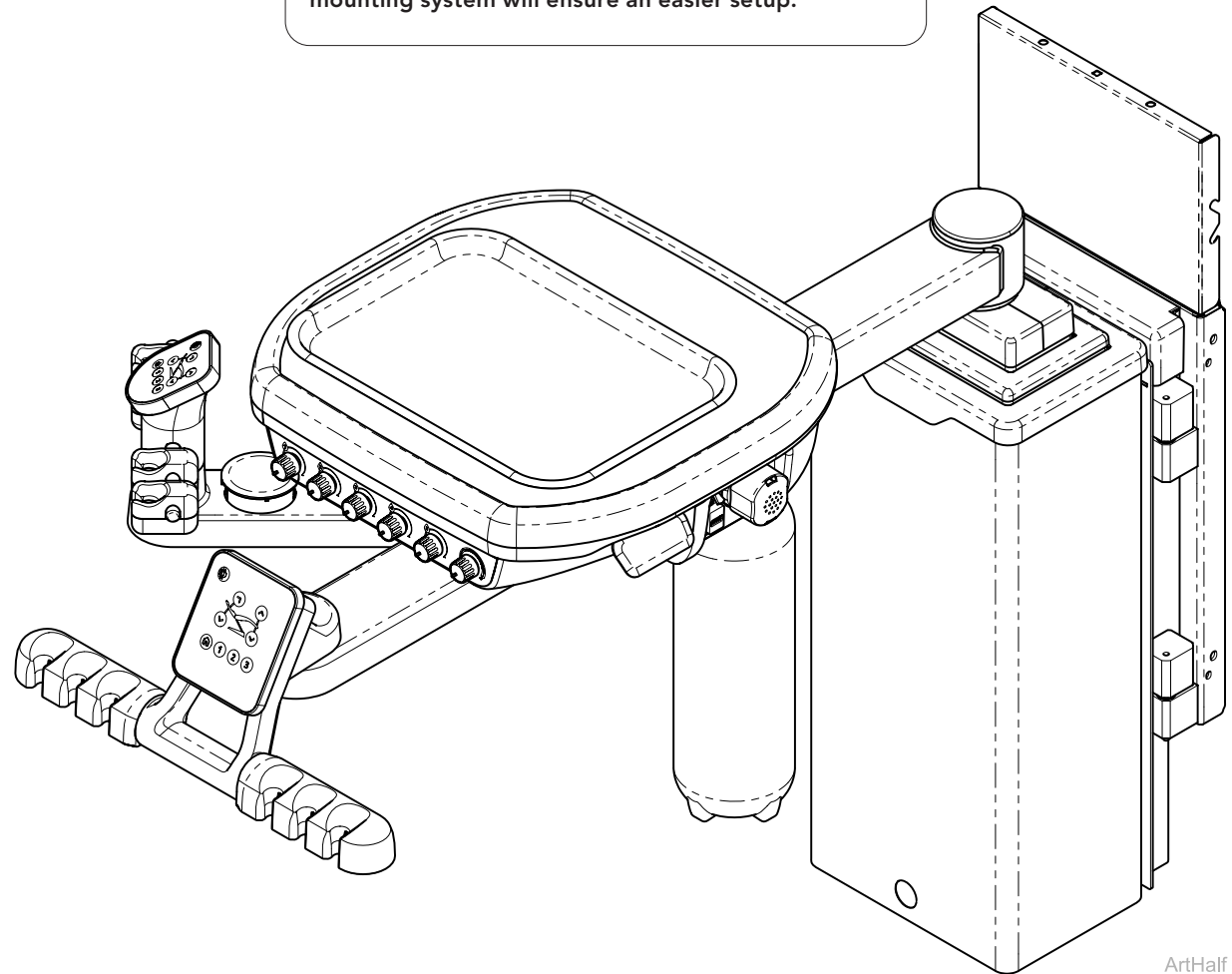
Equipment Alert

Before installing the Midmark Delivery System, ensure the 12:00 cabinet is level and secure to the floor.

Note

Midmark recommends multiple installers to lift.


Keeping Delivery System packaging wrap on while mounting system will ensure an easier setup.


Hardware Kit

THIS HARDWARE INSTALLATION KIT
(029-11816-00)
APPLIES TO MIDMARK 12:00 COLUMN

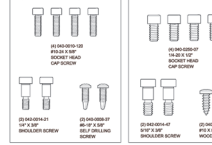SYNTHESIS CABINET


ARTIZAN EXPRESSIONS

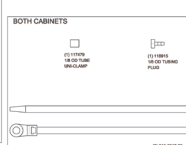BOTH CABINETS

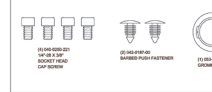BOTH CABINETS

BOTH CABINETS

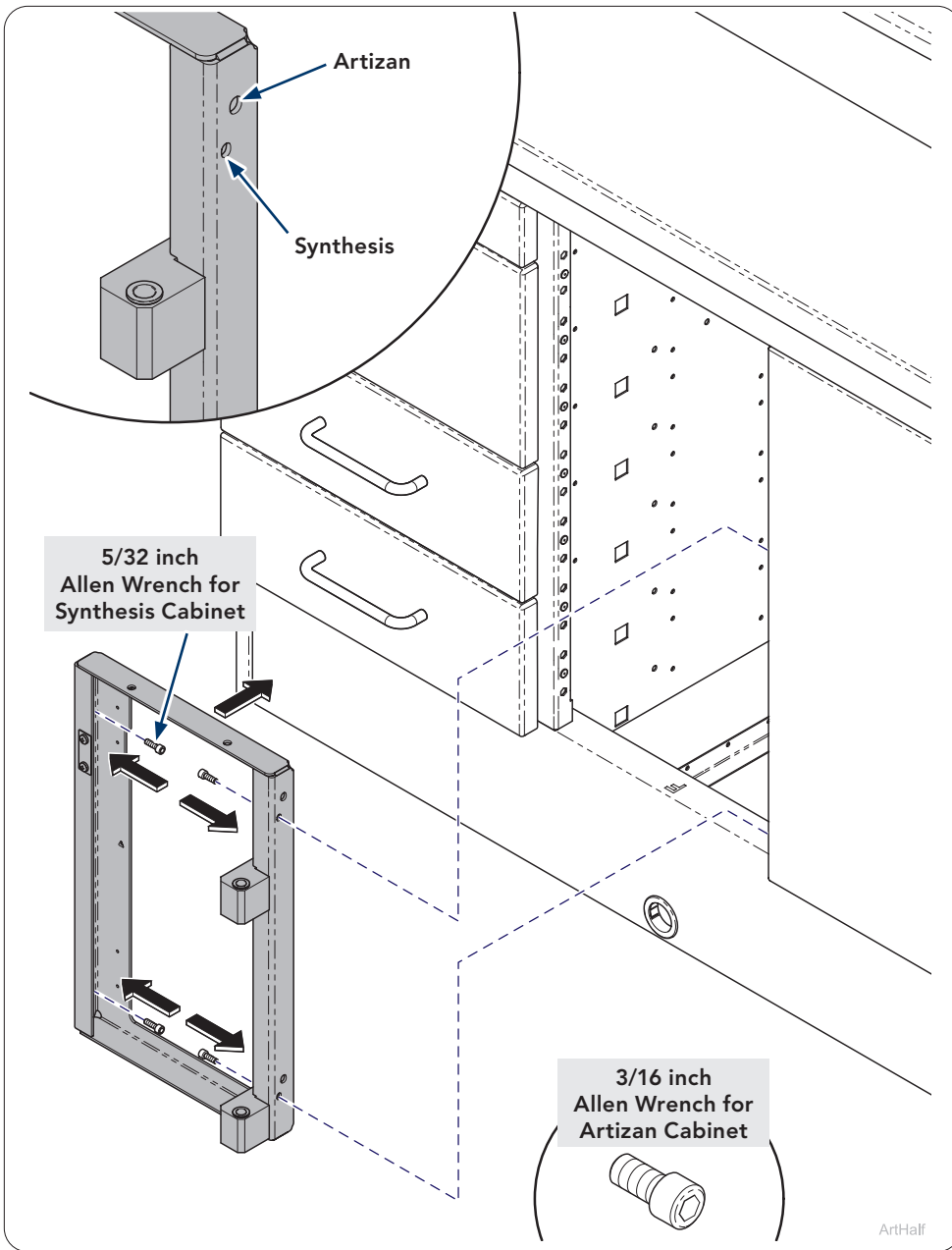
COLUMN MOUNTER


See installation manuals included in product for more detailed installation procedures or scan the QR code to download online.

Step 1: Synthesis Cabinet - shown

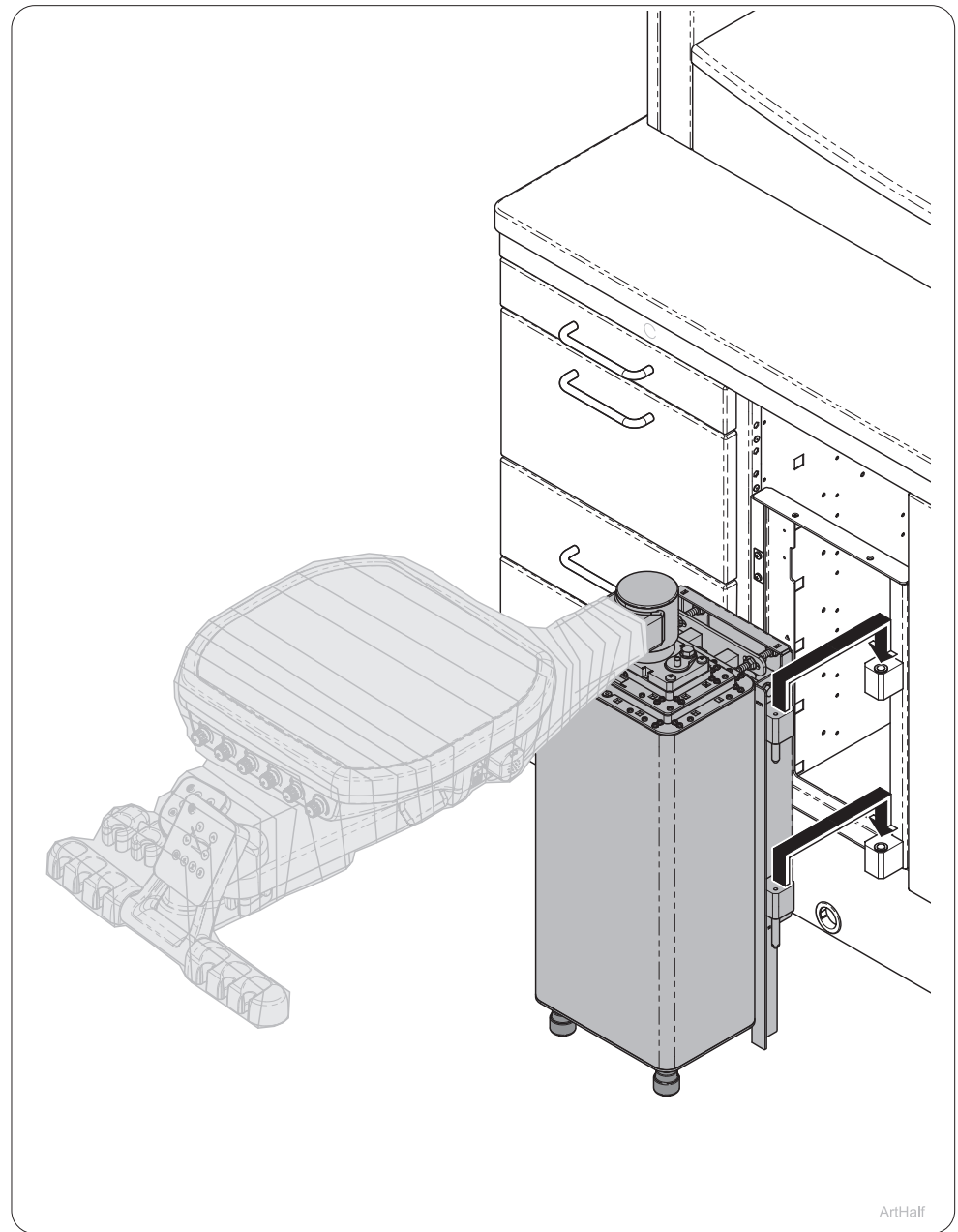
A) Install column mounting bracket with screws (4).

Step 2:

- A) Slide column delivery system into position and lift sliding pins into column mounting bracket.
- B) Latch column to mounting bracket.

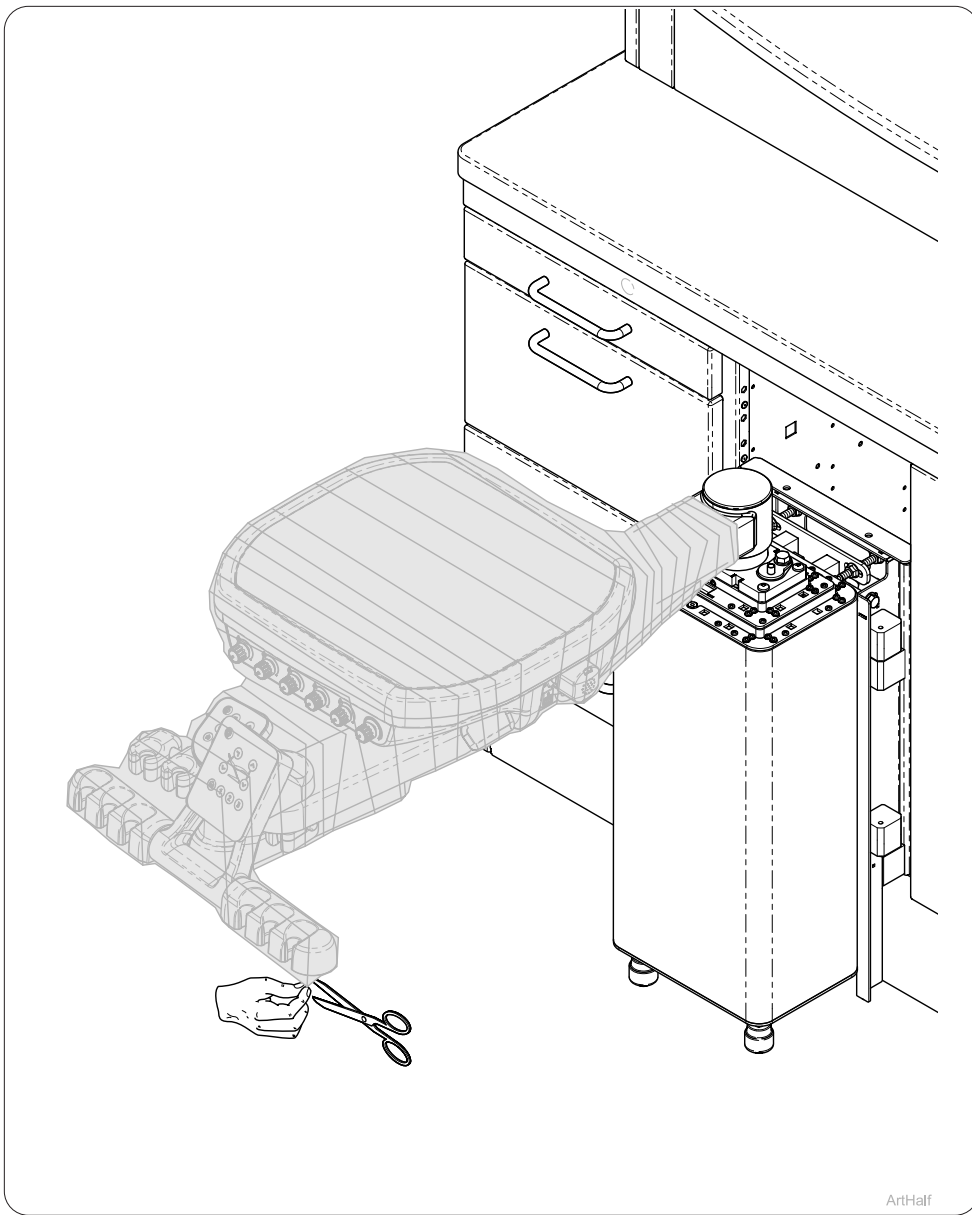
Note: Midmark recommends multiple installers to lift.

ArtHalf

Step 3:

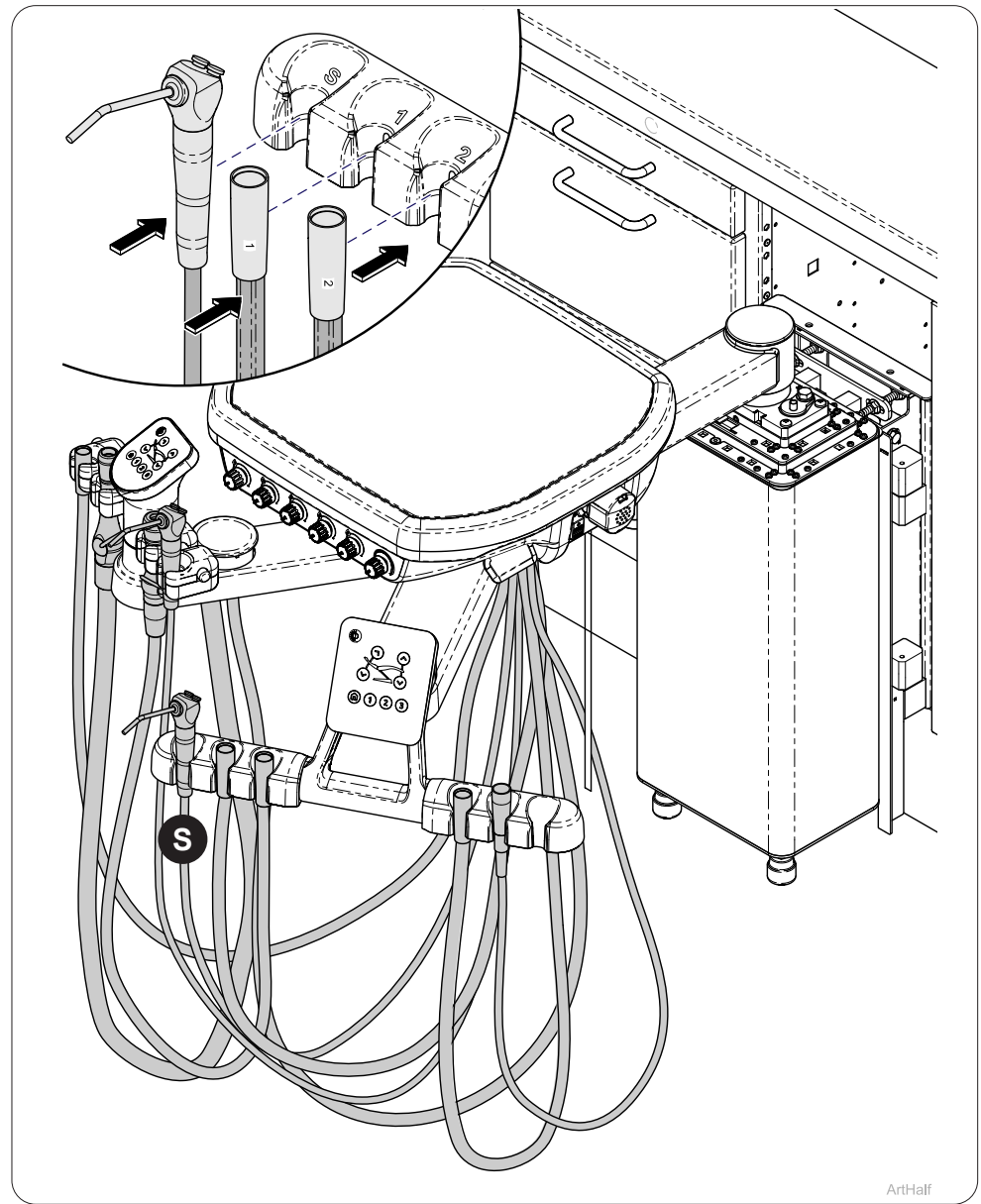A) Remove packaging from delivery system.


ArtHalf

Step 4:


- A) Place tubing and handpieces into handpiece holder.**
- B) The syringe is positioned in holder labeled "S". The holders after syringe are labeled with numbers from 1 starting left to right.**

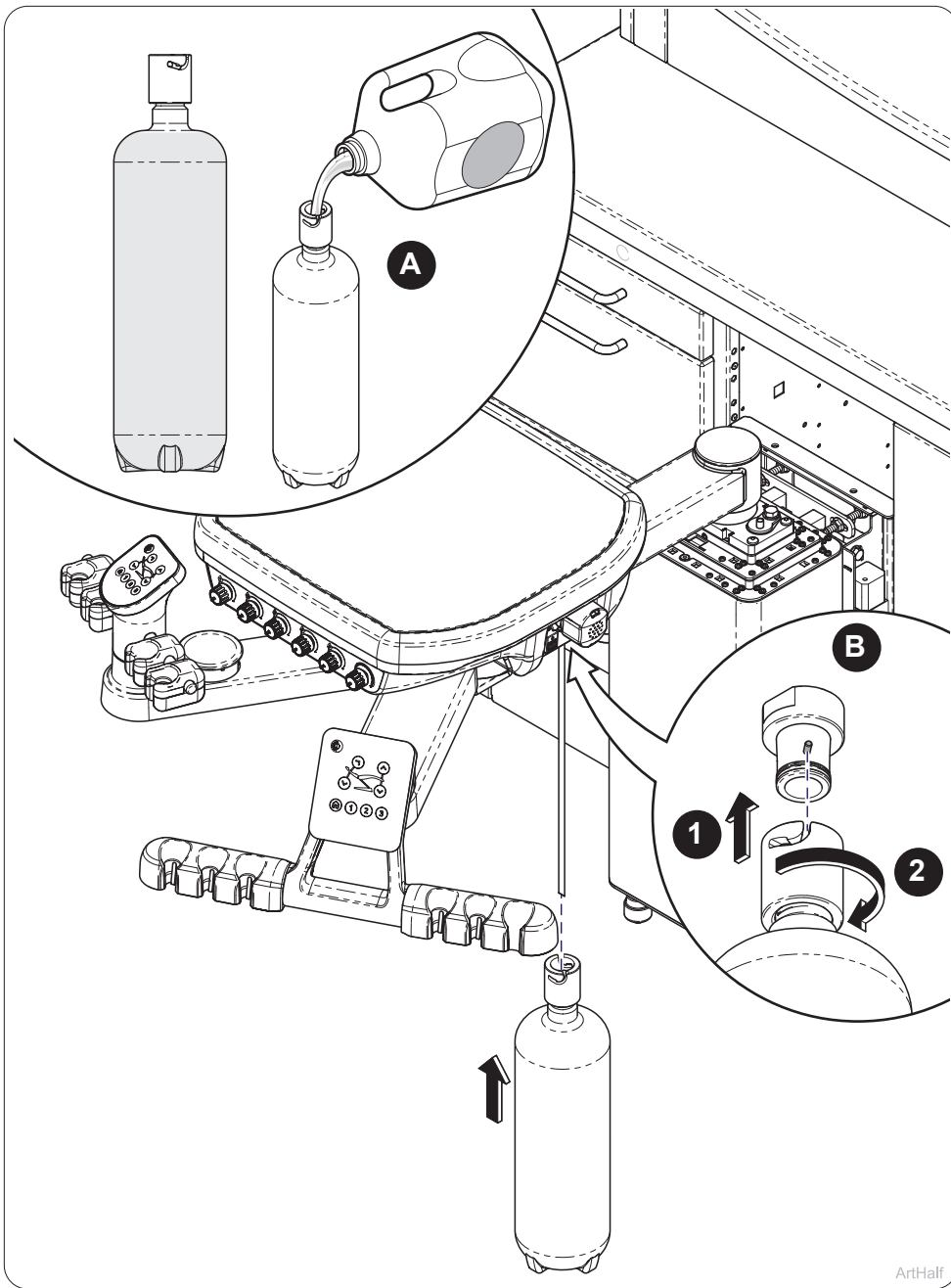Note: Handpiece locations are defined. Proper operation requires placing the correct tubing in the designated holder location.


Step 5:

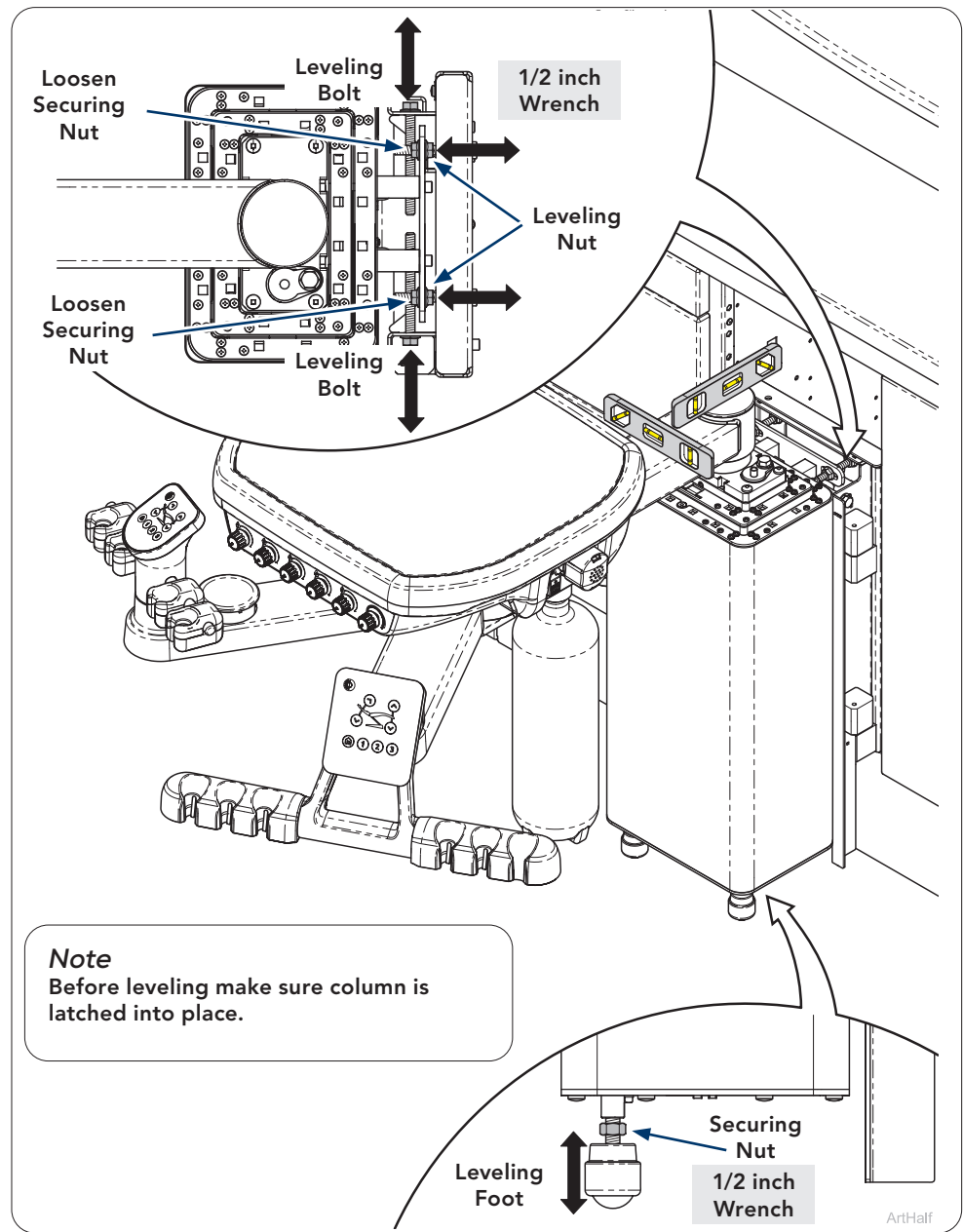- A) Fill water bottle with water that meets local drinking codes.
- B) Install water bottle as shown.


Note
Before leveling make sure column is latched into place.

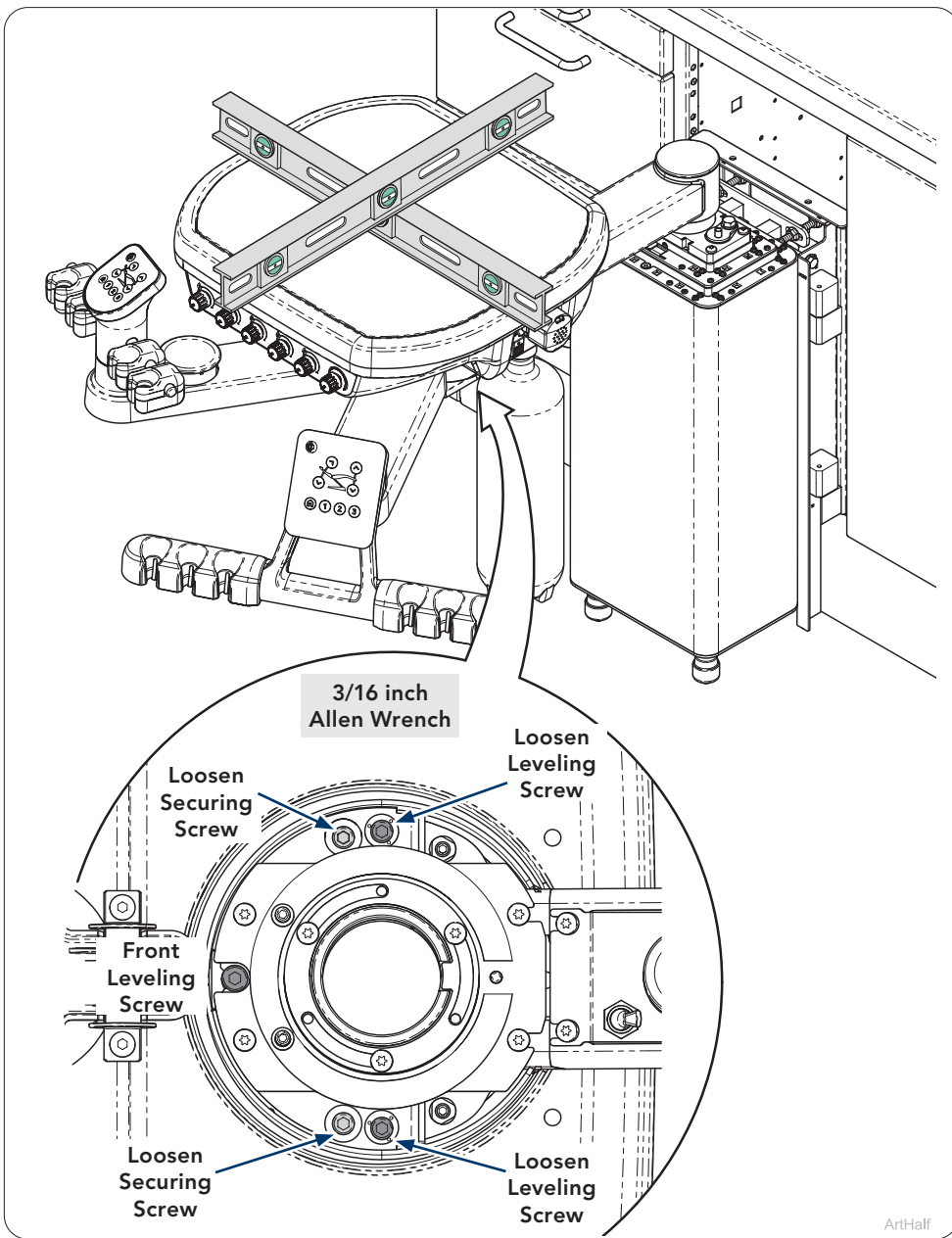- Step 6: Leveling column.**
- A) Loosen left and right securing nut.
 - B) Level delivery system column using left and right leveling bolts for side to side leveling.
 - C) Level column front to back with leveling nuts and leveling feet.
 - D) Retighten securing nuts to keep adjustments.


Step 7: Leveling Delivery System.

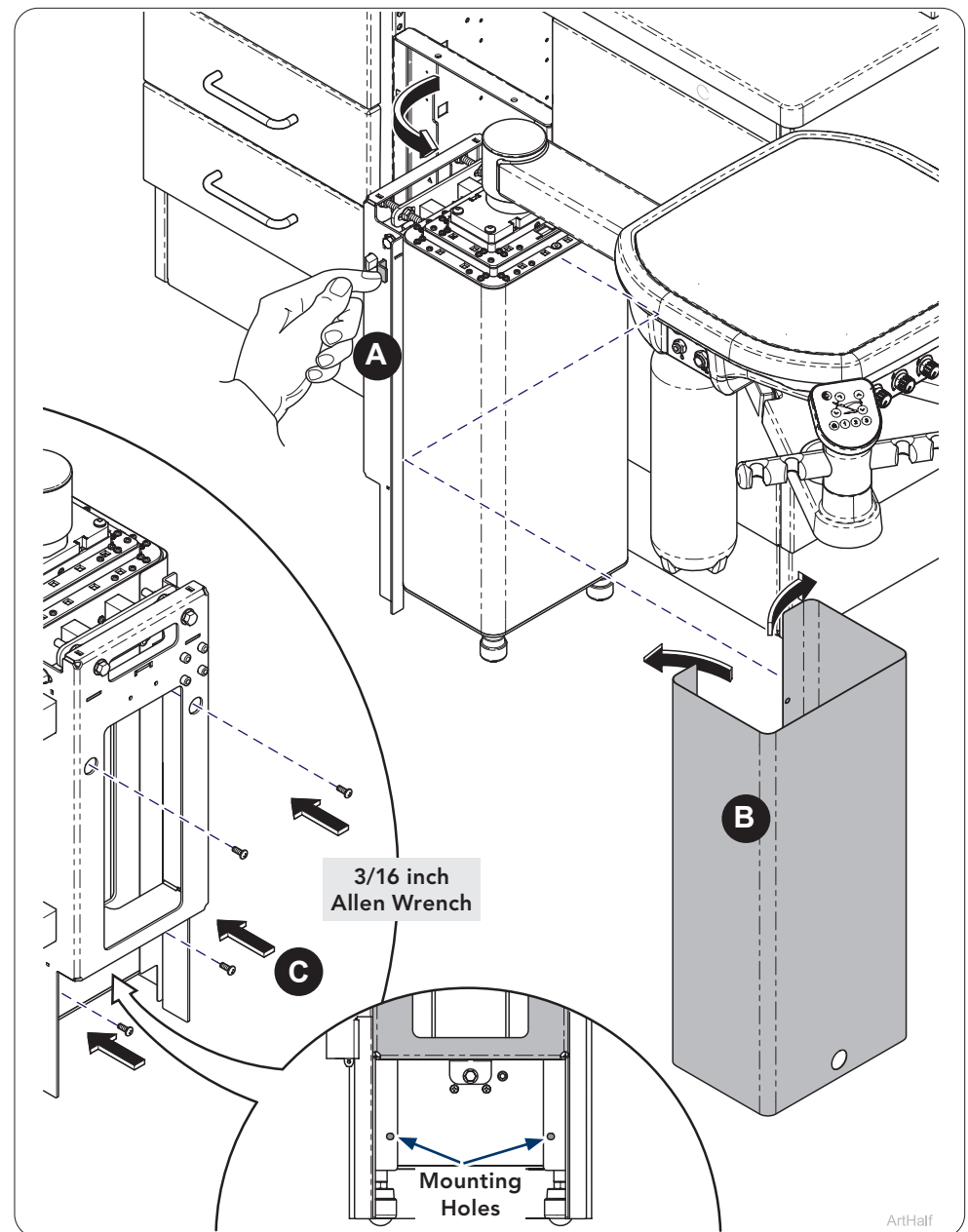- A) Loosen securing screws (2) and leveling screws prior to leveling.
- B) Level delivery system front to back with front screw (1).
- C) Level delivery system using left and right leveling screws for side to side leveling.
- D) Retighten securing screws to keep adjustments.

Step 8:

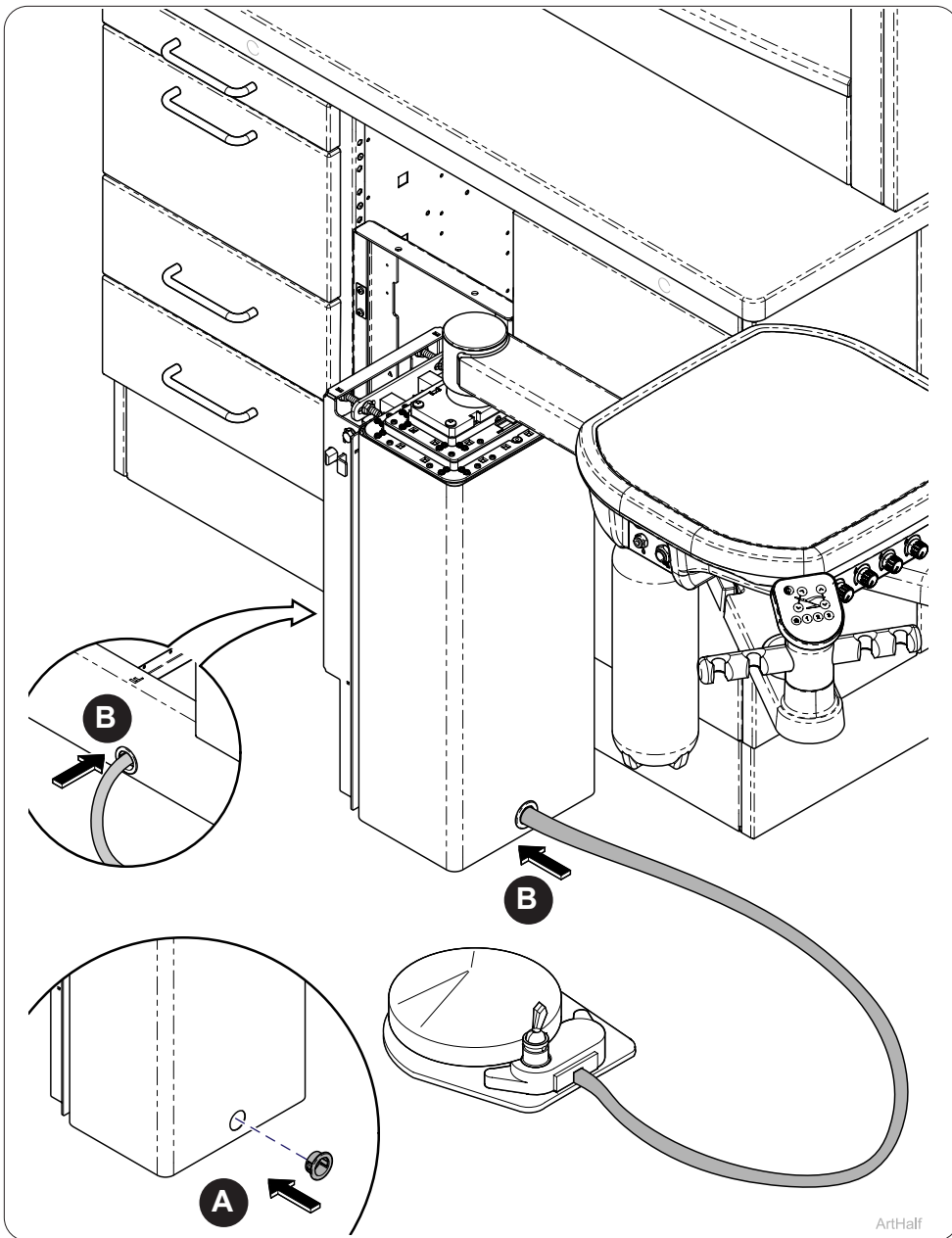
- A) Push button on side of mounting bracket and open column as shown.
- B) Bend sides of cover around column to install.
- C) Secure with screws (4).

Step 9:


- A) Install grommet into column cover.
- B) Route foot control through grommet into both column and cabinet.

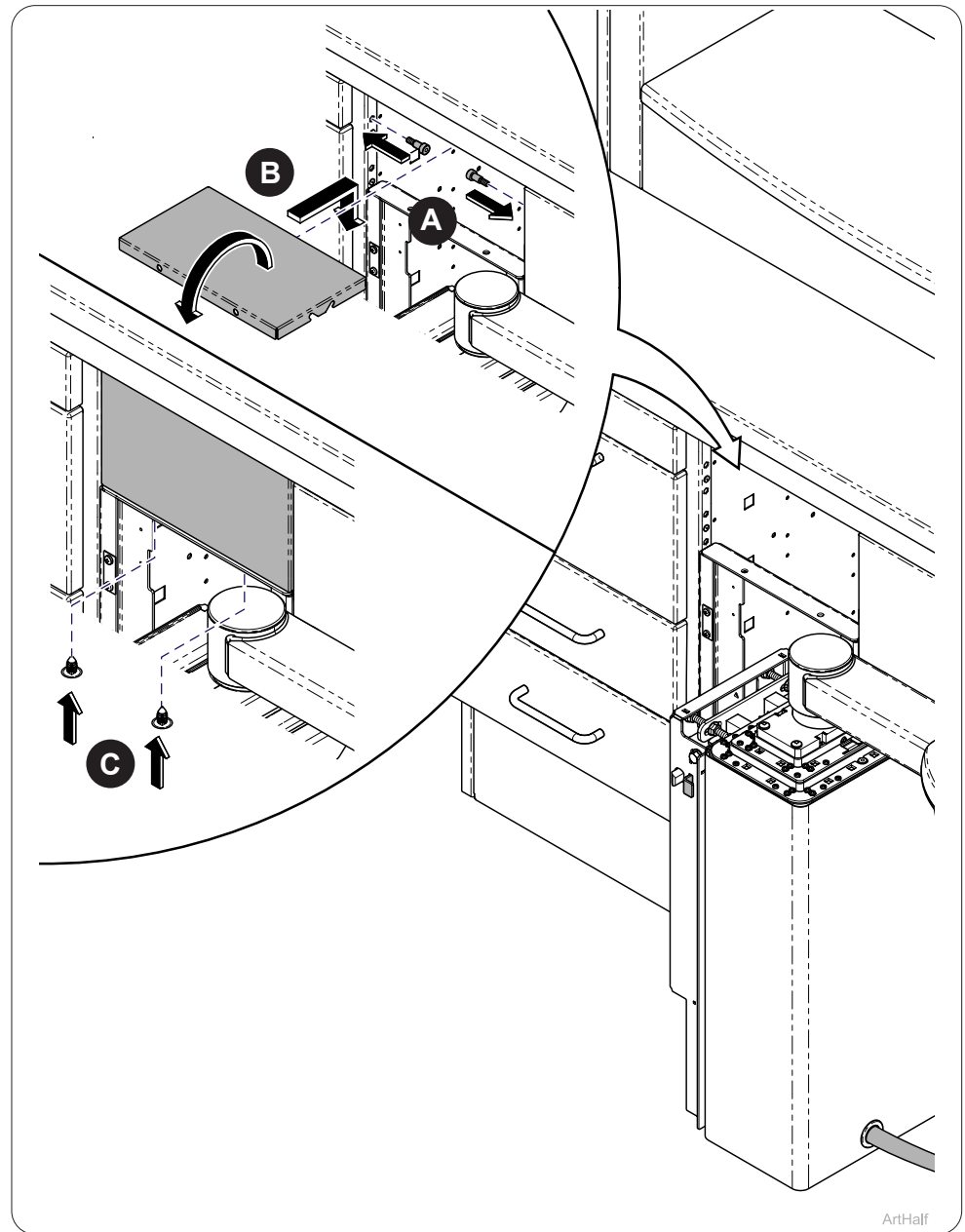Note: See page 9 labeled Tubing Connections (Cabinet).
 Long Rheostat Kit can be ordered, go to Midmark's Technical Library.

Step 10: (Optional, Not used when installing Nitrous Unit)

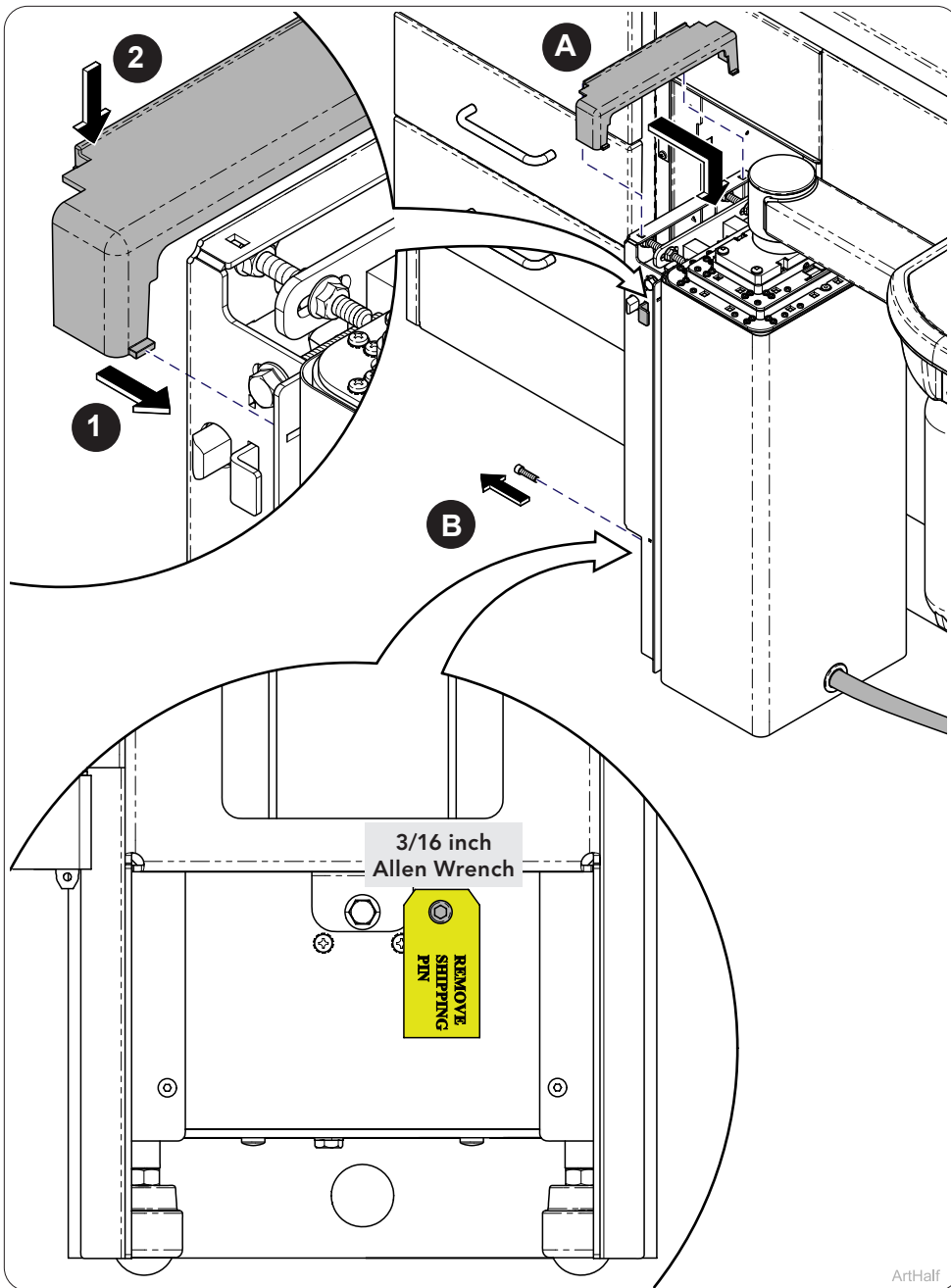
- A) Install mounting shoulder screws (2) hand tight into side of cabinet.
- B) Install top filler panel onto shoulder screws and rotate down.
- C) Secure bottom of filler with plastic fasteners (2).

Step 11:

- A) Install leveling bolt cover into back of column by installing tabs in front of cover and then push down on back tabs.
- B) Remove shipping pin and label from back of column.

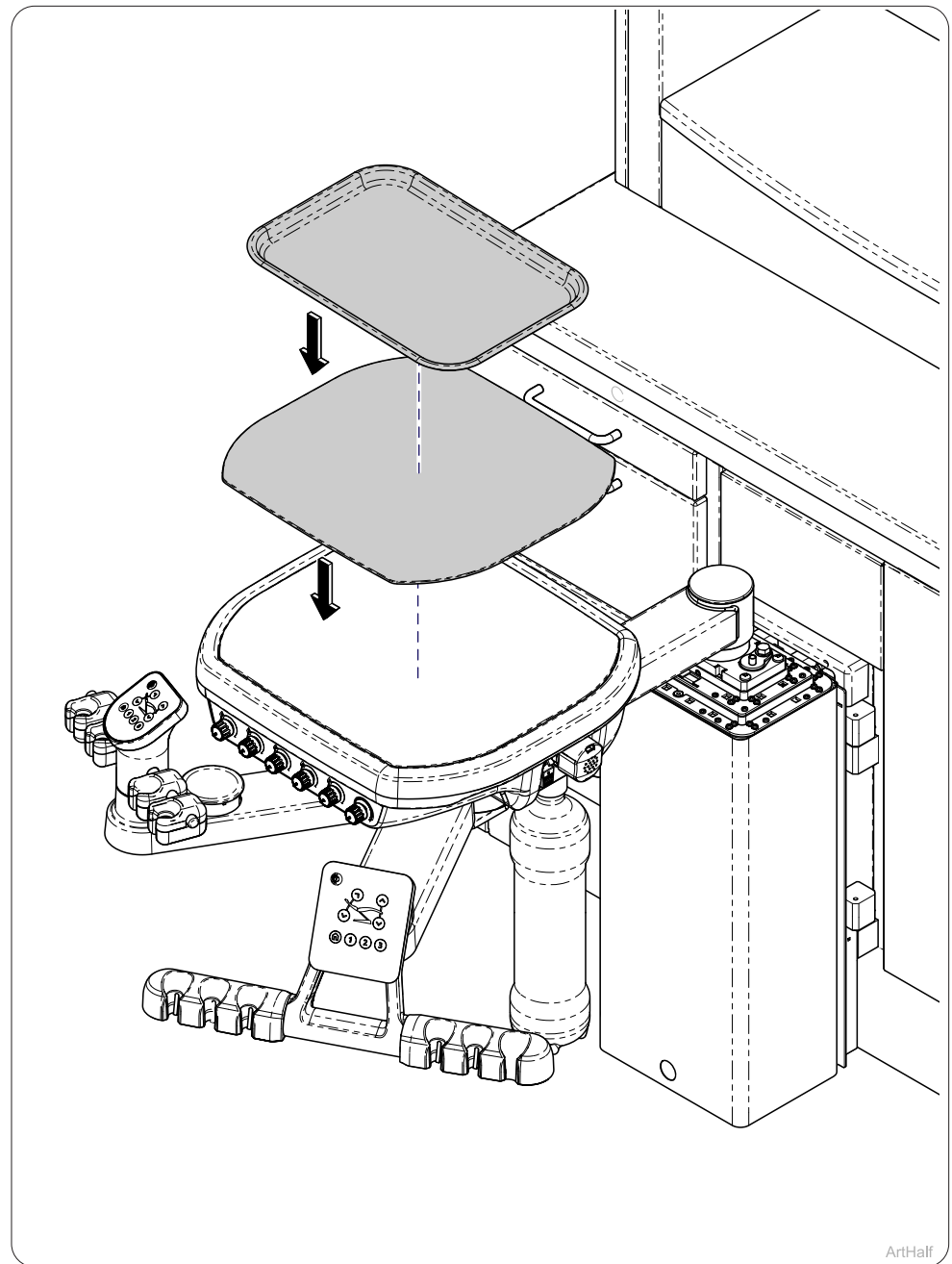
ArtHalf

Step 12:


- A) Close Column making sure all tubing is out of the way and does not get pinched.
- B) Place pad and tray onto delivery system.

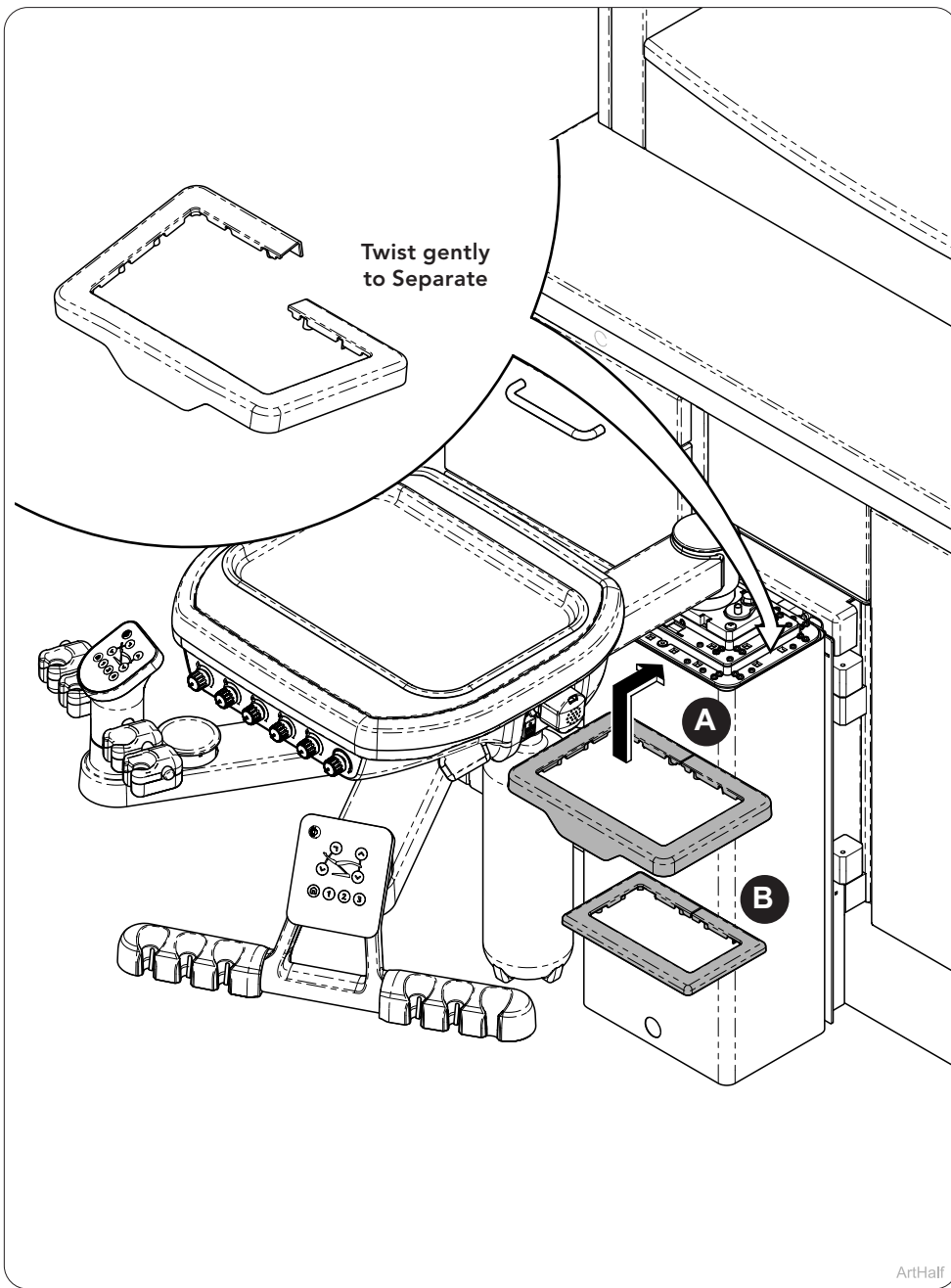ArtHalf

Step 13:

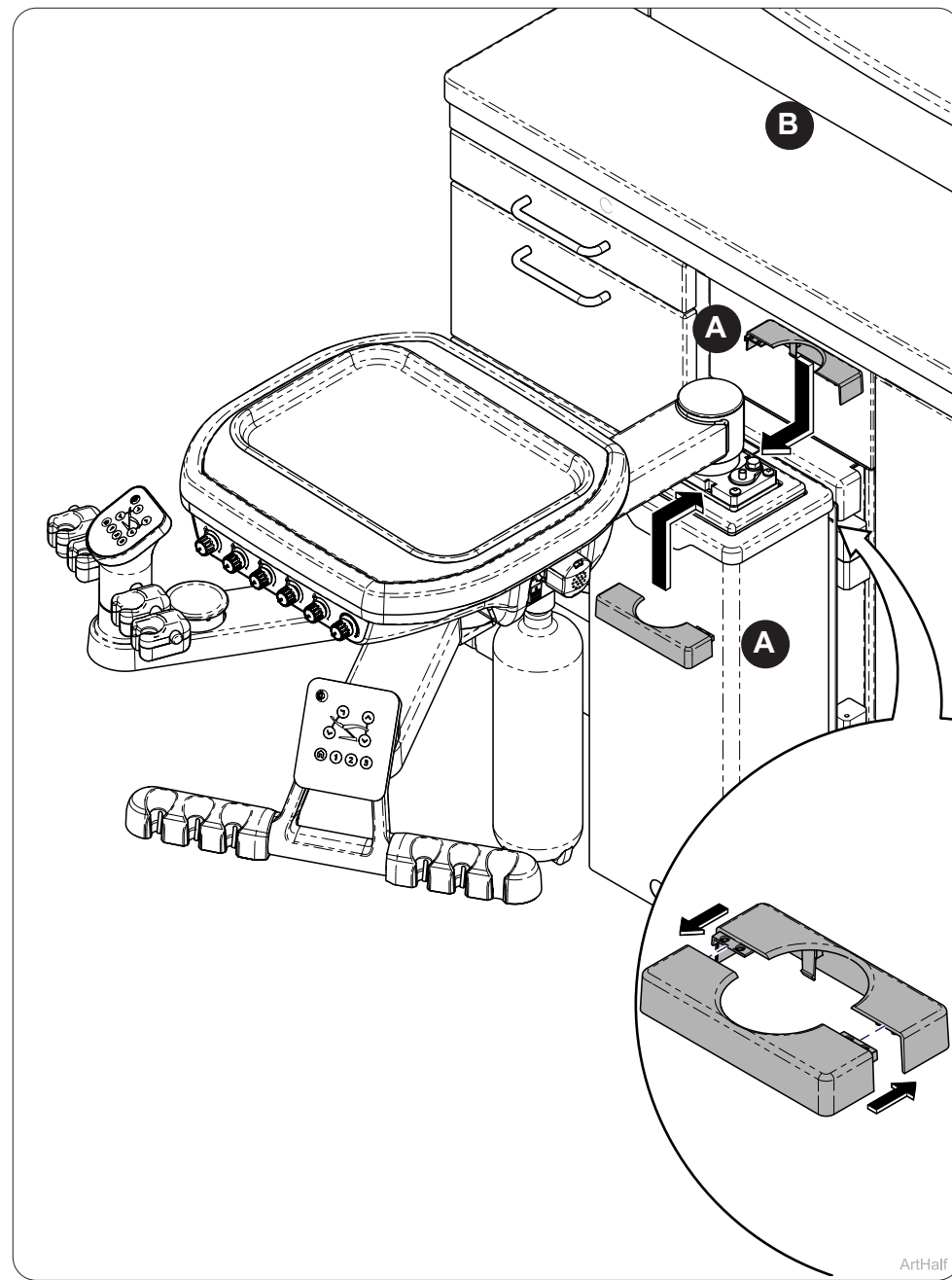- A) Install bottom larger plastic column cover as shown.
- B) Install next one piece medium plastic column cover.



Step 14:

- A) Install split cover onto top column snapping together.

Regulator Installation


Regulator installed to shutoff valve.
 Insert regulator into compression fitting of shut-off valve and tighten compression fitting.

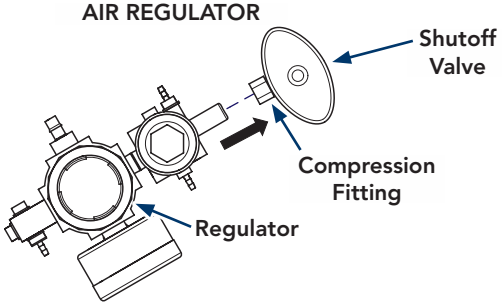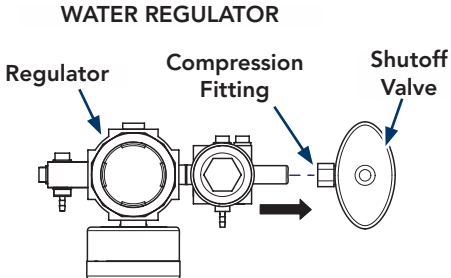Note: Delivery and Assistant's Unit Tubing Connections are located in cabinet base.

Tubing Connections (Cabinet)

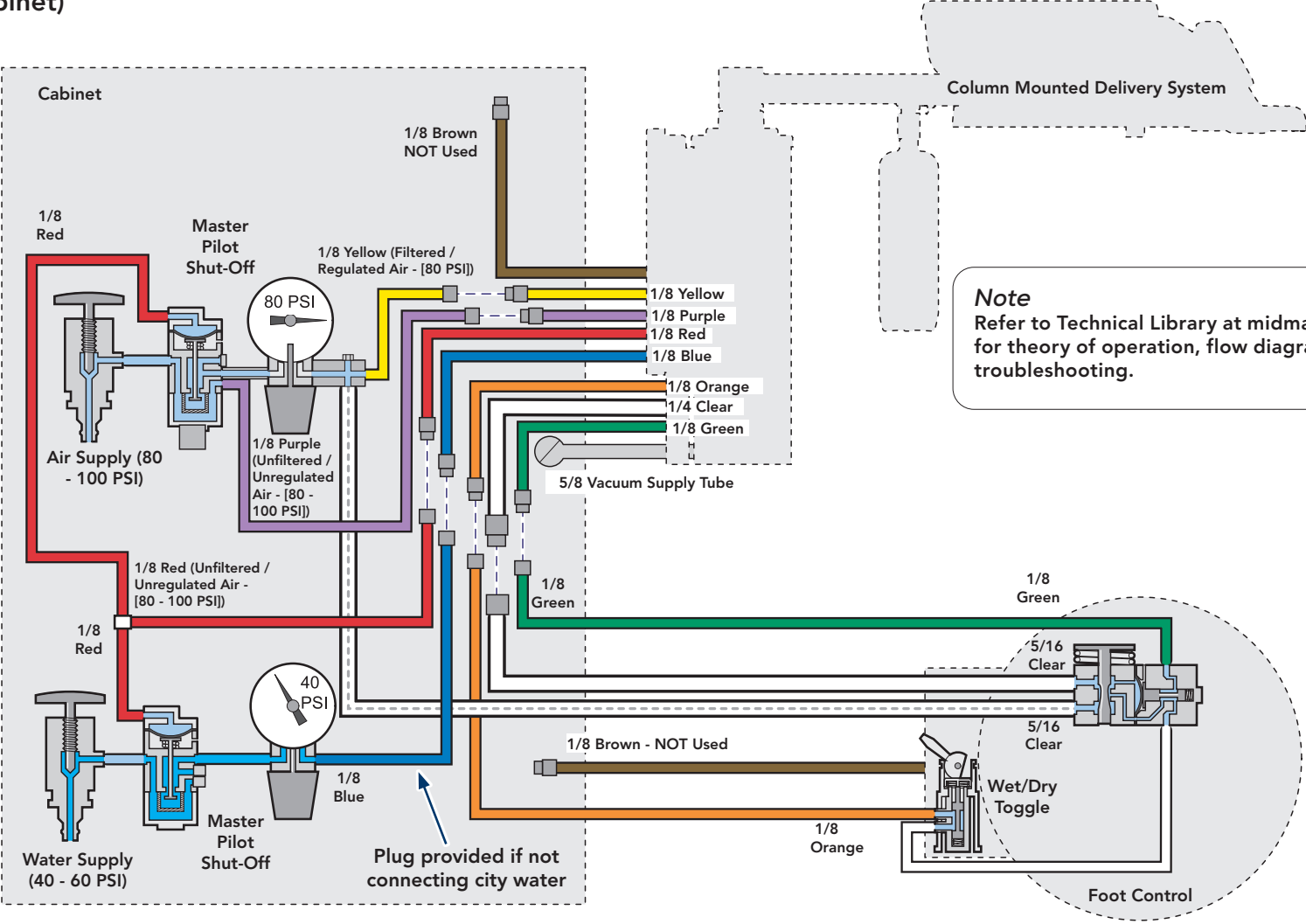
Step 15:



Note
 Refer to Technical Library at midmark.com for theory of operation, flow diagrams and troubleshooting.


Electrical Connections

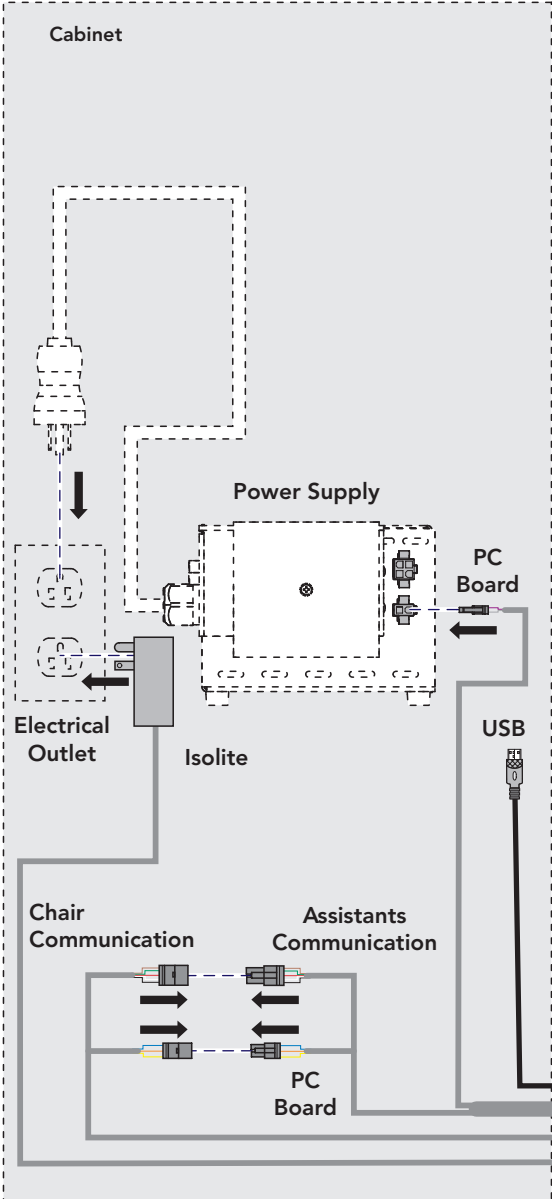Step 16:

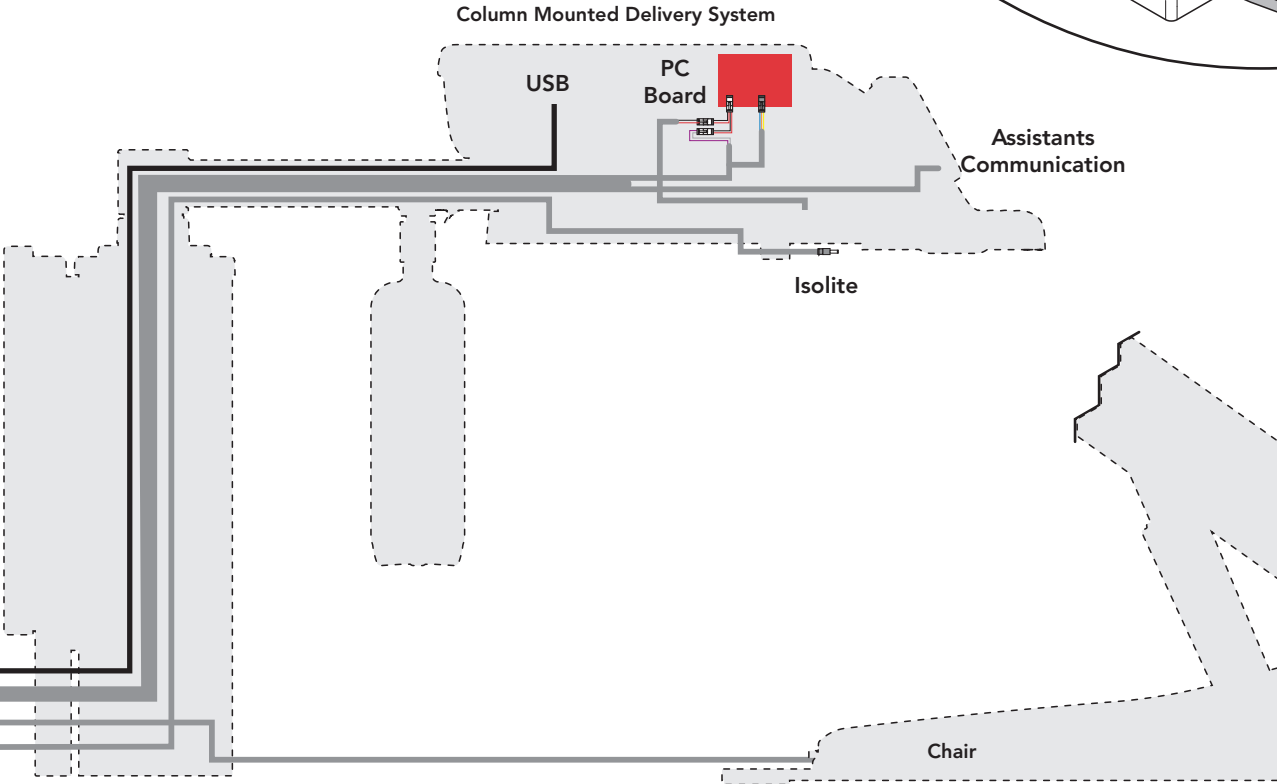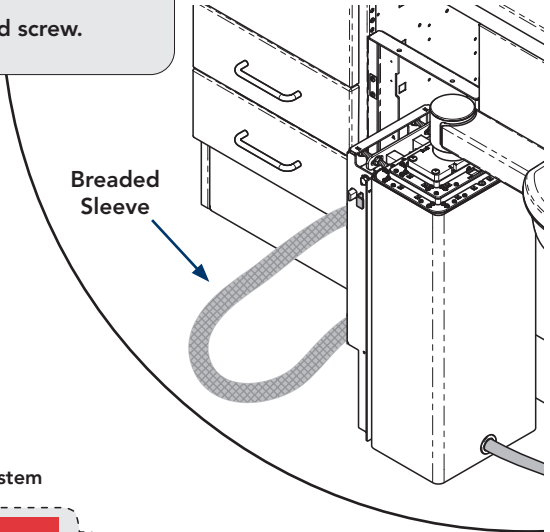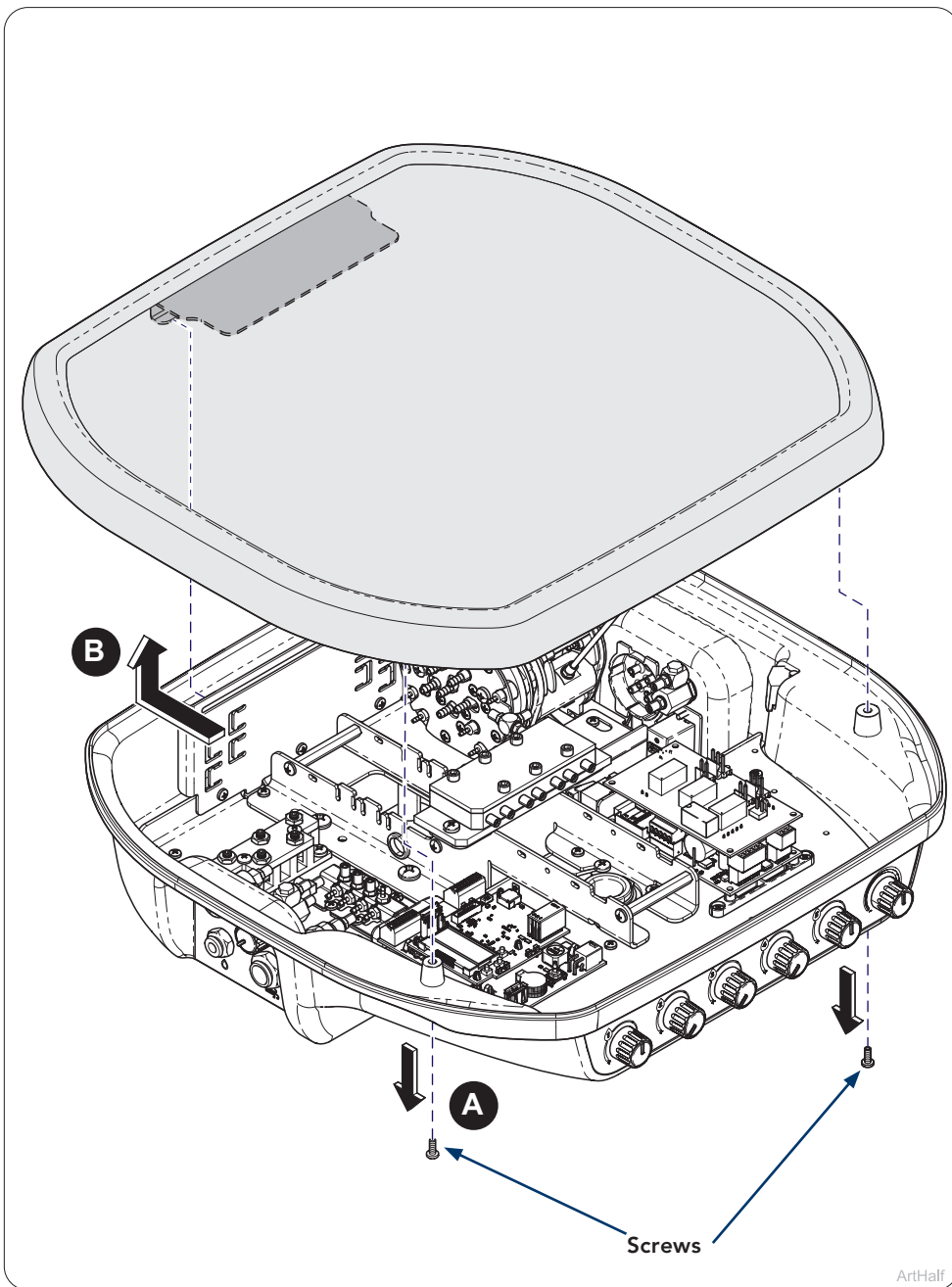
Step 17: Covering Tubing and Wiring.
 A) Use the hook and loop braided sleeve to wrap and secure the tubing and wiring within the cabinet.
 B) Secure braided sleeve to side of cabinet with cable tie and screw.

Step 17:

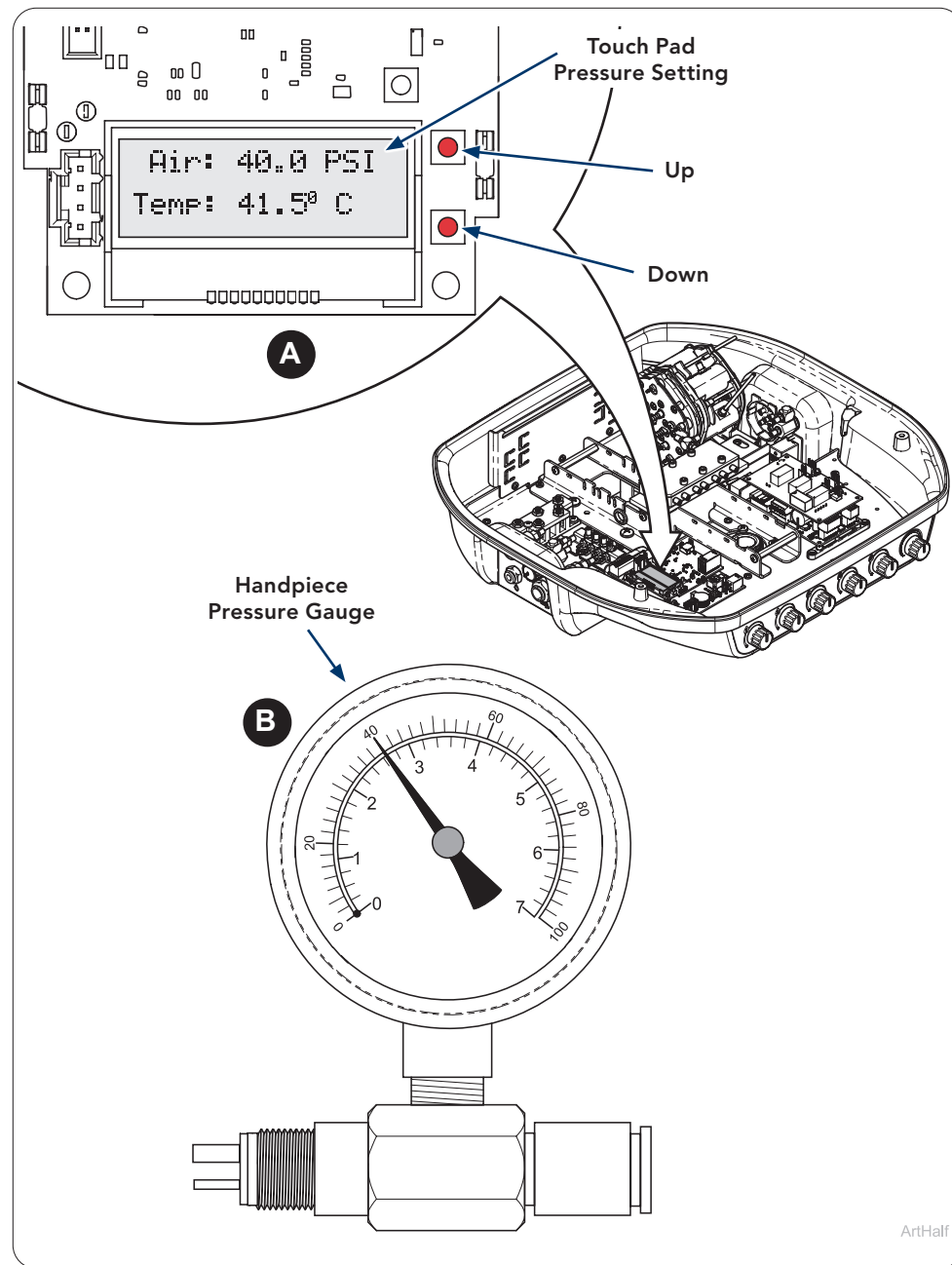- A) Remove screws (2) from the bottom front of the delivery system.
- B) Slide top cover back and lift up and off.

Step 18:

- Option A) (Touchpad) Use the up or down buttons on the PC board to navigate to the pressure screen until the pressure display appears.
- Option B) (Handpiece Pressure Gauge) Insert gauge between tubing and handpiece.

Equipment Alert

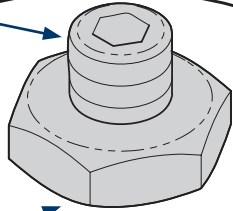
Do not restrict the drive air for electric handpieces, as it is essential for their cooling. Restricting the airflow will result in the handpiece overheating.

Equipment Alert

Always follow the handpiece manufacturer's recommended drive air settings. Failure to do so may result in damage to the handpiece.

Adjustment Set Screw

Lock Nut

Allen Wrench

D

Position #1

#3

#4

#5

#2

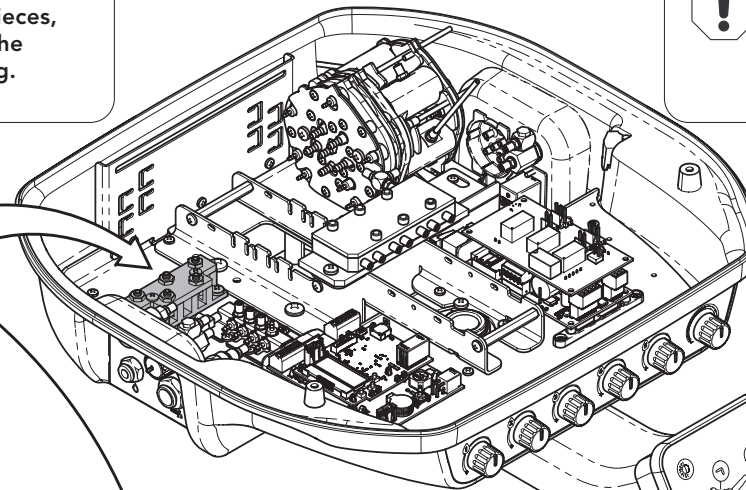
#3

#4

A

Wrench

F

Position #1

B

#2

#3

#4

#5

Foot Control

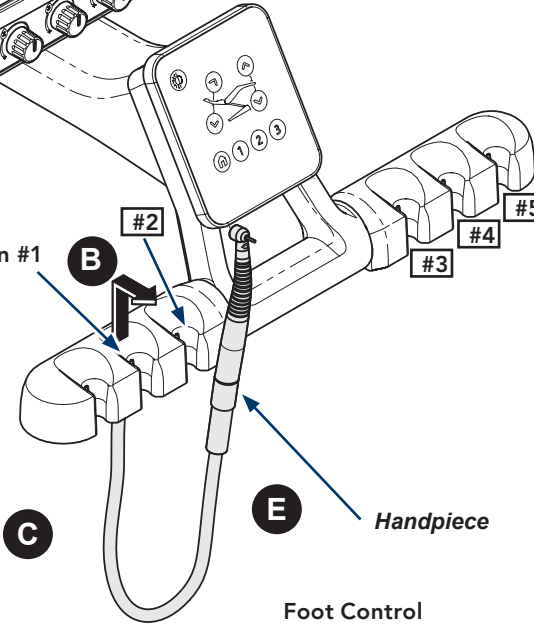
E

C

E

Handpiece


Foot Control


ArtFull

Step 19:

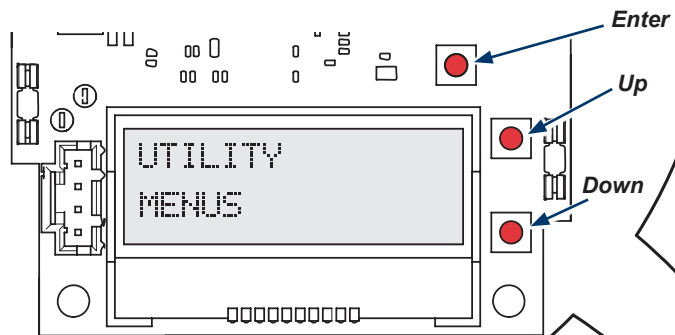
- A) Loosen the lock nut for the corresponding air-driven handpiece.
- B) Remove the selected air-driven handpiece from its holder.
- C) Press the foot control to run the handpiece.
- D) Adjust the air pressure by turning the adjustment set screw with an Allen wrench while monitoring the gauge or display.
- E) When the pressure reaches the handpiece manufacturer's recommended pressure, release the foot control and place handpiece into holder.
- F) Secure the setting by tightening the lock nut.
- G) Repeat for each handpiece steps A thru F.


A

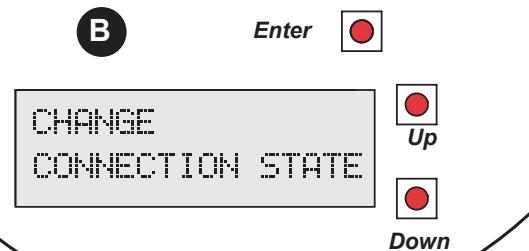B

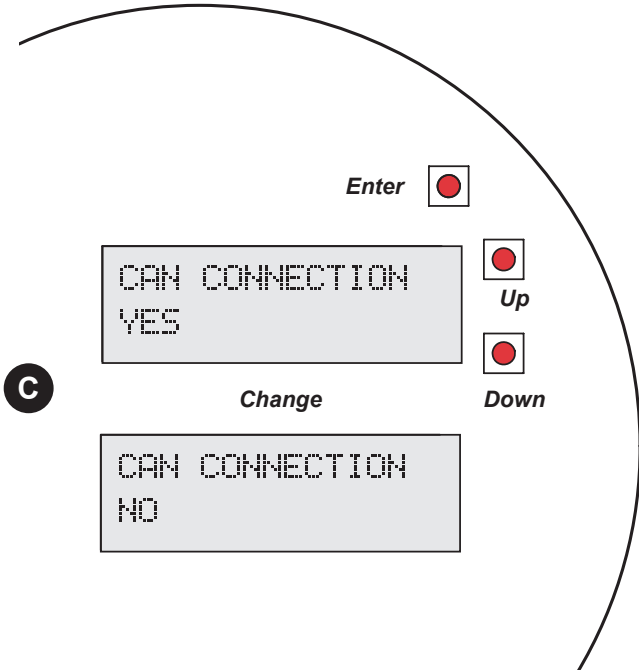
C


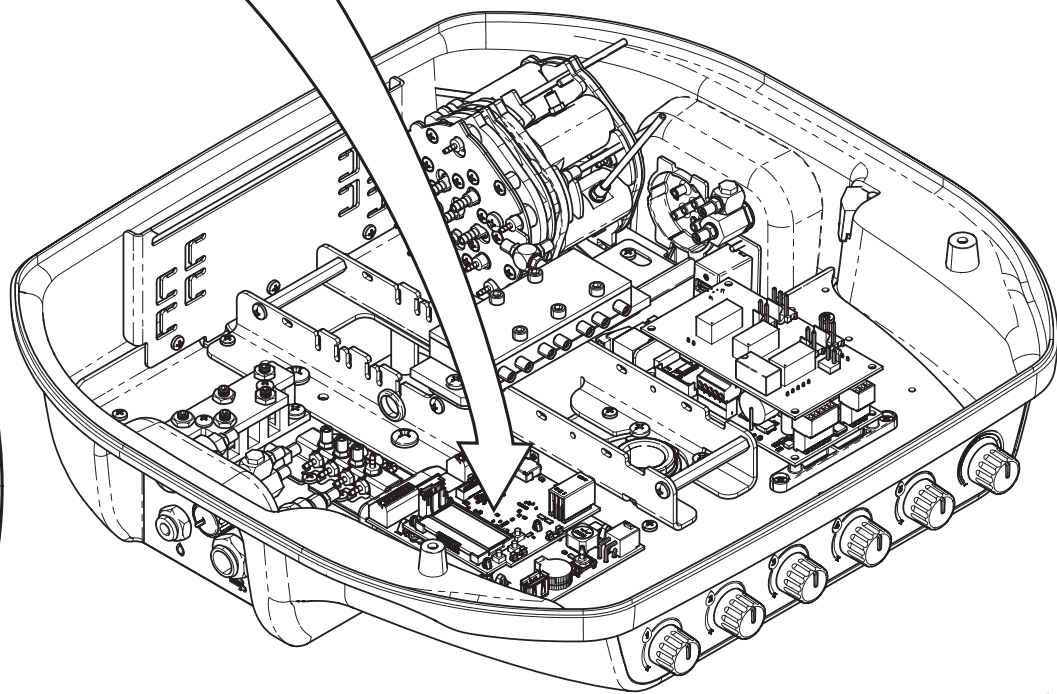
ArtFull

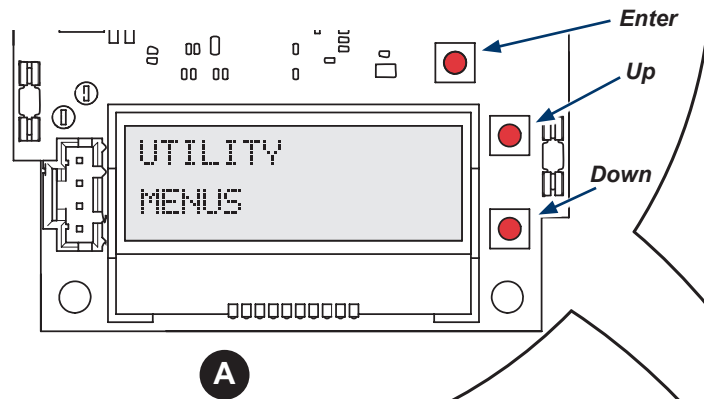Step 20: Change Connection State - If not connected to chair.
 A) Press up or down buttons on the PC board to navigate to Utility Menus, then press enter button.
 B) Press up or down buttons on the PC board to navigate to Change Connection Sate, then press enter button.
 C) Press up or down buttons on the PC board to change from Yes to No, then press enter button.

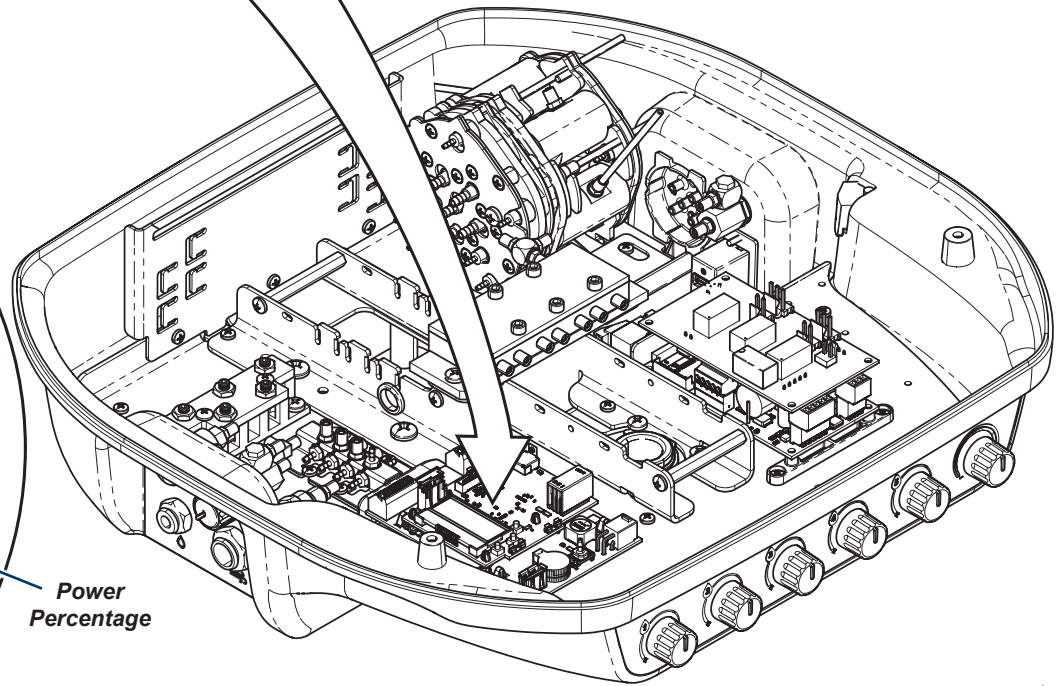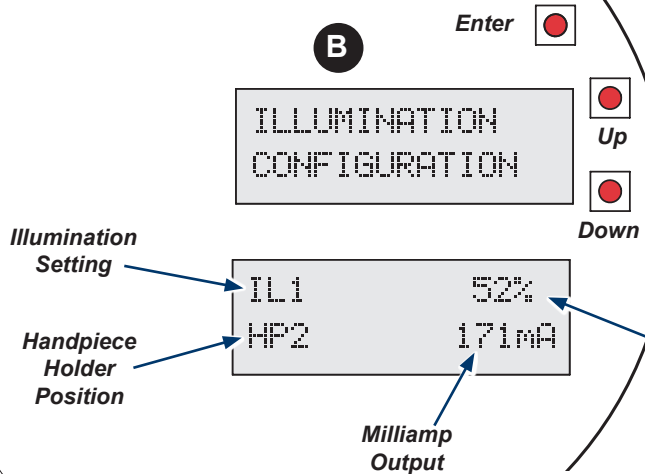Note
Starting from the left, IL1 corresponds to the first handpiece position configured with an illuminated handpiece. IL2 and IL3 follow in sequence, depending on the number of illuminated handpieces configured.

ArtFull

Step 21: Air Turbine Illuminating Handpiece Adjustment Setup.
 A) Press up or down buttons on the PC board to navigate to Utility Menus, then press enter.
 B) Press up or down buttons on the PC board to navigate to Illumination Configuration, then press enter.

Note: The display will cycle through all three available illumination settings (IL1, IL2, and IL3). It also shows the power percentage (0-100), the configured handpiece holder position, and the milliamp output (0-330) to the light.



Equipment Alert

Setting the illumination level higher than recommended by the handpiece manufacturer may cause the light to burn out prematurely.

PC Board Display

IL1	52%
HP2	171mA

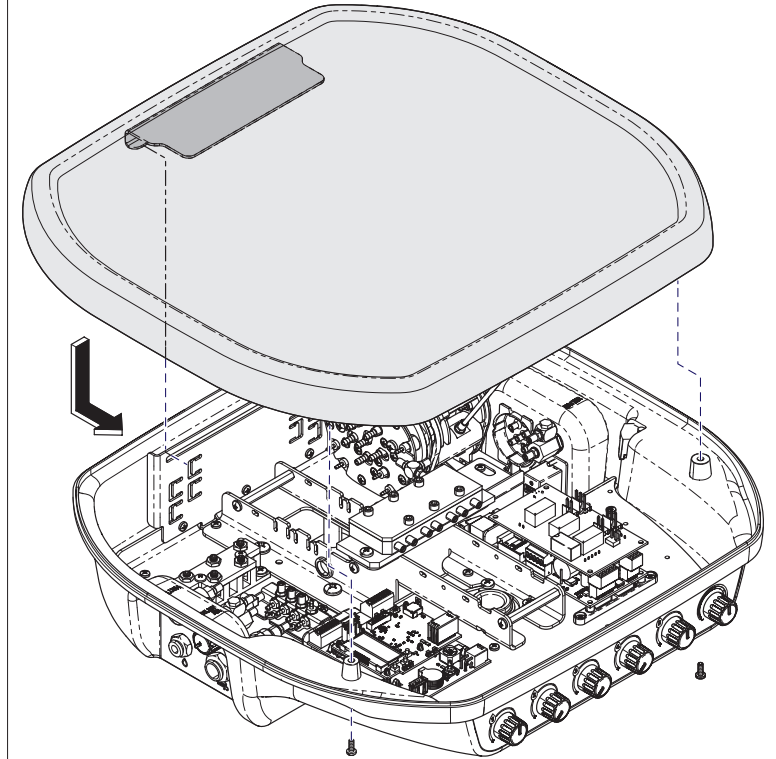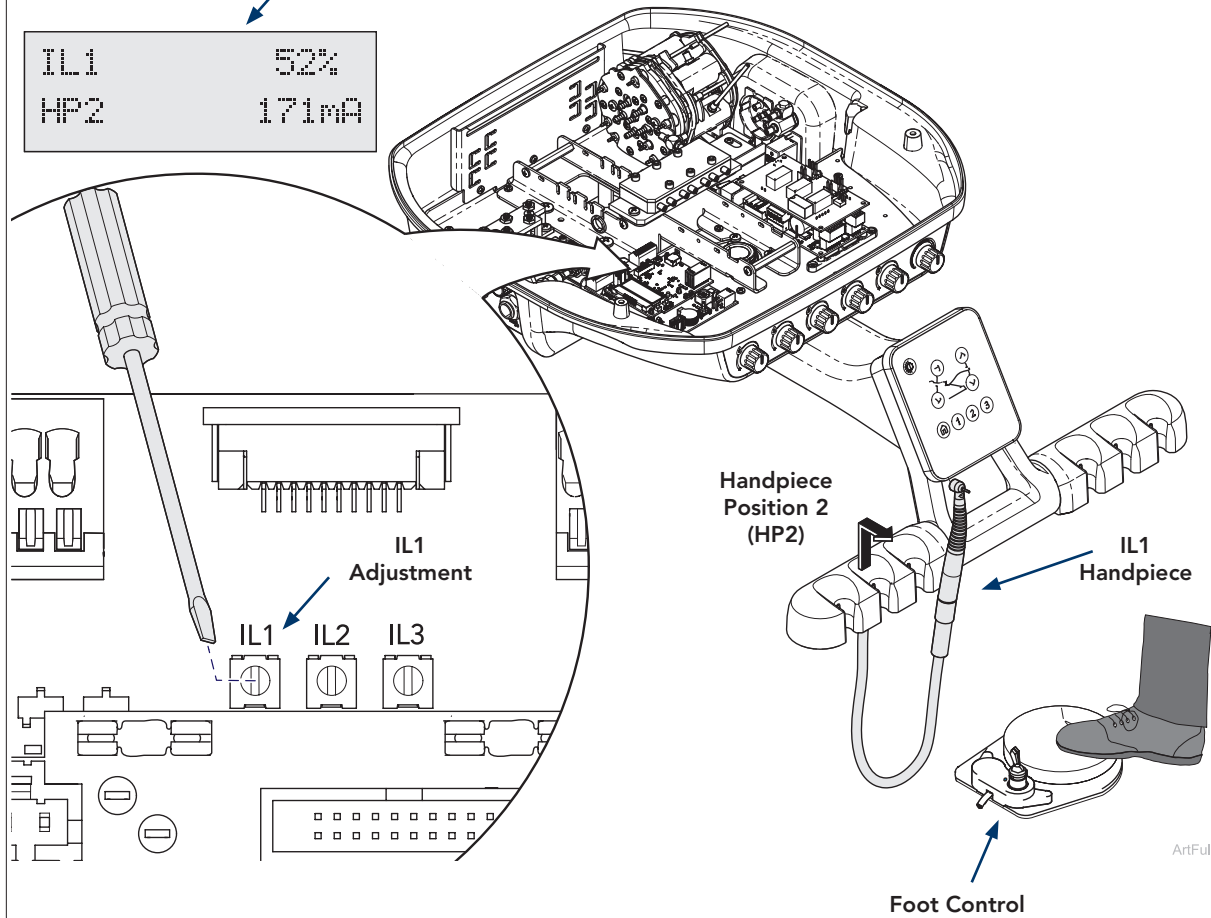ArtHalf

ArtFull

Step 22: Air Turbine Illuminating Handpiece Adjustment Setup.

- Using a small screwdriver, adjust IL1, IL2 or IL3 to the desired setting while observing the PC board display. Note that IL1, IL2 and IL3 each rotate up to a maximum of $\frac{3}{4}$ turn.
- Remove the handpiece from its holder and activate the foot control to verify the illumination level. Repeat the adjustment as needed until the desired setting is achieved.
- When Illumination settings are at the desired setting for all applicable handpieces.

Step 23:

- Place top cover onto delivery system and slide forward.
- Install screws (2) into the bottom front of the delivery system.
- Remove the number sticker after handpieces are positioned.

Tension Adjustments

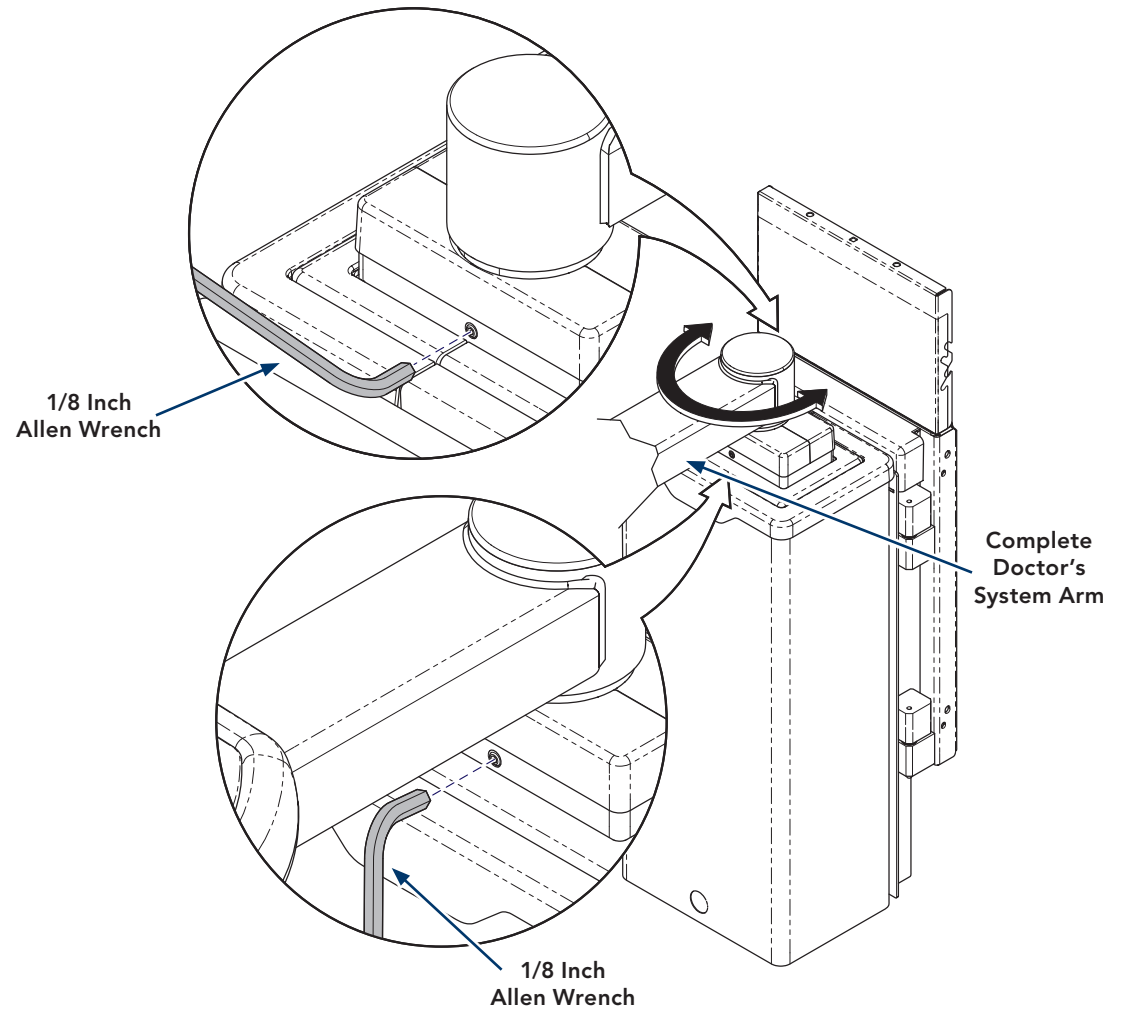
Complete Doctor's System Arm Rotation Tension Adjustment

(Doctor's 12 O'clock Column-Mounted)

The rotation tension on the complete doctor's system arm of the 12 O'clock column-mounted delivery system can be adjusted to your desired level.

Adjusting complete doctor's system rotation tension at the column.

A) To adjust the tension, use a 1/8 inch allen wrench to turn the two set screws. Rotate them clockwise to increase the tension and counterclockwise to decrease it.

ArtHalf

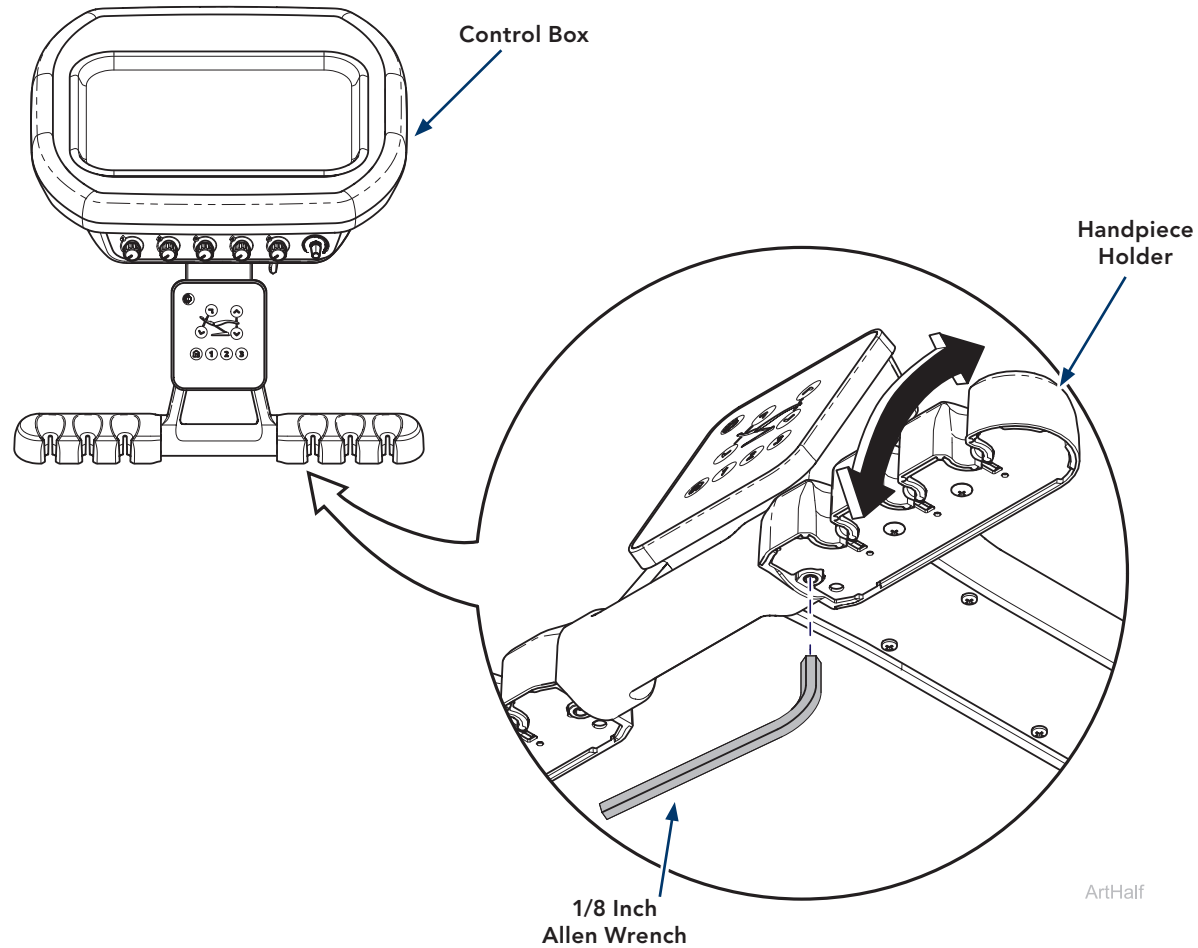
Tension Adjustments

Handpiece Holder Tension Adjustment (*Doctor's*)

The rotation tension on the handpiece holder of the doctor's delivery system can be adjusted to the desired level.

To adjust tension on the handpiece holder.

A) To adjust the tension, use a 1/8 inch Allen wrench to turn the set screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

Tension Adjustments

Handpiece Holder Arm and Handpiece Holder Pivot Tension

Adjustments (Doctor's 12 O'clock Cabinet-Mounted)

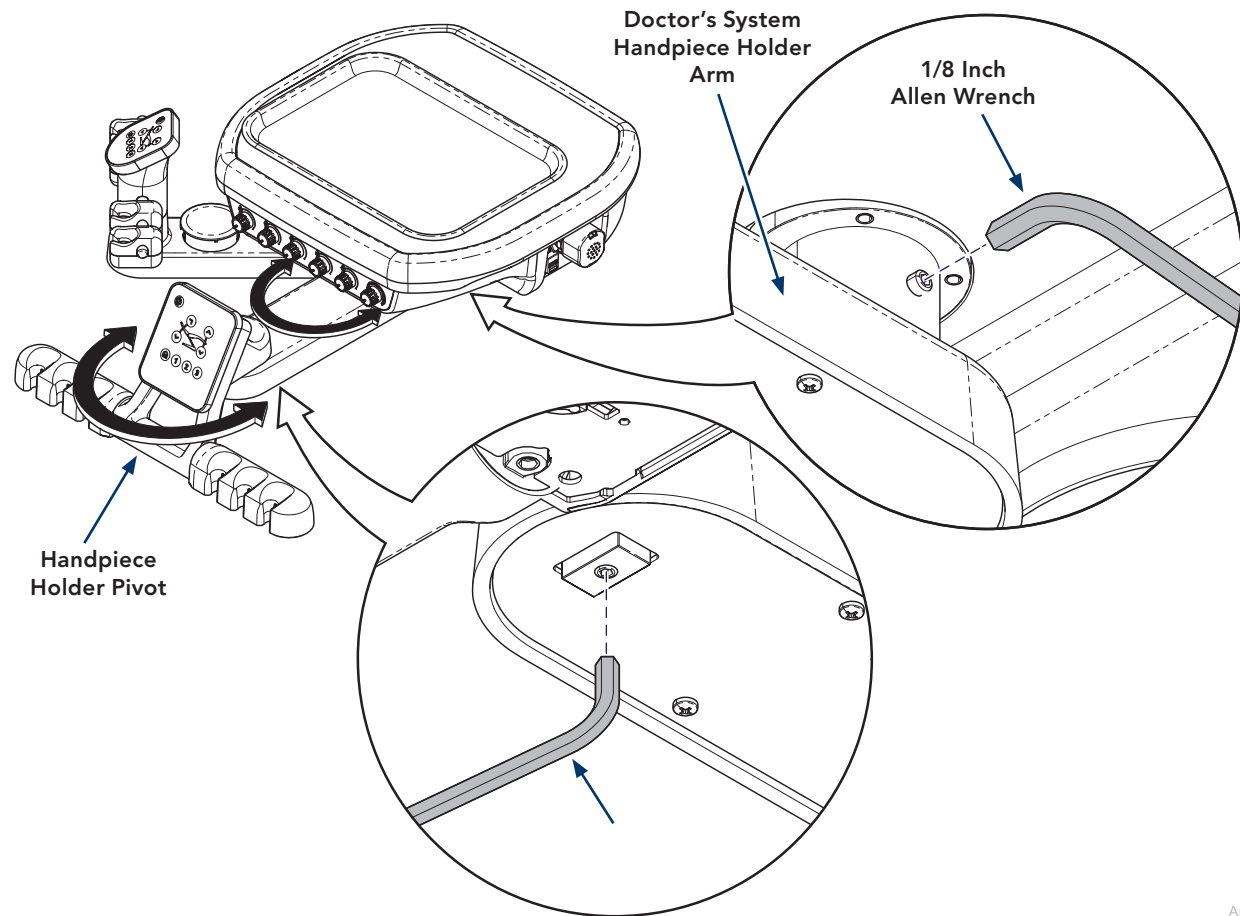
The rotation tension on the handpiece holder arm and handpiece holder pivot of the 12 O'clock cabinet-mounted delivery system can be adjusted to your desired level.

Adjusting tension on the doctor's system handpiece holder arm.

A) To adjust the tension, use a 1/8 inch allen wrench to turn the set screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

Adjusting tension on the handpiece holder pivot.

A) To adjust the tension, use a 1/8 inch allen wrench to turn the set screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

ArtFull

Tension Adjustments

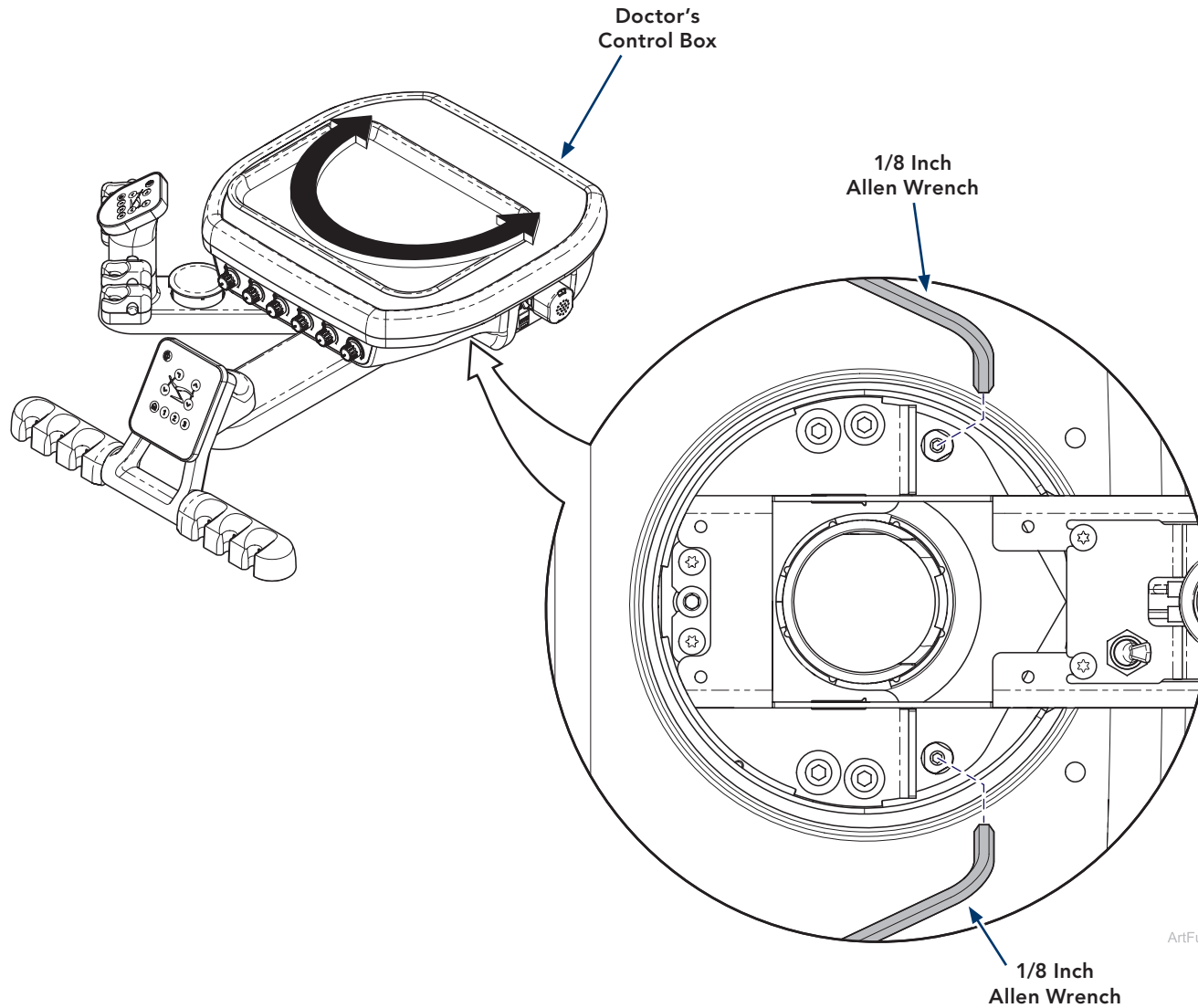
Control Box Tension Adjustment

(Doctor's 12 O'clock Cabinet-Mounted)

The rotation tension on the control box of the 12 O'clock cabinet-mounted delivery system can be adjusted to your desired level.

Adjusting tension on the doctor's control box rotation.

A) To adjust the tension, use a 1/8 inch allen wrench to turn the two set screws. Rotate them clockwise to increase the tension and counterclockwise to decrease it.

ArtFull

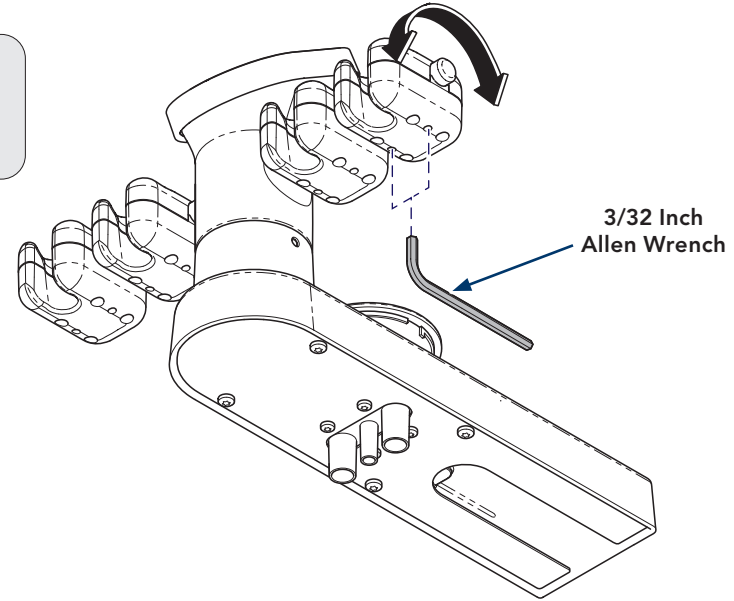
Tension Adjustments

Assistant's System Handpiece Holder Tension Adjustment

The rotation tension on the assistant's system handpiece holders can be adjusted individually to your desired level.

Adjusting tension on each handpiece holders rotation.

A) To adjust the tension, use a 3/32 inch allen wrench to turn the two set screws. Rotate them clockwise to increase the tension and counterclockwise to decrease it.


Assistant's System Head Rotation Tension Adjustment

The rotation tension on the assistant's system head rotation can be adjusted to your desired level.

Adjusting tension on the assistant's head rotation.

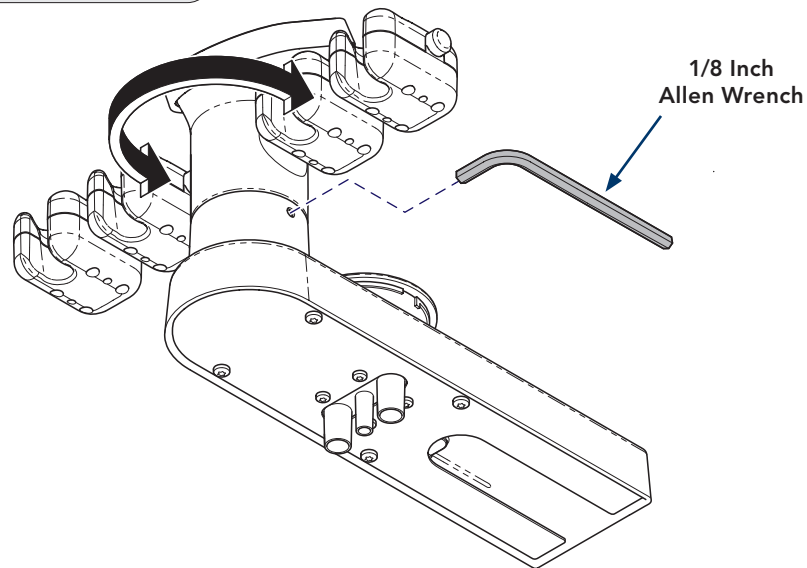A) To adjust the tension, use a 1/8 inch allen wrench to turn the set screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.


Assistant's System Arm Vertical Rotation Tension Adjustment

The vertical rotation tension on the assistant's system arm can be adjusted to your desired level.

Adjusting Assistant's System Arm Vertical Rotation Tension.

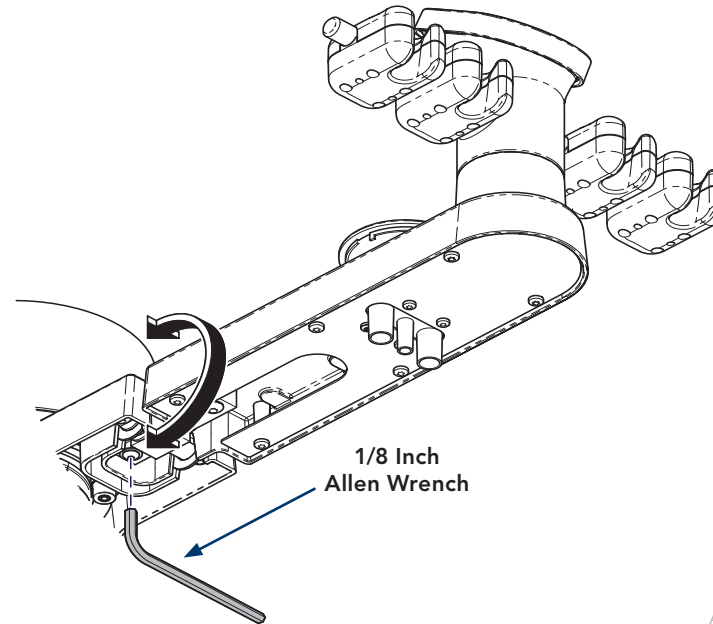- A) To adjust the tension, use a 1/8 inch Allen wrench to turn the button head screw. Rotate it clockwise to increase the tension and counterclockwise to decrease it.

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