

Accessory Shelf for Carts Installation (029-12025-00)

Applies to Models:
153740-012 & 153740-013

Special Tools:
Allen Wrenches (5/64",
1/8" and 5/32")
Level

Language of origin: English




Equipment Alert

Maximum shelf load limitation is 10 lbs (4.5 kg).

- Remove the socket head screws and separate the two halves of the tray clamp.
- Place the tray and clamp assembly on the delivery cart and install the socket head screws.
- Check operation.
If necessary, the amount of drag at the two pivot points on the tray arm can be adjusted by tightening or loosening the three socket head screws at each location.

Note

Check the accessory shelf to assure it is level.

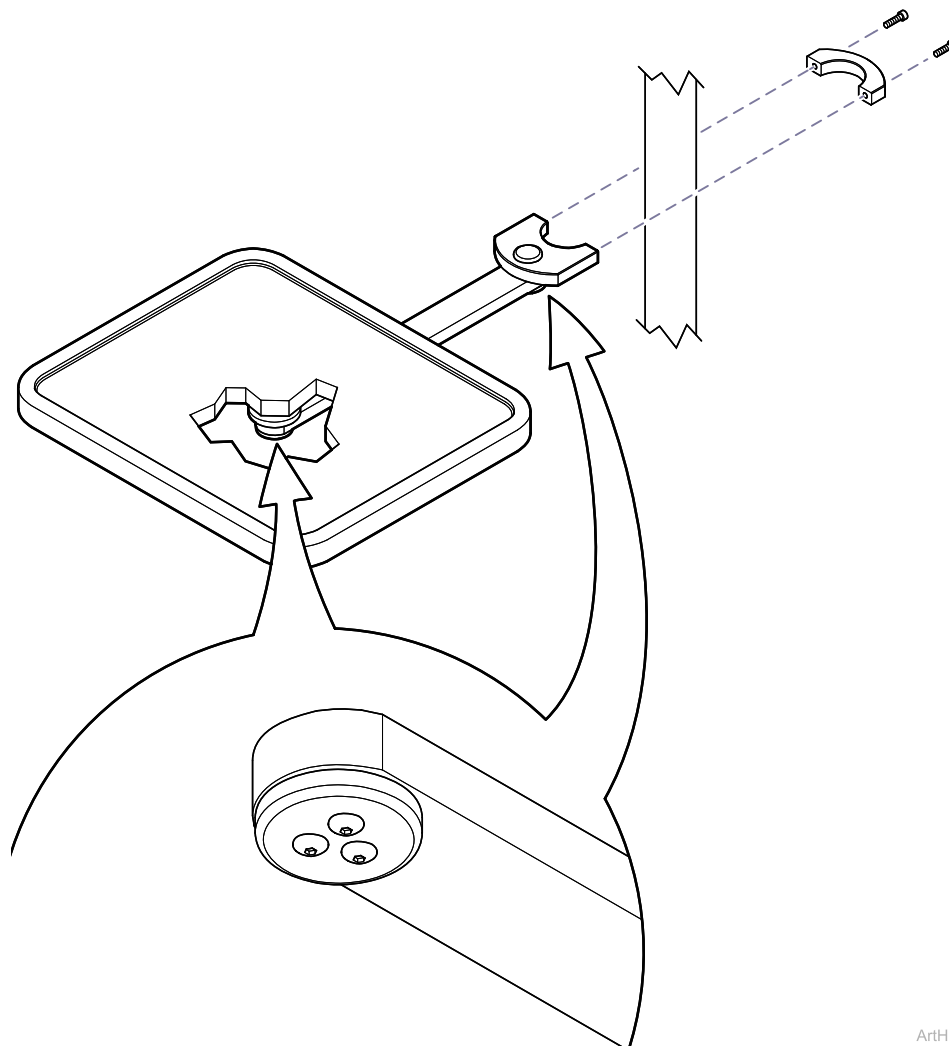

ArtHalf