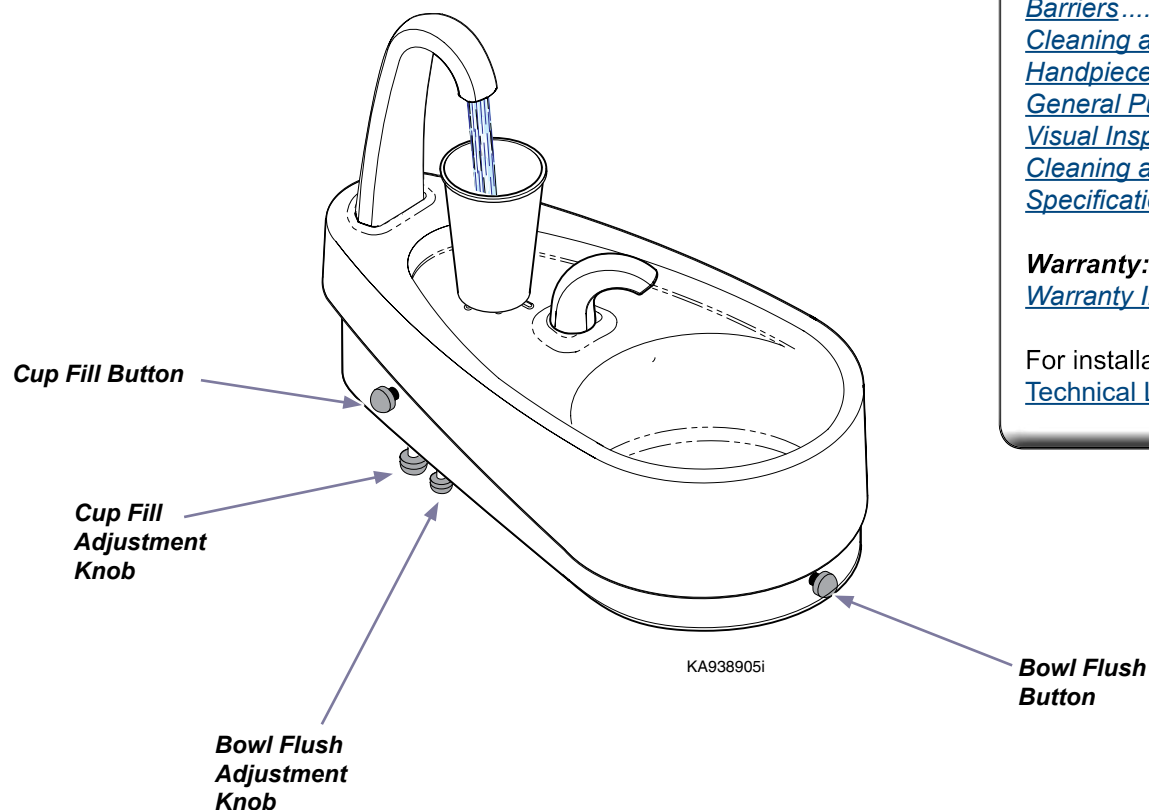
Procenter Cuspidor

[English](#)

[Español](#)

[Français](#)

Control Buttons and Knobs

Compliance Information and Symbols:

Intended Use	2
Electromagnetic Interference	2
Transportation / Storage /	
Operating Conditions	2
Disposal of Equipment	2
Calling for Service	2
Ordering Parts	2
Symbols	3

Operation and Adjustments:

Bowl Rinse Operation	4
Cup Fill Operation	4

Cleaning and Manitenance:

Barriers	5
Cleaning and Disinfecting	5
Handpiece Accessories	5
General Purpose Cleaning	5
Visual Inspection	5
Cleaning and Disinfecting Assistance ..	5
Specifications	6

Warranty:

Warranty Information	7
--	---

For installation information go to:
[Technical Library at Midmark.com](#)

Compliance Information and Symbols

Intended Use

To provide water for oral rinsing, and a drain to dispose of waste water and oral debris during dental examinations.

WARNING

Do not pour water into the Cuspidor from external sources. It is designed to safely eliminate only the amount of water supplied from the Cuspidor fill and flush operations.

Electromagnetic Interference

Midmark dental operatory components are designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this operatory, remove the interfering device from room and / or plug product into an isolated circuit.

Transportation / Storage / Operating Conditions

Transportation / Storage Temperature:.....23°F to 100°F (-5°C to 38°C)

Relative Humidity:..... 10% to 90% (non-condensing)

Atmospheric Pressure: 7.2 PSI to 15.3 PSI (50 kPa to 106 kPa)

Operating Temperature Range:59°F to 95°F (15°C to 35°C)

Disposal of Equipment

At the end of this product's life, the unit, accessories and other consumable goods may be contaminated from normal use. Consult local codes and ordinances for proper disposal of this equipment and other consumable goods.

Calling For Service

Direct all service inquiries to your authorized Midmark dealer. When calling for service, you must provide the following information:

Model / serial number
Date of purchase
Symptom(s) of malfunction

Ordering Parts

The following information is required when ordering parts:

Serial number & model number
Part number for desired part
(Refer to Exploded Views and Parts at midmark.com)

Non-warranty parts orders may be faxed to Midmark using the Fax Order Form in the back of this manual.

For warranty parts orders, call Midmark's Technical Service Department with the required information.

Hours: 8:00 am to 5:00 p.m. EST (Monday thru Friday)
Phone: 1-800-Midmark

Symbols

These symbols may appear on your equipment and/or in the manuals. Warnings and cautions are provided in the manuals where applicable.

Refer to instruction manual/booklet

Type B, Applied Part

Protective Earth Ground

Proper Shipping Orientation

Fragile

Catalogue Number

Serial Number

Manufacturer

Pressure Limit

Temperature Limit

Humidity Limit

Maximum stacking height (Refer to “n” number on package.)

Keep Dry

WARNING

Indicates a potentially hazardous situation which could result in serious injury if not avoided.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury if not avoided. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

NOTE

Amplifies a procedure, practice, or condition.

Operation and Adjustments

Bowl Rinse

To rinse the cuspidor bowl...

Press Bowl Rinse Button.

Note: Water will run for approximately 12 seconds, then shut off automatically.

To increase the flow time...

Screw Bowl Rinse adjustment counter-clockwise as necessary.

To decrease the flow time...

Screw Bowl Rinse adjustment clockwise as necessary.

**Bowl Rinse
Flow Knob**

To increase the flow volume...

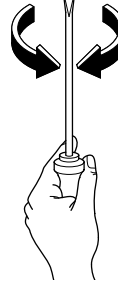
Rotate Bowl Rinse Flow Knob counter-clockwise as necessary.

To decrease the flow...

Rotate Bowl Rinse Flow Knob clockwise as necessary.

**Bowl Rinse
Flow Time
Adjustment**

**Cup Fill
Flow Time
Adjustment**

KA972001i

KA938904i

**Bowl Rinse
Button**

Cup Fill

To fill a cup with water...

Press Cup Fill Button.

Note: Water will run for approximately 6 seconds, then shut off automatically.

To increase the flow time...

Screw Cup Fill adjustment screw counter-clockwise as necessary.

To decrease the flow time...

Screw Cup Fill adjustment screw clockwise as necessary.

**Cup Fill
Button**

**Cup Fill
Flow Knob**

If cup is not full enough ...

Rotate Cup Fill Flow Knob clockwise as necessary.

If cup overflows...

Rotate Cup Fill Flow Knob counter-clockwise as necessary.

KA938903i

Cleaning, Disinfecting and Maintenance

Barriers

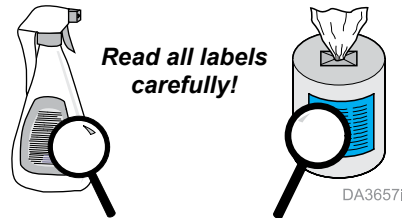
Midmark recommends the use of disposable barriers on all clinician controls that may be in contact with clinician hands and fingers during dental procedures. The use of barriers significantly reduces the need for chemical cleaners, thus prolonging the life of the equipment.

Only use barrier material that is intended for use with dental equipment. Midmark recommends the use of an FDA market-cleared barrier such as Pinnacle Cover-all™. Follow barrier manufacturer instructions for proper use of these products.

Cleaning and Disinfecting

In addition to the use of barriers Midmark recommends the use of an EPA registered and FDA market-cleared cleaner/disinfectant such as Cavicide™ to be used on all clinician controls or surfaces that may come in contact with dental instruments during dental procedures.

Follow cleaner/disinfectant manufacturer instructions for proper use of the product. Care should be taken to avoid excessive application and pooling of liquids.

Handpiece Accessories

Only use dental handpiece accessories with the delivery system that are FDA market-cleared and refer to manufacturer's instructions for proper cleaning and disinfecting. Either an autoclavable syringe tip or a single use disposable syringe tip may be used.

Equipment Alert

AUTOCCLAVABLE SYRINGE TIP STERILIZATION

The autoclavable syringe tips supplied with the delivery system must be sterilized prior to use with each patient, including initial use. Be sure to thoroughly rinse and clean syringe tips prior to sterilization, any debris may reduce the effectiveness of the sterilization. Recommended sterilization process is steam autoclave. Recommended parameters are 125°C (250°F) and 106 kPa (15 PSI) for 40 minutes at temperature and pressure.

General Purpose Cleaning

For general purpose cleaning, use cleaners that are appropriate for the situation, such as warm water and mild detergents, or a 10% solution of bleach with water.

Visual Inspection

After cleaning, visually inspect the product for deterioration of covers and touch pads. Do not use the delivery system if excessive discoloration, cracking, or other signs of wear are noticeable (see Calling for Service instructions).

Cleaning and Disinfecting Assistance

For assistance with cleaning and disinfecting instructions contact the Midmark Technical Service Department at 1-800-Midmark, it is helpful to provide the delivery system model number and serial number when asking for assistance.

Additional information is available from the organizations listed below:

- **Organization for Safety & Asepsis Procedures:**

<http://www.osap.org>

- **American Dental Association:**

<http://www.ada.org>

- **Dept. of Health & Human Resources
Centers for Disease Control & Prevention (CDC):**

<http://www.cdc.gov>

- **European Dental Association:**

<http://www.eda-eu.org>

Specifications	
Electrical Rating for Elevance Chair:	See power supply output (153941 ...)
Complies to the applicable requirements of:	ES/IEC/EN 60601-1, CAN/CSA C22.2 No. 60601-1 (Safety Standards), EN/IEC 60601-1-2 (EMC Standards)
Electrical Inputs for Knight and Midmark Ultra Chairs:	24 VAC, 3.2 amp 13 VAC, 2 amp
Complies to the applicable requirements of:	UL/IEC/EN 60601-1, CAN/CSA C22.2 No. 601.1, IEC 60601-1-1
Air Supply Requirements:	Maximum 80-100 PSI (5.52 - 6.9 bar)
Water Supply Requirements:	Maximum 30 - 50 PSI (2.1 -3.4 bar)
Classifications:	Class I, Type B Applied Part, Ordinary Equipment (IPX0), Continuous Operation
Certifications:	Midmark Corporation.....ISO-9001

WARNING

Do not modify this equipment without authorization of the manufacturer.

Caution

Federal law restricts this device to sale by or on the order of a licensed dental practitioner.

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Warranty Information

Midmark Limited Warranty - Dental Products

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will at Midmark's option repair or replace components of the dental products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this limited warranty is limited to the repair or replacement of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be as follows: Effective March 1, 2018 these applicable warranty periods are measured from the date of invoice to the original user, shall be as follows:

1. OPERATORY PRODUCTS

- a. Five (5) years for all products (except for the items in (b) through (e)).
- b. Two (2) years for upholstery (chairs and stools).
- c. "KINK-VALVE" module carries a ten (10) year warranty.
- d. The original light bulb on a new light carries a one (1) year warranty.
- e. Accessories not manufactured by Midmark are excluded including but not limited to Bien Air handpiece systems, Dentsply Cavitron scaler, Satelec scaler and curing light, and Sopro cameras.

2. ORAL SURGERY PRODUCTS are warranted for a period of one (1) year.

3. STERILIZER PRODUCTS are warranted for a period of one (1) year.

4. ULTRASONIC CLEANER PRODUCTS are warranted for a period of two (2) years.

5. AIR AND VACUUM PRODUCTS

- a. PowerAir® oil-less compressors – Five (5) years or 3,500 hours of use, whichever occurs first.
- b. PowerVac® and PowerVac® G dry vacuums – Five (5) years or 10,000 hours of use, whichever occurs first (except that the vacuum pump warranty term is ten (10) years or 20,000 hours of use, whichever occurs first).
- c. Classic Series® wet-ring vacuums – Five (5) years or 10,000 hours of use, whichever occurs first.
- d. PowerMax surgical suction – Two (2) years.
- e. Hg5 Series Amalgam Separator - One (1) year. (f) Midmark manufactured accessories - One (1) year.

6. SYNTHESIS™ DENTAL CASEWORK AND ARTIZAN® EXPRESSIONS PRODUCT

- a. Five (5) years for all products and components including door and drawer fronts, casters and slides, except for the items in (b), (c) and (d).
- b. Three (3) years for electrical components such as task lights/LED lights, cords, controls and accessories.
- c. Two (2) years for sliding track monitor mount and components and upholstery. (d) One (1) year for countertops and resin, including accessories.

7. IMAGING PRODUCTS are warranted for a period of two (2) years except for the ClearVision CR reader which is warranted for a period of one (1) year.

8. MIDMARK Replacement Parts and Accessories carry a ninety (90) day warranty

EXCLUSIONS

This warranty does not cover and Midmark shall not be liable for the following:

1. defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
2. products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environment conditions and electrical power requirements;
3. products considered to be of a consumable or sterile nature;
4. accessories or parts not manufactured by Midmark;
5. charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark
6. costs and expenses of routine maintenance and cleaning;
7. representations and warranties made by any person or entity other than Midmark;
8. matching of color, grain or texture except to commercially acceptable standards;
9. changes in color caused by natural or artificial light;
10. custom manufactured products;
11. alterations or modifications to the product by any person or entity other than Midmark; and
12. Products that would otherwise be covered under Sections 1 and 2 of this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical, animal health or dental market, as the case may be, in which purchaser intends to use the product.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER

MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

WARRANTY DISCLAIMER

THIS LIMITED WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUE OF LIMITATIONS

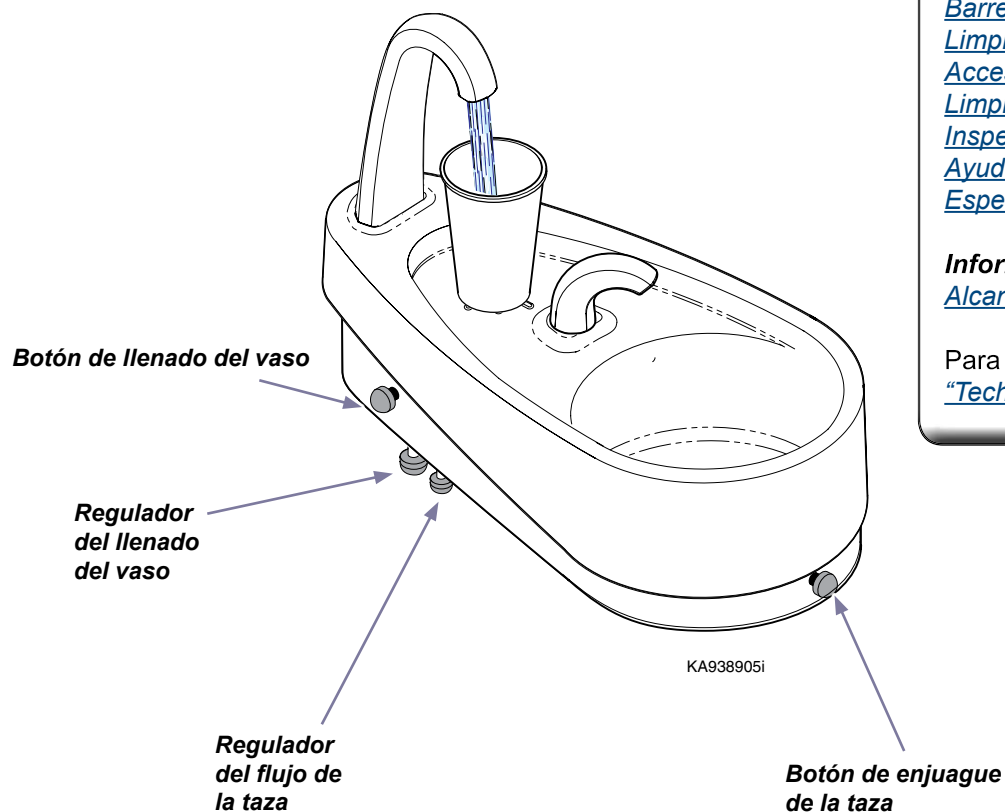
No action may be brought against Midmark for breach of this limited warranty, or implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

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