

Ultra Style J-Box Installation



Parts Bag Contents

none


Special Tools

none

Attention!

Unconnected tubing is for optional equipment. Refer to tubing diagram.

Step 1: Position J-box frame over utility connections.

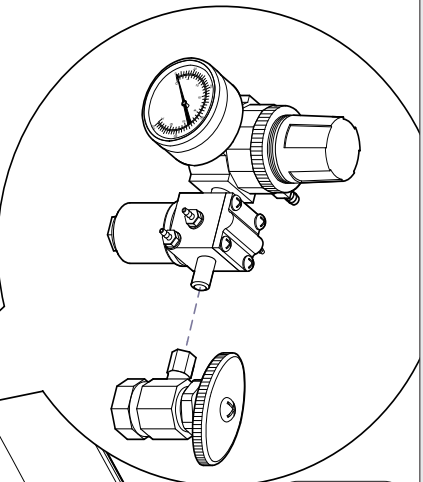

Step 2: Connect air regulator to air source.
Connect water regulator to water source.
Tighten compression nuts.

Step 3: Remove umbilical clamp.
Position umbilical as shown.
Replace umbilical clamp.

Optional
Step 4: Screw J-box frame to floor.

Step 5: Set lid on J-box frame.

