General Information:

To provide a safe means for removing trays and / or cassettes from the Midmark sterilizers through use of a non-heat conducting handle and related plates.

WARNING

Only use this tool with Midmark manufactured trays. Use care when removing or transporting the trays, or cassettes, as they may be hot. Hold the tray with the far end to prevent it from shifting and becoming dislodged. Failure to comply may result in personal injury due to burns.

Step 1: Removing trays from chamber.

A) To remove trays from the sterilizers insert the correct tray plate into end of the tool.

B) Hook saw-toothed tab onto center, top of tray lip while rotating tool downward until bottom plate is completely beneath tray. Remove the tray.

M11 - 05X
M9

Note

Use this tray plate for:

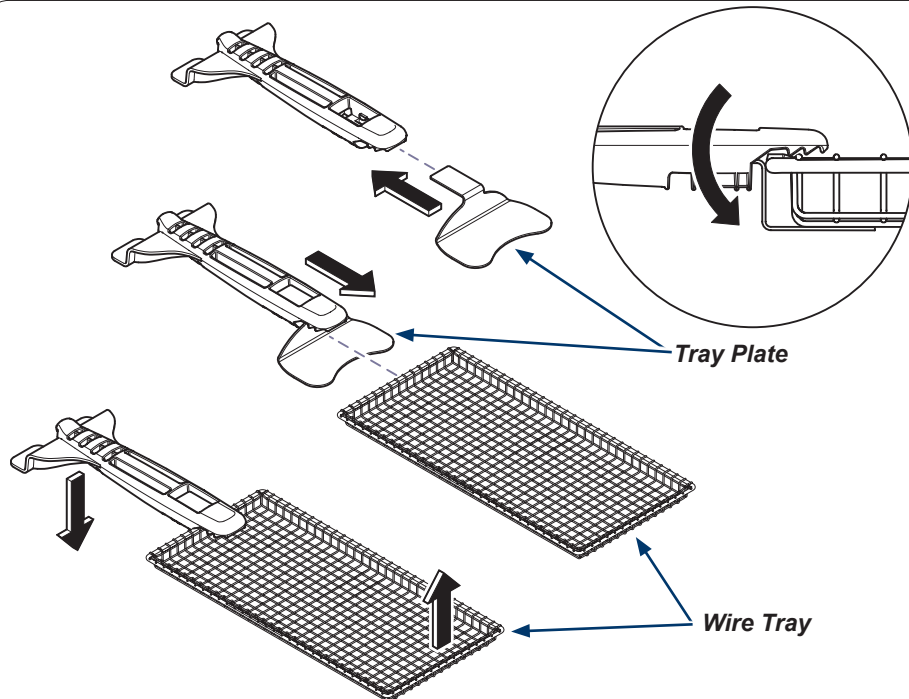
- All **M9**® and **M11**® -05X models with wire trays.
- All **M9** -02X and -04X models with stamped trays.

M11 - 02X / 04X


Note

Use this tray plate for:

- All **M11** -02X and -04X models with stamped trays.

ArtHalf

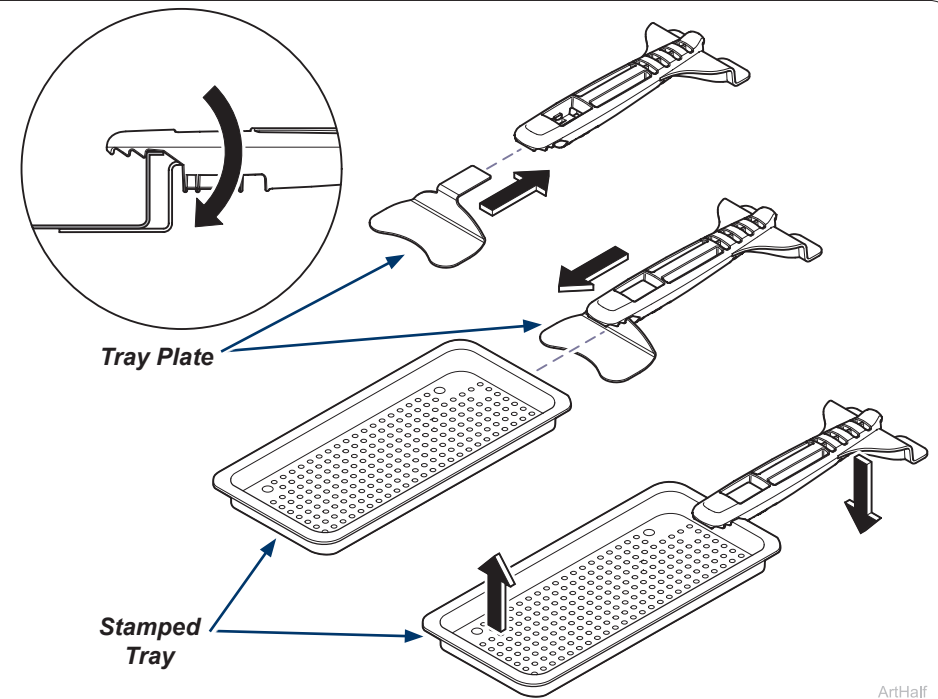ArtHalf

Note

The Cassette Tool is capable of handling cassettes up to 1 1/2" (3.8 cm) thick using the cassette plate attached to the handle.

**WARNING**

When removing the cassette, hold the cassette tool so the far end of the cassette is slightly elevated. Use care to prevent it from sliding off the cassette tool.

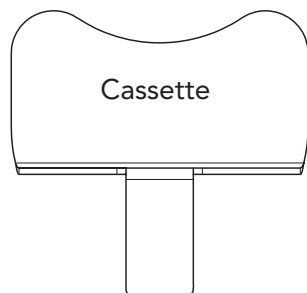
Step 1: Removing cassettes from chamber.

- A) To remove cassettes from the sterilizers insert the cassette plate into end of the tool.
- B) Hook saw-toothed tab onto center, top of cassette while rotating tool downward until bottom plate is completely beneath the cassette, then remove the cassette.

Note

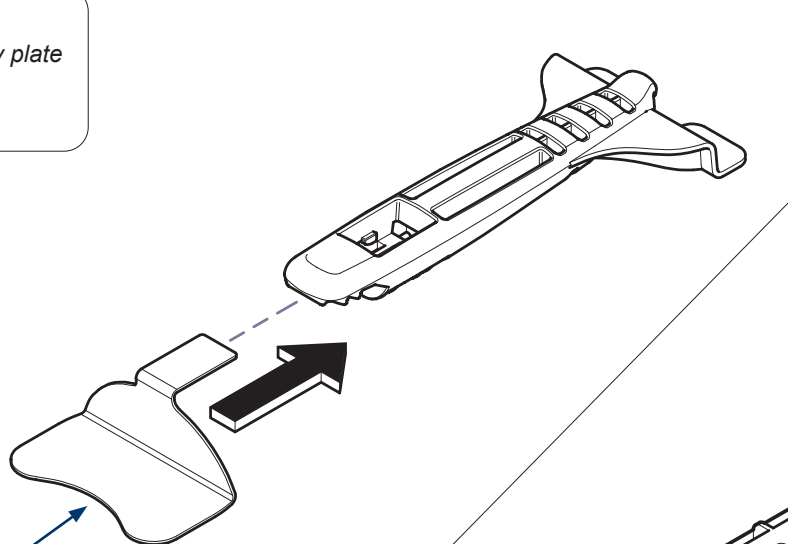
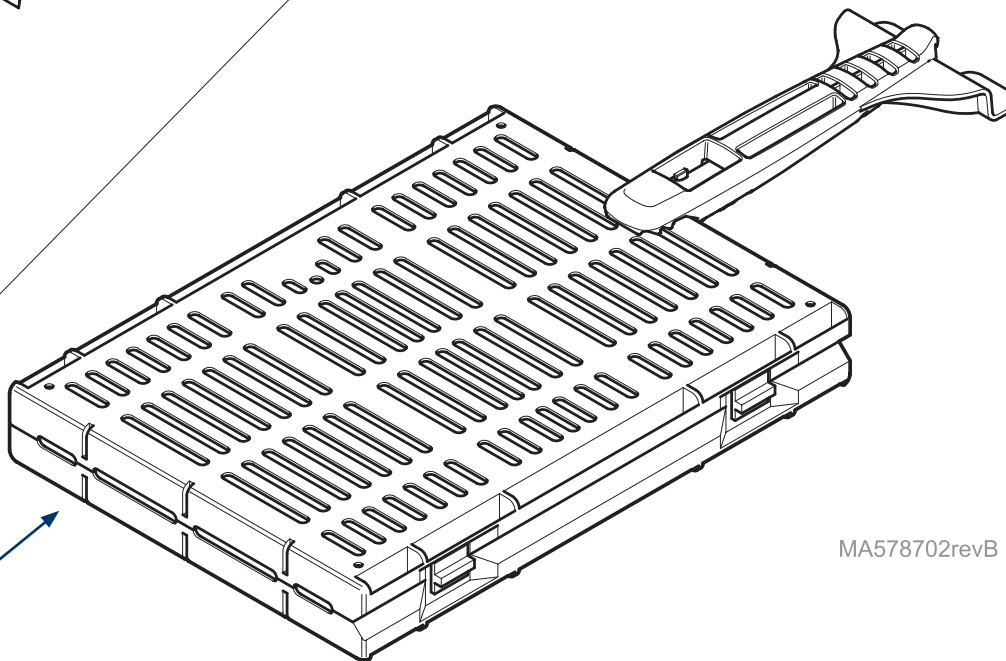
All models use the same tray plate for cassettes marked:

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