

Knee Crutches / Articulating Knee Crutches [9A352 / 9A361 / 9A362]

Contents:

Installation 1
Exploded Views 3

Special Tools:

Power Drill
7/16" Drill Bit

Intended Use:


To support patient's legs during examination.

Requirements / Restrictions:

Midmark models require a special cladding kit to accommodate the knee crutch brackets. Refer to Midmark pricing catalog to determine proper kit.

Step 1: Remove side cladding.

Note: Models without side cladding, go to step 2.

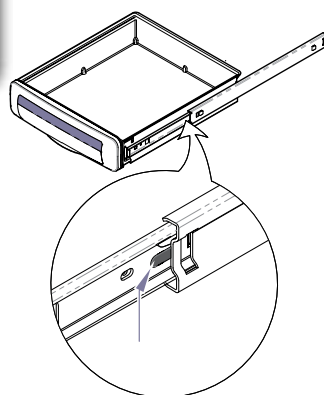


To remove top drawer, pull drawer out, then:
Press tab on RH slide down.
Press tab on LH slide up.

Midmark 604

604 shrouds are secured with screws from inside cabinet.

Midmark 622 / 623

622 / 623 shrouds are magnetic.
Pull outward to remove.

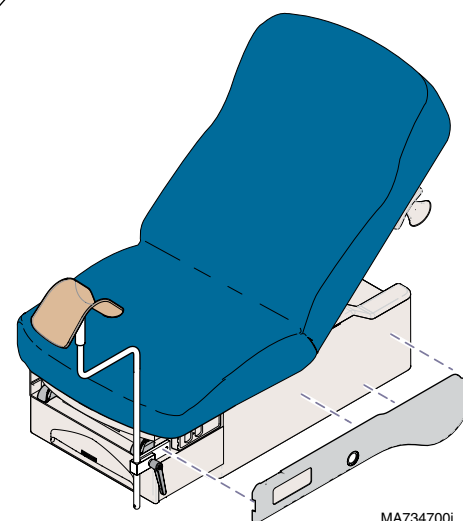

MA734700i

Note


Tables ordered with “factory installed” knee crutches will have holes pre-drilled. Go to Step 5.

1/16 in.
(1.5 mm)

Step 2: Position mounting block as shown.
Mark mounting holes.

Step 3: Drill two 7/16” mounting holes.

Step 4: Repeat steps 2 & 3 for other mounting block.

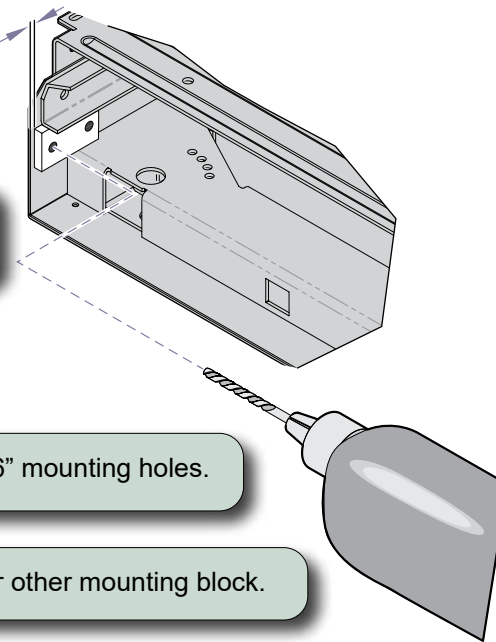


Step 5: Secure mounting block and clamp assembly with two hex bolts.

Step 6: Install knee crutch.
Tighten clamp handle.

Midmark Models Only

Step 7: Install side cladding w/ notches.
(Refer to illustration on page 1)

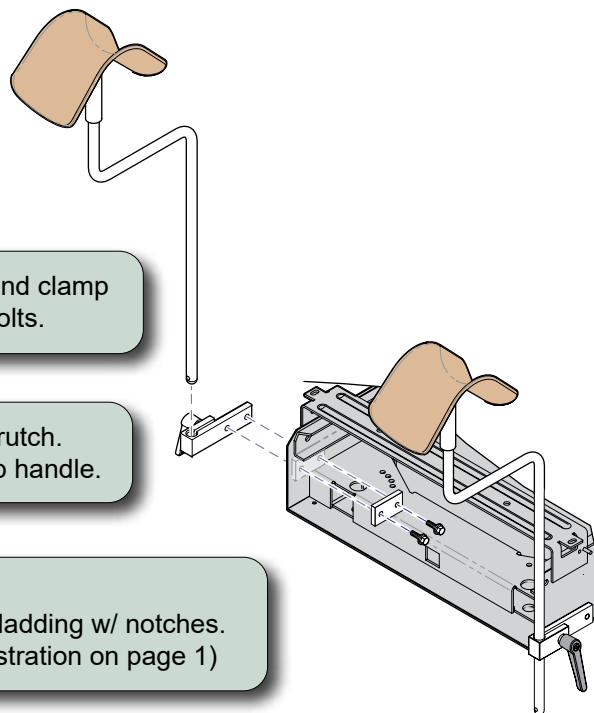

MA734700i