


# Procedure Chair Installation


**Applies to Models:**  
230 (all)  
641 (all)  
Language or origin: English

**Step 1:** Cut cable ties.  
Remove shipping brace.

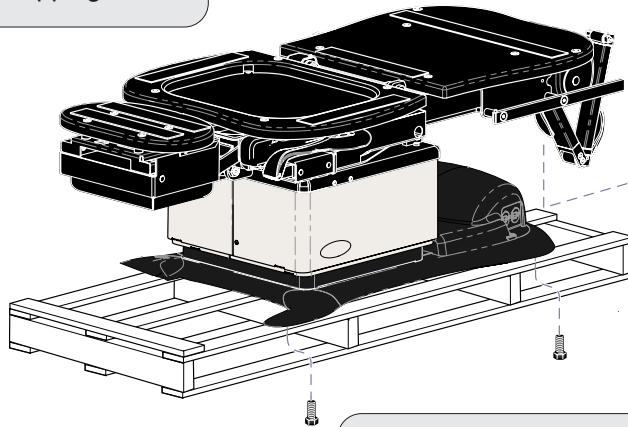

**NOTE:** Shipping brace may not be present on some models.

**Models w/Rotational Base only**  
**Step 2:** Remove shipping bracket.

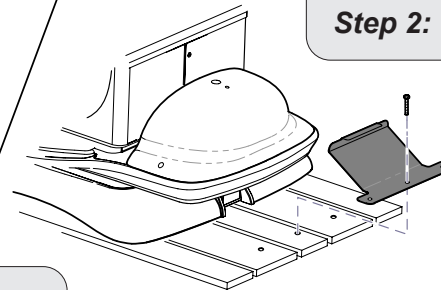

**Step 3:** Remove shipping bolts.  
Remove chair from skid.

**Does the voltage setting that appears in the fuse holder window match the facility supply voltage?**

If **YES**, go to Step 4.

If **NO**, you must perform  
Voltage Conversion Procedure.




## Equipment Alert

Voltage setting that shows thru window must match supply voltage. Incorrect setting will result in damage to the chair.

### Voltage Conversion Procedure:


- Open fuse holder cover.
- Remove fuse holder.
- Flip fuse holder 180°.

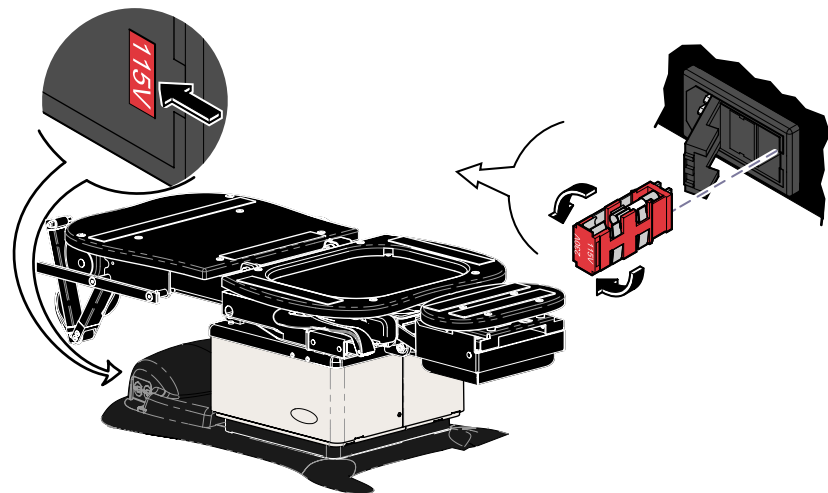

Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.


#### Attention

Slide narrow portion of cord connector into bracket as shown.

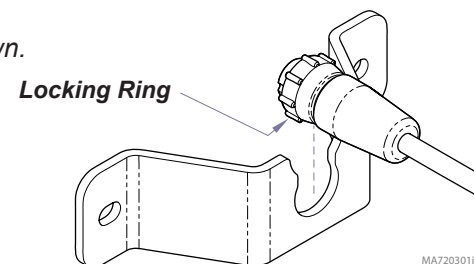

**Step 4:** Connect hand/foot control to cord inlet. Tighten locking ring.

**Note:** The cord connectors are “keyed” and must be oriented properly to connect.

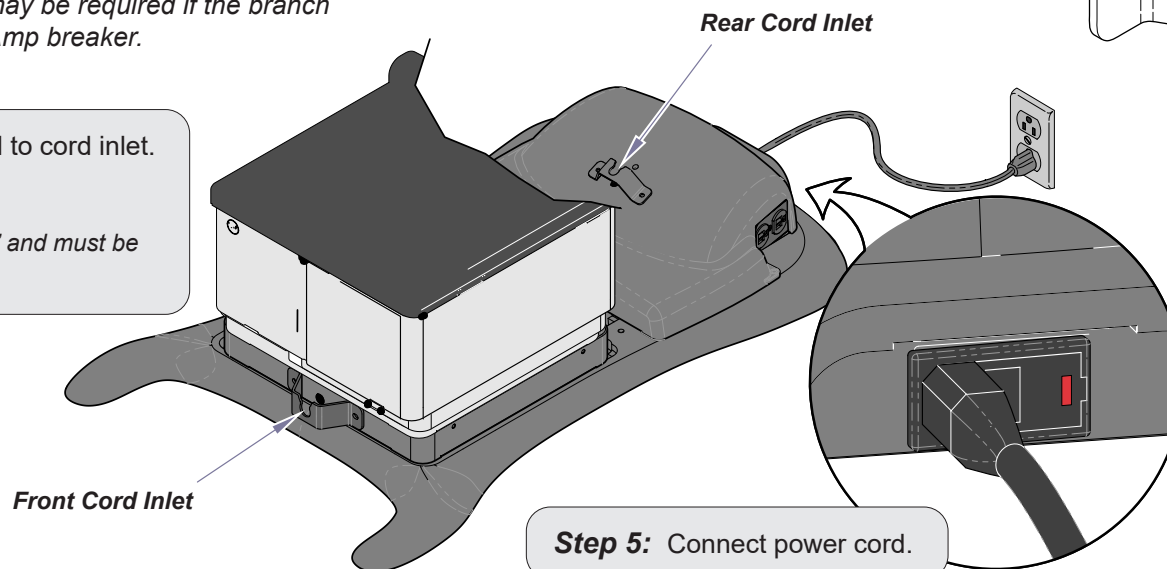


#### Equipment Alert

The chair must be level to ensure proper operation.

**Step 7:** Install hand control mounting discs (2), and holster.

#### Step 6: Level the chair.

- Using a level and Allen wrench, adjust leveling screws as necessary.
- Install four hole plugs.

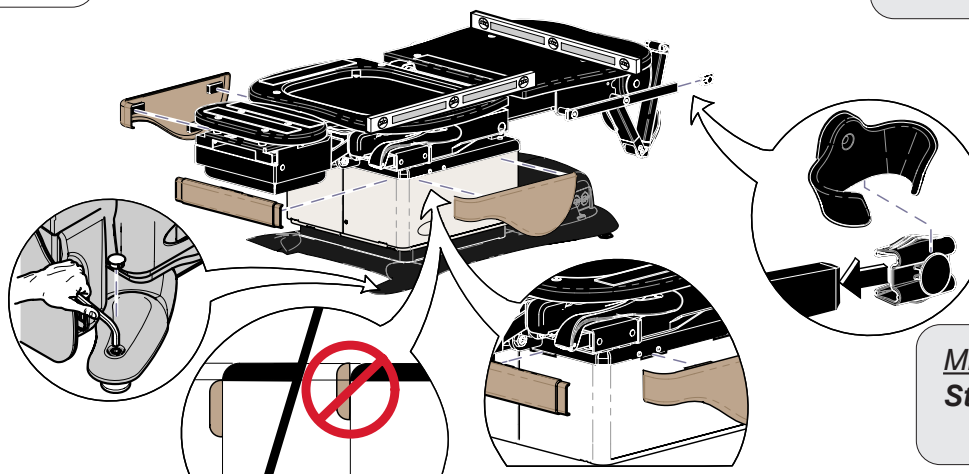

#### Midmark models only

**Step 8:** Install magnetic base covers (3 pcs.)