# Ritter®

## 244 Barrier-Free™ Bariatric Power Treatment

[English](#)

[Español](#)

[Français](#)

# User Guide

003-1456-99 Rev. AA6 (10/19/23)  
TP202 20-42-FO-00014 Rev A3 C2169 Style P

# **Product Information**

*(The information below is required when calling for service.)*

**Model Number :**

**Serial Number:**

**Date of Purchase:**

**Name of Owner / Facility:**

**Name of Dealer:**

**Dealer's Phone Number:**

**Midmark Authorized Service Company:**

**Model / Serial  
Number Label**  
*(on frame)*

# Table of Contents

## Important Information

|  |   |
|--|---|
| <i>Safety Symbols</i> .....                      | 4 |
| <i>Intended Use</i> .....                        | 5 |
| <i>Patient Positioning</i> .....                 | 5 |
| <i>Electromagnetic Interference</i> .....        | 5 |
| <i>Transportation / Storage Conditions</i> ..... | 5 |
| <i>Authorized Representative</i> .....           | 6 |
| <i>Safety Instructions:</i>                      |   |
| <i>Hand Control Connection</i> .....             | 7 |

## Operation

|                              |    |
|------------------------------|----|
| <i>Power Switch</i> .....    | 8  |
| <i>Table Up / Down</i> ..... | 9  |
| <i>Back Up / Down</i> .....  | 10 |
| <i>Tilt Up / Down</i> .....  | 11 |
| <i>Paper Roll</i> .....      | 12 |

## Cleaning and Maintenance

|                                   |    |
|-----------------------------------|----|
| <i>Cleaning</i> .....             | 13 |
| <i>Periodic Maintenance</i> ..... | 13 |
| <i>Calling for Service</i> .....  | 13 |

## Specifications

|   |    |
|---|----|
| <i>Weights, Dimensions, Electrical Specifications</i> ..... | 14 |
| <i>Regulatory Compliance Chart</i> .....                    | 15 |
| <i>Upholstered Top Dimensions</i> .....                     | 15 |
| <i>Range of Motion</i> .....                                | 16 |

## Warranty Information

|                               |    |
|-------------------------------|----|
| <i>Limited Warranty</i> ..... | 17 |
|-------------------------------|----|

# Important Information

## Safety Symbols

### WARNING

Indicates a potentially hazardous situation which could result in serious injury.

### CAUTION

Indicates a potentially hazardous situation which could result in injury if not avoided.

### Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

### Note

Amplifies a procedure, practice or condition.

## Symbol Glossary

Consult User Guide for important information

Proper shipping orientation

Fragile

Type B, Applied Part

Protective earth ground

Keep dry

Maximum stacking height (palletted units)

Fuse rating

Dangerous voltage / shock hazard

Duty Cycle (motor run time):  
1 minute ON, 9 minutes OFF

This product is Listed by Underwriters Laboratories, Inc. Representative samples of this product have been evaluated by UL and meet the applicable U.S. safety requirements.

This product has been evaluated with respect to electric shock, fire and mechanical hazards only, in accordance with UL60601-1, IEC60601-1, CAN/CSA C22.2, No. 601.1.

This product has been evaluated with respect to electrical shock, fire, and mechanical hazards only, in accordance with CAN/CSA C22.2, No. 601.1.

This component (or material) is Recognized by Underwriters Laboratories, Inc. Representative samples of this component have been evaluated by UL and meet the applicable U.S. safety requirements.

## Intended Use

This product is intended to be used as a chair / table to provide positioning and support for patients during examinations and procedures conducted by qualified medical professionals (*primarily podiatrists*).

## Patient Positioning

The table is designed to accommodate the following examination positions:

- Prone
- Supine
- Lateral Sims

### CAUTION

*Never seat a patient on the back section of the table. Doing so may cause table to tip.*

## Electrical Requirements

### CAUTION

*Most exposed metal parts of the table are electrically grounded. When performing a cauterization or similar treatment, the patient must be insulated from the metal portions of the table by nonconductive material. Failure to do so could result in electric shock or burns to patient.*

## Electromagnetic Interference

This table is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this table, remove interfering device from room and/or plug table into an isolated circuit.

## Transportation / Storage Conditions

Ambient Temperature Range:.....-5°C to +38°C (+23°F to 100°F)

Relative Humidity..... 10% to 90% (non-condensing)

Atmospheric Pressure .....500hPa to 1060hPa (0.49atm to 1.05atm)

### WARNING

*This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to [www.p65warnings.ca.gov](http://www.p65warnings.ca.gov)*

## ***Authorized Representatives***

### **WARNING**

***In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.***

Customers in Colombia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

# Safety Instructions

## Hand Control Connection

The hand control connection port is located on the table base at the head end of the table.

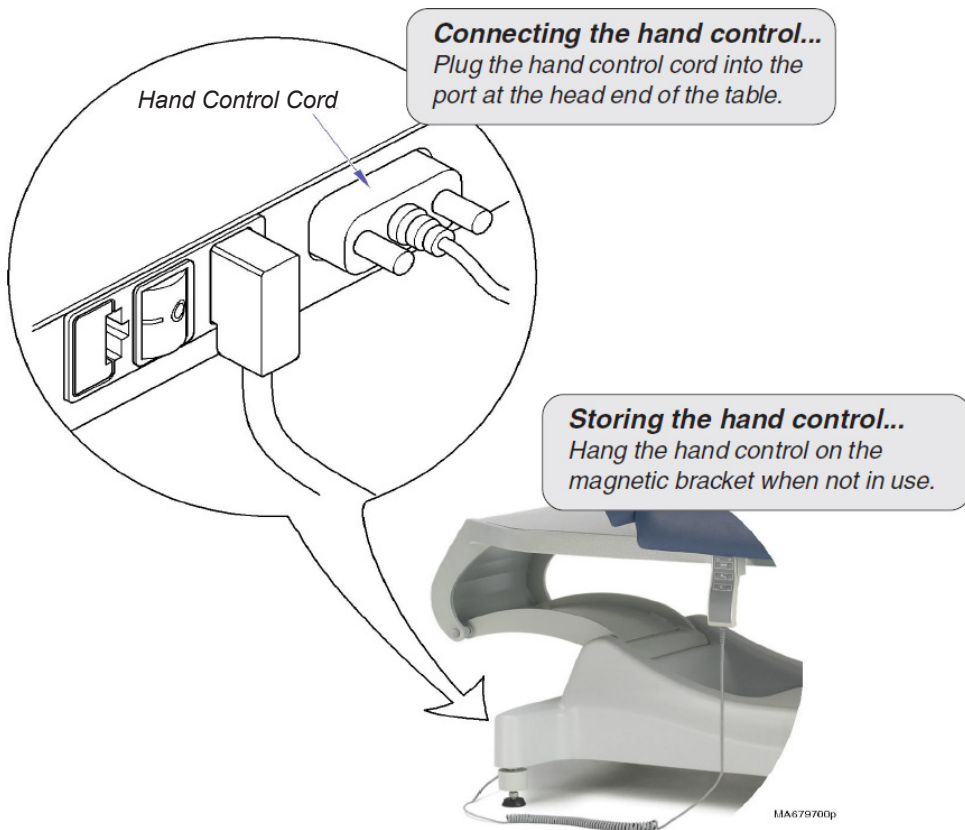
### **Equipment Alert**

Never plug the hand control into any other outlet. Doing so may result in damage to the equipment.

### **CAUTION**

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.


# Operation

## Power Switch

The foot control cord inlet is located on the base column at the foot end of the chair.

**To turn the table ON...**  
Move the power switch to ON (I).

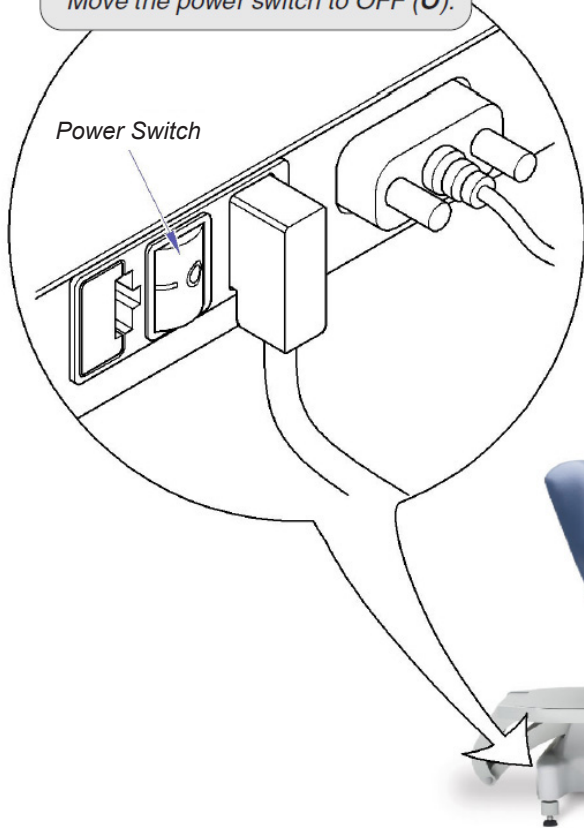**To turn the table OFF...**  
Move the power switch to OFF (O).

MA679100p

## Table Up / Down

**To move the table top UP or DOWN...**  
Press and hold the appropriate button.

### Note

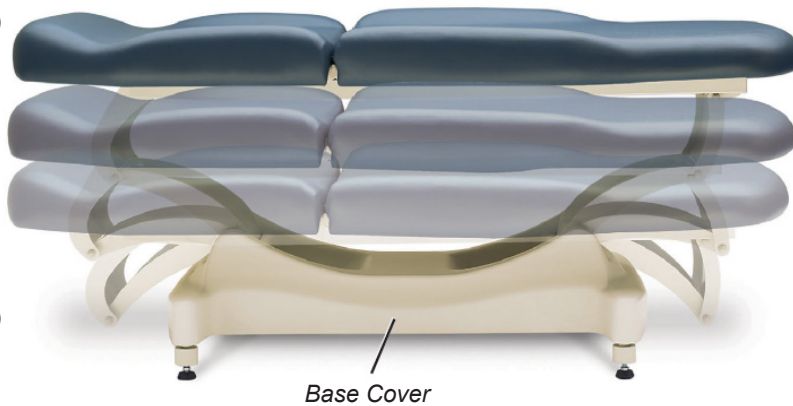
Movement automatically stops at the maximum / minimum height.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].

**Max.**  
34 in.  
(86.4 cm)

**Min.**  
18 in.  
(45.7 cm)



## Back Up / Down

**To move the Back section UP or DOWN...**  
Press and hold the appropriate button.

### Note

Movement automatically stops at the maximum / minimum angle.

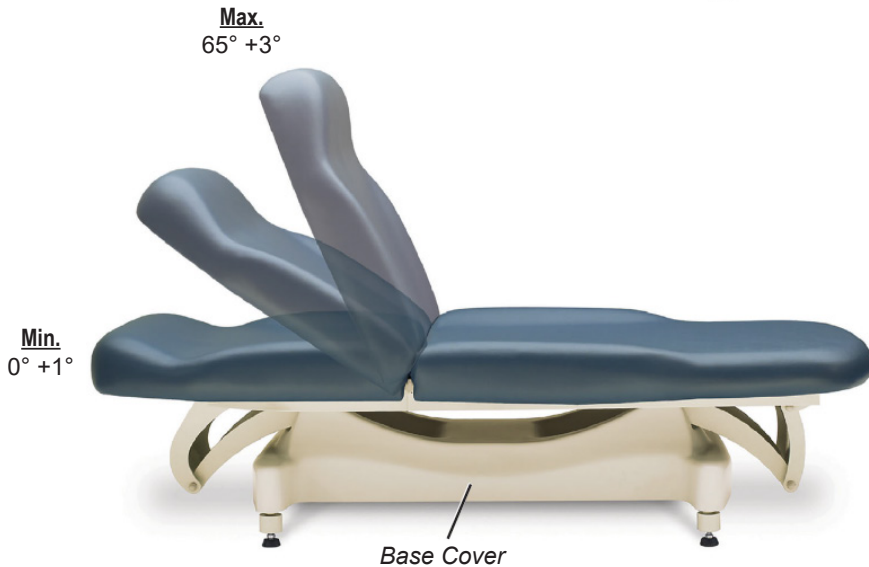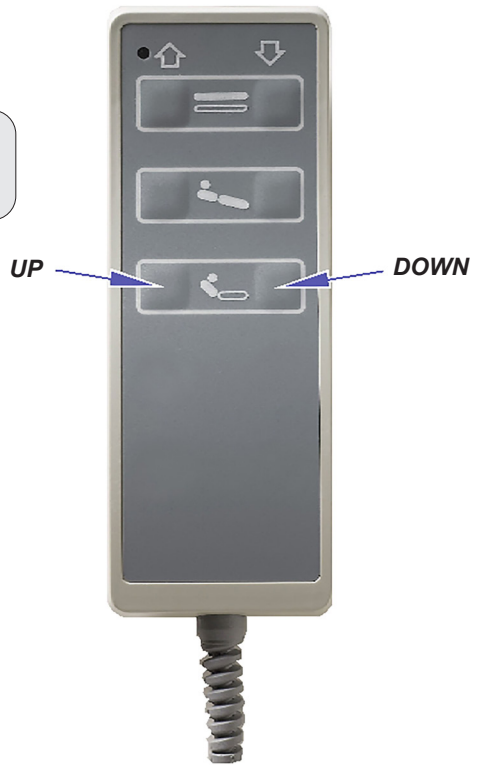[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].


## Tilt Up / Down

**To tilt the table top UP or DOWN...**  
Press and hold the appropriate button.

### Note

Movement automatically stops at the maximum / minimum angle.

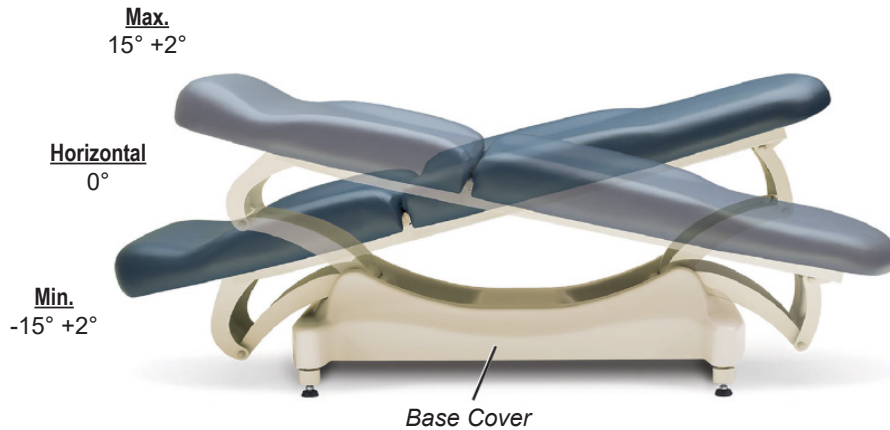[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].


# Paper Roll

Paper Roll Size (max.) ..... 24 in. wide x 6 in. diameter  
(61 cm x 15.2 cm)

## **WARNING**

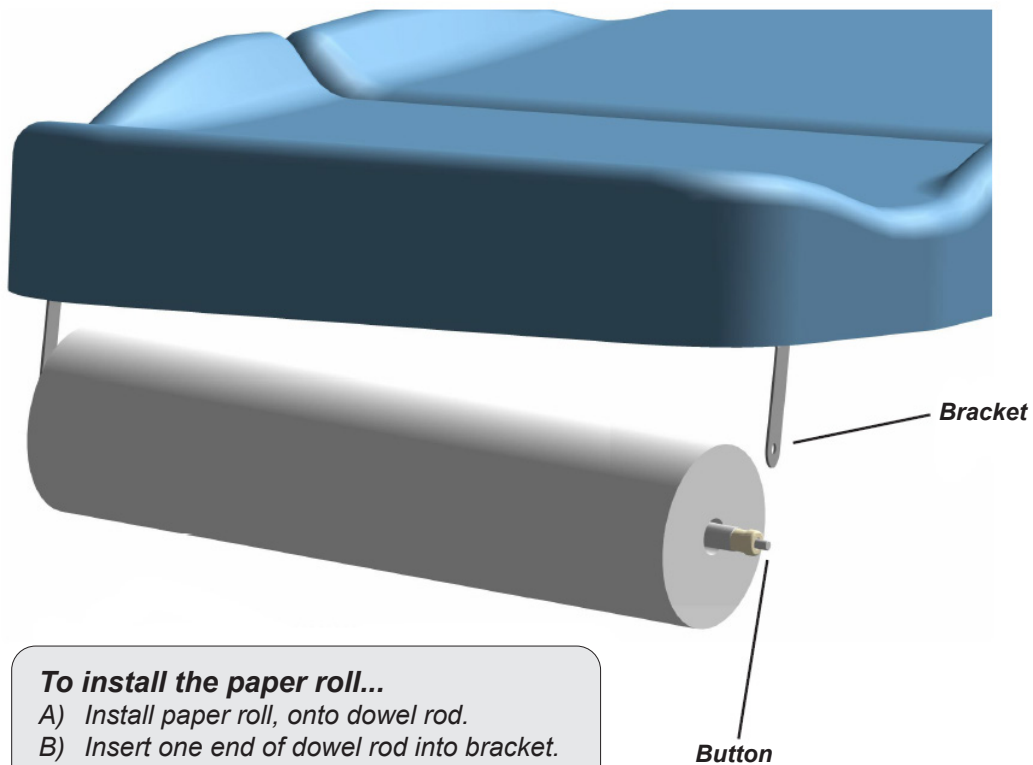*To avoid injury, do not put head or extremities under the paper tear strap.*

### **To install the paper roll...**

- A) *Install paper roll, onto dowel rod.*
- B) *Insert one end of dowel rod into bracket.*
- C) *Press button on other end of dowel rod, then insert rod into other bracket.*

## ***Cleaning Upholstery***

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](http://midmark.com) in the Technical library under the User Information tab for your product.

### ***Painted Metal / Plastic Surfaces***

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

## ***Periodic Maintenance***

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

## ***Calling for Service***

### **Note**

Model / Serial Number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585  
Monday-Thursday 8:00 AM until 6:00 PM  
Friday 8:00 AM until 5:00 PM (ET)  
[midmark.com](http://midmark.com)

# Specifications

## Weights, Dimensions, Electrical Specifications

Maximum Patient Weight: ..... 850 lbs (385 kg)  
w/Caster Base Accessory ..... 650 lbs (295 kg)

Paper Roll (maximum size): ..... (See Paper Roll page)

Range of Motion and  
Dimensions ..... (See Range of Motion and Dimensions page)

Weight of Table: ..... 340 lbs (154 kg)

Power Cord: ..... 8 ft. (243.8 cm) long

### Fuses:

115 VAC models:

located in IEC inlet (2) ..... 6.3A, 250V, Slo-Blo, 5 x 20 mm

Duty Cycle (motor run time): ..... Intermittent Operation  
[1 minute ON / 9 minutes OFF]

Classifications: ..... Class 1, Type B Applied Part,  
Ordinary Equipment, Intermittent Operation

Electrical Requirements: ..... [See Regulatory Compliance Charts]

### **WARNING**

***Equipment is not suitable for use in the presence of a flammable anesthetic Mixture.***

***Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.***

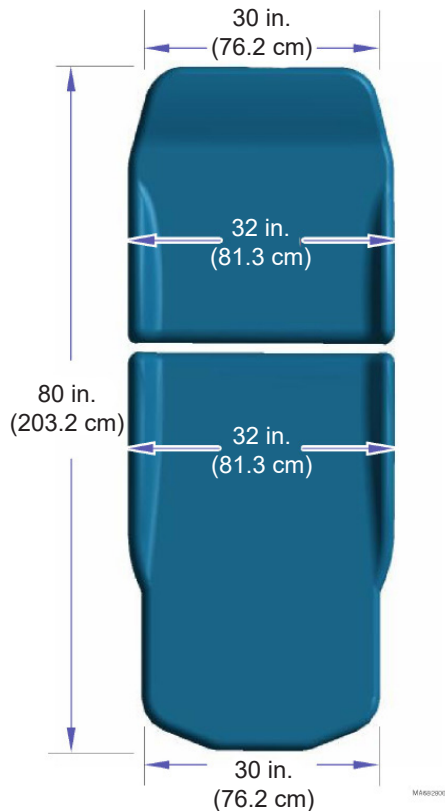
# Regulatory Compliance Chart

**Fire Code Ratings:** All upholstery complies with California Bureau of Home Furnishing Technical Bulletin 117.

Optional upholstery is available that complies with California Bureau of Home Furnishing Technical Bulletin 133.

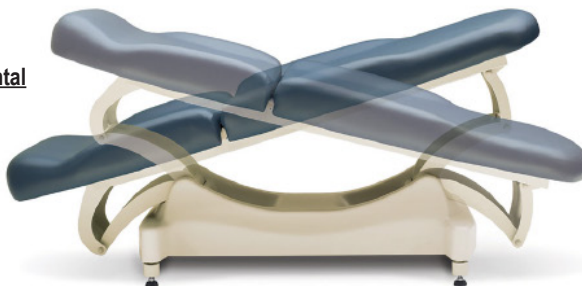
| Model   | Description               | Serial Number Prefix | Complies To: |                          |                    |            |    | Electrical Ratings: |      |             |
|---------|---------------------------|----------------------|--------------|--------------------------|--------------------|------------|----|---------------------|------|-------------|
|         |                           |                      | UL 60601-1   | CAN/CSA 22.2, #601.1-M90 | EN 60601-1-2 (EMC) | EN 60601-1 | CE | VAC +/-10%          | Amps | Cycles (Hz) |
| 244-001 | Bariatric Treatment Table | n/a                  | X            | X                        |                    |            |    | 120                 | 2.5  | 60          |

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**Midmark Corporation**