


Barrier-Free™ Bariatric Power Treatment Table Installation

For models:
244 (all)

Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

Step 1: Connect hand control cord.
Connect power cord.

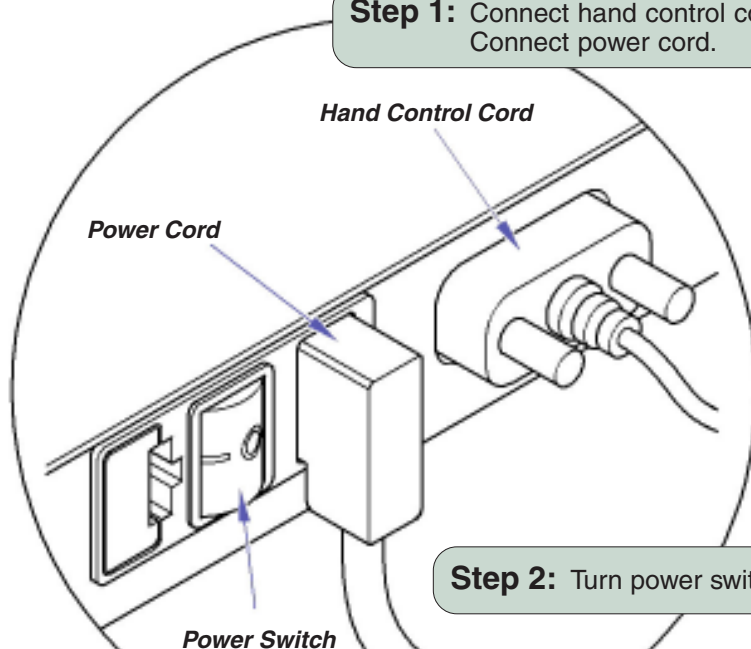

Step 2: Turn power switch to ON (I)



Equipment Alert

Table must be leveled to ensure proper operation.

Step 3: Adjust four leveling feet until table is level.