


Note

Oversized Headrest Accessories for Midmark® 630 models:
9A331001-xxx Standard and UltraFree upholstery
9A342001-xxx Ultra-Premium upholstery

To install headrest upholstery...

- Insert mounting studs into holes in pivot mount.
- Push three lock levers inward to lock in place.

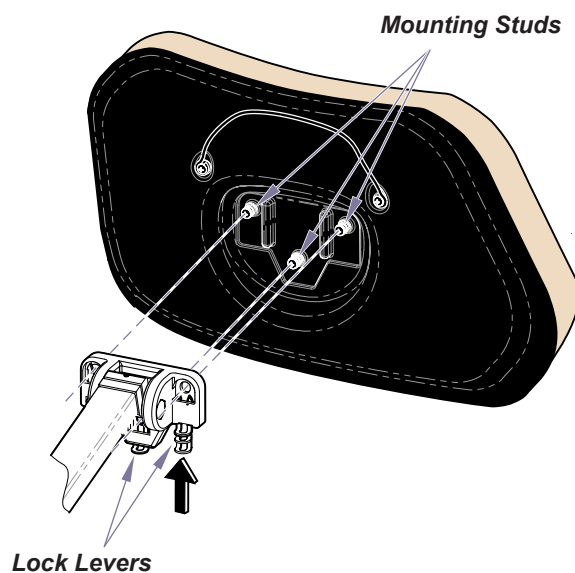


To install seat / back upholstery...

- Position front of seat section against mounting board.
- Lay upholstery flat, then press down firmly.

To install foot upholstery...

- Align hook and loop fastener strips.
- Press down firmly.

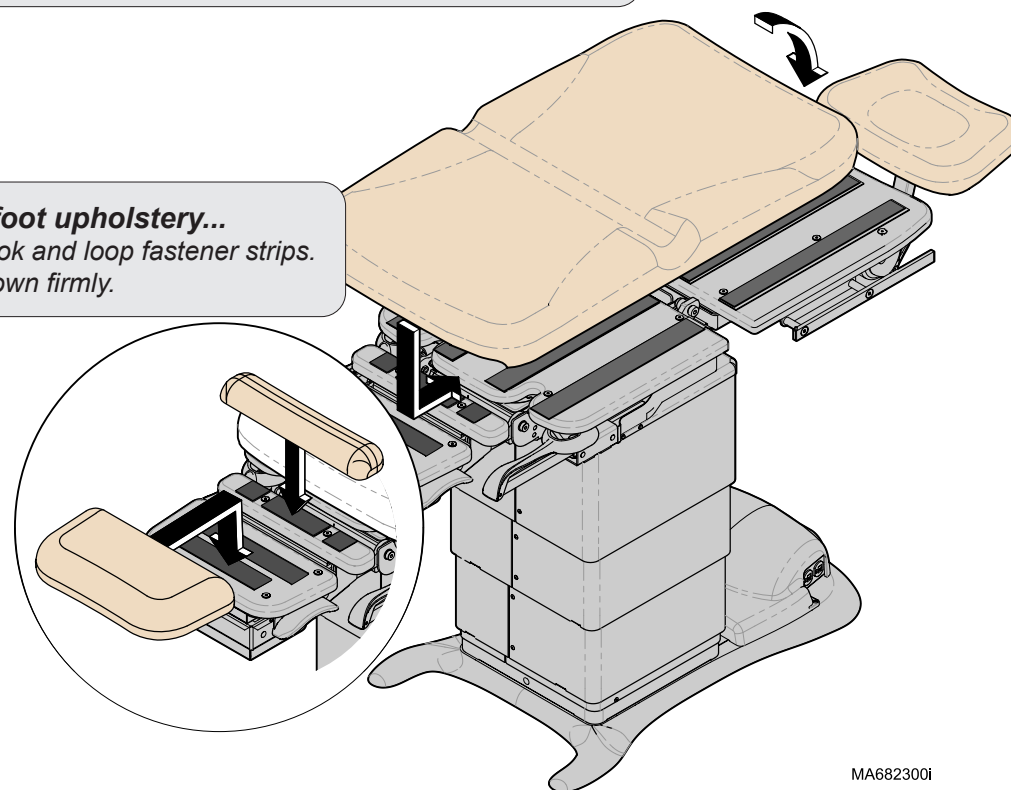

MA682300i