230

Universal Procedures Chair

Ritter®

[English](#)

[Español](#)

[Français](#)

For Models:

230 (All)

User Guide

Product Information

(The information below is required when calling for service)

Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company [name / phone]:

Model / Serial Number
Location

Table of Contents

Important Information

Safety Symbols	4
Intended Use	5
Electrical Requirements	5
Electromagnetic Interference	5
Disposal of Equipment	5
Transportation / Storage Conditions	5
Authorized Representatives	6
Safety Instructions	7
Hand / Foot Control Connection	7

Operation

Power To The Chair	8
Base Up / Down	9
Back Up / Down	10
Foot Up / Down	11
Tilt Up / Down	12
Paper Roll	13
Stirrups	14
Treatment Pan	15
Duplex Receptacle (optional - not available on 230 volt models)	16
Headrest Positioning	17
Knee / Chest Position	18
Rotational Base (optional)	19

Maintenance

Cleaning	20
Periodic Maintenance	20
Calling for Service	20

Troubleshooting

No Foot Up Function	21
---------------------------	----

Specifications

Weights, Dimensions, Electrical Specifications	22
Model Identification / Compliance Chart	23
Range of Motion & Dimensions	24

Warranty Information

Limited Warranty	26
------------------------	----

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note
Amplifies a procedure, practice, or condition.

Refer to manual

Proper Shipping Orientation

Fragile

Type B, Applied Part


Protective earth ground

Keep dry

Catalogue Number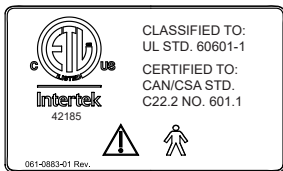

Serial Number

Manufacturer

Maximum stacking height
(palletted units)

Fuse rating specification

Protected against dripping water

Duty Cycle (motor run time):
30 seconds ON, 5 minutes OFF

Pan must be fully retracted to
enable Foot Up function.

Patient weight limit

This product has been evaluated with respect to electrical shock, fire & mechanical hazards only, in accordance with UL60601-1 and CAN/CSA C22.2 NO. 601.1.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during general examinations and procedures conducted by qualified medical professionals.

Electrical Requirements

Caution

When using high frequency surgical devices or endocardial catheters:

- Use non-conductive material to insulate patient from metal portions of the chair.
- Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This Midmark chair is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this chair:

- Remove interfering device from room
- Plug chair into an isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range..... +41°F to 100°F (+5°C to +38°C)
Relative Humidity..... 10% to 90% (non-condensing)
Atmospheric Pressure 500hPa to 1060hPa (0.49atm to 1.05atm)

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofia Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading

Midmac Roundabout

Opposite Al Mana Petrol Station

Doha, Qatar

Tel: +974-44337000

Fax: +974-44337100

Customers in Kuwait should direct all questions, incidents, and complaints to the Midmark Authorized Representative: Central Circle Co.

P.O. Box 1015 Salmieh

22011 Kuwait

Al-Sharq

Khalid Bin Al-Waleed Street

Al-Kazmi Building 10

Third Floor

Email : cencico@centralcircleco.com

Phone : +965 22411758, +965 22411748, +965 22444228

Fax : +965 22445457

Safety Instructions

Caution

Be sure that all personnel and equipment are clear of the chair before activating any function. Failure to do so could result in personal injury.

Hand / Foot Control Connection

There are two hand / foot control cord inlets. One is located on the base column at the foot end of the chair, and one is located on the PC board cover at the head end. These cord inlets are interchangeable.


Equipment Alert

Never plug hand / foot control into any other outlets. Doing so may result in damage to the equipment.


To connect hand / foot control cord...

- A) Slide narrow portion of cord connector into bracket.
- B) Connect cord to inlet (see Note).
- C) Tighten locking ring.

Note: The cord connector is "keyed" and must be oriented properly to connect.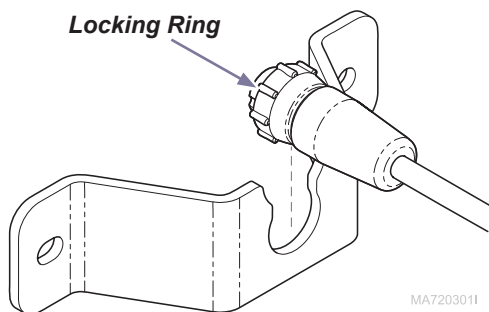

MA7203011

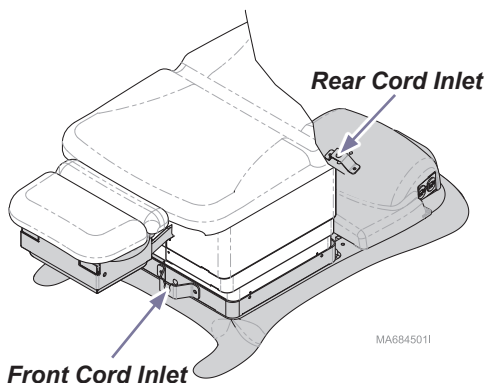MA6845011

Operation

Power To The Chair

If the chair has no power...

All functions will be inoperable.


Before calling for service...

- A) Check power cord connections.
- B) Check / reset facility circuit breaker.
- C) Check / replace chair fuse(s).

If Power Indicator Light is...

ON: Chair has power.

OFF: No power to chair.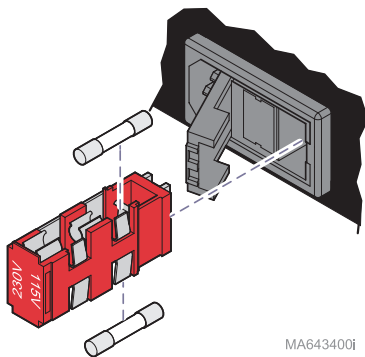

Before replacing fuses...


Disconnect power cord from chair.

To replace fuses...

[Refer to Specifications section for correct fuse size and ratings]

- A) Open the fuse holder cover.
- B) Remove fuse holder and faulty fuse(s).
- C) Install new fuse(s), then replace fuse holder.

Attention: Voltage setting that shows thru window of fuse holder cover must match facility supply voltage.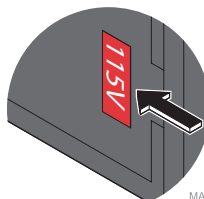


Equipment Alert

Incorrect voltage setting will result in damage to the chair.

Base Up / Down

To move the Base UP or DOWN...
Press and hold the appropriate button.

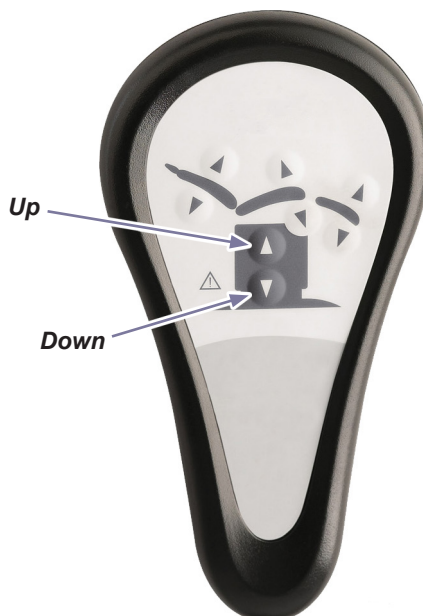Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.


Max.
Standard base: 40 in. (101.6 cm)
Rotational base: 43.5 in. (110.5 cm)

Min.
Standard base: 22.5 in. (57 cm)
Rotational base: 26 in. (66 cm)

Back Up / Down

To move the Back section UP or DOWN...
Press and hold the appropriate button.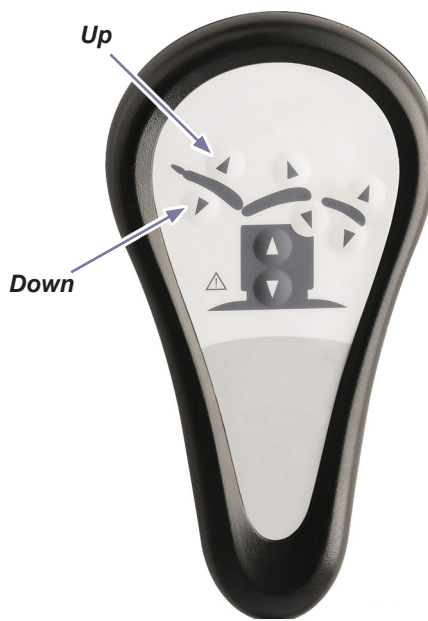

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.


Foot Up / Down

Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” when Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.

To move the Foot section UP or DOWN...
Press and hold the appropriate button.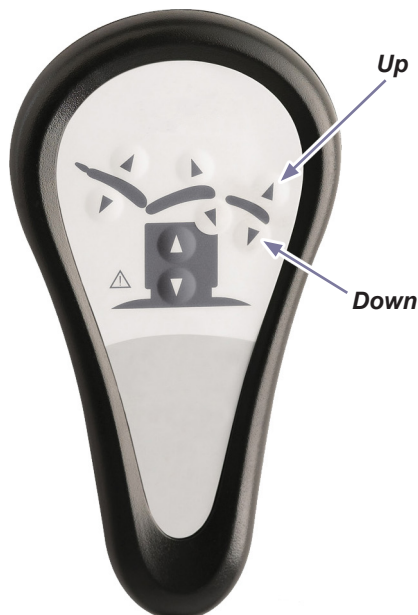

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.


Max.
 $0^{\circ} \pm 0.5^{\circ}$

Min.
 $90^{\circ} \pm 1^{\circ}$

Tilt Up / Down

To Tilt the seat section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.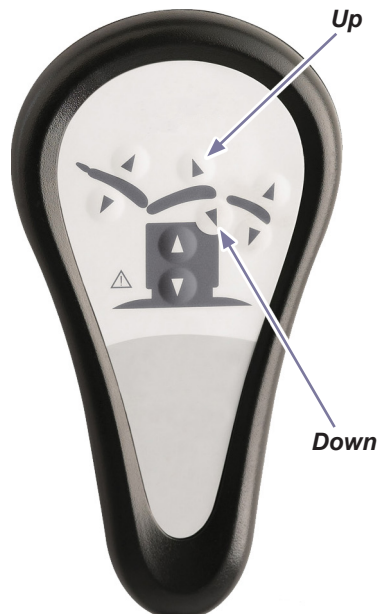


Paper Roll

*Paper Roll Size (max.)..... 18 in. long x 3.5 in. diameter
(45.7 cm x 8.9 cm)*

To install the paper roll...

- A) *Pivot paper roll cover back to open.*
- B) *Install paper roll, then pull paper over chair.*
- C) *Pivot paper roll cover forward to close.*

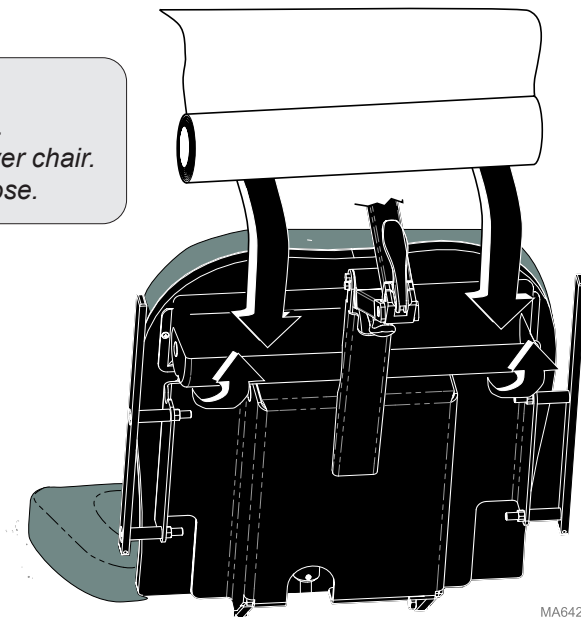MA642601I

Stirrups

Caution

Never use stirrups to lift chair.

Be sure the stirrups are locked in place before using.


The stirrups will not support the patient's entire weight / excessive force.

To position the stirrups...

A) Pull the stirrup out, then unfold.

B) Lift the stirrup slightly, then move it left or right as desired.

C) Release stirrup to lock in position.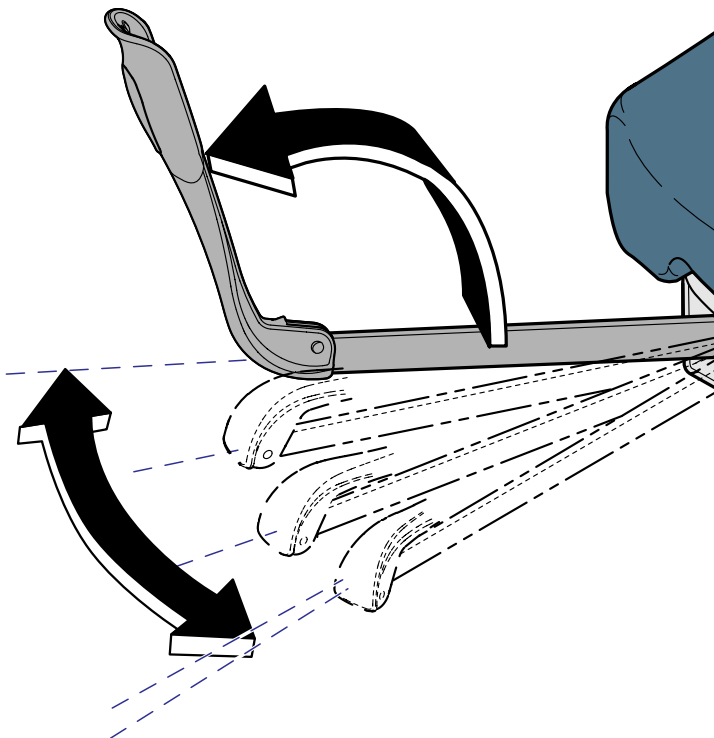

MA602500i

Treatment Pan


Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” if Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.

To access the treatment pan...

- A) Lower the foot section.
- B) Pull the treatment pan slide out.

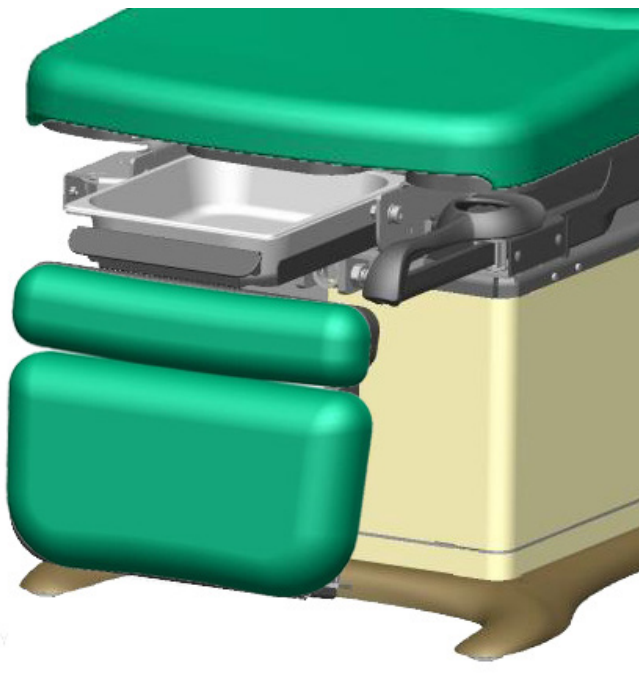Duplex Receptacle (optional - not available on 230 volt models)

The duplex receptacle provides power for accessories used during medical procedures. Power to the receptacle will be interrupted if the maximum load is exceeded.

Maximum Load [both receptacles combined] 115 VAC, 3 amps, 50/60 Hz

Equipment Alert

*Exceeding the maximum load for the receptacle may result in blown fuses. If this occurs, all chair functions will be disabled until fuses are replaced. [Fuse replacement procedure is shown on the **Power To The Chair** page.]*

Headrest Positioning


Caution

Be sure headrest is locked securely in position before use.

To adjust the headrest at pivot points 1 and/or 2...

- A) Pull upper handle outward to release.
- B) Move pivot point(s) to desired position.
- C) Push upper handle inward to lock in place.

[For clarity, the upholstery is not shown]

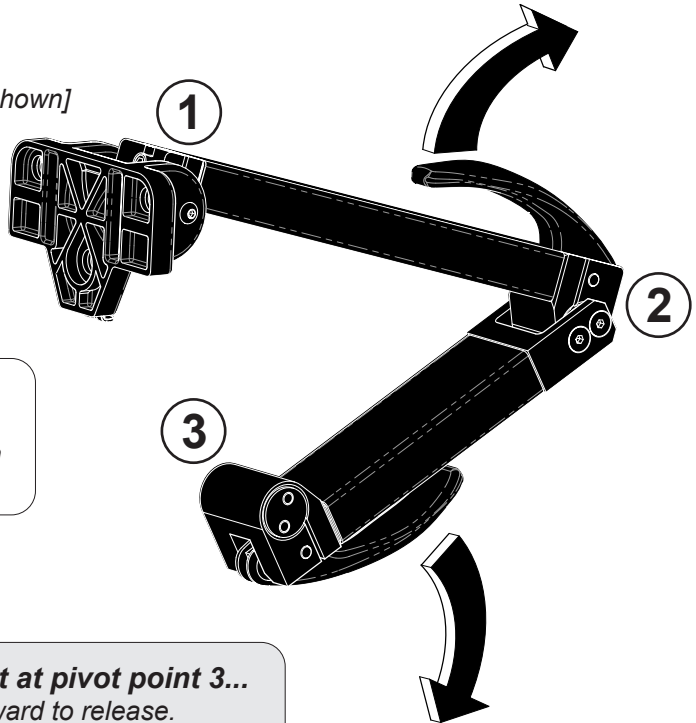Note

The handle tension is adjustable.
Contact your service provider if an
adjustment is required.

To adjust the headrest at pivot point 3...

- A) Pull lower handle outward to release.
- B) Move pivot point to desired position.
- C) Push lower handle inward to lock in place.

MA642200i

Knee / Chest Position


Caution

Be sure foot section is locked securely in position before use.

To remove the foot section...

- A) Lift release levers on either side of foot section.
- B) Pull foot section straight out.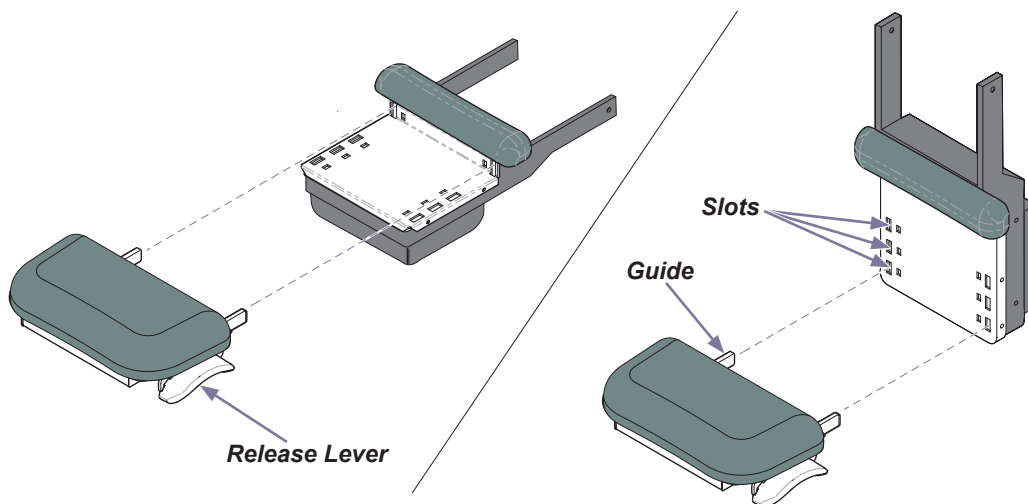

MA6847001

To install foot section in Knee / Chest Position...

- A) Lower the foot section.
- B) Insert foot section guides into desired slots.
- C) Be sure foot section is securely engaged before using.

Rotational Base **(optional)**

To rotate the chair base...

- A) *Press and release the brake pedal.*
- B) *Rotate the chair to desired position.*

To lock in position...

- A) *Press and release the brake pedal.*

Note: The base will automatically lock in position after two minutes.

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted / Plastic Surface

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling for Service

[Product serial number is required]


If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 am until 6:00 pm
Friday 8:00 an until 5:00 pm (ET)
midmark.com

Troubleshooting

No Foot Up Function

Function	Status	Problem	Correction
Foot Up		No movement ("beeps")	Push treatment pan in.
All other functions		OK	-

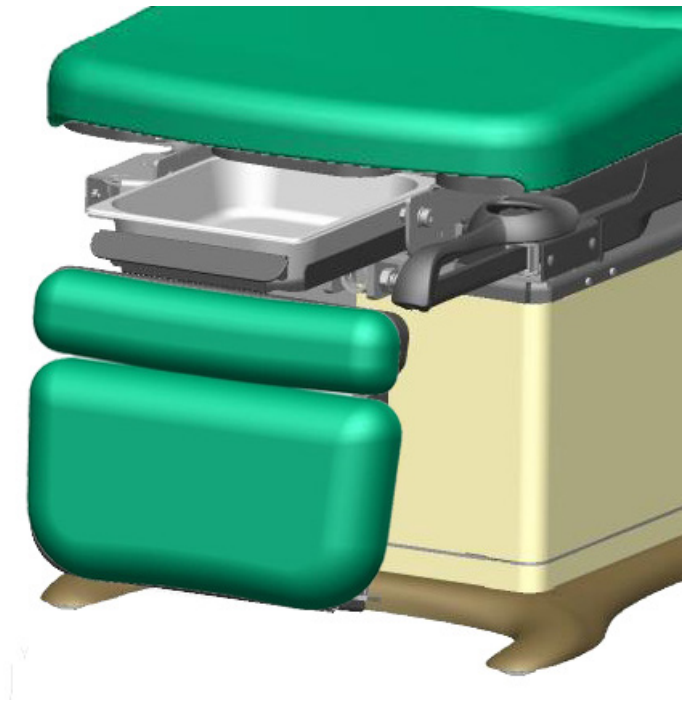Specifications

Weights, Dimensions, Electrical Specifications

Patient Weight (<i>maximum</i>):	450 lbs (204.1 kg)
Paper Roll (<i>maximum size</i>):	18 in. long x 3.5 in. diameter (45.7 x 8.9 cm)
Range of Motion & Dimensions:	[See <i>Range of Motion & Dimensions</i> page]
Weight of Chair: 230 (w/standard base) w/packaging & skid 230 (w/rotational base) w/packaging & skid	455 lbs (206.4 kg) 510 lbs (231.3 kg) 540 lbs (244.9 kg) 600 lbs (271.1 kg)
Power Cord:	8 ft. (243.8 cm) long
Electrical Requirements:	[See <i>Compliance Charts</i>]
Foot Control Voltage:	10 VAC, SELV (Safety Extra Low Voltage)
Duplex Receptacle (optional) maximum load:	115 VAC, 3A, 50/60 Hz
Fuses: <i>Located at power cord inlet:</i> <i>F1 & F2</i>	6.3 A, 250V, Type T, 5 x 20 mm
Duty Cycle (motor run time):	Intermittent Operation [30 seconds ON / 5 minutes OFF]
Classifications:	Class 1, Type B Applied Part
Protection against ingress of fluids: <i>Foot Control <u>only</u>:</i>	Ordinary Equipment IPX1
Regulatory Compliance:	[See <i>Compliance Chart</i>]

WARNING


Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.



Model Identification / Compliance Chart

Model	Description	Serial Number Prefix	Complies To:				Electrical Ratings:		
			UL 60601-1	CAN/CSA 22.2, #601.1-M90	IEC 60601-1	EN 60601-1-2 (EMC)	VAC +/-10%	Amps	Cycles (Hz)
230-001	Ritter Universal Proc. Chair (Non-Programmable)	V	X		X	X	Configurable 115 230	Configurable 7 3.5	50 / 60
230-002	Ritter Universal Proc. Chair (Non-Programmable) w/ Receptacle	V	X		X	X			
230-003	Ritter Universal Proc. Chair (Non-Programmable) w/ Rotational Base	V	X	X	X	X	Configurable 115 230	Configurable 7.5 3.8	50 / 60
230-010	Ritter Universal Proc. Chair (Non-Programmable)	V	X	X	X	X			

Range of Motion & Dimensions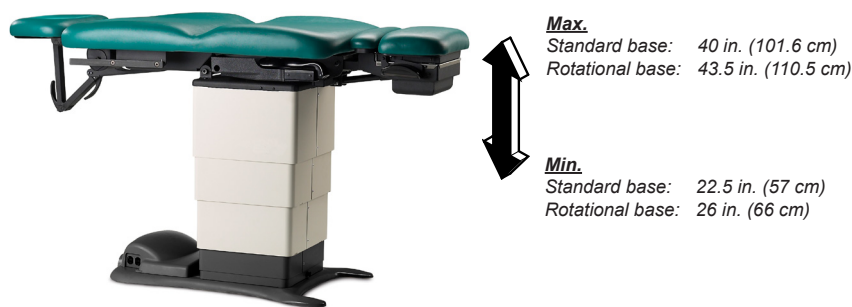

Range of Motion & Dimensions - (continued)

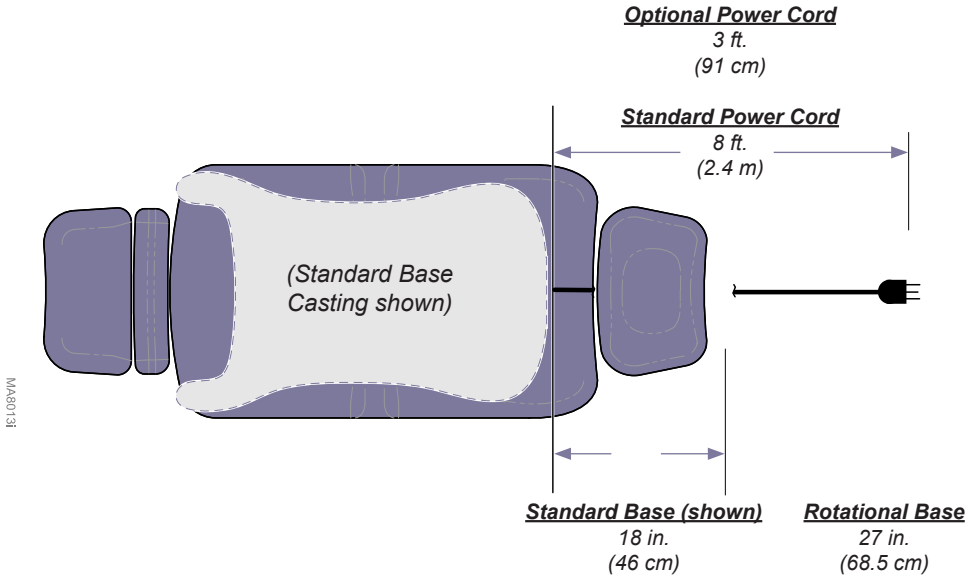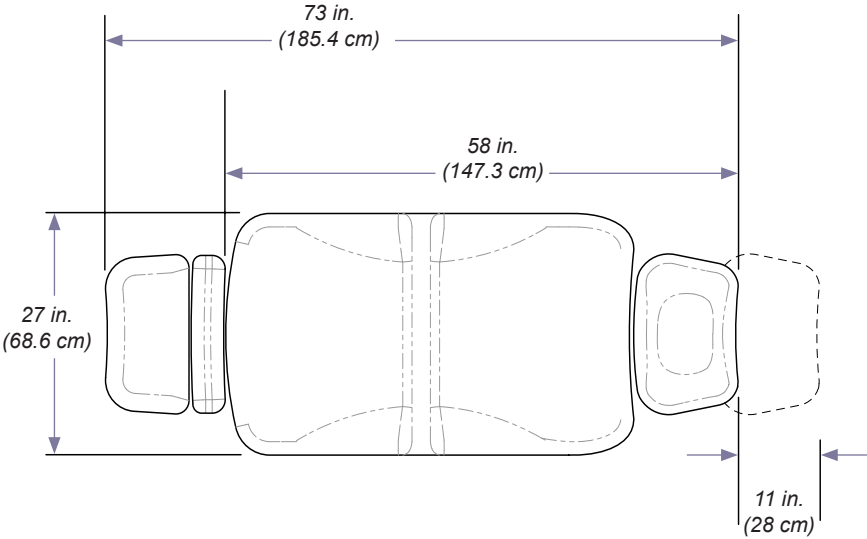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Midmark Corporation