

Casters [9A346]



Caution

Always disconnect power to table before performing any service procedures.



Caution

The table weighs approx. 340 lbs (154 kg). Use an assistant to perform this procedure.

Intended Use:

To provide mobility for Bariatric Power Treatment Table.

Note

Addition of Casters will raise the table no more than 5/8" (15.8 mm).

Contents:


Casters
(2.953" [75 mm] dia x 2.244 [57 mm] wide) 4
Washers 8

Special Tools:

none

Step 1: Remove leveling foot.


- Disconnect power to table.
- Securely prop up table.
- Raise base cover to access mounting screws.
- Remove two mounting screws.
- Remove leveling feet taking care not to loose spring and plunger. (early models only)

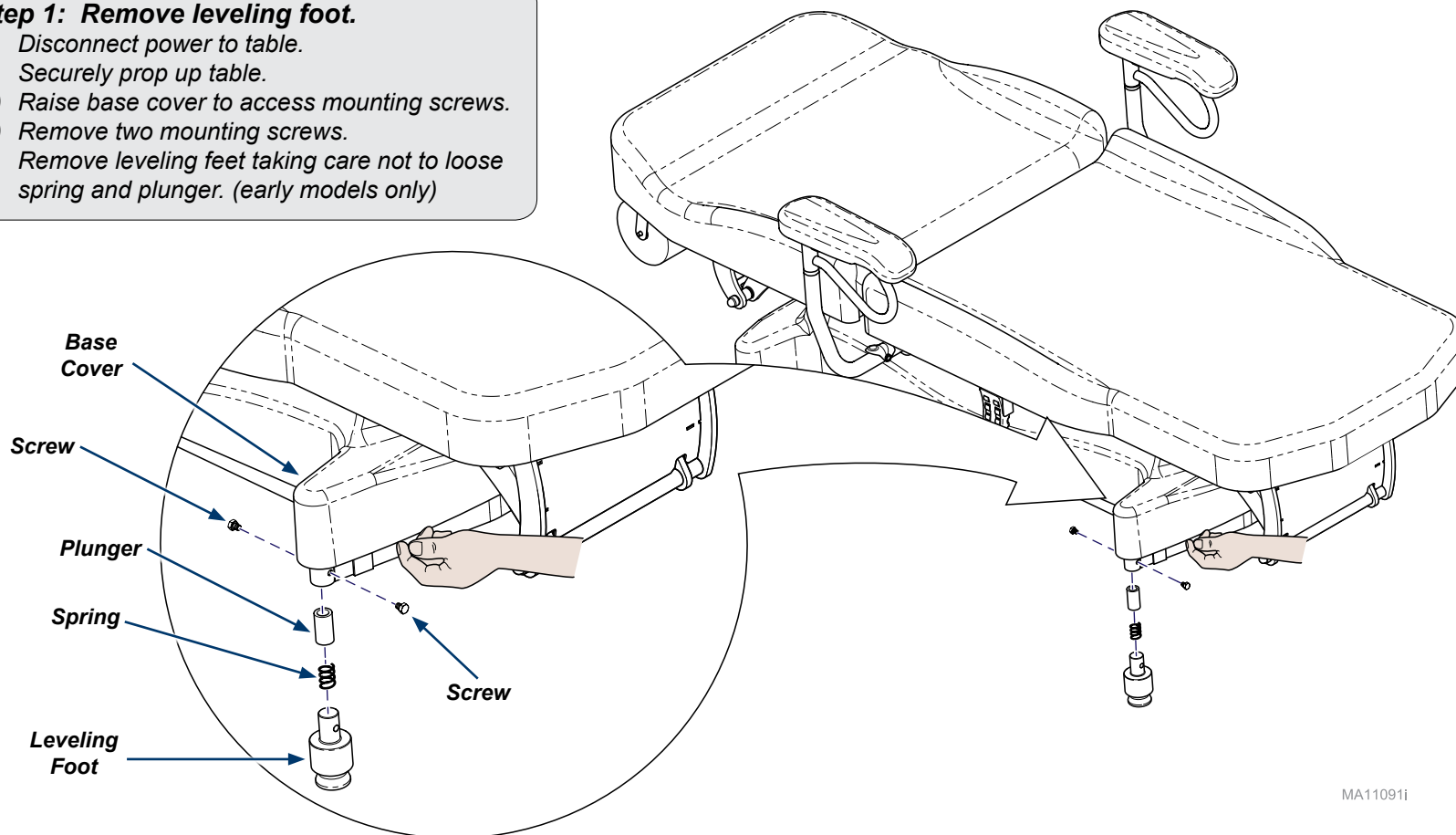

MA11091i

Step 2: Install Caster.

- A) Insert plunger and spring into table base.
- B) Install caster assembly.
- C) Secure caster with two washers and screws.
- D) Repeat steps for remaining casters.

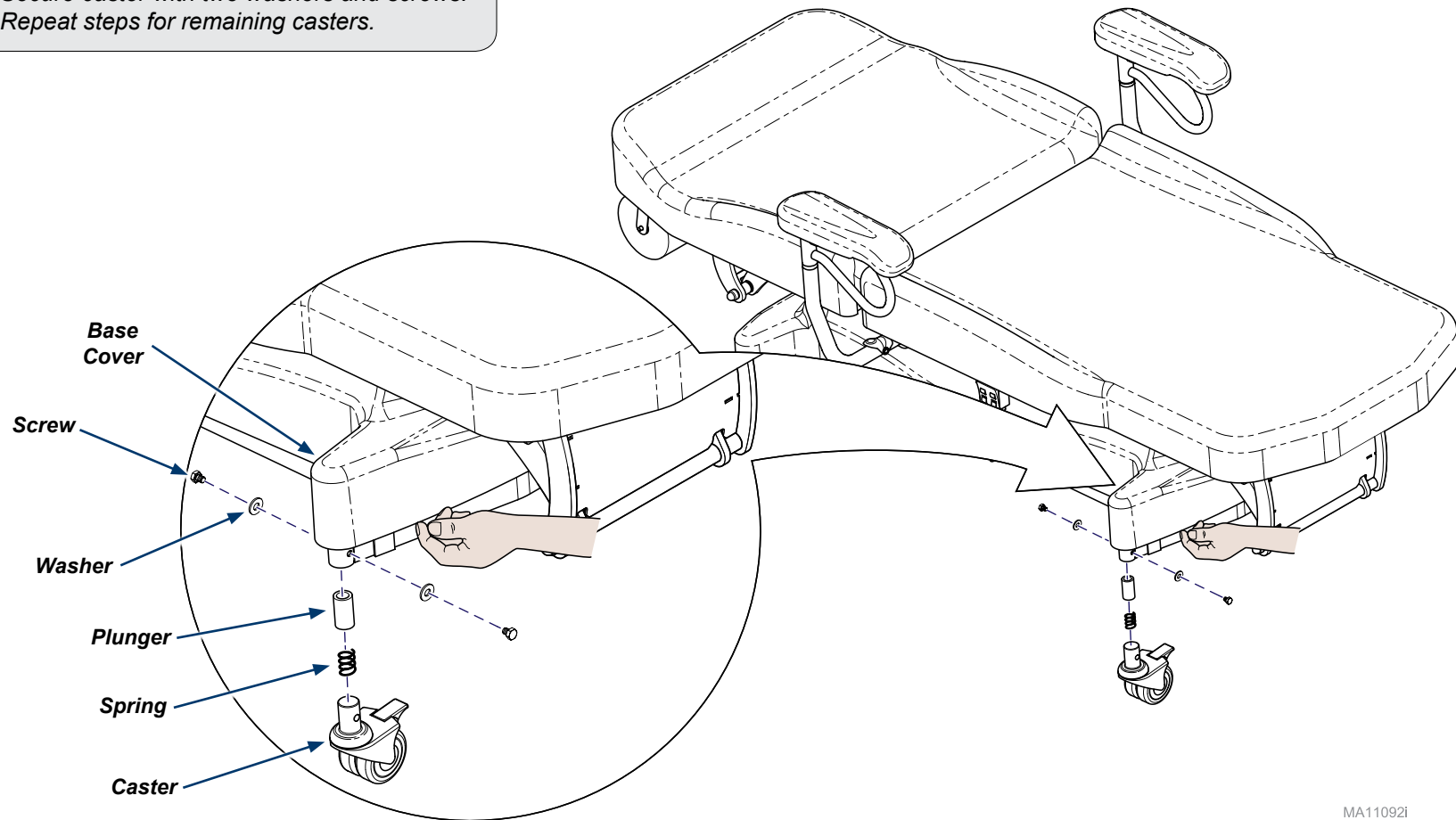

MA11092i