Manual Examination Table

[English](#)
[Español](#)
[Français](#)

For models:
604 (all)

MA706600i

Refer to compliance chart
in this manual.

User's Guide

Product Information

(The information below is required when calling for service)

Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company [name / phone]:

**Model / Serial
Number Location**

MA706500i

Table of Contents

Important Information

<u>Safety Symbols</u>	4
<u>Warnings</u>	5
<u>Intended Use</u>	5
<u>Electrical Requirements</u>	5
<u>Electromagnetic Interference</u>	6
<u>Disposal of Equipment</u>	6
<u>Authorized EU Representative</u>	6
<u>Transportation / Storage Conditions</u>	6

Operation

<u>Back Section Positioning</u>	7
<u>Stirrups</u>	8
<u>Footrest & Treatment Pan</u>	9
<u>Footstep</u>	9
<u>Paper Roll & Tear Strap</u>	10
<u>Drawer Heater (optional)</u>	11
<u>Pelvic Tilt (optional)</u>	12
<u>Receptacle (optional)</u>	13
<u>Upholstery Heater (optional)</u>	14
<u>Exam Assistant® Drawer System</u>	15
<u>Reversing the Side Drawers</u>	16

Maintenance

<u>Cleaning</u>	17
<u>Preventative Maintenance</u>	17
<u>Calling For Service</u>	17

Specifications

<u>Weights & Dimensions</u>	18
<u>Voltage, Current, Frequency Ratings & Compliance Chart</u>	19

Warranty Information

<u>Limited Warranty</u>	20
---	----

Important Information

Safety Symbols

Warning

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Proper shipping orientation

Fragile

Type B, Applied Part
(Upholstery)

Protective earth ground

Patient Weight Limit

Manufacturer

Serial Number

Keep dry

Maximum stacking height
(**palletted** units)

Fuse rating specification

Dangerous voltage / shock hazard

Refer to manual

Catalogue Number

Warnings

Warning

To avoid injury, do not put head or extremities under the paper tear strap.

Warning

Connecting equipment to the multiple socket outlet effectively leads to creating a Medical Electrical System and the result can be a reduced level of safety.

Warning

Table must be positioned in normal use such that the power cords can be easily accessed to unplug.

Warning

To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

Warning

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: *Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.*

Warning

No modification of this equipment is allowed.

Intended Use

This product is intended to be used as a table to provide positioning and support of patients during examination procedures conducted by qualified medical professionals in an office environment.

Electrical Requirements

Caution

To completely isolate the table from electrical mains supply, both power cords must be unplugged.

When using high frequency surgical devices or endocardial catheters:

- Use non-conductive material to insulate patient from metal portions of the table.*
- Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room.
- Plug chair into an isolated circuit.
- Increase separation between chair and interfering device.
- Contact Midmark if interference persists.

Disposal of Equipment

At the end of product life, the table, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized EU Representative

Countries in the EU should direct all questions, incidents, and complaints to Midmark's Authorized EU representative listed below.

Midmark EMEA Ltd
Beech House
First Floor, East Wing
Ancells Business Park
Fleet
Hampshire GU51 2UN
United Kingdom
Tel: + 44 (0) 1252 360 940
Fax: + 44 (0) 1252 360 941

Transportation / Storage Conditions

Ambient Temperature Range:.....-5°C to +38°C (+23°F to 100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Operation

Back Section Positioning


To position the back section...

- A) While holding the back section in place, squeeze the Back Release Handle (either side).
- B) Move the back section to desired position.
- C) Release handle to lock back section in position.

Back Release Handle

MA696700p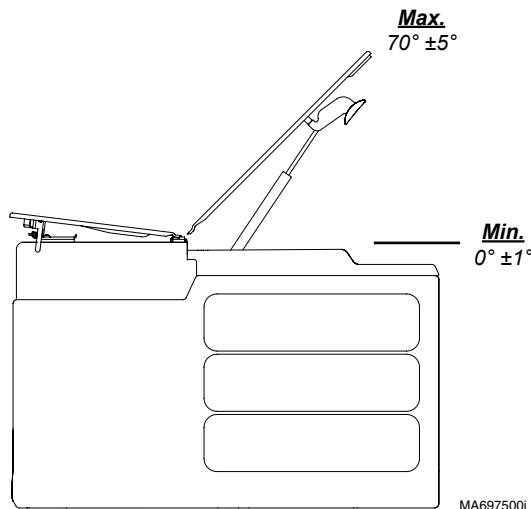

Stirrups


Caution

*Be sure the stirrups are locked in place before using.
The stirrups will not support the patient's entire weight.*

To position the stirrups...

- A) *Pull the stirrup out, then unfold.*
- B) *Lift the stirrup slightly, then move it left or right as desired.*
- C) *Release stirrup to lock in position.*

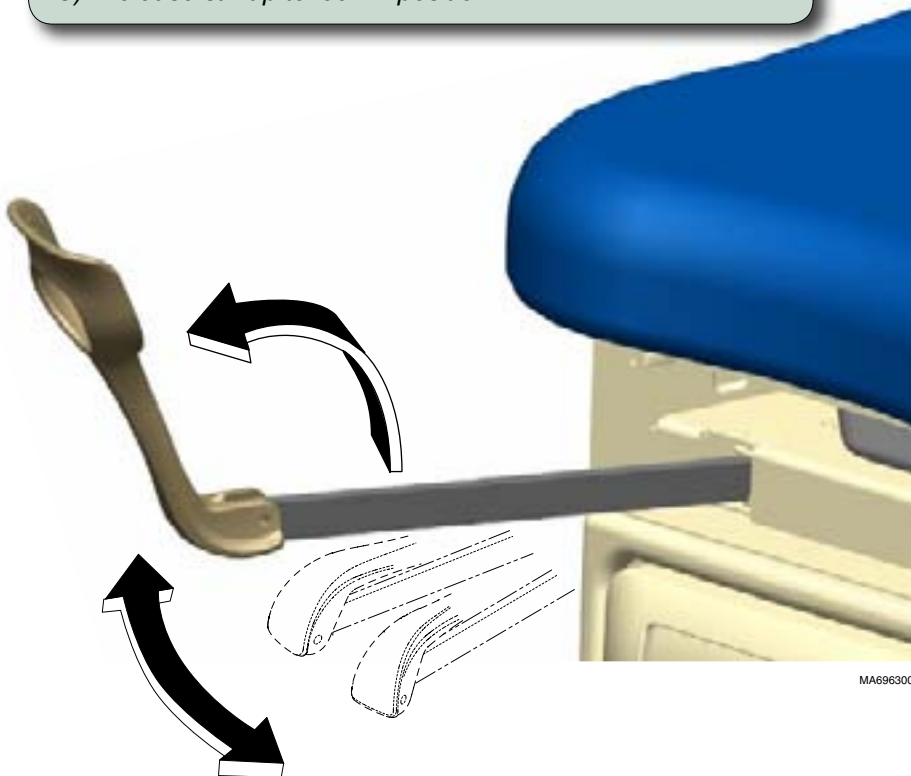Footrest & Treatment Pan

Caution

Do not use footrest to support patient's entire weight.

To access the treatment pan...

Lift footrest pad slightly, then slide pad back into table.

MA696401p


To extend footrest...

Pull footrest out to desired position.

Footstep

To access the footstep...

Lift footstep slightly, then pull outward.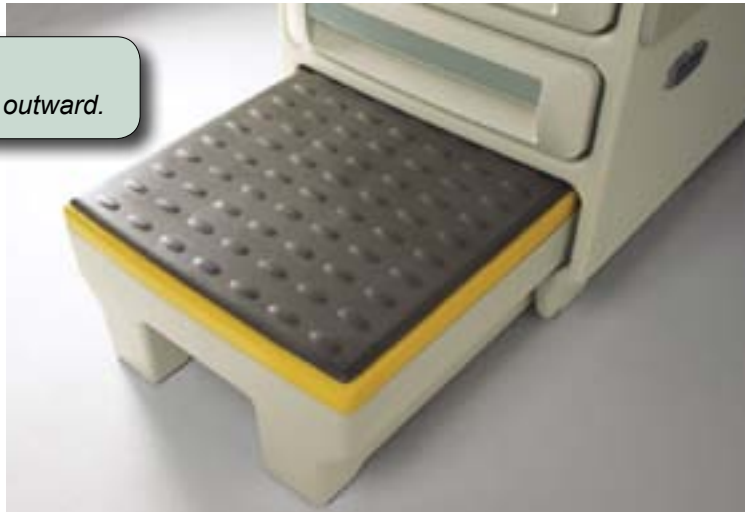

MA696600i

Paper Roll & Tear Strap


Note

Up to four paper rolls may be stored under the back section.

Paper Roll Size (max.):.....21 in. wide x 3.5 in. diameter
(53 cm x 9 cm)

To install the paper roll...

Insert rod into sockets under head end of upholstery.

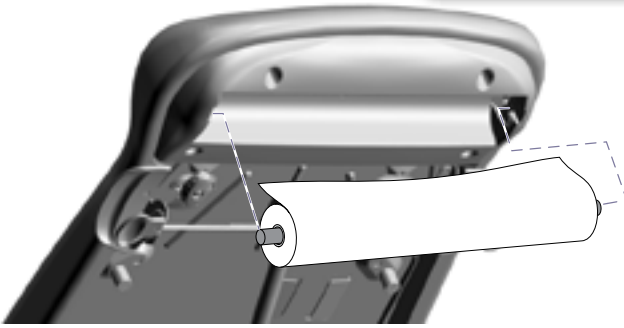To install the paper tear strap...

Attach strap to snaps on each side of table.

MA696800i

Drawer Heater (optional)

Note

The drawer heater switch illuminates when heater is ON (I).

The drawer heater warms the contents of the drawer to approximately 98°F (37°C).

***To turn the drawer heater ON/OFF..
Press the drawer heater button.***

Drawer Heater Button (white)

MA697000p

Pelvic Tilt (optional)

To elevate seat section...


Lift seat until pelvic support drops into place.

To return to flat position...

- Lift seat section slightly.
- Pivot trigger handle upward.
- Lower seat section.

MA702300i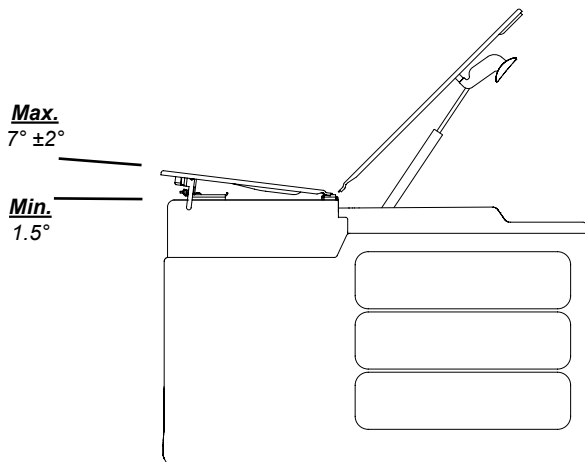



MA697500i

Receptacle (optional)

The duplex receptacle on the side of the table, provides power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacle's maximum load is exceeded, the circuit breakers interrupt power to the receptacle.

Maximum Load 5 amps

To reset circuit breaker(s)...
Push the circuit breaker toggle switch.

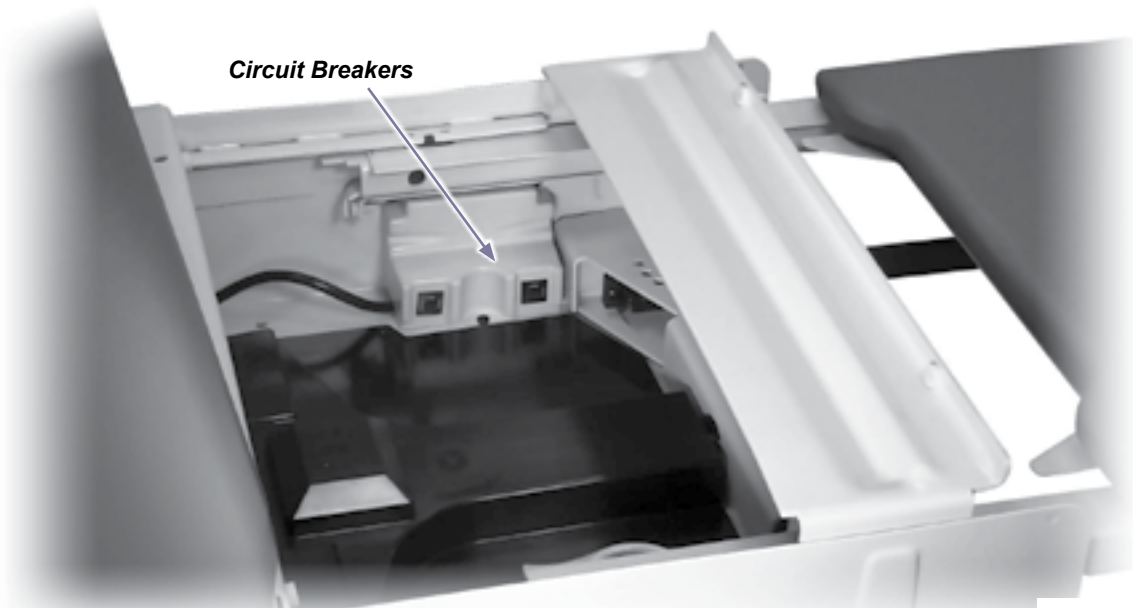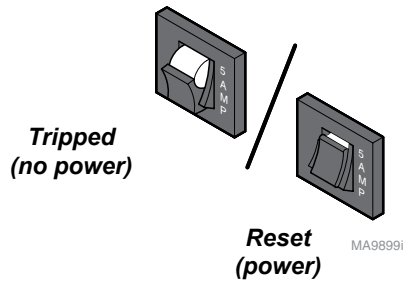MA9894p

Upholstery Heater (optional)

To turn upholstery heater ON / OFF...

Press the upholstery heater button.

Note: The upholstery heater will ...

- reach operating temp. in 15 - 20 minutes.
- automatically shut off after 8 hours.

Note

The upholstery heater button illuminates when heater is ON (I).

Upholstery Heater Button (green)

MA697000p

Equipment Alert

The maximum power loading of the heating element is 45 W, and should only be powered by the connection provided.

If the heated upholstery is punctured, it must be repaired or replaced to ensure proper operation.

Exam Assistant® Drawer System

To use the drawer front as a shelf...

Pull drawer front down to release magnetic latch.

If drawer front folds past horizontal...

Pull drawer front upward until it snaps back into the horizontal position.

MA7045001


Equipment Alert

The maximum weight the shelf will support is 10 lbs (4.5 kgs).

Reversing the Side Drawers

The side drawers can be reversed so that they open from the opposite side of the table.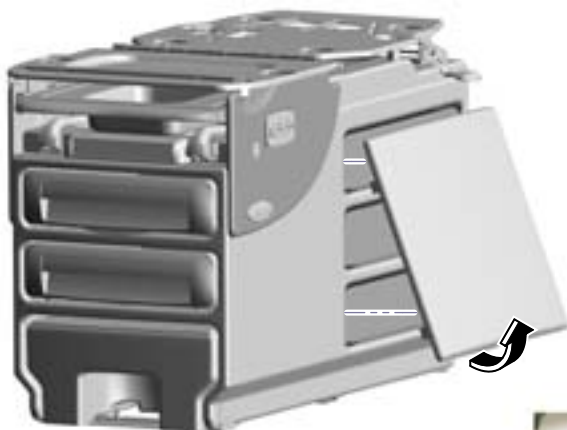

Step 1: Remove side panel.
(Pull panel out from the bottom)

Step 2: Remove drawer fronts.
(Pull tabs down to release)

Tabs

Step 3: Install drawer fronts on opposite side of table.

Step 4: Install side panel on opposite side of table.
(Align panel, then push straight in)

MA705400i

Maintenance

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Preventative Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your table every six months.

Calling For Service

If service is required, contact your authorized Midmark dealer.

To contact Midmark directly:


1-800-MIDMARK (1-800-643-6275)

8:00 am until 5:00 pm Monday - Friday (EST) [excluding standard U.S. holidays]

Specifications

Weights & Dimensions

Maximum Patient Weight:	500 lbs (226 kg)
Paper Roll (maximum size):	(See Paper Roll & Tear Straps page)
Range of Motion & Dimensions:	(See illustration below)
Weight of Table:	220 lbs (99.8 kg)
Power Cord:	6 ft. (1.8 m) long

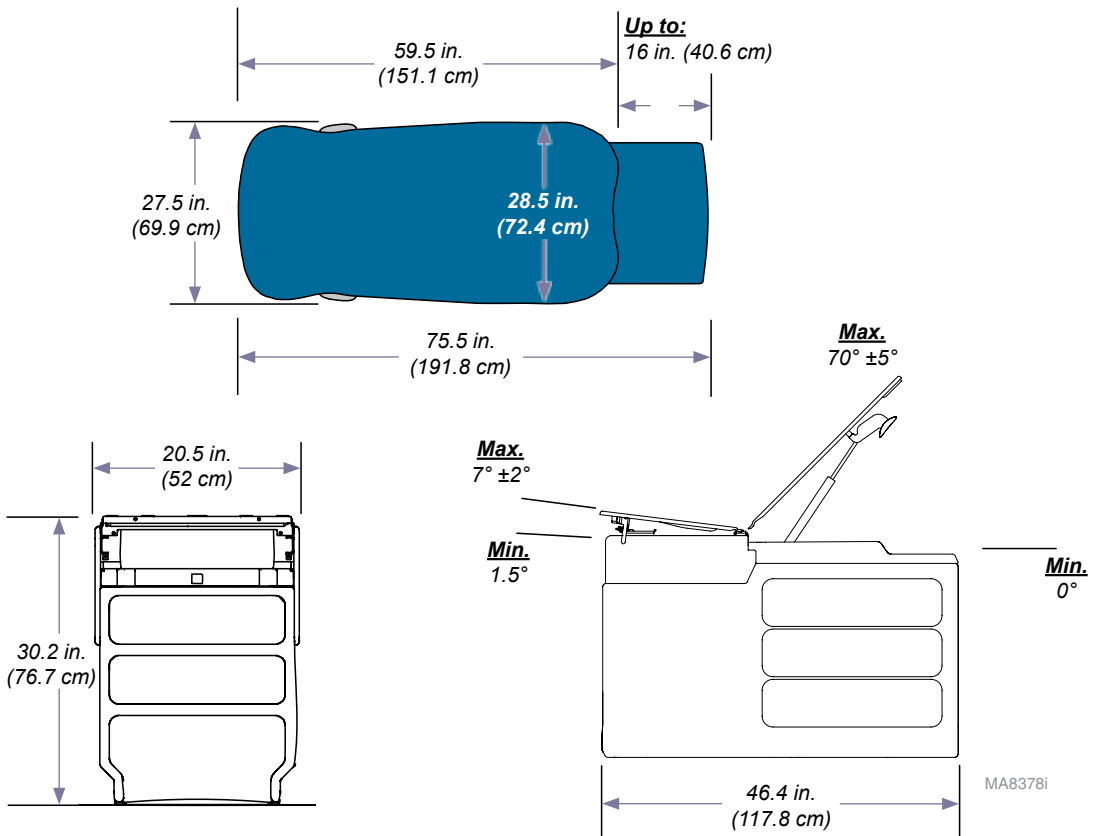Specifications *(continued)*

Voltage, Current, Frequency Ratings & Compliance Chart

Model	Description	Electrical Ratings:					
		ES 60601-1	CAN / CSA C22.2, 60601-1	CE	VAC +/- 10%	Amps	Cycles (Hz)
604-001	Midmark Exam Table w/ Receptacle	•	•		115	5	50/60
604-002	Midmark Exam Table w/ Receptacle, Pelvic Tilt, & Drawer Heater	•	•		115	5.5	50/60
604-003	Midmark Exam Table w/ Receptacle, Pelvic Tilt, Drawer Heater, & Heated Upholstery	•	•		115	6.5	50/60
604-004	Midmark Exam Table - Export w/ Pelvic Tilt	•	•	•	-	-	-
604-005	Midmark Exam Table - Export w/ Receptacle & Pelvic Tilt	•	•	•	230	5	50/60

Warranty Information

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com; by phone at 1-800-MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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