


## Upholstery Installation (Ritter)

### Applies to Models:

204 (all)  
222 (-015 thru -017)  
223 (-015 & -016)

Language of origin: English

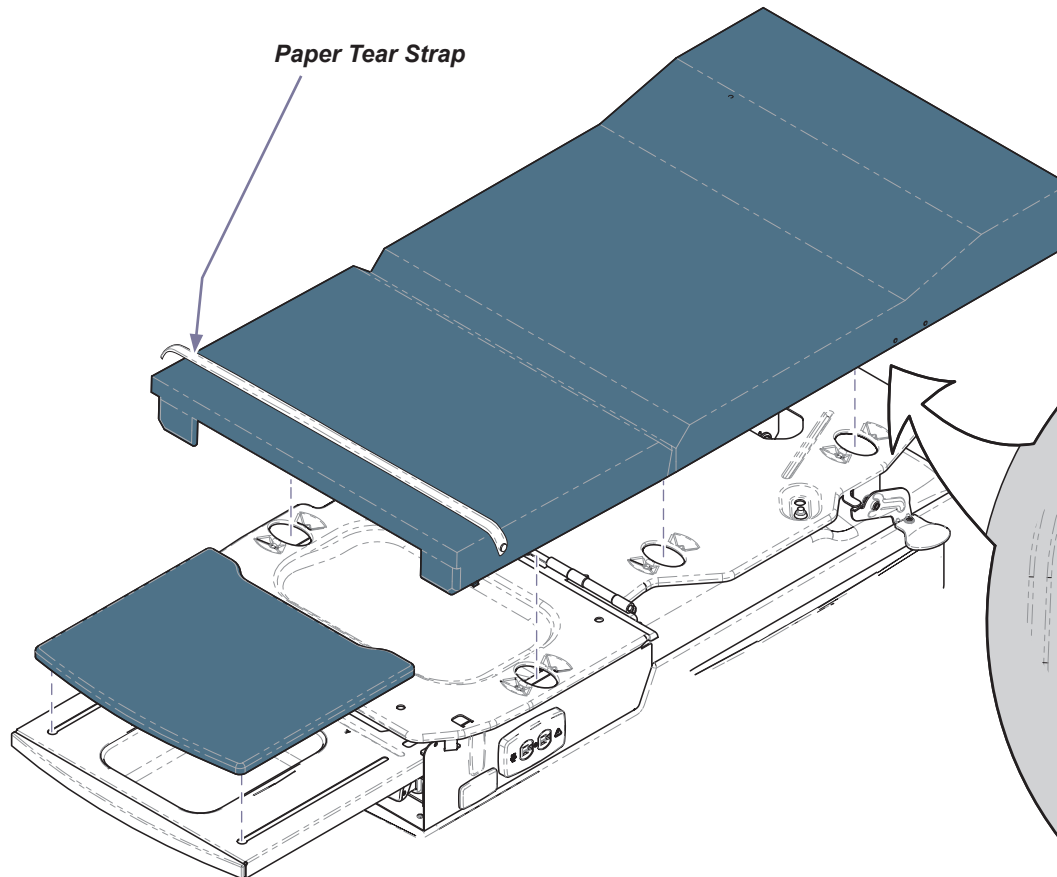


### Equipment Alert

Be sure paper roll rod has been removed from the seat / back hinge before installing upholstery top.


### To install seat / back upholstery...

- Position upholstery so that locking knobs align with holes in mounting plates.
- Turn all locking knobs to **LOCKED** position.

### Upholstery Locking Knobs

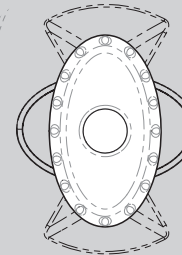

**Locked**

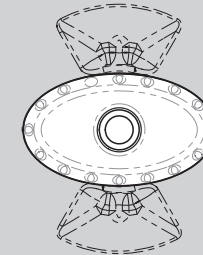

**Unlocked**

### To install footrest pad...


- Extend footrest shelf.
- Place footrest glides in shelf slots.

MA8096i

**To install paper roll rod...**

- A) Slide paper roll onto rod.
- B) Insert one end of rod into socket.
- C) Insert other end of rod into opposite socket.

Note: The "Soft Touch"-style upholstery is shown.  
The procedure for other upholstery styles is similar.

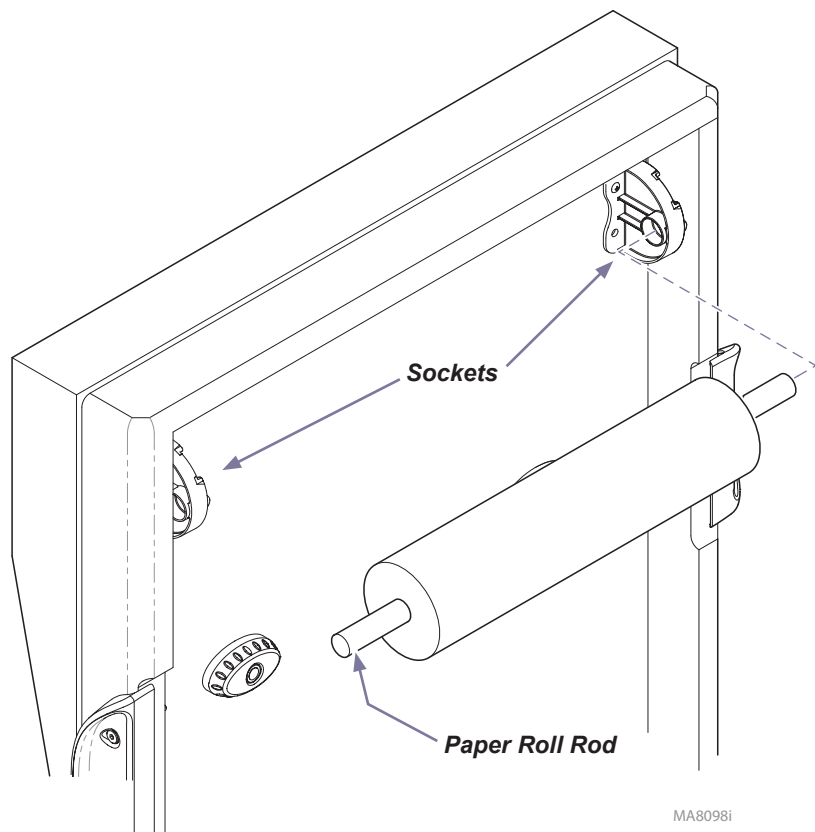

**204 Models only:**

**To install color inserts onto drawer fronts...**

- A) Insert one end of color insert into slot on drawer front.
- B) Bend insert, then slide other end into opposite slot.

