[English](#)
[Español](#)
[Français](#)

For models:
120 VAC: 152647-01, 153102-01
240 VAC: 153039-01, 152723-01, 153335-01

Contents

Important Information

Intended Use 1
Serial Number Location 1
Power To The Chair 1
Safety Symbols 2
Electromagnetic Interference 2
Certifications 2

Operation / Adjustments

Touch Pad & Chair Operation 3
Foot Control Operation 4
Base Programming Procedure 5
Back Programming Procedure 5
Rotating The Chair 6
Headrest Adjustments 6
Swing Out Armrest 7

Cleaning / Maintenance

Disinfecting External Surfaces 8
Upholstery (Cleaning & Removal) 9
Preventive Maintenance 9
Calling For Service 9

Specifications


Weights, Dimensions, Electrical 10
Fuse Characteristics 10
Chair Dimensions 11
Warranty 11

Important Information

Intended Use

To provide patient support and positioning during dental exams.


Model / Serial Number Location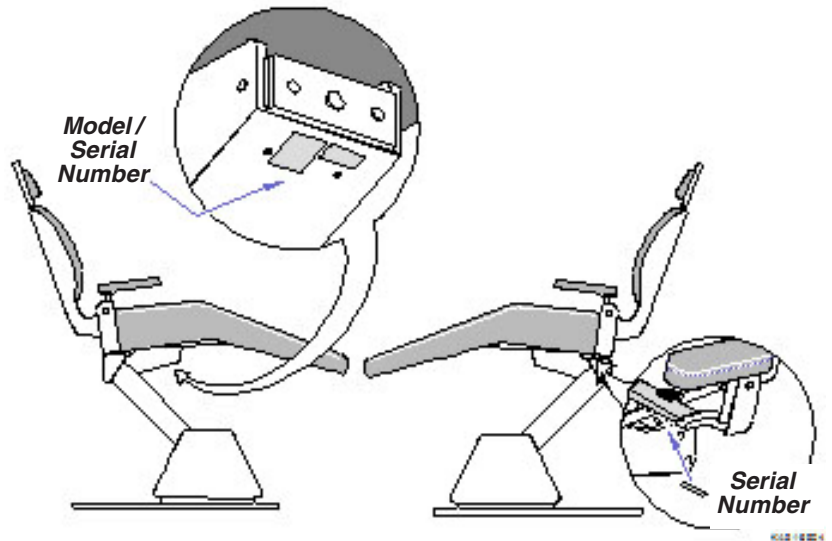

Power To The Chair

If the chair has no power ...

- A. Check / reset facility circuit breaker.
- B. Check Primary Line Fuse. Replace with same size fuse if necessary.
- C. Contact your dealer for service if necessary.

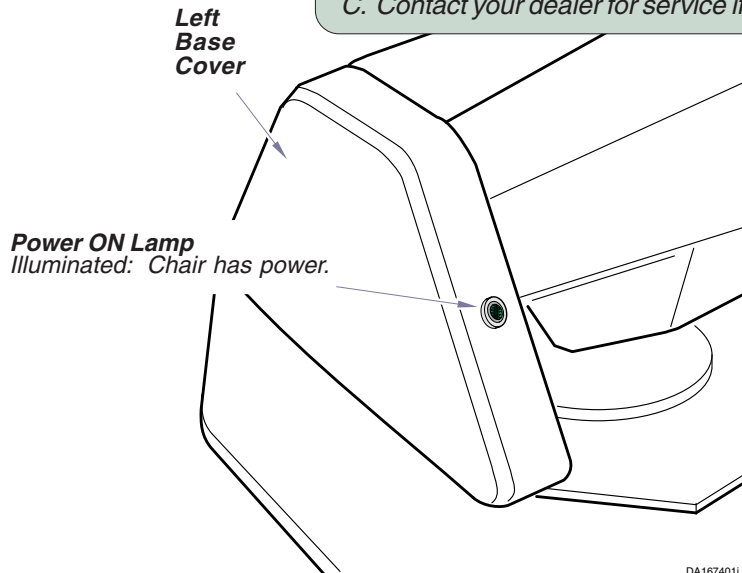DA167401i

IMPORTANT INFORMATION

Safety Symbols

CAUTION

Indicates a potentially hazardous situation which could result in injury if not avoided.

EQUIPMENT ALERT

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Safety Instructions

CAUTION

Equipment not suitable for use in the presence of a flammable anesthetic mixture.

Assure chair is attached to a dedicated circuit suited for the electrical specifications of the chair.

Do not use the chair if the supply cord is frayed or worn.


Ground reliability can only be achieved if this equipment is connected to an equivalent receptacle marked "Hospital Grade".

Electromagnetic Interference

This Midmark table is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this table:

- Remove interfering device from room
- Plug table into an isolated circuit
- Increase separation between table and interfering device

Certifications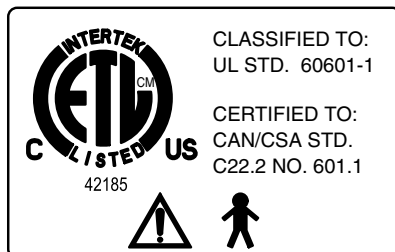

For Knight Dental Chair Model:


| | | |
|-----------|----------------------|---------|
| 152192-01 | ENT Chair | 120 VAC |
| 152647-01 | Biltmore Classic | 120 VAC |
| 152375-01 | Ophthalmic Chair | 120 VAC |
| 153102-01 | Biltmore Classic L/R | 120 VAC |

MA3922CS


TOUCH PAD & CHAIR OPERATION

Touch Pad Location

The chair position touch pads allow you to position your patient from either side of the chair. Additional touch pads are available as options on delivery units and casework.

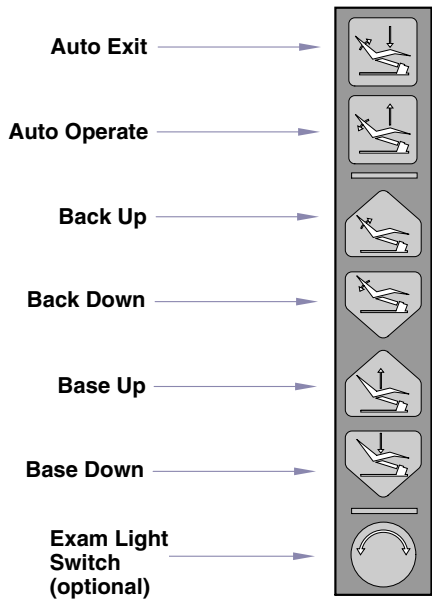
Note
The Automatic Functions may be stopped during operation by depressing any switch on the touch pad or operating the foot control.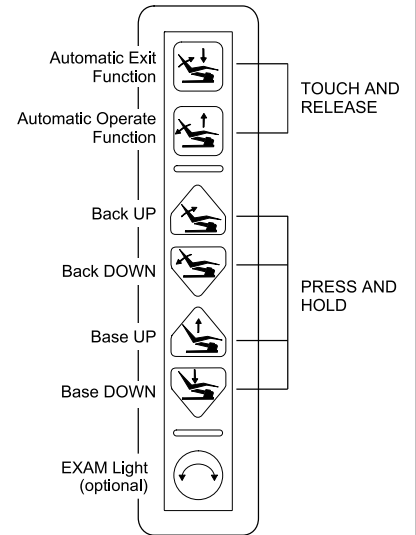

Rotation Lock Lever
(Biltmore Classic Only)


Foot Switch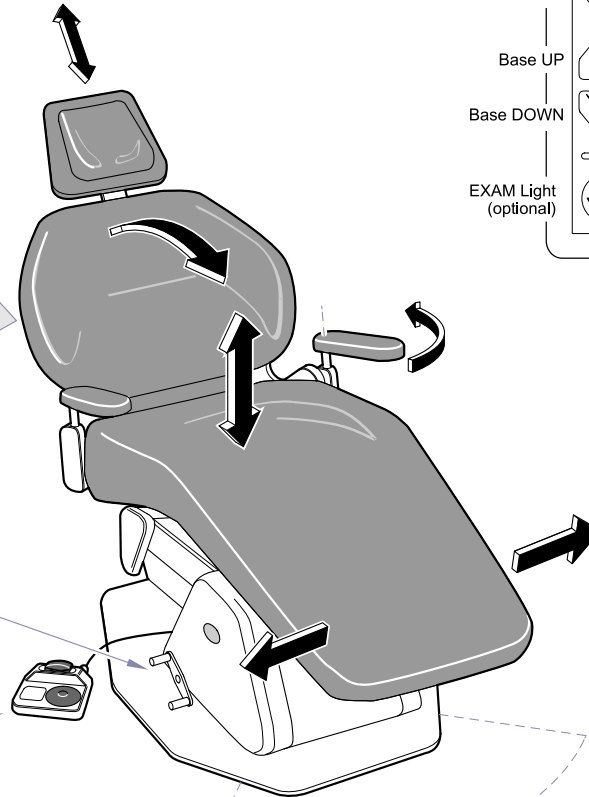

90°

(Biltmore Classic Only)

KA917002i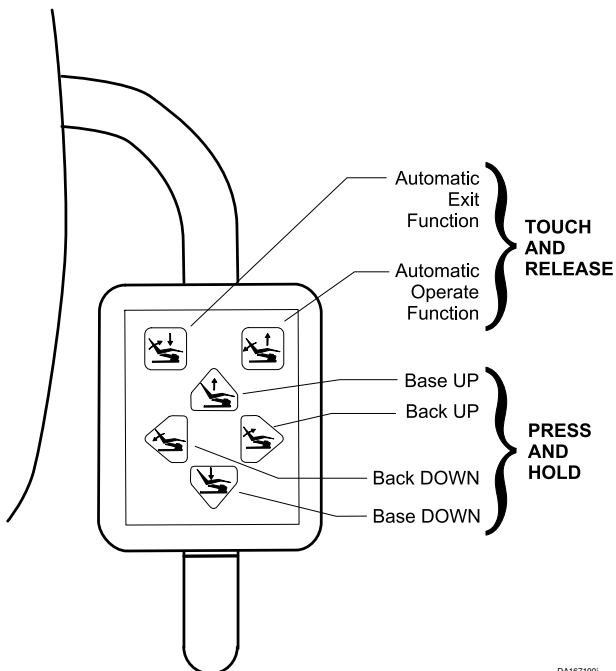

Optional Unit Mounted Chair Control

Allows Operator to control positions of chair from the unit as well as by the foot control and chair back controls.

DA167100i

Foot Control Operation

Note

The Automatic Functions may be stopped during operation by depressing any switch on the touch pad or operating the foot control.

Automatic Operate

Momentarily pressing and releasing to the Green position activates the Pre-programmed Automatic Positioning.

Manual Operation

Place Selector in Neutral (Middle) position and Operate Control Disk to position chair manually.

Selector Switch


Toggles from its Neutral (Manual) position to

- Automatic Operate (Green)
- or
- Automatic Exit (Yellow) positions

Automatic Exit

Momentarily pressing and releasing to the Yellow position activates the Pre-programmed Exit Positioning.

Illustration Of Disk Functions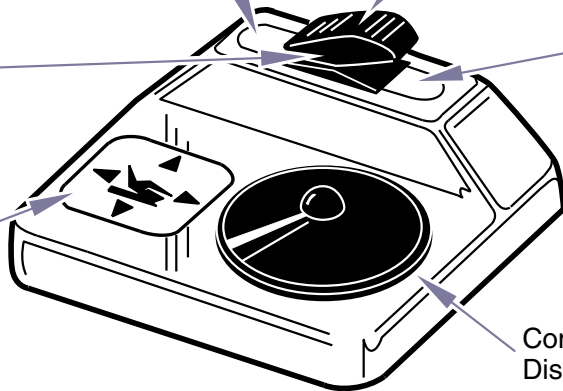


DA167500i

Manual, Two Position Operation

Place Selector in Neutral (Middle) position.

To activate two functions at once, slide the Control Disk "Diagonally" between the two positions.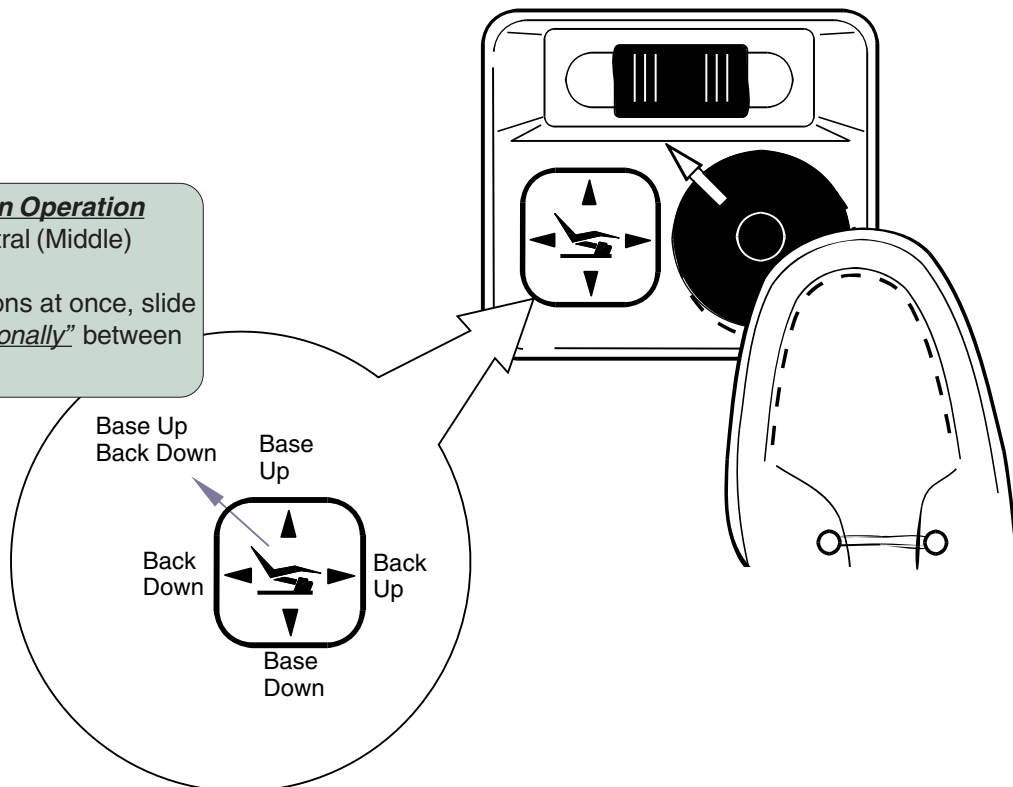


DA167600i

Operation / Adjustments

Base Programming Procedure.

The Automatic Operate function positions can be adjusted.


The Automatic Exit function positions have been preset at the factory and cannot be adjusted.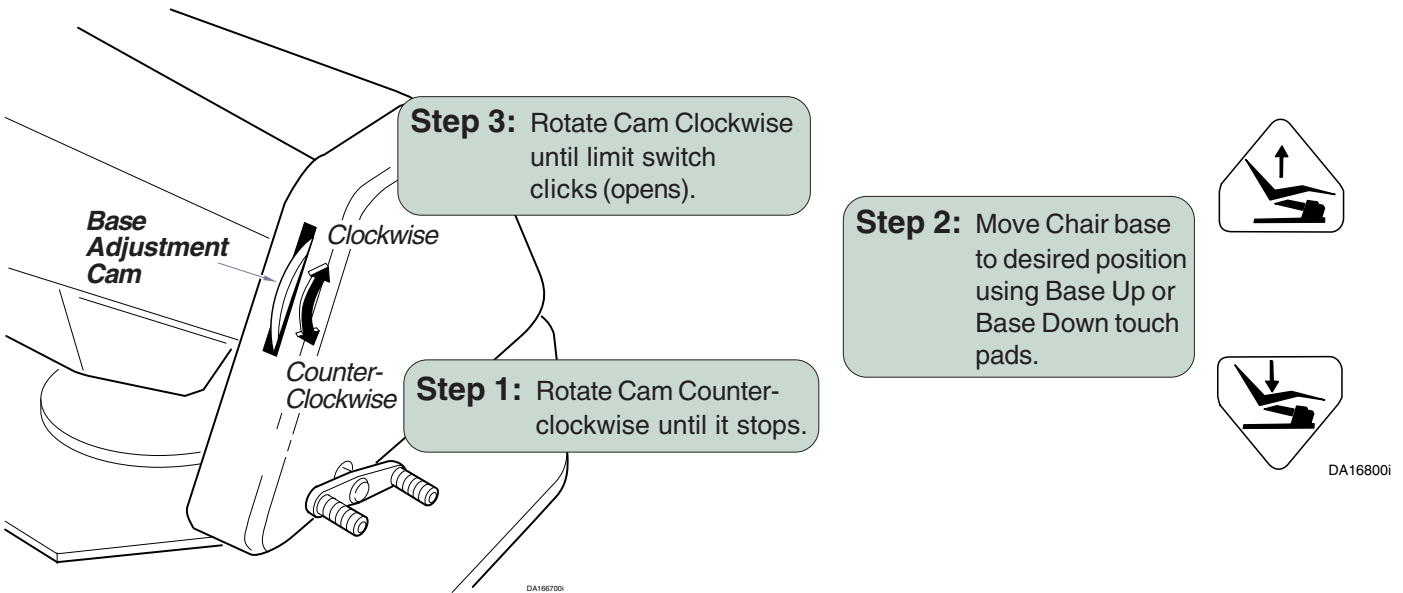

Back Programming Procedure.

The Automatic Operate function positions can be adjusted.


The Automatic Exit function positions have been preset at the factory and cannot be adjusted.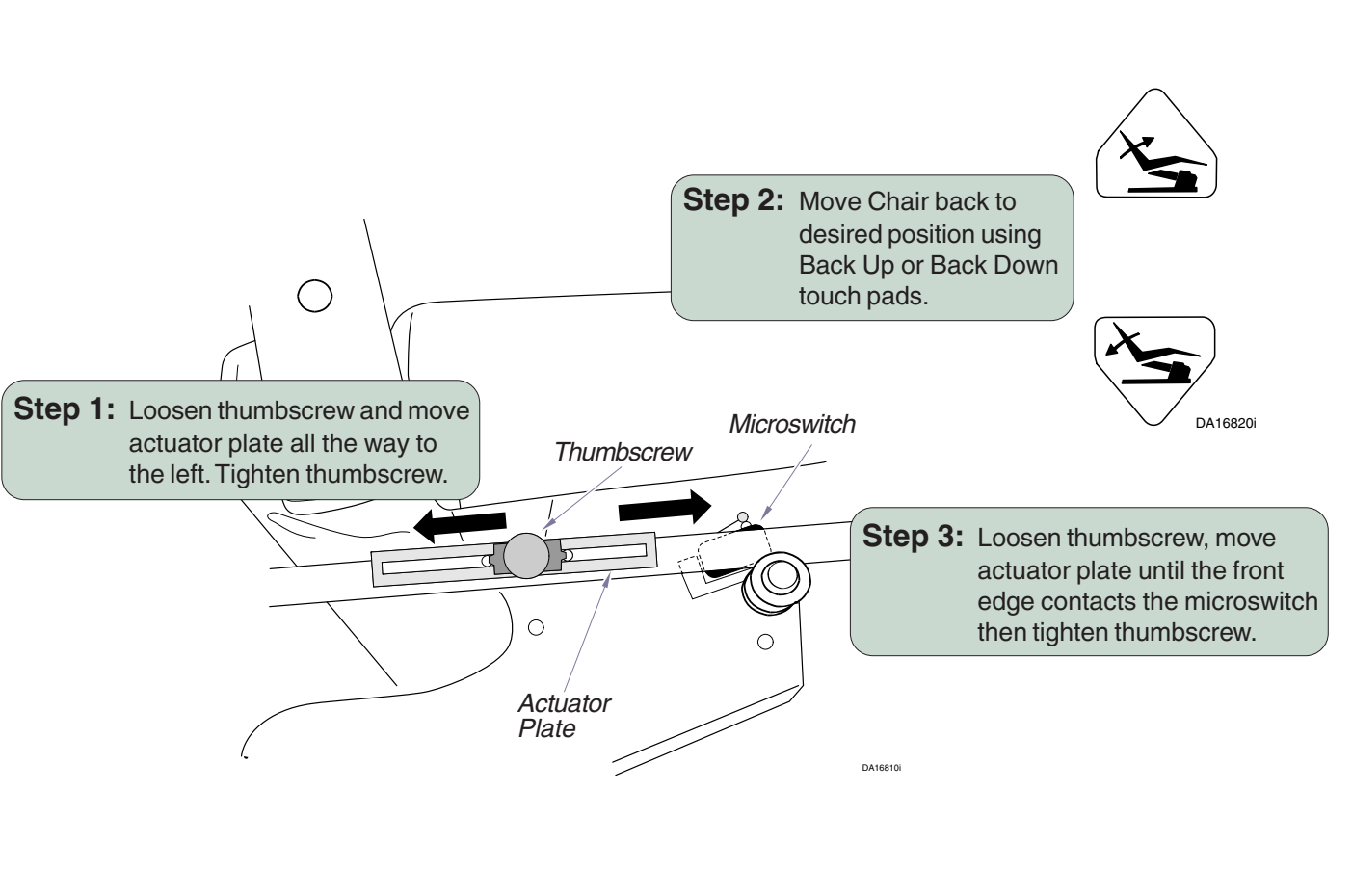


Operation / Adjustments

Rotation Base.

The chair will rotate at the base a total of 90° (45° on either side of center line).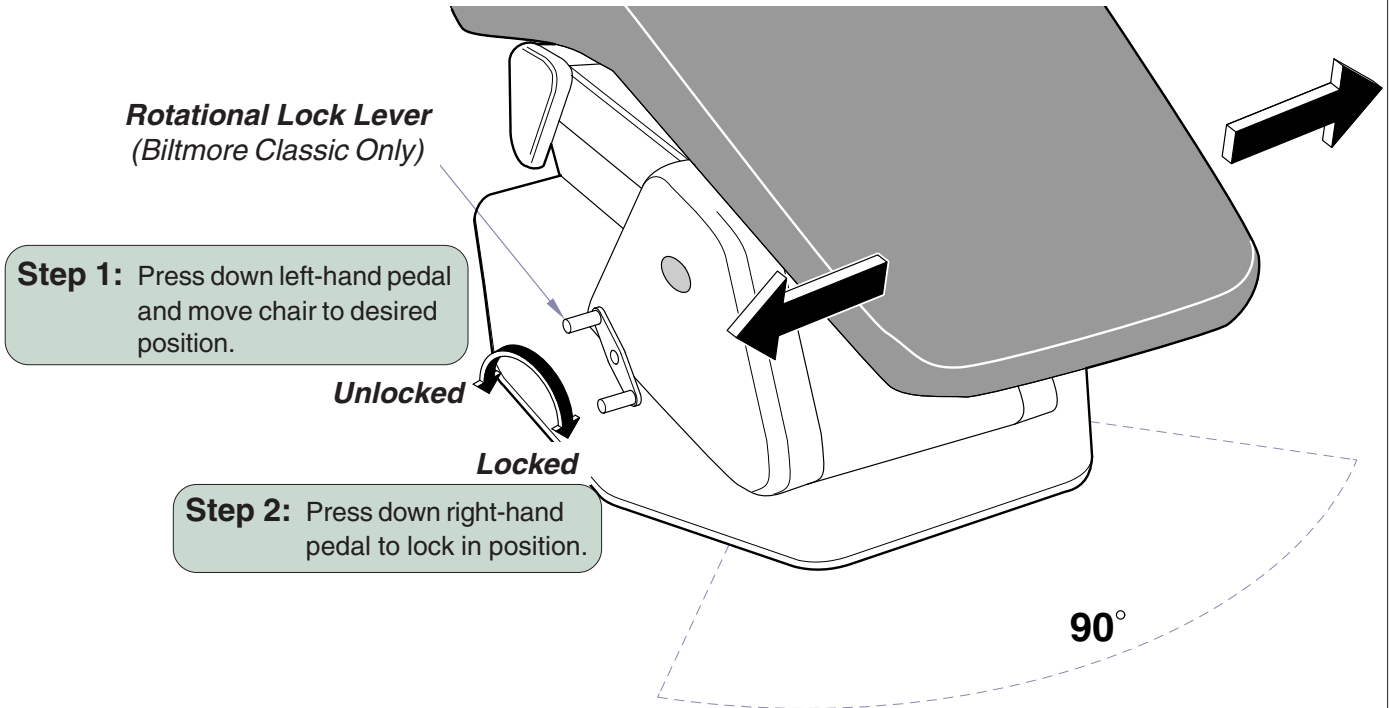


Headrest Adjustments

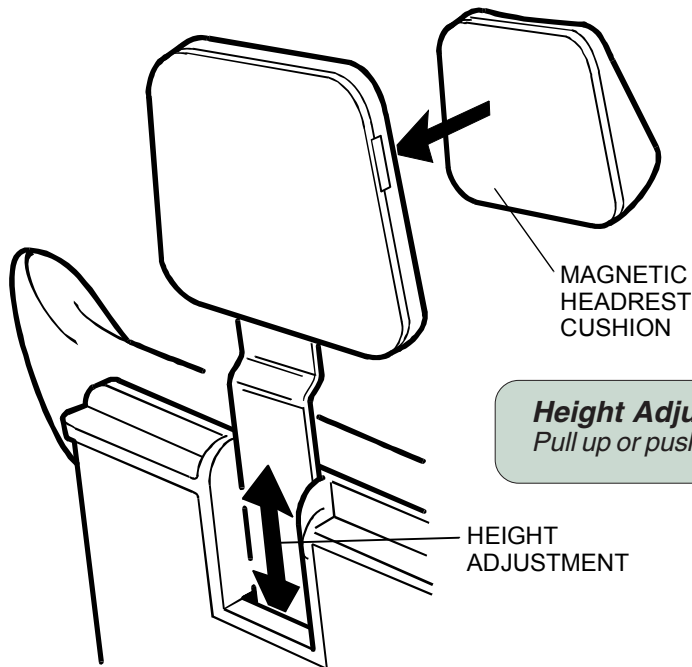Operation / Adjustments

Swing Out Armrest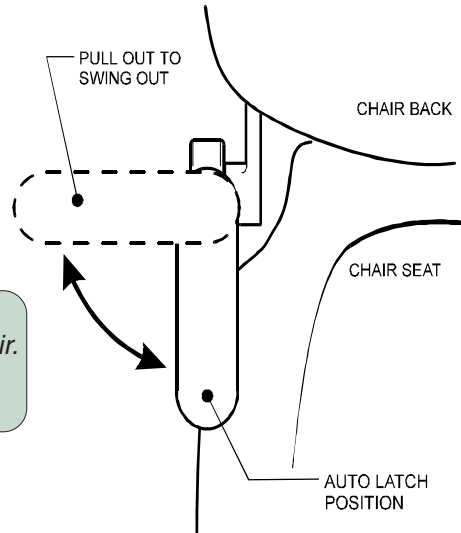


Swing Out Armrest

- A. Grasp armrest and pull out, away from chair.
- B. Have patient be seated.
- C. Push armrest back until it latches.

Note

It is recommended to disable (lock) the swing out feature on the side of the chair opposite patient access.

Left Armrest Swing Out Position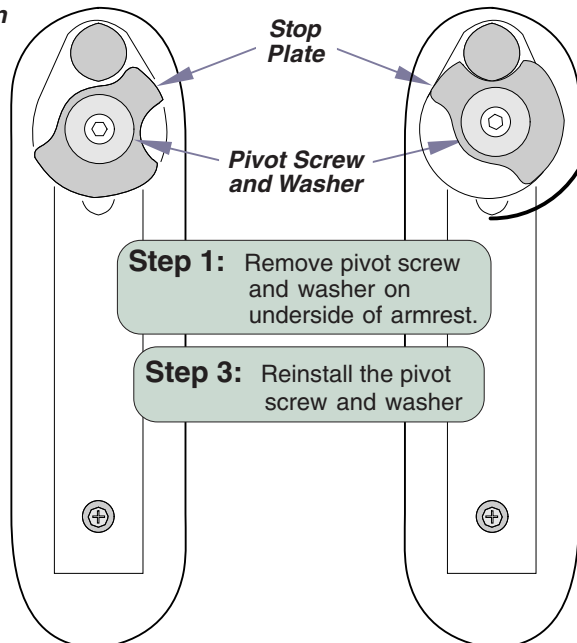

Left Armrest Locked Position

Step 2: Rotate stop plate 90°

Step 1: Remove pivot screw and washer on underside of armrest.

Step 3: Reinstall the pivot screw and washer


DA169700i

Cleaning / Maintenance

Disinfecting Procedures for External Surfaces

Attention!

Midmark assumes no responsibility or liability for any result, expressed or implied. These are suggested practices, based on the best information available at this time.

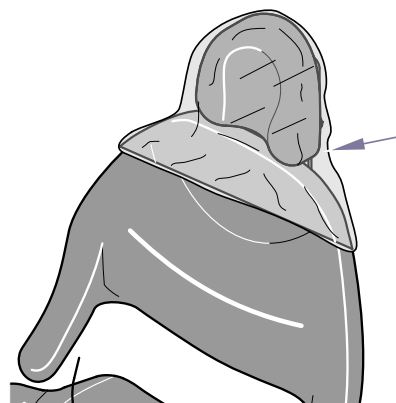DA177500i

Barriers:

Note

The use of single-use barriers and disposable items will significantly reduce the frequency and need for chemical usage, thus prolonging the life of the equipment. Barrier material must be impervious (does not allow penetration or passage) to moisture / fluids.

Some examples:

- Plastic covers available at your dealer, or equipment manufacturer.
- Clear plastic wrap
- Plastic bags
- Plastic sheets
- Plastic tubing
- Plastic-backed paper
- Similar materials

At the beginning of each workday...
Use barrier protection to cover any surfaces that must be protected from contamination.

Note: Barriers should always be changed between patients.

WARNING

If a surface has been contaminated, it will need to be cleaned and disinfected using proper techniques, and EPA-registered hospital disinfectants.

Chemical Disinfectant:

Note

Every dental practice setting is different and no one disinfectant is the best choice for every facility. Due to this fact there are several organizations, educated in disinfection procedures, with web sites that can assist dental personnel in choosing what is best for their practice.

Listed below are several links to some of those sites.

Organization for Safety & Asepsis Procedures

<http://www.osap.org/index.cfm>


American Dental Association

<http://www.ada.org/prof/resources/topics/icontrol/index.asp>

Dept. of Health & Human Resources

Centers for Disease Control & Prevention (CDC)

<http://www.cdc.gov/OralHealth/infectioncontrol/index.htm>

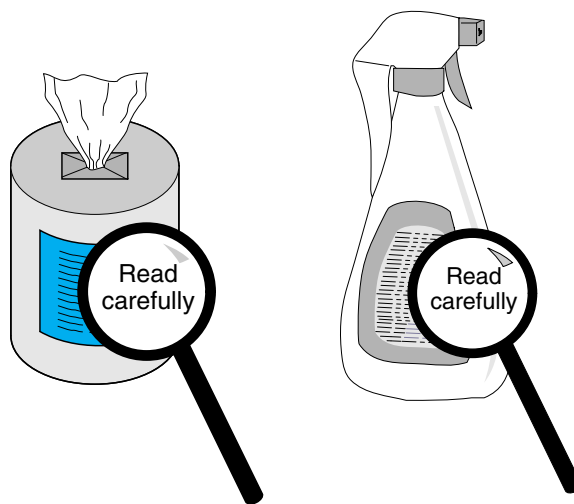DA177400i

DA148100

When using a disinfectant:

- Carefully read the product label and directions for use.
- Do not exceed the dilution rate.

Equipment Alert

No materials available for the manufacture of dental equipment are impervious to every chemical. Midmark does incorporate the most chemical-resistant materials available in our dental line of equipment.

Use of barriers are the most effective means to prevent equipment damage.

Cleaning / Maintenance

Upholstery (Cleaning & Removal)

After following the recommended disinfection procedures outlined by the listed organizations on the previous page wipe off the remaining disinfectant residue from the upholstery. Use a mild soap and water solution for this procedure.

Spot cleaning use mild soap and water

Difficult stains use a mild solvent such as Fantastik® and 409® brand cleaners

[Allow upholstery to air dry, or use a hair dryer on the lowest setting]

Preventive Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Calling For Service [Product serial number is required]

If service is required, contact your Midmark dealer or to locate your nearest service provider, log onto:

http://www.midmark.com/?url=trnd_srv_providers.asp

To contact Midmark directly:

1-800-Midmark (1-800-643-6275) or 937-526-3662

8:00 am until 5:00 pm. Monday through Friday (EST)

[excluding standard U.S. holidays]

Specifications

Weights, Dimensions, Electrical Specifications

Patient Weight (Maximum) 325 lbs (147.4 kgs)

Weight of Chair:

Without Shipping Carton 355 lbs (161.0 kg)

With Packaging & Skid 367 lbs (166.5 kg)

Electrical Requirements: [See Chart]

Fuses: [See Chart]

Protection against ingress of fluids: Ordinary Equipment (IPX1)

Fire Code Ratings: All Midmark Standard Upholstery sets comply with:
California Bureau of Home Furnishing Tech. Bulletin 117.

Classifications Class 1, Type B Applied Part, Ordinary Equipment

Duty Cycle: Intermittent Operation, On 30 secs.- Off 5 min.

Certifications:

Midmark Corporation ISO 9001 Certified

Models 152192-01, 152647-01, 152375-01, ETL Classified, conforms to UL STD. 60601-1
153102-01, 153085-01 Certified to CAN/ CSA STD. C22.2 NO. 601.1

Models 152267-01, 152723-01, Conforms to EN60601-1, CE
153039-01, 153335-01

Fuse Characteristics


Caution

Only people qualified in electrical service should attempt to replace the fuse(s).
Always disconnect power to the chair before removing any component or panel.

| Model Number | Input Rating (+/- 10% V) | Fuse Characteristics | | | |
|--|-------------------------------|-------------------------|-------------------------|-------------------------|-------------------------|
| | | PCB Transformer (F1) | Motor Fuses (F2, F3) | Exam Light (F5) | Main's Fuses (F6, F7) |
| 152647-01 153102-01 152192-01 152375-01 | 120 VAC 50/60 Hz 8 Amps | Type T 125mA 250V | Type T 4 Amp 250V | Type T 315mA 250V | Type T 8 Amp 250V |
| 152267-01 152723-01 153039-01 153335-01 | 240 VAC 50/60 Hz 4 Amps | Type T 50mA 250V | Type T 2 Amp 250V | Type T 160mA 250V | Type T 4 Amp 250V |

Specifications

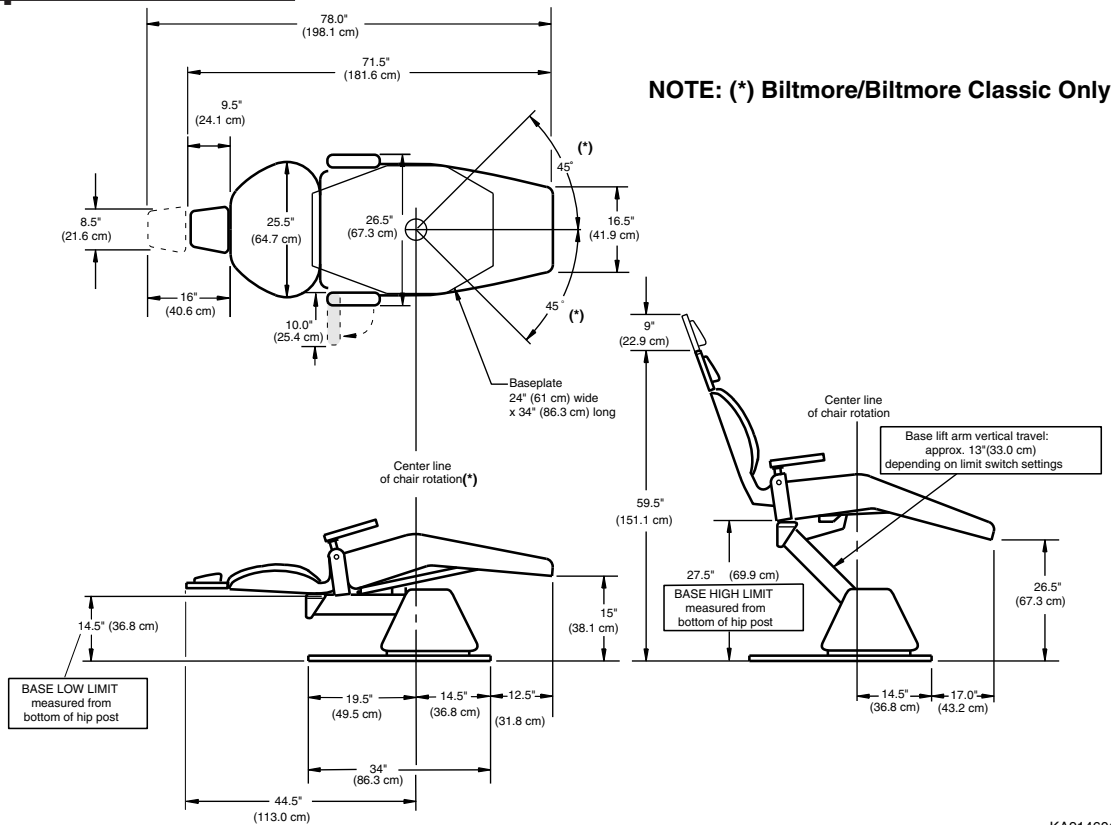KA914601I

Warranty

WARRANTY

Knight by Midmark products are warranted against defect in material and workmanship for a period of 2 years, unless otherwise indicated in writing, from the time of delivery.

Exceptions:

- 1) "KINK-VALVE" module carries a 10 year warranty.
- 2) The original light bulb on a new light carries a 1 year warranty.
- 3) Accessories not manufactured by Knight are excluded (i.e. Fiber optic systems, Scalers, Electric micromotor systems, Curing light systems, X-ray viewers, etc.)
- 4) Replacement parts and accessories carry a 90 day warranty.

Knight's sole obligation under the warranty is either to provide parts for the repair or to provide the replacement product (excluding labor and shipping). The buyer shall have no other remedy. All special, incidental and coincidental damage is excluded. Written notice of breach of warranty must be given to Knight within the warranty period.

The warranty does not cover damage resulting from improper installation or maintenance, accident or misuse. The warranty does not cover damage resulting from the use of cleaning, disinfecting or sterilization chemicals and processes.

No other warranties as to merchantability, fitness for use or otherwise are made.

Midmark Corporation
60 Vista Drive
P.O. Box 286
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
www.midmark.com

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Midmark Corporation Certification ISO 9001