

Language of origin: English



Equipment Alert

Failure to follow these instructions may result in damage to the table.

Applies to Models

222

223

623




Caution

Heavy object. Can cause muscle strain or back injury. Use lifting aids and proper lifting techniques when moving.

NEVER push on base covers.


NEVER use stirrups to lift table.

NEVER use upholstered top to lift table.

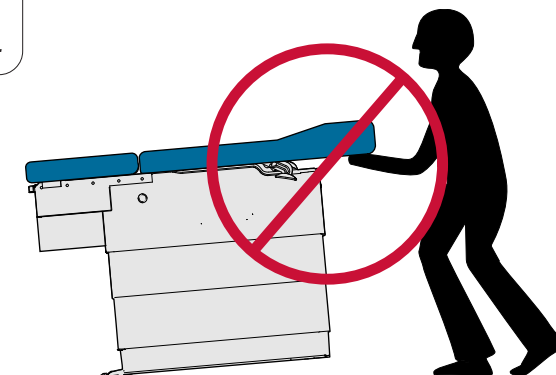

ALWAYS transport table in “full-down” position.

ALWAYS leave (4) base cover supports in place until final delivery.

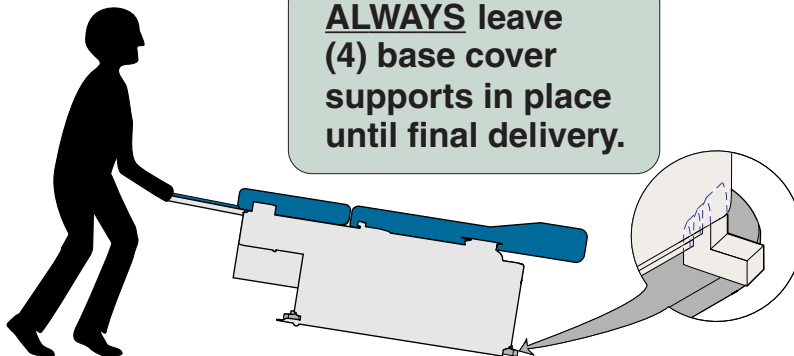


Note

Upholstery installation instructions are packaged with upholstered top.


Step 1: REMOVE two lag screws going thru shipping straps securing the table to the skid. Slide shipping straps towards the center of the table allowing them to fall free from slots in the table baseplate.

Lag Screw

Lag Screw

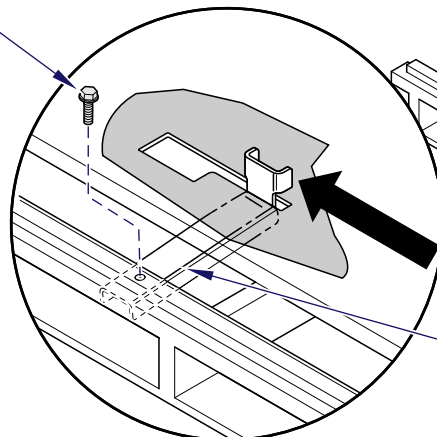

Shipping Strap

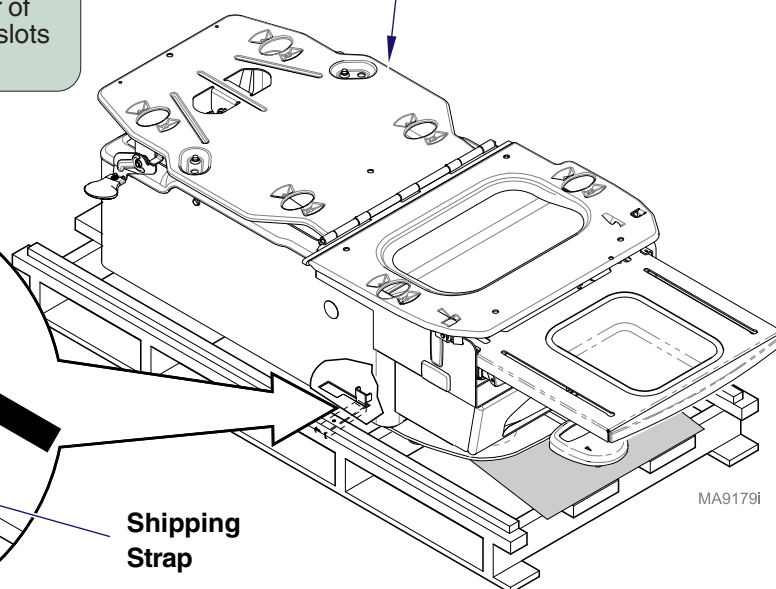


MA9179i

**Equipment Alert**

The table must be leveled to ensure proper operation.

Step 2: Adjust leveling screws at head end of table as necessary

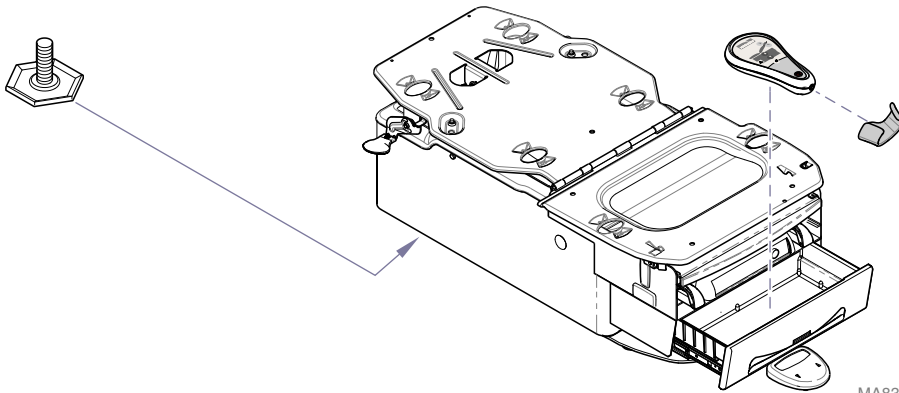

MA8331i



623 only

Step 3: Remove hand control & hand control holster from drawer.


Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

Step 4: Connect power cord.

Note: Models with the optional receptacle have two power cords. Both cords must be connected for proper operation.

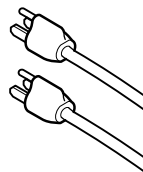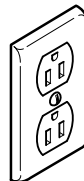


The power cord on models w/o receptacle must be connected to the table as shown.

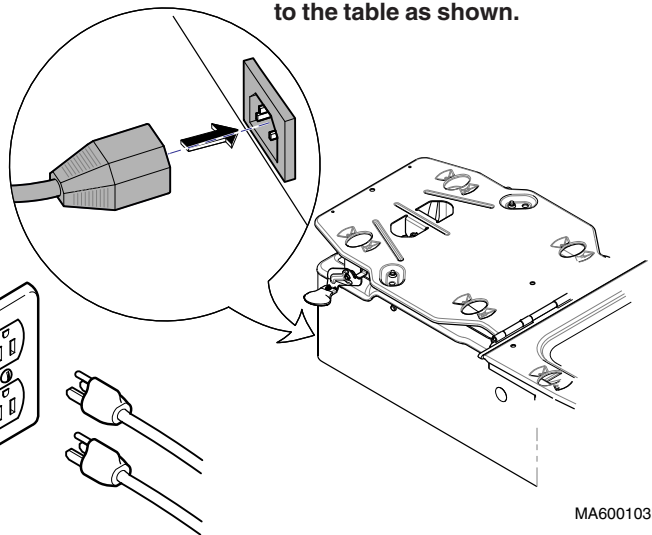

MA600103i

223/623 only

Step 5: Connect foot / hand control.
223: foot end.
623: foot or head end

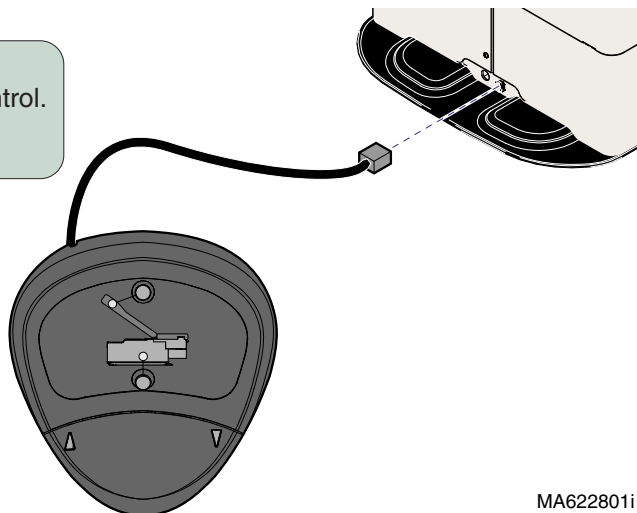

MA622801i

Midmark Corporation
60 Vista Drive
Versailles, OH 45380 USA
1-800-643-6275
1-937-526-3662
www.midmark.com