222 / 223

Barrier-Free® Examination Table

Ritter®
by MIDMARK

[English](#)

[Español](#)

[Français](#)

CE

Refer to compliance chart
in this manual.

User's Guide

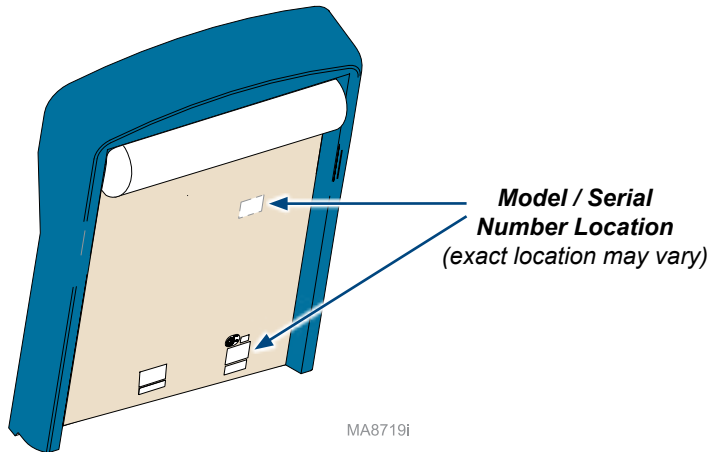
Product Information

Dealer :

Date of Purchase:

Model / Serial Number:

**Midmark Authorized Service
Company:**

Product Registration

To register your product warranty go to www.Midmark.com.

Table of Contents

Important Information

<u>Safety Symbols</u>	4
<u>Warnings</u>	5
<u>Intended Use</u>	5
<u>Electrical Requirements</u>	5
<u>Electromagnetic Interference</u>	6
<u>Disposal of Equipment</u>	6
<u>Authorized EU Representative</u>	6
<u>Transportation / Storage / Operating Conditions</u>	6
<u>Safety Instructions</u>	7

Operation

<u>Table Up / Down Operation</u>	8
<u>Back Up / Down Operation</u>	
<u>222 Model</u>	9
<u>223 Model</u>	10
<u>Exam Assistant® Drawer System</u>	11
<u>Paper Roll & Tear Strap</u>	12
<u>Stirrups</u>	13
<u>Footrest & Treatment Pan</u>	14
<u>Drawer Heater (optional)</u>	14
<u>Pelvic Tilt</u>	15
<u>Duplex Receptacle</u>	16

Maintenance

<u>Cleaning</u>	17
<u>Preventative Maintenance</u>	17
<u>Calling For Service</u>	17

Specifications

<u>Weights & Electrical Specifications</u>	18
<u>Model Identification / Compliance Chart</u>	19
<u>Range of Motion & Dimensions</u>	20

Warranty Information

<u>Limited Warranty</u>	22
---	----

Important Information

Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Duty Cycle (motor run time)
Domestic: 30 seconds ON, 5 minutes OFF
Export: 25 seconds ON, 7 minutes OFF

No Pushing

Proper shipping orientation

Keep dry

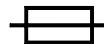
Fragile

Maximum stacking height
(palletted units)

Type B, Applied Part
(Upholstery)

Fuse rating specification

Protective earth ground

Dangerous voltage / shock hazard

Patient Weight Limit

Refer to manual

Warnings

Warning

To avoid injury, do not put head or extremities under the paper tear strap.

Warning

Connecting equipment to the multiple socket outlet effectively leads to creating a Medical Electrical System and the result can be a reduced level of safety.

Warning

Table must be positioned in normal use such that the power cords can be easily accessed to unplug.

Warning

To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Warning

No modification of this equipment is allowed.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during general examinations and procedures conducted by qualified medical professionals in an office environment.

Electrical Requirements

Caution

To completely isolate the table from electrical mains supply, both power cords must be unplugged.

When using high frequency surgical devices or endocardial catheters:

- Use non-conductive material to insulate patient from metal portions of the table.*
- Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room.
- Plug chair into an isolated circuit.
- Increase separation between chair and interfering device.
- Contact Midmark if interference persists.

Disposal of Equipment

At the end of product life, the table, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized EU Representative

Countries in the EU should direct all questions, incidents, and complaints to Midmark's Authorized EU representative listed below.

Midmark EMEA Ltd
Beech House
First Floor, East Wing
Ancells Business Park
Fleet
Hampshire GU51 2UN
United Kingdom
Tel: + 44 (0) 1252 360 940
Fax: + 44 (0) 1252 360 941

Transportation / Storage Conditions

Ambient Temperature Range:.....+5°C to +38°C (+41°F to 100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to 104°F)
Relative Humidity.....30% to 75% (non-condensing)
Atmospheric Pressure700hPa to 1060hPa (0.69atm to 1.05atm)

Safety Instructions

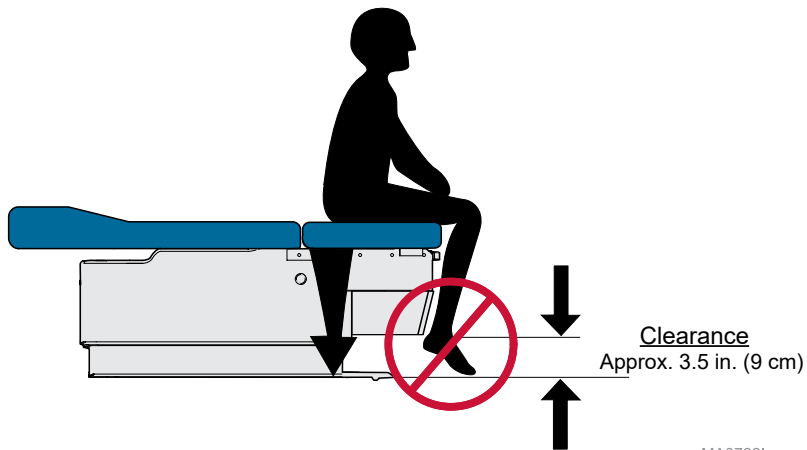
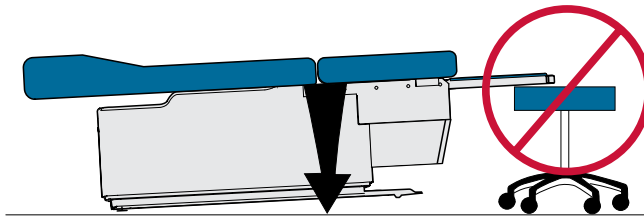
Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.

At the table's lowest height [18 in. (45.7 cm)], the clearance between the bottom of the drawer and the floor is approximately 3.5 in. (9 cm). Make sure patient (esp. feet) and all objects are clear of this area before lowering table.

If the table malfunctions, immediately release foot pedal, unplug the table power cord, and assist the patient from the table. If table continues to malfunction, call for service.

Failure to comply with these instructions may result in injury to patient and/or damage to equipment.

MA8722i

Operation


Table Up / Down Operation

Note


The table automatically stops at its maximum and minimum height.

To move the table Up or Down...
Press and hold the appropriate button.

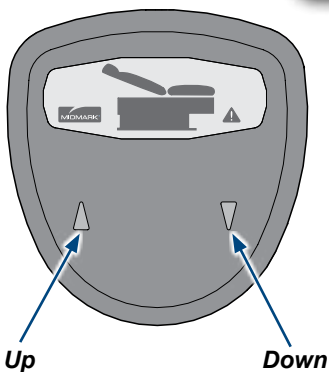222

223

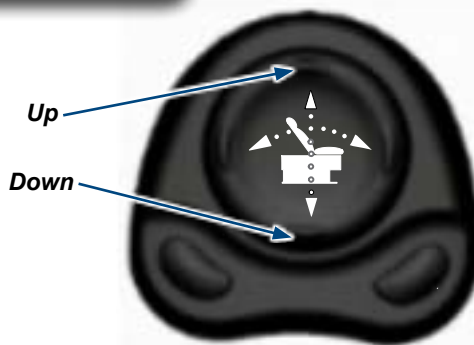
MA8741i

Equipment Alert

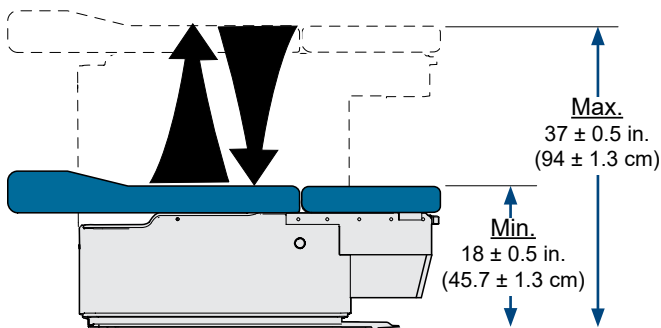If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.


MA8723i

Operation - continued

Back Up / Down Operation - 222

To position the back section...

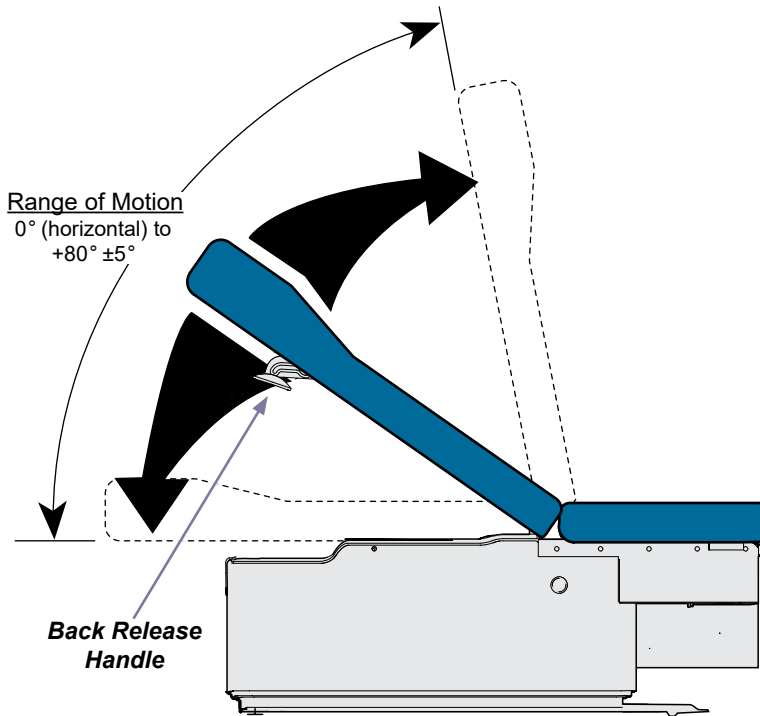- A) While holding the back section in place, squeeze the Back Release Handle (either side).
- B) Move the back section.
- C) Release handle to lock back section in position.

MA8724i

Operation - continued

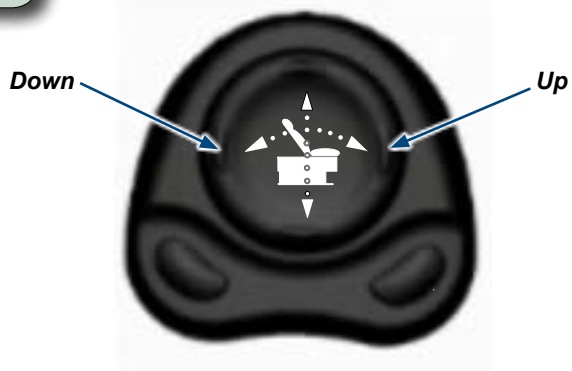
Back Up / Down Operation - 223

Note

The back section automatically stops at the top and bottom of its travel.

To position the back section...

Press and hold the appropriate button.


MA8728i

Equipment Alert

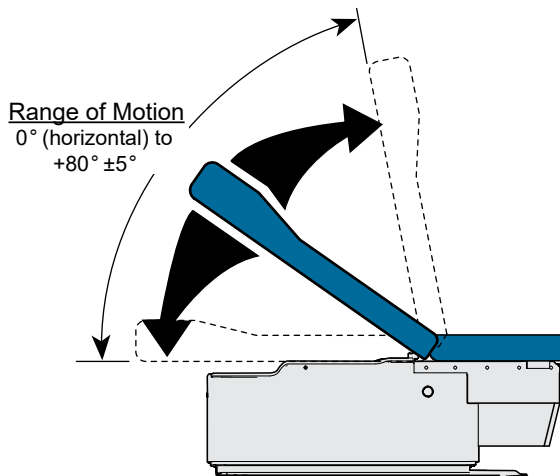If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.

MA8725i

Exam Assistant® Drawer System

To use the drawer front as a shelf...
Pull drawer front down to release the magnetic latch.

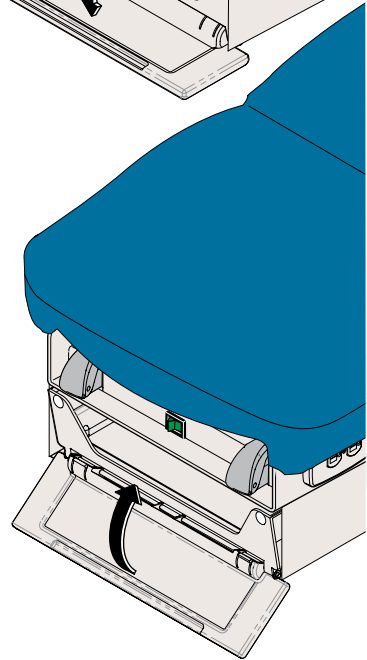
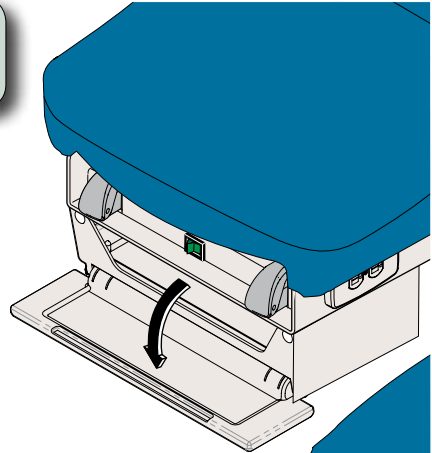
Equipment Alert
The maximum weight the shelf
will support is 10 lbs (4.5 kg).

If drawer front folds past horizontal...
Pull drawer front upward until it snaps back into
the horizontal position.

Equipment Alert
If drawer front folds past horizontal,
do not operate table.

MA8729I


Paper Roll & Tear Strap

Note

Up to four paper rolls (max.) may be stored under the back section.

Paper Roll Size (maximum):

222 & 223 (-016).....	21 in. wide x 3.5 in. diameter (53.3 cm x 8.9 cm)
222 (-017)	19.7 in. wide x 4.7 in. diameter (50 cm x 12 cm)

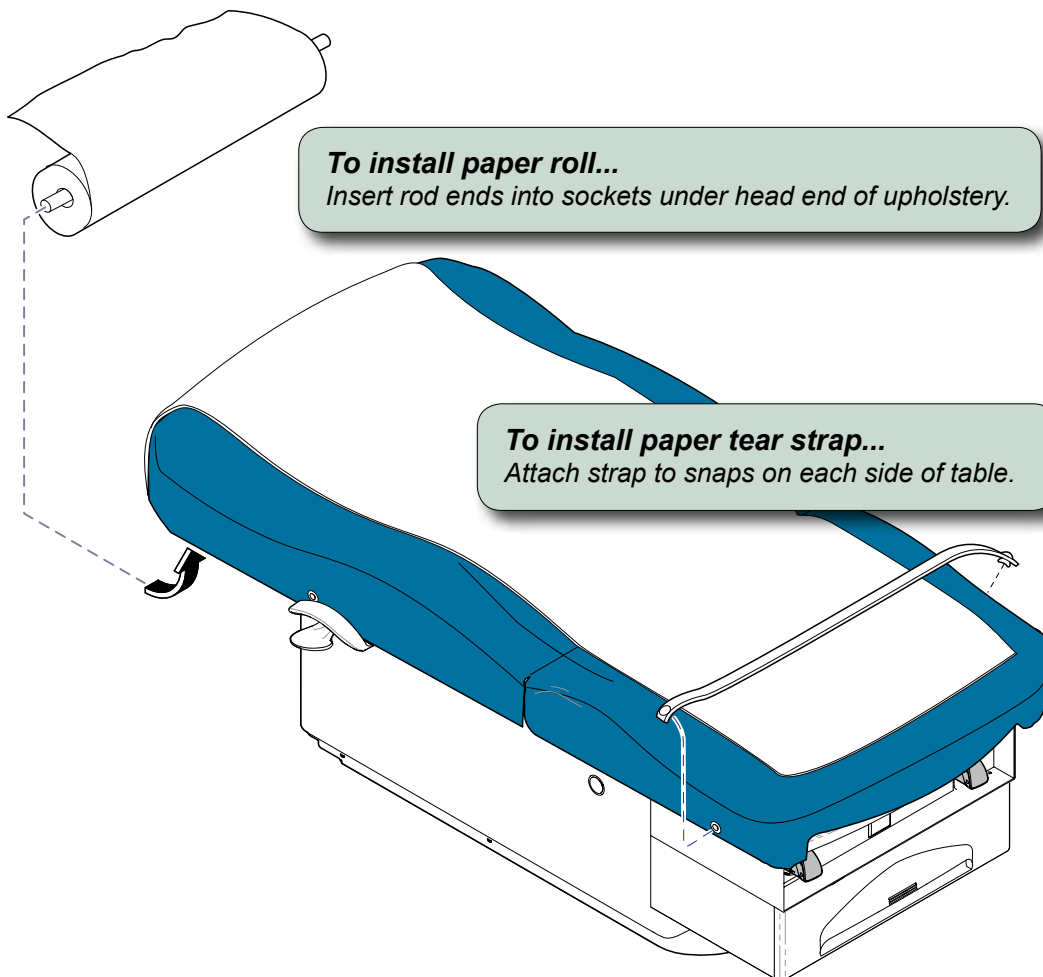MA8730i

Stirrups

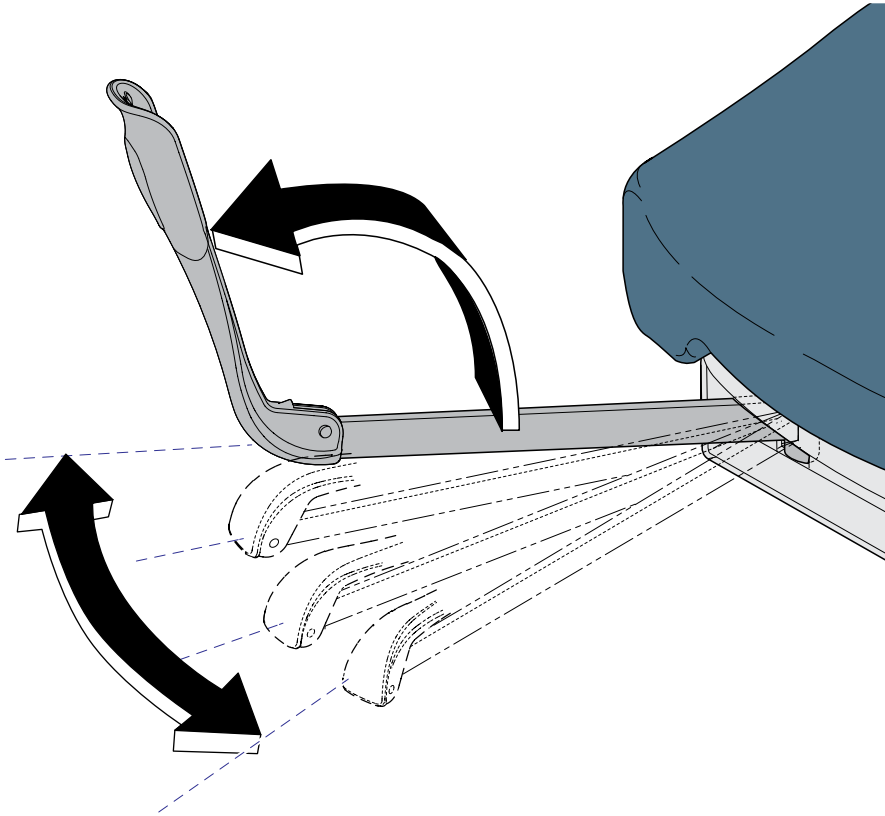
Caution

*Be sure the stirrups are locked in place before using.
The stirrups will not support the patient's entire weight.*

To position the stirrups...

- A) *Pull the stirrup out, then unfold.*
- B) *Lift the stirrup slightly, then move it left or right as desired.*
- C) *Release stirrup to lock in position.*


MA87311

Footrest & Treatment Pan

Caution

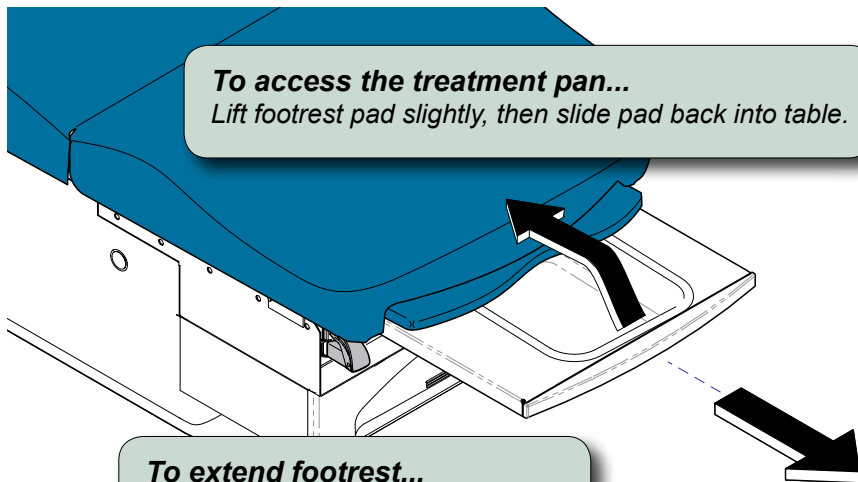Do not use footrest to support patient's entire weight.

To access the treatment pan...

Lift footrest pad slightly, then slide pad back into table.

To extend footrest...

Pull footrest out to desired position.

MA8733i

Drawer Heater (optional)

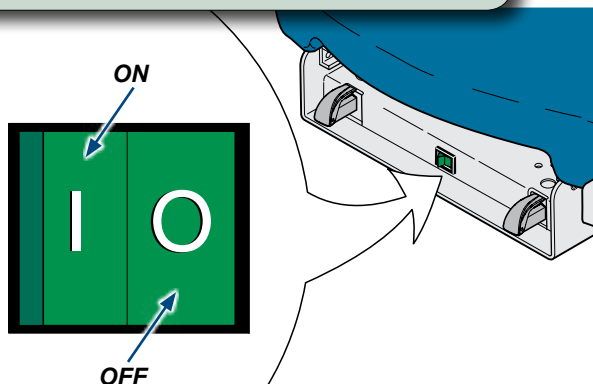
Note

The drawer heater switch illuminates when heater is ON.

The drawer heater warms the contents of the drawer to approximately 98°F (37°C).

To turn the drawer heater ON / OFF...

Press the heater switch to desired position.


MA8734i

Pelvic Tilt


To elevate seat section...

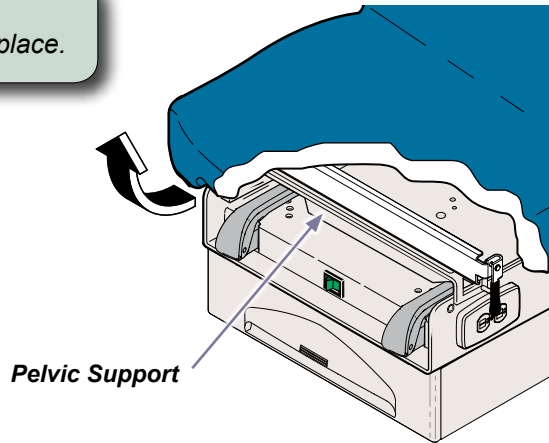Lift seat until pelvic support drops into place.

To return seat section to flat position...

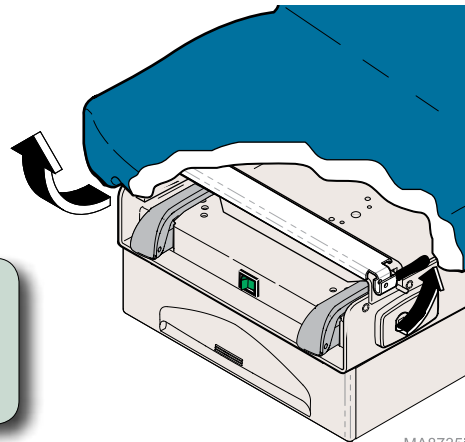- A) *Lift seat section slightly.*
- B) *Pivot pelvis support handle upward.*
- C) *Lower seat section.*


MA8735i

Duplex Receptacle (not available on 230 volt models)

The duplex receptacle on the side of the table provides power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacle's maximum load is exceeded, the circuit breakers interrupt power to the receptacle.

Maximum Load:..... 115 VAC, 5 amps

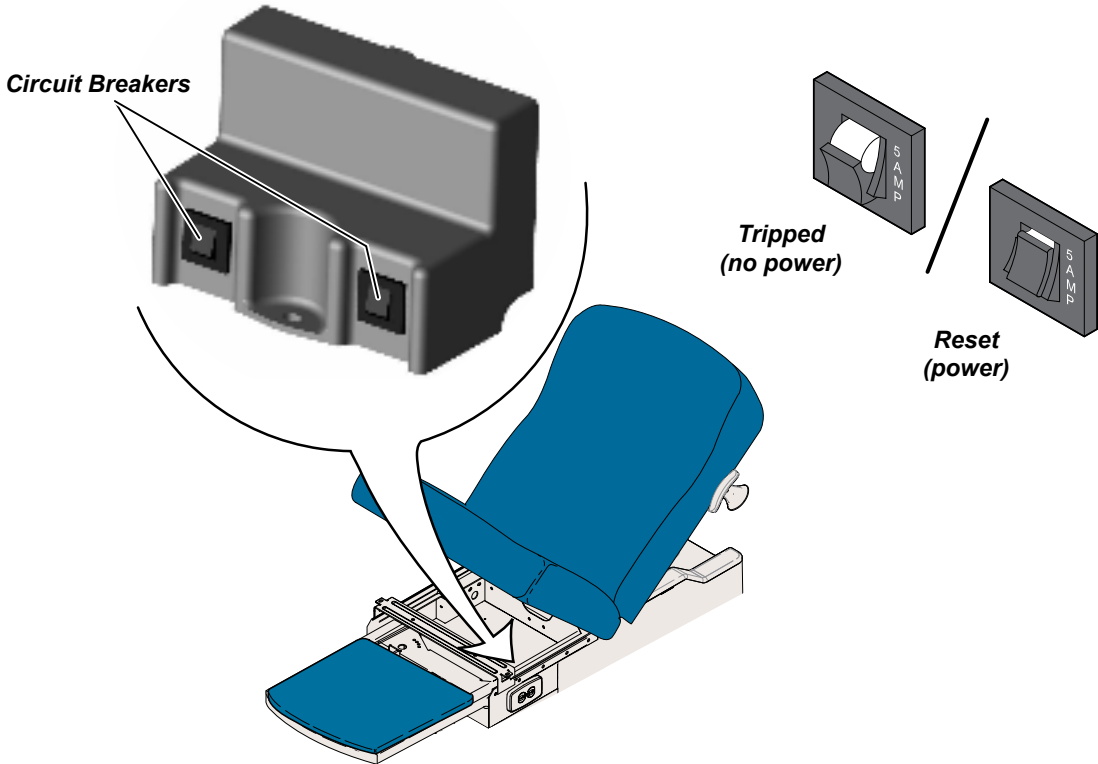To reset the circuit breaker(s)...
Push circuit breaker switch.

MA8736i

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Preventative Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Periodically lubricate the following areas to maintain quiet, smooth, operation:

- Back hinge (*use light machine oil*).
- Footrest slide (*use household furniture wax*).

To ensure safe use of table, have a qualified service technician inspect your table every six months.

Calling For Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

To contact Midmark directly:

1-800-MIDMARK (1-800-643-6275)

8:00 am until 5:00 pm Monday - Friday (EST) [excluding standard U.S. holidays]

Specifications

Weights & Electrical Specifications



Patient Weight (<i>maximum</i>)	400 lbs (181.4 kg)
Paper Roll (<i>maximum size</i>)	(See <i>Paper Roll & Tear Strap page</i>)
Range of Motion & Dimensions	(See <i>Range of Motion & Dimensions page</i>)
Weight of Table <i>[Weights listed are for 115V models. The 230V transformer adds 14.5 lbs (6.5 kg)]</i>	
w/upholstery	325 lbs (147 kg)
w/ <i>packaging & skid (no upholstery)</i>	375 lbs (192.7 kg)
Upholstery w/<i>packaging (shipped separately)</i>	40 lbs (18.1 kg)
Power Cord(s) <i>[115V models have two cords]</i>	8 ft. (243.8 cm) long
Electrical Requirements	<i>[See Compliance Chart]</i>
Foot Control Voltage	14 VAC, SELV (Safety Extra Low Voltage)
Duplex Receptacle (<i>maximum load</i>) <i>(not available on 230 volt models)</i>	115 VAC, 5A, 50/60 Hz
Fuses <i>222/223 (all)</i> <i>F1 & F2 (on main PC board)</i> <i>F3 (on main PC board)</i> <i>F4 (on main PC board)</i> <i>223 (only)</i> <i>F5 (on main PC board)</i> <i>Models w/ drawer heater</i> <i>distribution board fuses (2)</i> <i>222 (-017)</i> <i>located next to IEC inlet (2)</i>	10A, 250V, Slo-Blo, 1/4" x 1 1/4" 5A, 250V, Slo-Blo, 1/4" x 1 1/4" 187 mA, 250V, Slo-Blo, 1/4" x 1 1/4" 4A, 250V, Slo-Blo, 1/4" x 1 1/4" 800 mA, 250V, Slo-Blo, 5 x 20 mm 4A, 250V, Slo-Blo, 5 x 20 mm
Duty Cycle (Motor Run Time)	Intermittent Operation <i>[Domestic: 30 seconds ON / 5 minutes OFF]</i> <i>[Export: 25 seconds ON / 7 minutes OFF]</i>
Receptacles / Drawer Heater	<i>Continuous Operation</i>
Classifications	Class I, Type B Applied Part Intermittent Operation <i>[Domestic: 30 seconds ON / 5 minutes OFF]</i> <i>[Export: 25 seconds ON / 7 minutes OFF]</i>
Protection against ingress of fluids	IPXO <i>[Foot Control only: IPX1]</i>

Model Identification / Compliance Chart

Model	Description	Complies To:					Electrical Ratings:		
		UL 60601-1	CAN / CSA 22.2, #601.1 - M90	IEC 60601-1	EN 60601-1-2 (EMC)	CE	VAC +/- 10%	Amps	Cycles (Hz)
222-016	Ritter Basic Table w/ Receptacle, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	11	50/60
222-017	Ritter Basic Table - Export w/ Pelvic Tilt	•	•	•	•	•	230	2.75	50/60
223-016	Ritter Power Back Table w/ Receptacle, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	12.5	50/60




Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishings Technical Bulletin 117. Optional upholstery is available that complies with California Bureau of Home Furnishing Technical Bulletin 133.

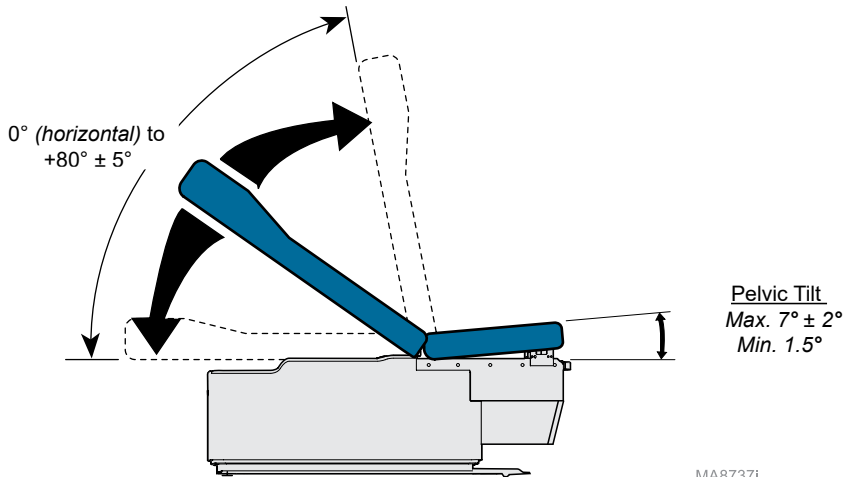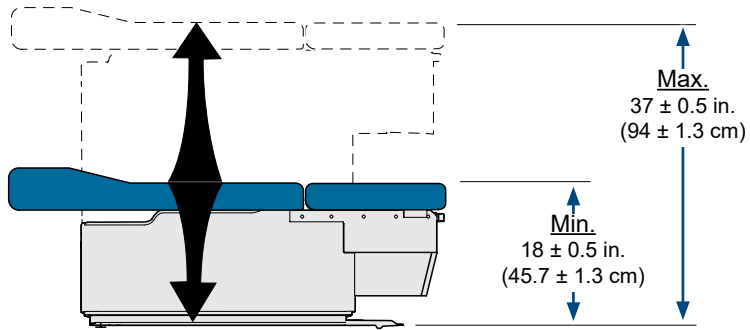Range of Motion & Dimensions

MA87371

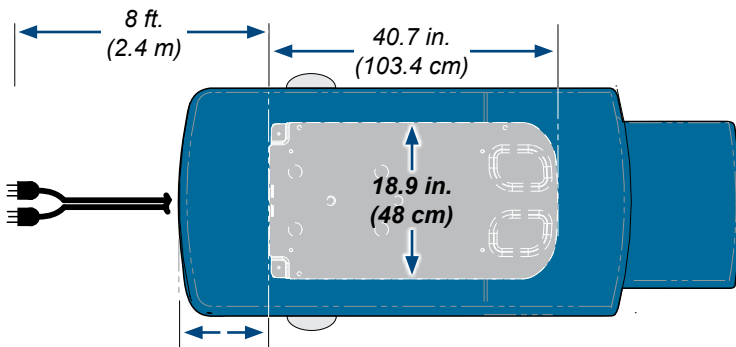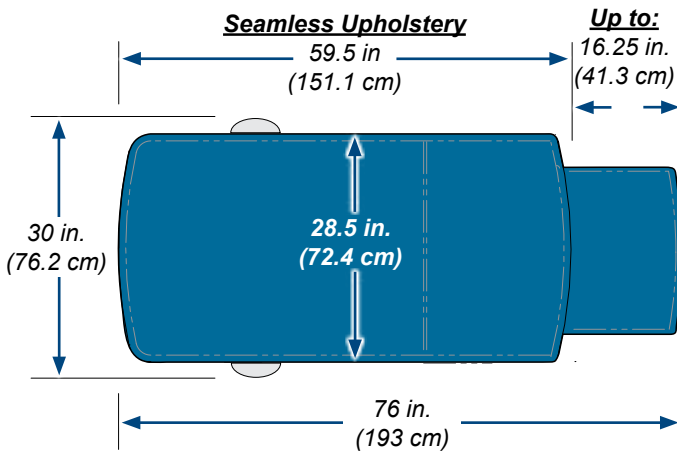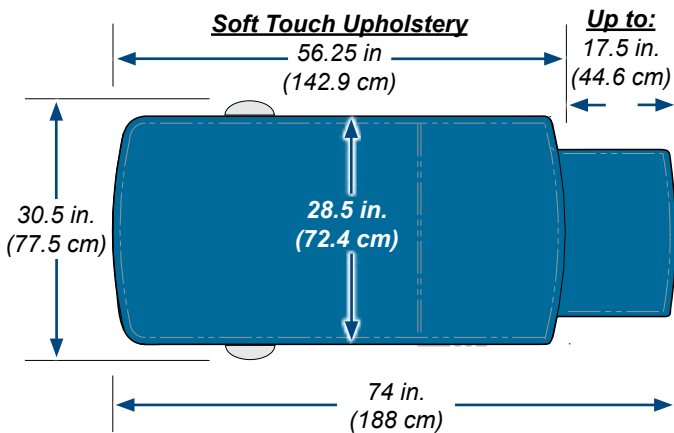Range of Motion & Dimensions (continued)

Soft Touch: 12 in (30.5 cm)
Seamless: 13.5 in (34.3 cm)

MA8738i

Warranty Information

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com; by phone at 1-800-MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. **WARRANTY DISCLAIMER** THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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