



Socket Accessory Installation

Applies to: Assist Arms [9A354, 9A632]

Components:

Sockets:

Screws..... 4
Accy. Sockets 2

Assist Arms:

Assist Arms..... 2

Intended Use:

To assist patient mounting / dismounting table


Restrictions:

9A354, 9A632 Assist Arms to be used on 204/604, 222/223, and 625 models with 28" upholstery only.

Midmark models

Note

The cladding on 604 models with serial numbers Vxxxxxx thru present is magnetic. It is **not** necessary to remove drawers / screws.

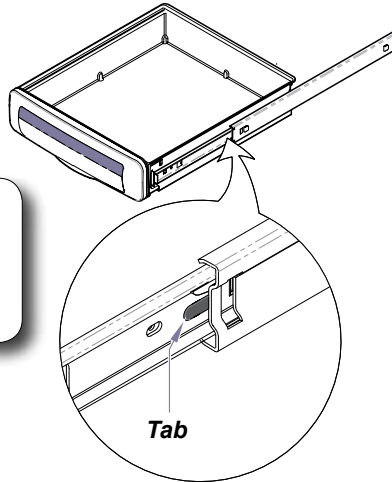


Step 1: Remove two drawers from foot end.

*Note: To remove top drawer, pull drawer all the way out, then:
Press tab on RH side down.
Press tab on LH side up.*


Step 2: Remove side cladding.

*Serial #'s prior to V1177445:
Cladding is secured with screws from inside cabinet.*

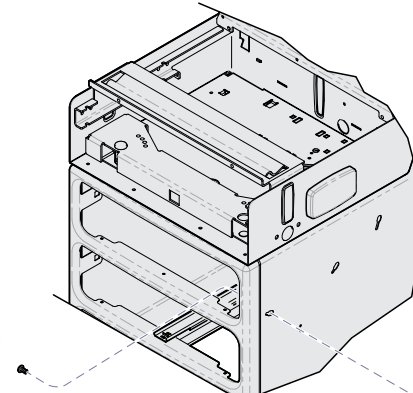


Step 3: Remove hole plugs.

Go to next page for socket installation.

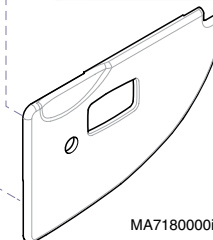

Ritter models

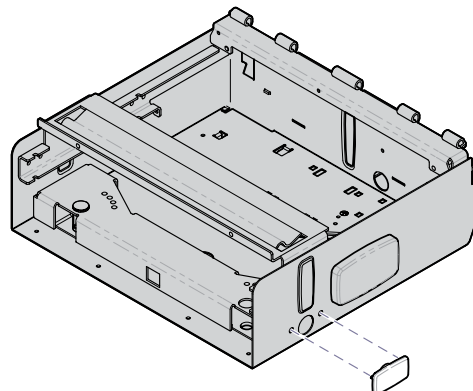


Step 1: Carefully pry hole plugs out of upperwrap.

Go to next page for socket installation.

MA721100i

Socket Installation

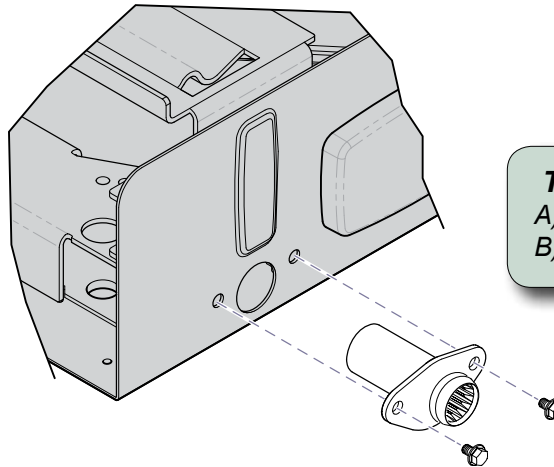

To install accessory socket...

- A) Insert socket into upperwrap.
- B) Secure with two screws.

MA7212000i

Accessory Installation


To install accessory...

- A) Position accessory as shown.
- B) Insert into socket until release lever locks into place.

To remove:

Squeeze release lever, then pull accessory out of socket.

Assist Arms (9A354*, 9A632)

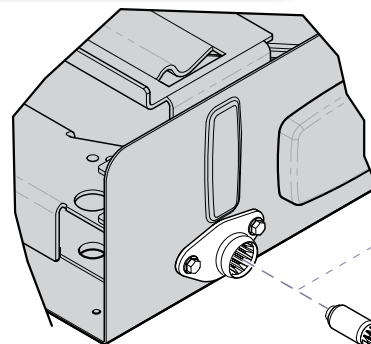

Operating Position

Storage Position


Assist Arms only

To reposition...

- A) Squeeze release lever & pull partially out of socket
- B) Rotate to desired position.
- C) Push arm into socket until release lever locks into place.

Release Lever


MA7215000i

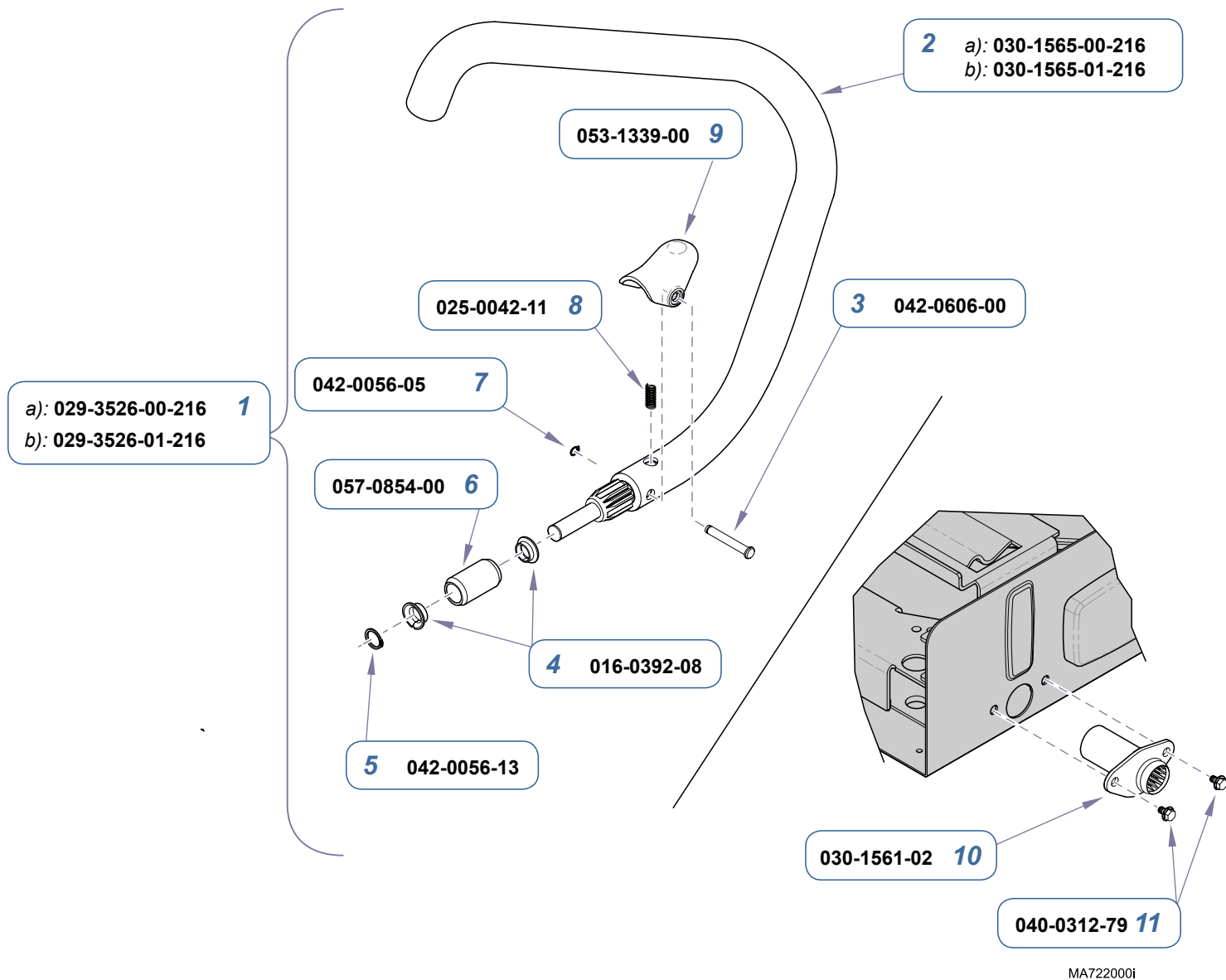

Item	Description	Qty.
9A354001 Assist Arms only (set- LH/RH) (includes items 1 thru 9)		
1	• Assist Arm Assembly (includes items 2 thru 9): A) LH- shown B) RH- not shown	1 1
2	• • Arm Weldment A) LH- shown B) RH- not shown	1 1
3	• • Clevis Pin	1
4	• • Flange Bearing	2
5	• • Retaining Ring	1
6	• • Pivot Sleeve	1
7	• • Retaining Ring	1
8	• • Spring	1
9	• • Retainer Clip	1
9A354003 Accessory Socket Kit (includes items 10 & 11)		
10	• Socket Weldment	2
11	• Flange Bolt (5/16-18 x 3/4")	4

Serial Numbers / Dates:

9A354
all

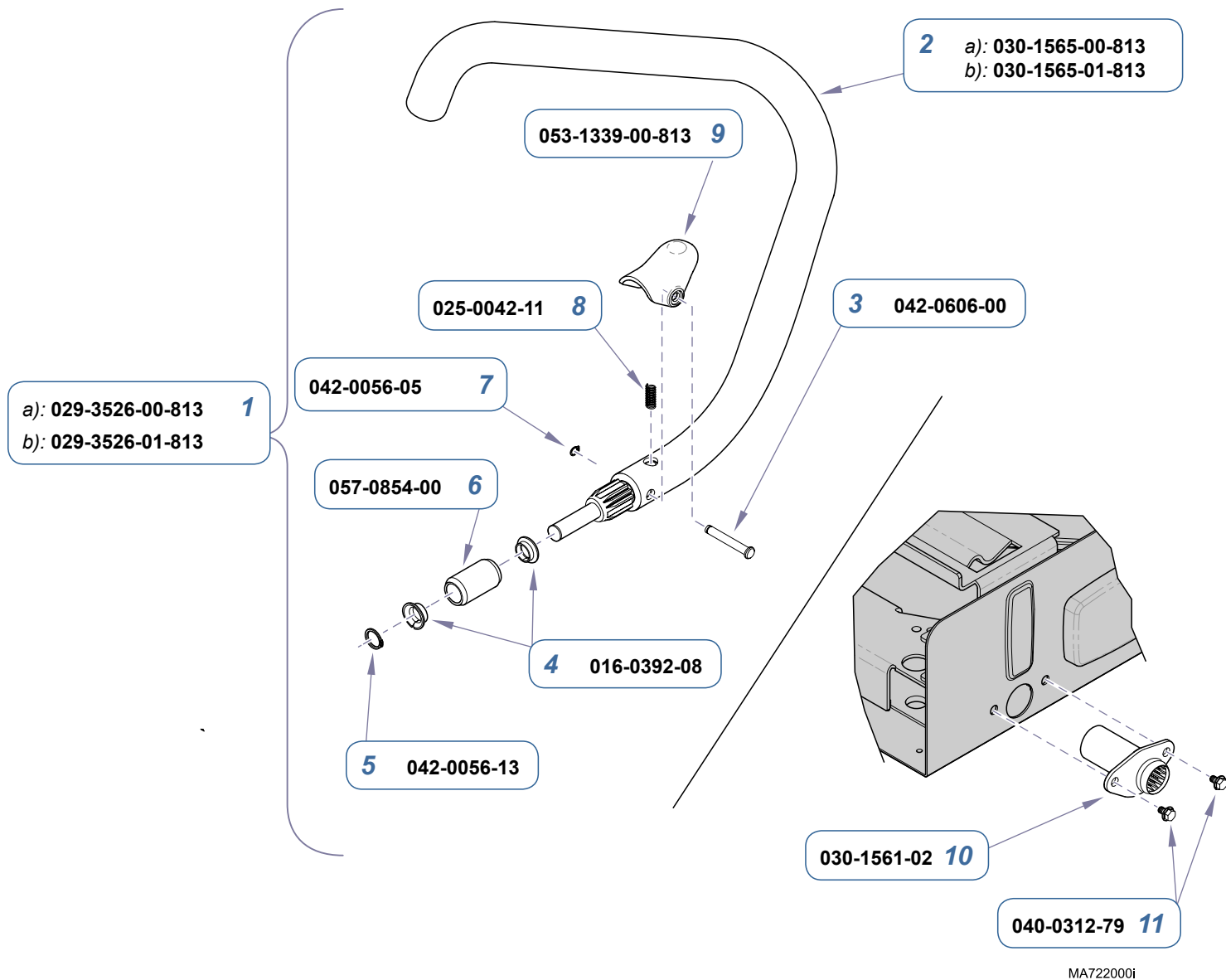

Item	Description	Qty.
9A632001 Assist Arms only (set- LH/RH) (includes items 1 thru 9)		
1	• Assist Arm Assembly (includes items 2 thru 9): A) LH- shown B) RH- not shown	1 1
2	• • Arm Weldment A) LH- shown B) RH- not shown	1 1
3	• • Clevis Pin	1
4	• • Flange Bearing	2
5	• • Retaining Ring	1
6	• • Pivot Sleeve	1
7	• • Retaining Ring	1
8	• • Spring	1
9	• • Retainer Clip	1
9A632003 Accessory Socket Kit (includes items 10 & 11)		
10	• Socket Weldment	2
11	• Flange Bolt (5/16-18 x 3/4")	4

Serial Numbers / Dates:

9A632
all