

## Flat and Narrow Upholstery Kit for 230 [002-0950-xxx and 002-10135-xxx]

### NOTE

The foot and leg sections of the flat & narrow upholstery set are the same size as the standard upholstery. These sections are provided for new installations and color changes, it is not necessary to replace them.

### Components:

#### Support Boards:

Back Section ..... 1

Seat Section ..... 1

#### Upholstery Sections:

Back / Seat ..... 1

Head ..... 1

Foot ..... 1


Leg ..... 1

**Step 1:** Remove standard back and seat support boards.


**Step 2:** Remove paper roll holder from standard back support board.

**Step 4:** Install paper roll holder on narrow back support board.

**Step 3:** Install narrow back and seat support boards.

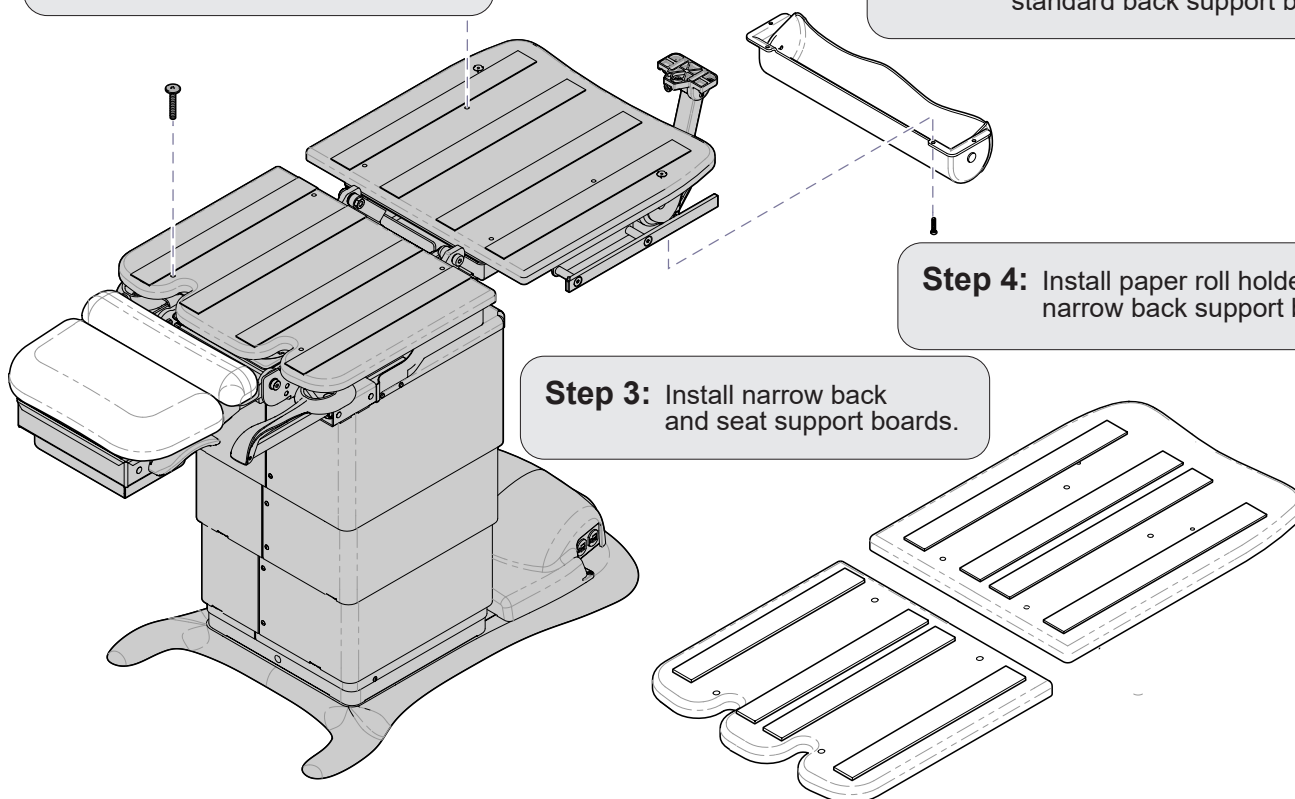

### ATTENTION! Applies to 230 Models before serial number V1831829

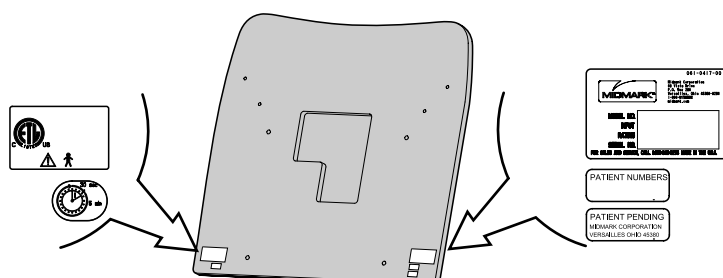


The labels on the standard back support board are essential for model identification and regulatory compliance.

Retain the standard support board with labels intact.


**To install seat / back section upholstery...**

- A. Position front of seat section against mounting board.  
B. Lay upholstery flat, then press down firmly.

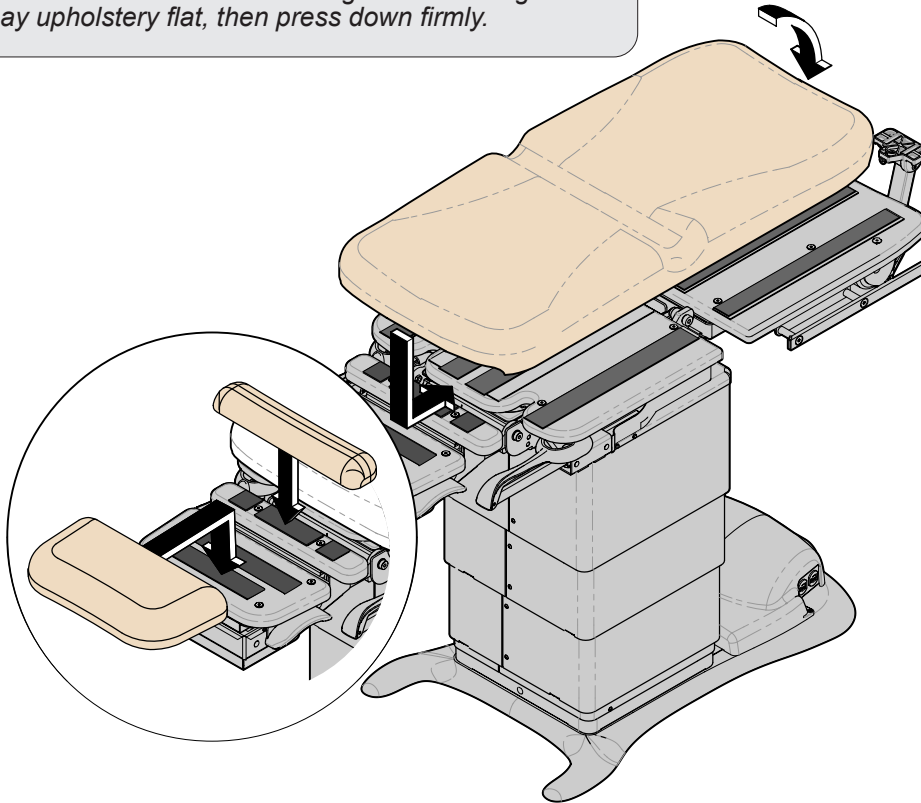

**To install foot and leg section upholstery...**

- Align hook and loop fastener strips - press down firmly.

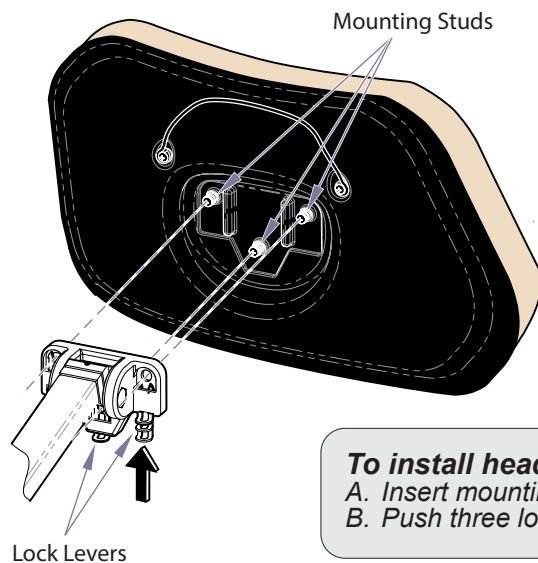

**To install headrest upholstery...**

- A. Insert mounting studs into holes in pivot mount.  
B. Push three lock levers inward to lock in place.