

### Drawer & Upholstery Heaters (optional)

#### **To turn the heater(s) ON / OFF...**

Press the appropriate heater switch button.  
(The switch illuminates when heater is ON).

Note: The upholstery heater automatically shuts off after 8 hours.

**Upholstery Heater  
Switch (green)**

**Drawer Heater Switch (white)**



### Exam Assistant® Drawer System



#### **Equipment Alert**

Max. weight the shelf will support: 10 lbs. (4.5kg)

#### **To lower drawer front...**

Pull drawer front down to release magnetic latch.

#### **If drawer front folds past horizontal...**

Pull drawer front upward until it snaps back into the horizontal position.

## Maintenance

### **Periodically inspect the following:**

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate

### **Cleaning**

#### **Upholstery**

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [www.midmark.com](http://www.midmark.com) in the Technical library under the User Information tab for your product.

#### **Painted Metal / Plastic Surfaces**

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

### **Calling for Service**

If service is required, contact your Midmark dealer. To contact Midmark directly:

1-800-MIDMARK (1-800-643-6275)

Monday-Friday 8:00 am until 5:00 pm (EST)

• Français : Référez-vous à [www.midmark.com](http://www.midmark.com)

• Deutsch: Siehe [www.midmark.com](http://www.midmark.com)

• Español: Refiera a [www.midmark.com](http://www.midmark.com)

Midmark Corporation  
60 Vista Drive  
Versailles, OH 45380-0286  
Phone: 937-526-3662  
Fax: 937-526-5542  
[www.midmark.com](http://www.midmark.com)



Because we care.



## Quick Reference Guide 623

### **Barrier-Free® Exam Tables**

Model Number:  
Serial Number:

Dealer Information:

Purchase Date:

Upholstery Color:

003-1613-00 Rev E 7/10/15

This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



**To operate the table's power functions...**

Press & hold the appropriate button on the foot / hand control.



**Equipment Alert**

If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.



**Note**

**To stop movement of the Home function...**

Press the Stop button on the hand or foot control.

**'Home' Function**

This function automatically lowers the base to 20 in. (51 cm), so that your patient may easily mount / dismount the table.

**To activate the Home function...**

Press, then release the Home button.