


Table Receptacle (optional)

If the maximum load is exceeded, the circuit breakers remove power from the receptacle.
Maximum Load: 5 amps

To reset circuit breaker(s)...

Push circuit breaker toggle switch.

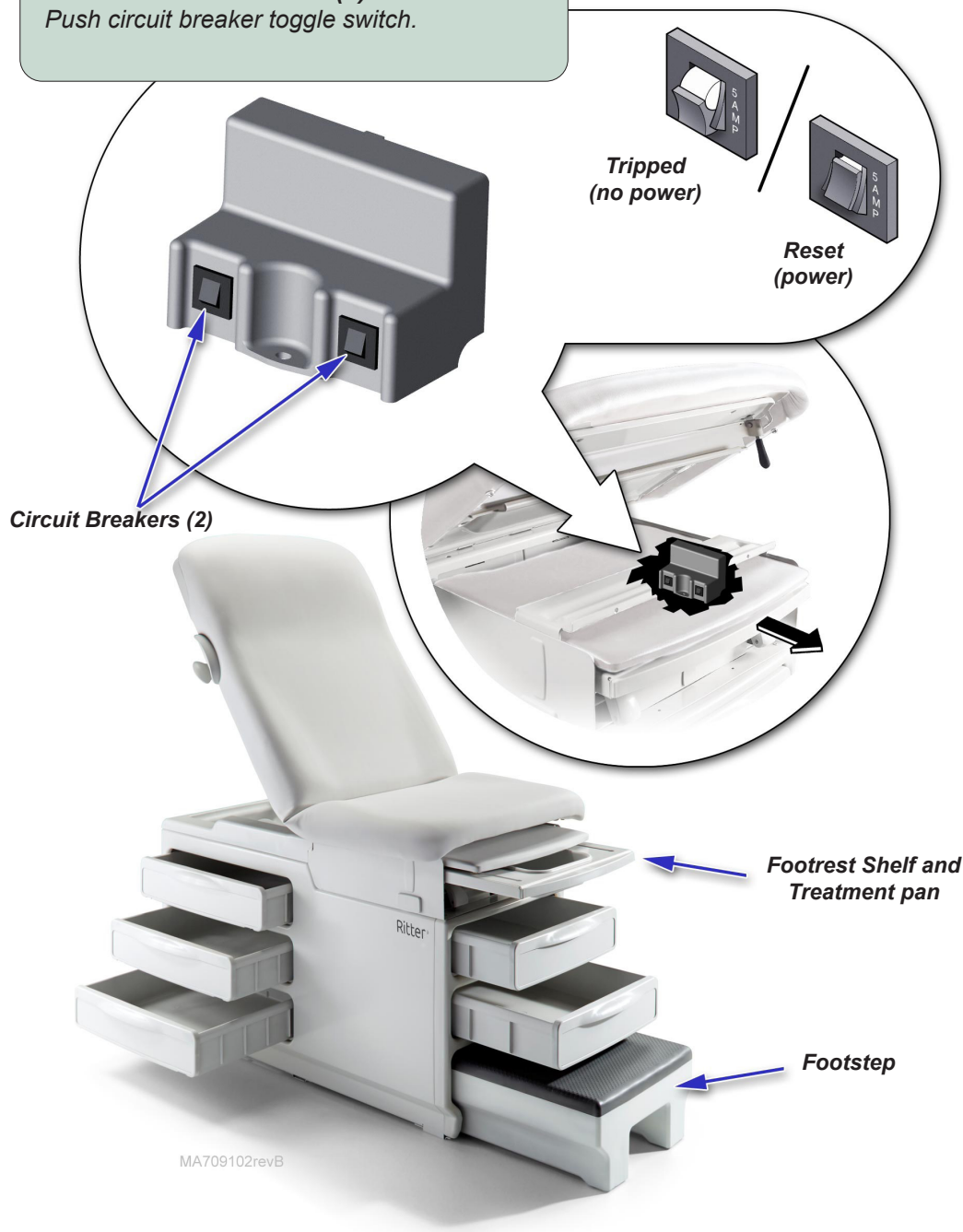

MA709102revB

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Calling for Service

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

Français : Référez-vous à midmark.com

Deutsch: Siehe midmark.com

Español: Refiera a midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937.526.3662
Fax: 937.526.5542
midmark.com



Ritter®

Quick Reference Guide

204

Manual Exam Table

Model Number:
Serial Number:

Dealer Information:

Purchase Date:

Upholstery Color:

003-1616-00 Rev DA3 12/18/23

This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



MA10616revB

Pelvic Tilt feature (optional)

To elevate seat section...

Lift seat section until pelvic support drops into place.

To return to flat position...

A. Lift seat section slightly.

B. Pivot trigger handle backward.

C. Lower seat section.

Trigger Handle

Back Release Handle

To position the back section...

A. Squeeze the Back Release Handle (either side).

B. Move the back section to desired position.

C. Release handle to lock back section in position.

Stirrups



MA709004revB

Drawer Heater (optional)

To turn the drawer heater ON / OFF...

Press the drawer heater switch.