Ritter®

95 Treatment Table

[English](#)
[Español](#)
[Français](#)

Applies to Models:

95-001

Special Tools:

none

Set Up / User Guide

Style P

003-1649-99 Rev. AA3 (10/10/23)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

***Midmark Authorized Service
Company:***

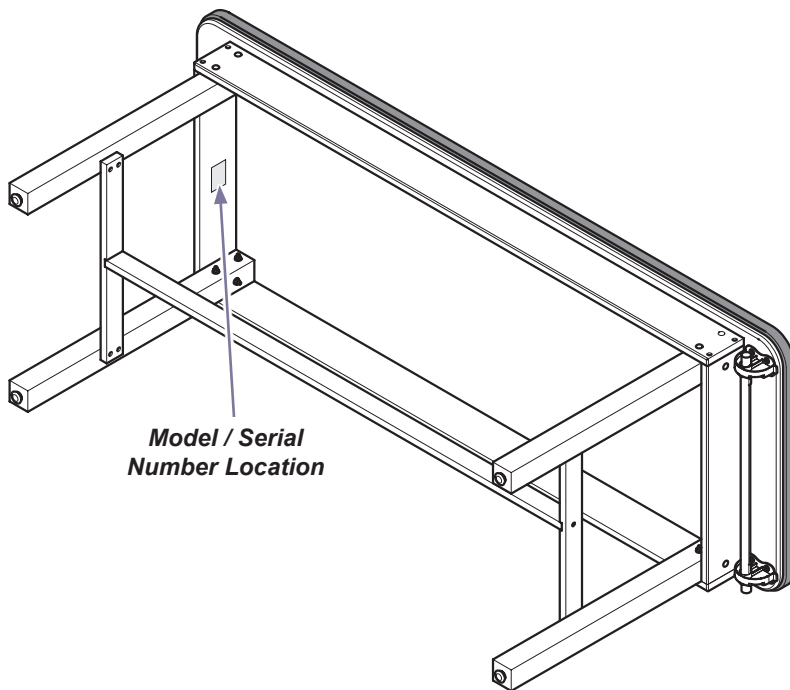
Table of Contents

Important Information

Product Information 2

Intended Use 4

Safety Instructions 4

Product Registration 4

Transportation / Storage Conditions 4

Disposal of Equipment 4

Important Information 5

Authorized Representatives 6

Set Up

Set Up Instructions 8

Operation

Operating Instructions 9

Maintenance

Cleaning 10

Periodic Maintenance 10

Calling for Service 10

Warranty Information

Limited Warranty 11

Intended Use

This product is intended to be used as a table to provide positioning and support of patients during examination procedures conducted by qualified medical professionals in an office environment.

Safety Instructions

WARNING

No modification of this equipment is allowed.

Product Registration

Transportation / Storage Conditions

Ambient Temperature Range:.....-30°C to +60°C (-22°F to +140°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Disposal of Equipment

At the end of product life, the table, accessories and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

	Fragile		Keep dry
	Patient Weight Limit		Maximum stacking height (palletted units)
	Manufacturer		Catalogue Number
	Serial Number		

Authorized Representatives

WARNING

***In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.***

Customers in India should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

Midmark India

Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Qatar should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974-44337000
Fax: +974-44337100

Customers in Israel should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3-925-4040
Fax: +972 3-924-9977

Authorized Representatives - continued

Customers in Colombia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,


Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Set Up

Assembly Instructions

- A) Lay table top assembly upholstered side down on a clean surface.
- B) Secure leg assemblies with carriage bolts & nuts.
Note: Do not tighten nuts at this time.
- C) Secure cross brace with drywall screws.
- D) Tighten nuts securing leg assemblies.
- E) Install four leveling screws.
- F) Install paper roll brackets & dowel rod as shown.
Note: There are pre-drilled holes at one end. Install roll brackets as shown to allow notch access for paper roll refills.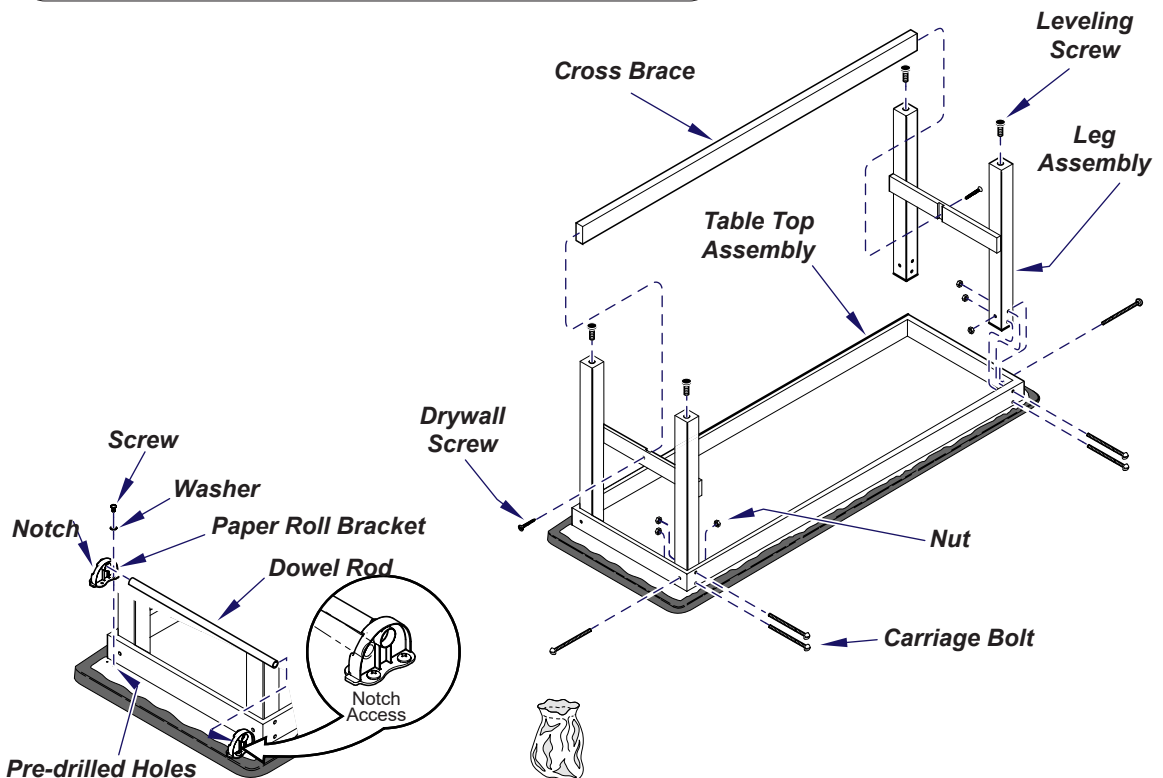

Parts Bag contains the following parts:

Leveling Feet (4pcs)
Drywall Screws (2 pcs)
Screws (8 pcs)

Carriage Bolts (12 pcs)
Nuts (12 pcs)
Washers (8 pcs)

Operation


WARNING

To avoid injury, do not put head or extremities under the paper tear strap.

To install paper tear strap

Slide strap over table as shown.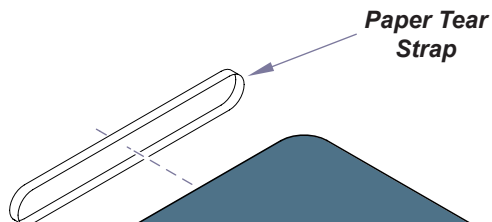

To level the table.

Adjust four leveling feet as necessary.

Maximum Patient Weight

Max. Patient Weight...350 lbs (159 kg)

**Leveling
Feet**

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Wood / Plastic Surfaces

Clean the wood and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Periodic Maintenance

Periodically inspect the following areas:

- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling For Service

Note

Model / Serial Number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

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Midmark Corporation