

Drawer Stop Installation [9A372]

Components:

Slide Clips 3

Special Tools:

none

Restriction

Early models have a drawer hole cover in the bottom of the table.
The drawer stop **cannot** be installed on the bottom slide of these models.

Attention

Slide clips must be installed on the RH cabinet slides.
Each drawer requires one slide clip.


Step 1: Hook long “leg” of slide clip over sheet metal edge as shown.

Step 2: Align notches with cabinet slide.

Notches

Step 3: Press slide clip down until it snaps into place.

Note: Slide clip should rest flat on slide / cabinet.

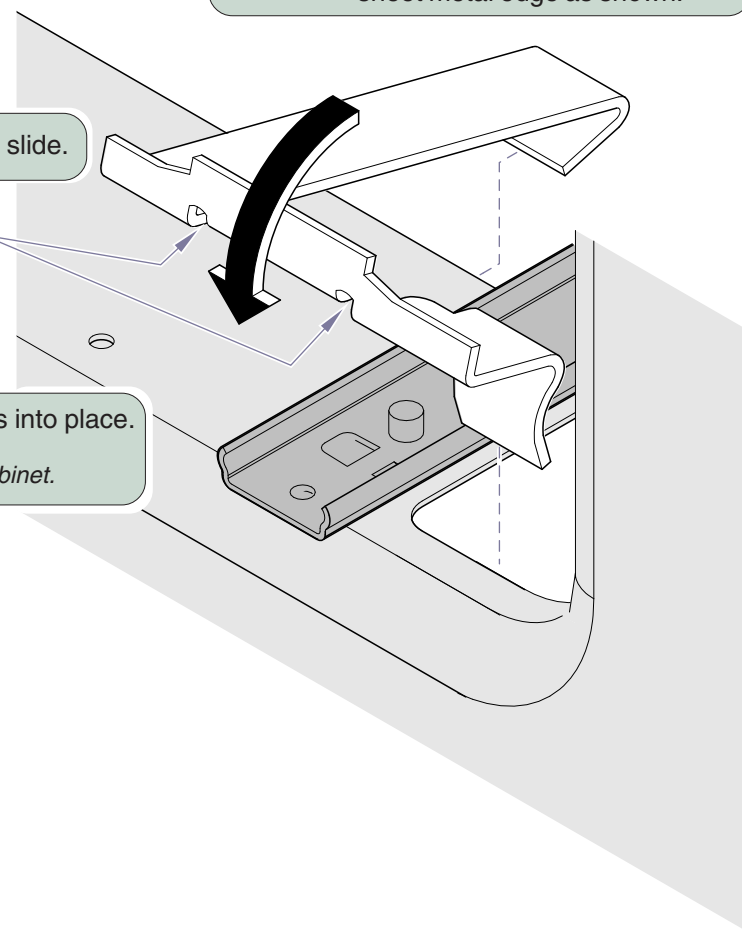

MA747500i