

UPT Actuator Grounding Kit

Components:


Ground Wire 1
Ground Screw 1
Star Washer 1

Special Tools:

none

Models w/ Grounding Plate:


To ensure proper ground, the metal braided insulation on the actuator wires must be in direct contact with the grounding plate.

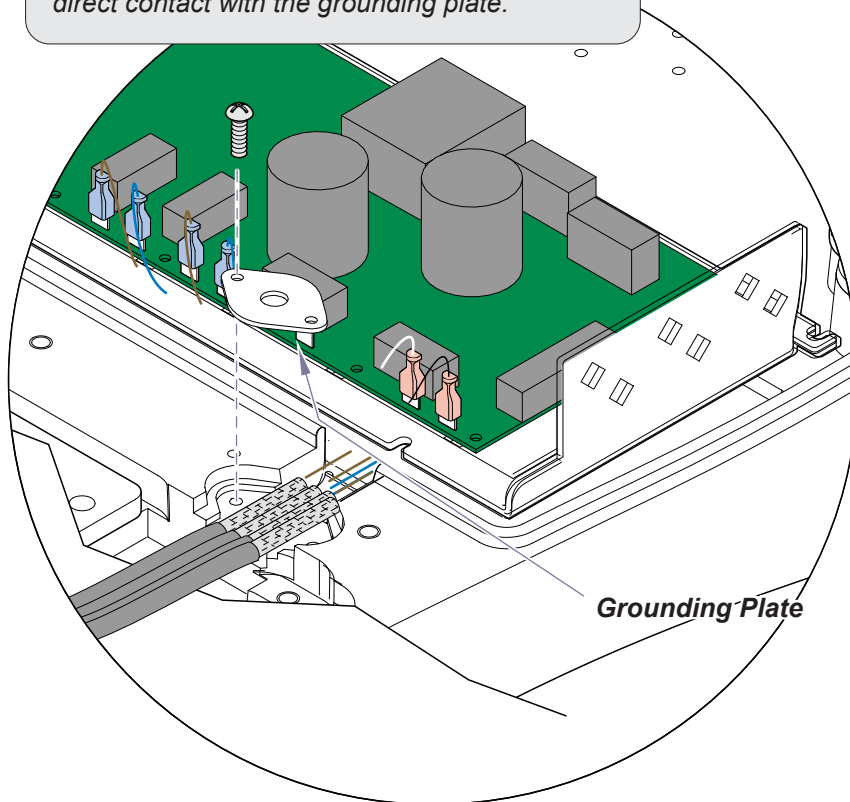

ArtHalf

Models w/o Grounding Plate:

Separate the strands of the braided insulation on actuator wires.
Twist the strands together.
Connect the ground wire (provided) as shown.
Connect ground wire to base casting w/screw (provided).

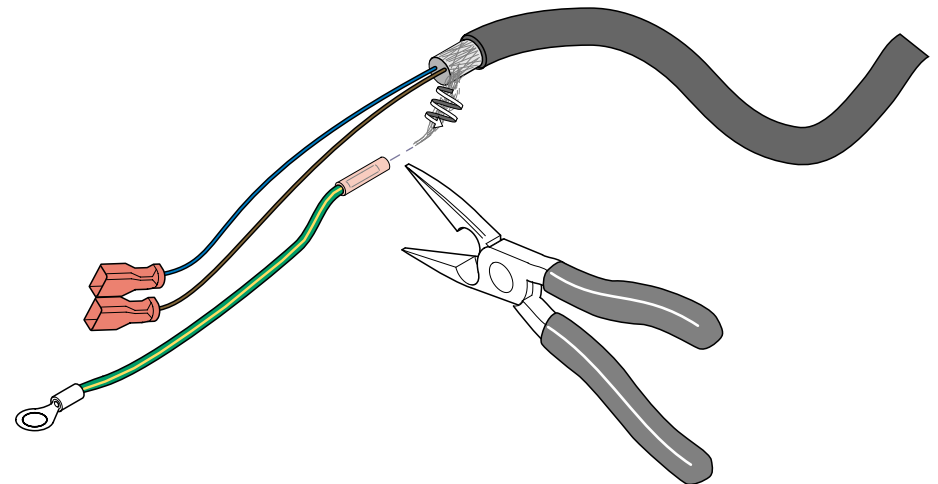

ArtHalf