



Power Procedure Chair Upholstery Installation

Applies to Models:
641 (all)

Language of origin: English

To install seat / back section upholstery...

- Raise back section of table to approx. 30°.
- Position front of seat upholstery against mounting board.
- Press seat section down firmly, then lay back section into position and press down firmly.

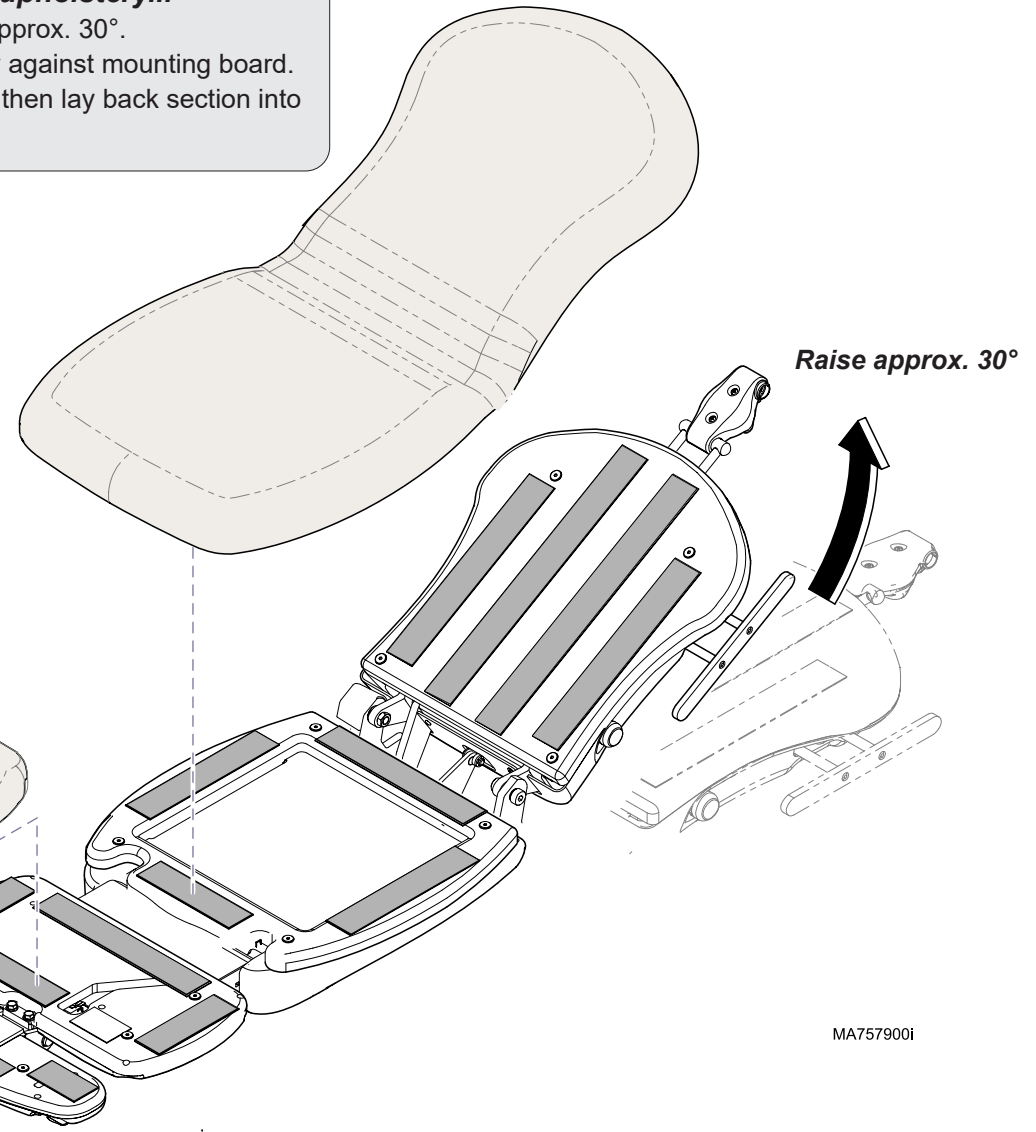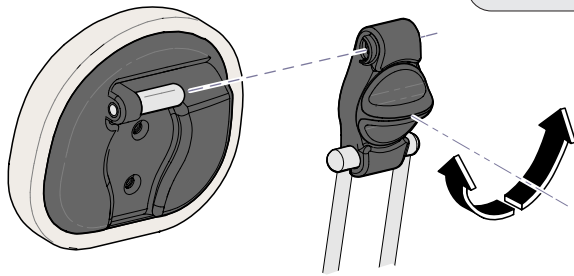


To install headrest assembly...

- Loosen headrest adjustment knob.
- Insert mounting stud into headrest fixture.
- Position headrest as desired.
- Tighten headrest adjustment knob.

To install foot rest / extension upholstery...

- Position upholstery section with mounting board.
- Lay upholstery flat, then press down firmly.

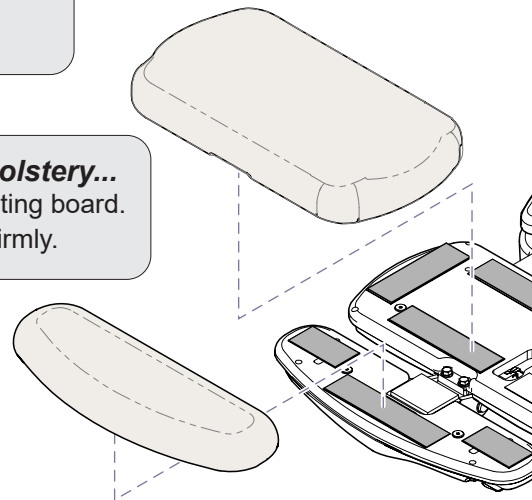

MA7579001