

**Quick Reference Guide
230
Procedure Chair**

To adjust the headrest at pivot points A and/or B

- A) Pull upper handle outward to release.
- B) Move pivot point(s) to desired position.
- C) Push upper handle inward to lock in place.

Note

The handle tension is adjustable.
Contact your service provider if
an adjustment is required.

To adjust the headrest at pivot point C


- A) Pull lower handle outward to release.
- B) Move pivot point to desired position.
- C) Push lower handle inward to lock in place.

Note

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” if Foot Up is activated with pan pulled out).

To resume Foot Up operation...

Push treatment pan all the way in, then press Foot Up button.

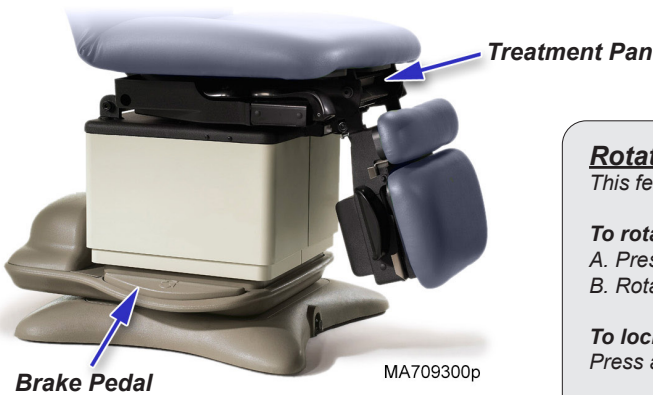

MA709300p

Rotational Base (optional)

This feature allows you to rotate the chair base 350°.

To rotate the chair base

- A. Press and release brake pedal.
- B. Rotate the chair to desired position.

To lock base in position

Press and release brake pedal.

Note: The base will lock automatically after two minutes.

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner. Use of chlorine based disinfectant can cause damage to the hand control.

Calling for Service

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)
midmark.com

Français : Référez-vous à midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937.526.3662
Fax: 937.526.5542
midmark.com




Model Number:	<input type="text"/>
Serial Number:	<input type="text"/>
Dealer Information:	<input type="text"/>
Purchase Date:	<input type="text"/>
Upholstery Color:	<input type="text"/>
003-1705-00 Rev EA3 10/18/23	

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information.



To operate the Base / Back / Tilt / Foot functions...
Press and hold the appropriate button on the hand control.

Note: The Foot Up function is disabled when the treatment pan (under seat) is pulled out.

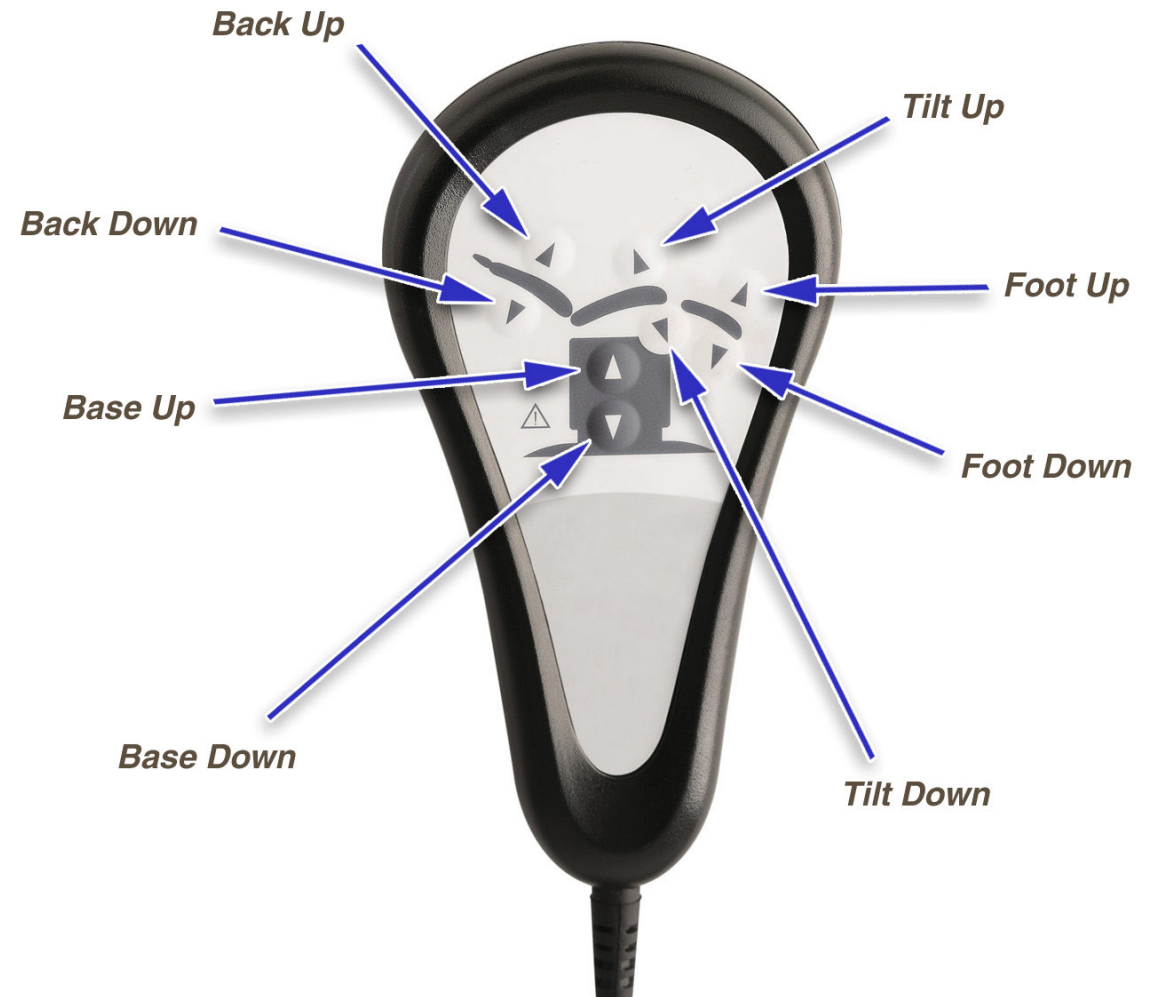

MA730001p