Power Procedures Chair

[English](#)

[Español](#)

[Français](#)

For Models:
641 (all)

User Guide

0 0 3 - 1 7 0 9 - 9 9

Product Information

(The information below is required when calling for service)

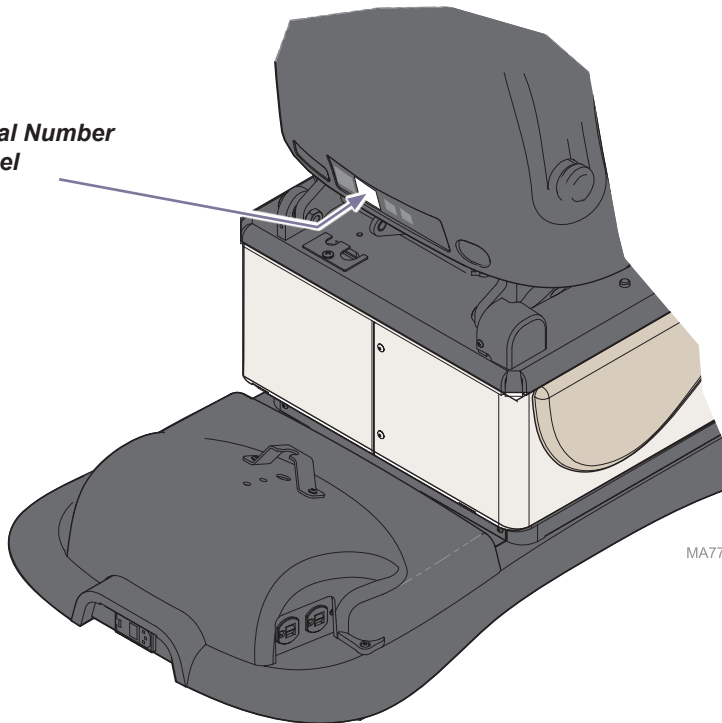
Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

***Midmark Authorized Service
Company*** [name / phone]:

***Model / Serial Number
Label***

MA774500i

Table of Contents

Important Information

Safety Symbols	4
Intended Use	5
Electrical Requirements	5
Electromagnetic Interference	5
Disposal of Equipment	5
Transportation / Storage Conditions	5
Authorized Representatives	6
Safety Instructions	7
Hand / Foot Control Connection	7
Crash Avoidance System	7

Operation

Power To The Chair	8
Chair Fuses	8
Base Up / Down	9
Back Up / Down	10
Foot Up / Down	11
Tilt Up / Down	12
Activating Multiple Functions	13
Home Function	14
Quick Chair® Function	15
Headrest Positioning	16
Side Rail Adjustment	17
Foot Extension	18
Chair Receptacle (options feature)	19
Programming Chair Positions (optional feature)	20
One-Touch Positioning (Programmable models only)	21
Rotational Base (optional feature)	22

Maintenance

Cleaning	23
Periodic Maintenance	23
Calling for Service	23

Troubleshooting

Crash Avoidance System	24
------------------------------	----

Specifications

Specifications Chart	25
Model Identification / Compliance Chart	26
Range of Motion	27
Dimensions:	28

Warranty Information

Warranty	29
----------------	----

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note
Amplifies a procedure, practice or condition.

Refer to manual

Proper Shipping Orientation

Fragile

Type B, Applied Part

Protective earth ground

Emergency Stop

Catalogue Number

Serial Number

Manufacturer

Keep dry

Maximum stacking height
(palletted units)

Fuse rating specification

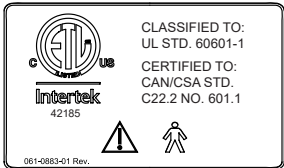
Protected against dripping water

Duty Cycle (motor run time):
30 seconds ON, 5 minutes OFF

Patient weight limit

This product has been evaluated with respect to electrical shock, fire & mechanical hazards only, in accordance with UL60601-1 and CAN/CSA C22.2 NO. 601.1.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during general examinations and procedures conducted by qualified medical professionals.

Electrical Requirements

Caution

When using high frequency surgical devices or endocardial catheters:

- *Use non-conductive material to insulate patient from metal portions of the chair.*
- *Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This Midmark chair is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this chair:

- Remove interfering device from room
- Plug chair into an isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range.....	+41°F to 100°F (+5°C to +38°C)
Relative Humidity.....	10% to 90% (non-condensing)
Atmospheric Pressure	500hPa to 1060hPa (0.49atm to 1.05atm)

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer.

For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofia Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading

Midmac Roundabout

Opposite Al Mana Petrol Station

Doha, Qatar

Tel: +974-44337000

Fax: +974-44337100

Customers in Kuwait should direct all questions, incidents, and complaints to the Midmark Authorized Representative: Central Circle Co.

P.O. Box 1015 Salmieh

22011 Kuwait

Al-Sharq

Khalid Bin Al-Waleed Street

Al-Kazmi Building 10

Third Floor

Email : cencico@centralcircleco.com

Phone : +965 22411758, +965 22411748, +965 22444228

Fax :+965 22445457

Safety Instructions

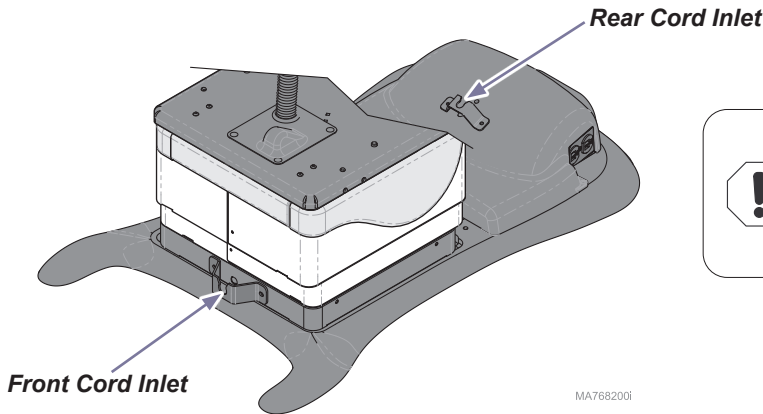
Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.

Hand / Foot Control Connection

There are two hand / foot control cord inlets. One is located on the base column at the foot end of the chair, and one is located on the PC board cover at the head end. These cord inlets are interchangeable.


Equipment Alert

Never plug hand / foot control cord into any other receptacle. Doing so may damage equipment.

To connect the hand and foot control cords...

- Slide narrow portion of cord connector into bracket.
- Connect cord to inlet. (See Note)
- Tighten locking ring.

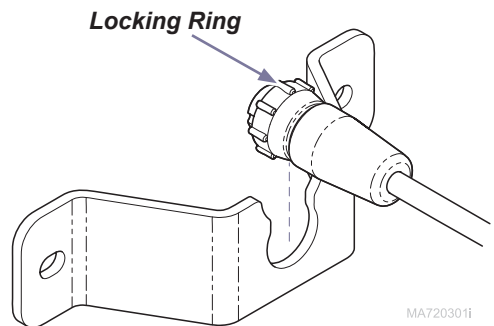Note: The cord connector is "keyed" and must be oriented properly to connect.

Crash Avoidance System

The Crash Avoidance System prevents the headrest and foot extension from colliding with the floor (or chair shrouds) by disabling certain functions if a potential crash position is detected. When this system is activated, the PC board will sound several audible "beeps" and disable any or all of the following functions (depending on the chair position): Base Down, Back Down, Tilt Up / Down and Foot Down.

Operation

Power To The Chair

If Power Indicator Light is...

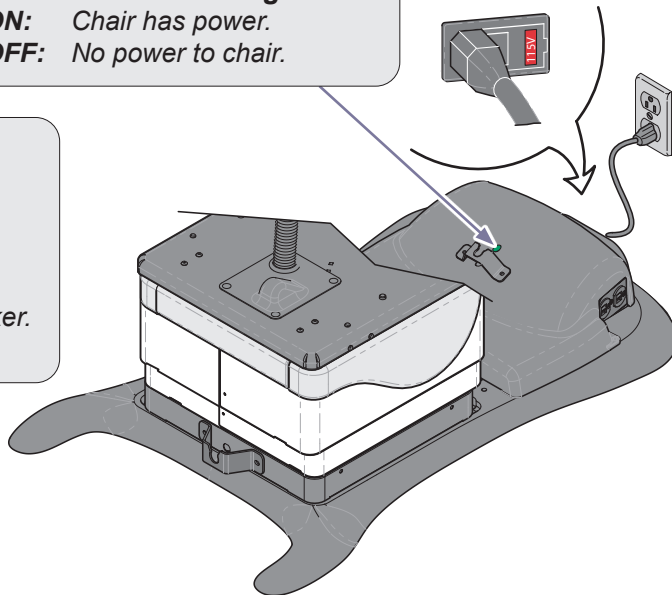
ON: Chair has power.
OFF: No power to chair.

If the chair has no power...

All functions will be inoperable.


Before calling for service...

- A) Check power cord connections.
- B) Check / reset facility circuit breaker.
- C) Check / replace chair fuse(s).

Before replacing fuses...

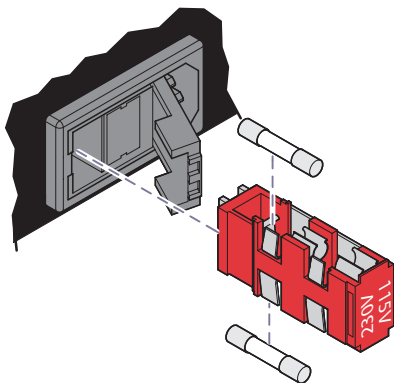Disconnect power cord from chair.


To replace fuses...

[Refer to Specifications section for correct fuse size and ratings]

- A) Open the fuse holder cover.
- B) Remove fuse holder and faulty fuse(s).
- C) Install new fuse(s), then replace fuse holder.

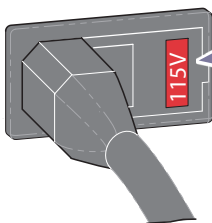Attention: Voltage setting that shows thru window of fuse holder cover must match facility supply voltage.

Voltage Setting

Equipment Alert



Incorrect voltage setting will result in damage to the chair.

MA768400i

Base Up / Down

To move the Base UP or DOWN...
Press and hold the appropriate button.

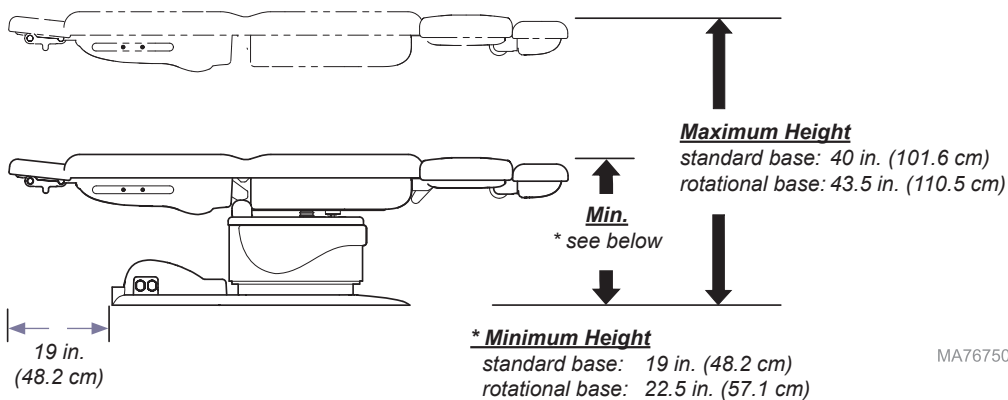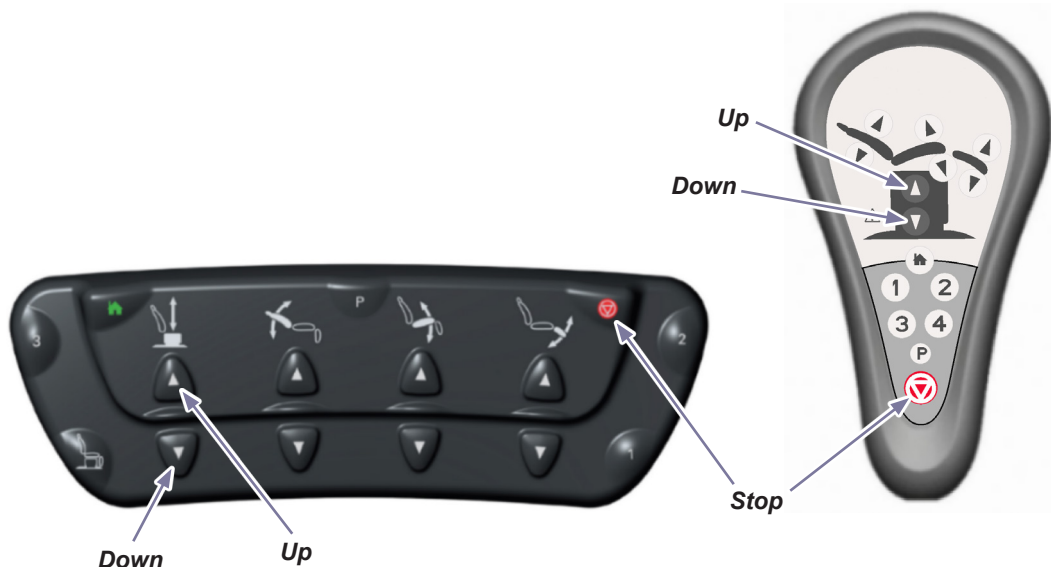Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.
Stop button is pressed.



MA767500i

Back Up / Down

To move the Back UP or DOWN...
Press and hold the appropriate button.

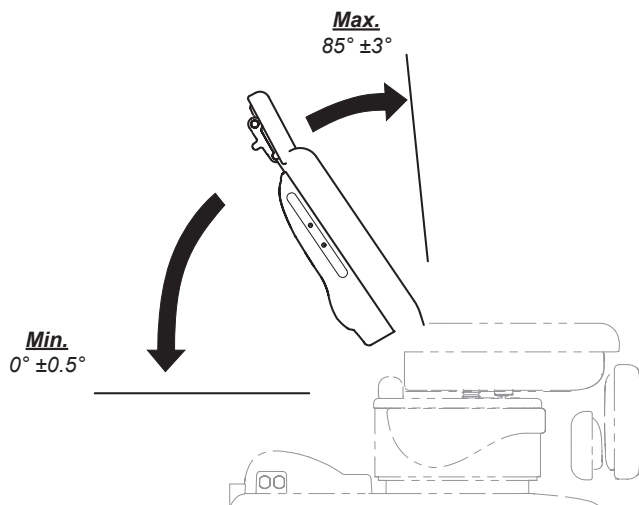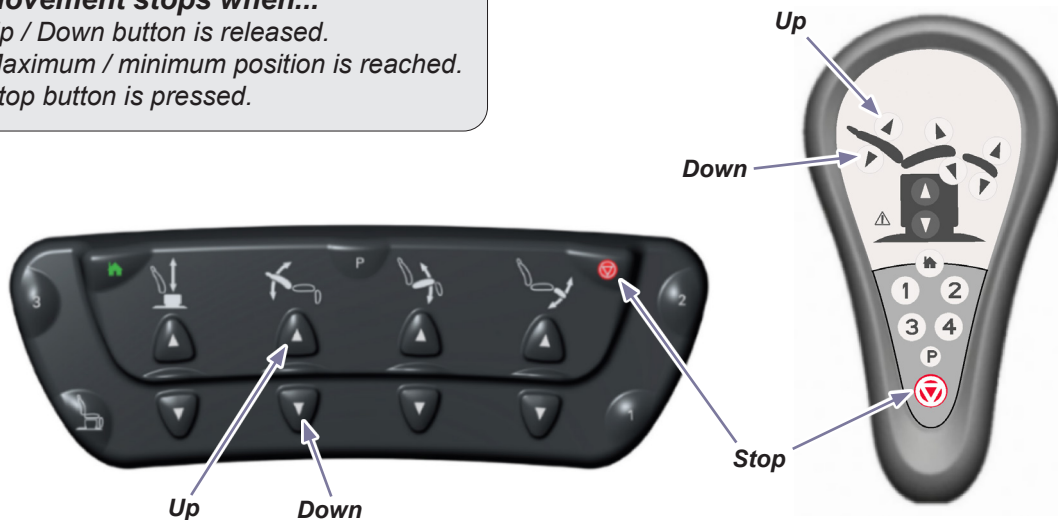Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.
Stop button is pressed.



MA767600i

Foot Up / Down

To move the Foot UP or DOWN...
Press and hold the appropriate button.

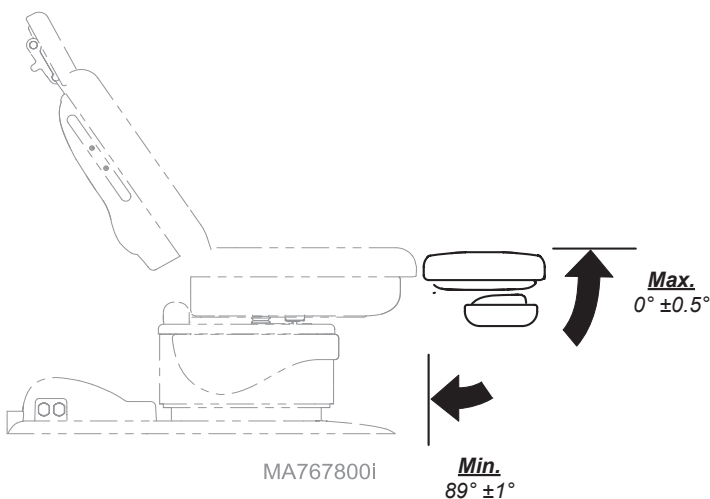
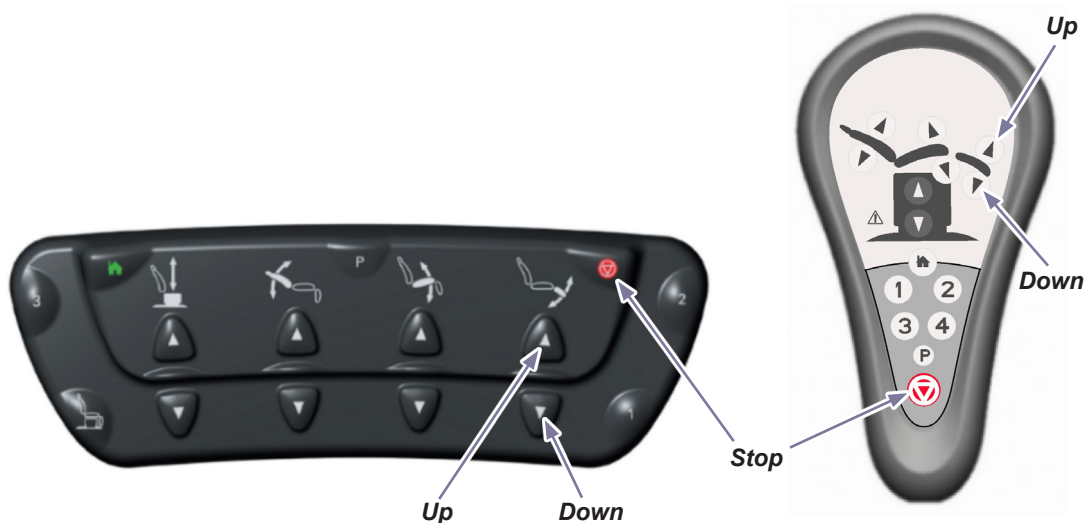
Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.
Stop button is pressed.

Tilt Up / Down

To Tilt the seat section UP or DOWN...
Press and hold the appropriate button.

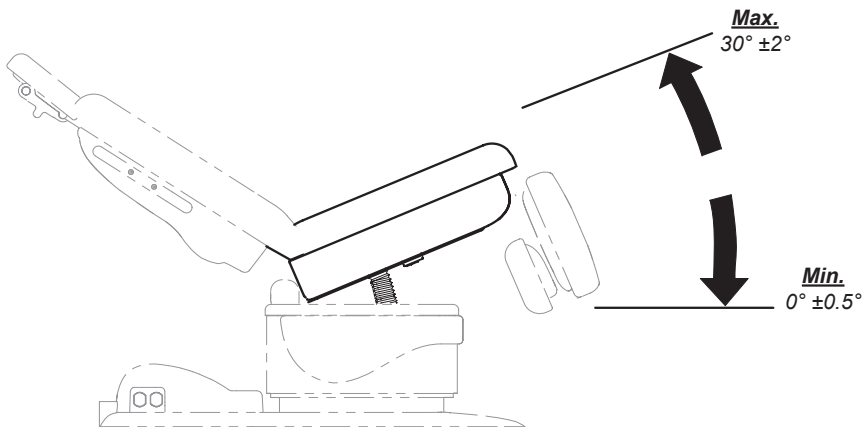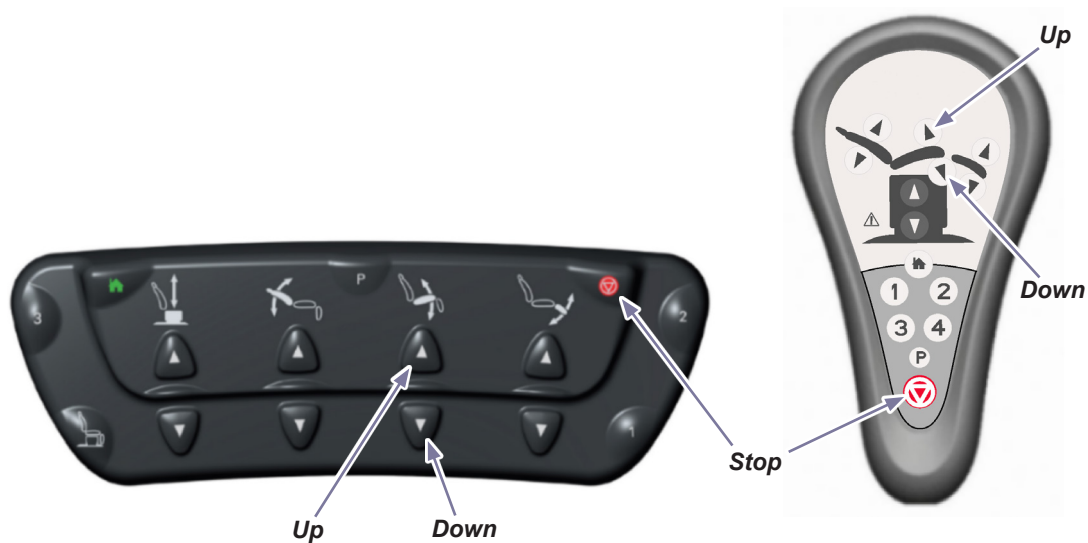Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.
Stop button is pressed.



MA767700i

Activating Multiple Functions

The Base / Back / Foot / Tilt functions may be activated at the same time.

To activate multiple functions ...

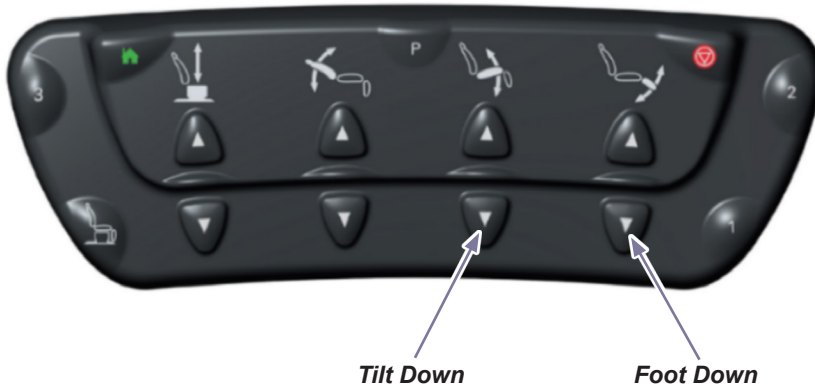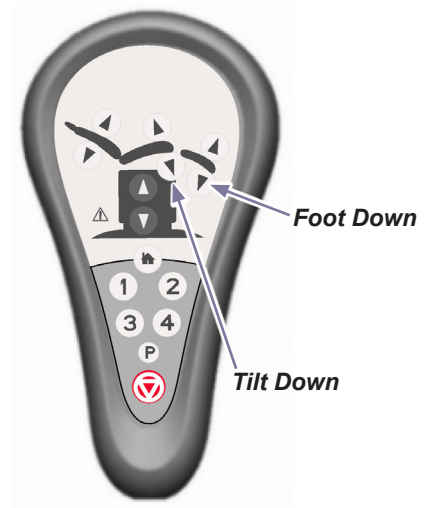*Simultaneously press and hold the desired buttons.
(ex. Tilt Down and Foot Down)*



Home Function

The Home function automatically lowers the chair so that your patient may easily mount or dismount the chair.

To lower the base to 23 in. (58.4 cm)...
Press, then immediately release the Home button.

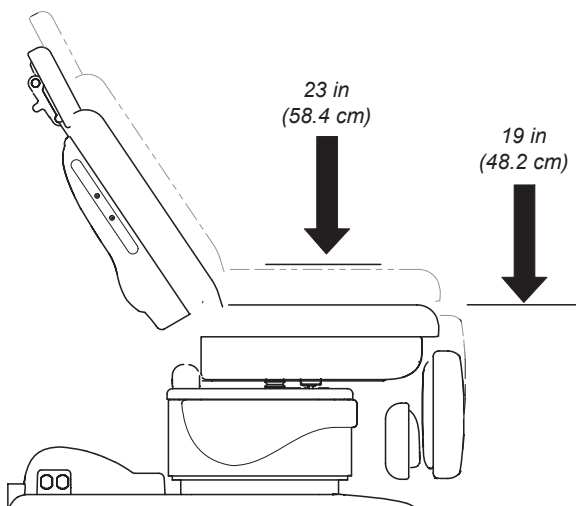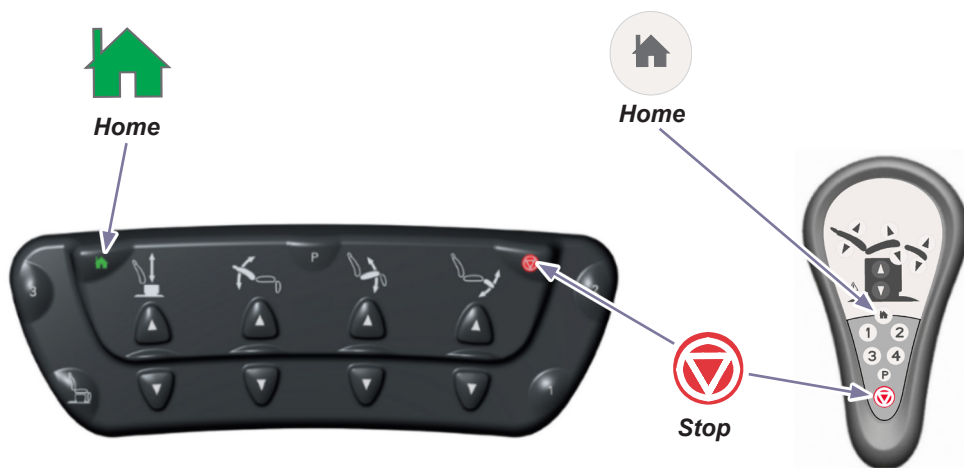To lower the base to 19 in. (48.2 cm)...
Press and hold the Home button until you hear a single “beep”.

Movement stops when...
“Home” position is reached.
Stop button is pressed.

MA767900i

Quick Chair Function

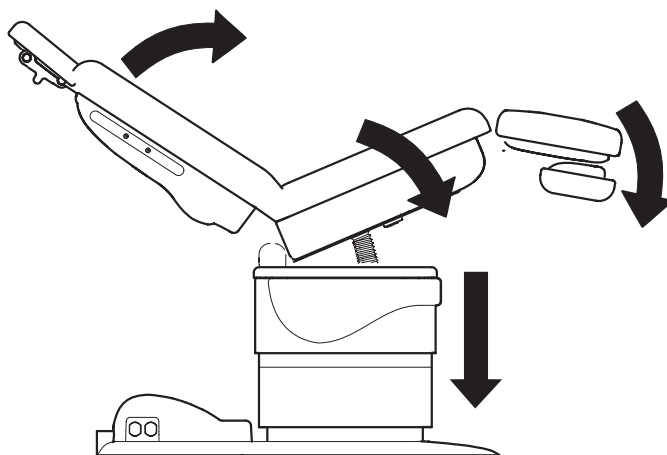
The QuickChair function activates the Base Down / Tilt Down / Foot Down / Back Up functions simultaneously. This function can only be activated using the foot control.

To activate the Quick Chair function...
Press and hold the Quick Chair button.

Movement stops when...
“Chair” position is reached.
Quick Chair button is released.
Stop button is pressed.

**Quick Chair
Button**

MA768000i

Headrest Positioning

Caution

Headrest must be securely locked in place before performing any exam or procedure.

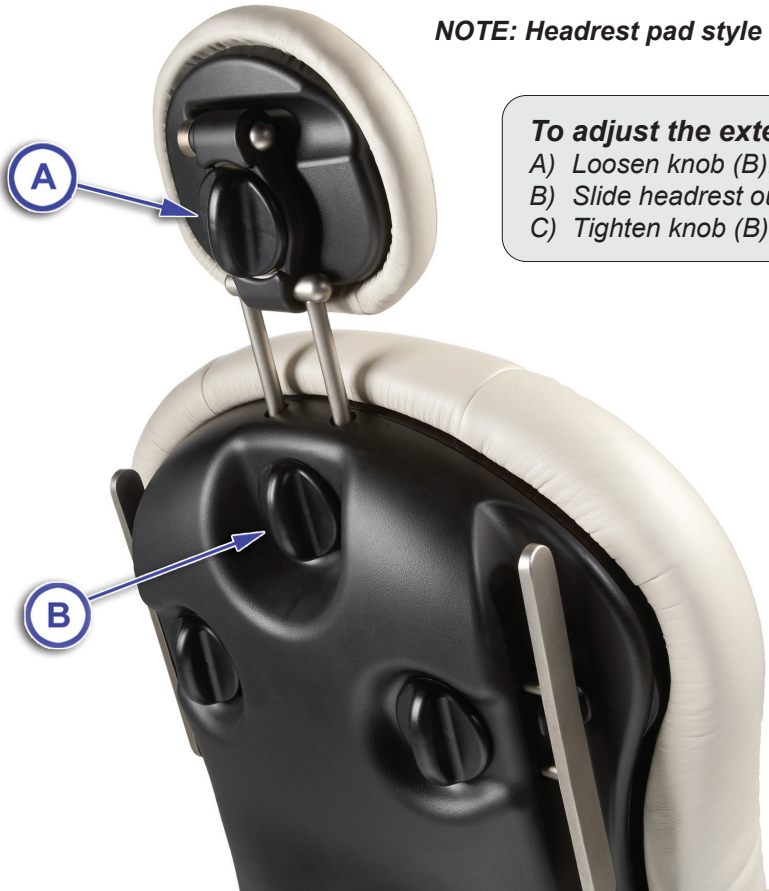
To adjust the pivot angle...

- A) Loosen knob (A).
- B) Pivot headrest to desired position.
- C) Tighten knob (A) to lock in place.

NOTE: Headrest pad style may vary.

To adjust the extension length...

- A) Loosen knob (B).
- B) Slide headrest out/in to desired position.
- C) Tighten knob (B) to lock in place.

MA774000p

Side Rail Adjustment

To adjust the side rail (out / in)...

- A) Loosen adjustment knob for desired side rail.
- B) Slide side rail out/in to desired position.
- C) Tighten adjustment knob to lock in place.

MA774100b

Foot Extension

The retractable foot extension lengthens the patient support area to accommodate taller patients.

Equipment Alert

The weight capacity of the foot extension is 120 lbs (54kg).
Exceeding this limit may result in equipment damage.

To extend the foot section...

- A) Squeeze release lever on either side of foot section.
- B) Pivot foot extension upward until it locks into place.

To retract foot extension...

- A) Squeeze release lever on either side of foot section.
- B) Pivot foot extension back to stowed position.

Chair Receptacle
(optional feature)

The chair receptacle provides power for accessories used during medical procedures. Power to the receptacle will be interrupted if the maximum load is exceeded.

Maximum Load 115 VAC, 3 amps, 50/60 Hz

Equipment Alert

*Exceeding the maximum load for the receptacles may result in blown fuses. If this occurs, all chair functions will be disabled until fuses are replaced. Refer to **Power To The Chair** section for fuse replacement procedure.*

MA774400p

Programming Chair Positions (optional feature)

The Programming feature allows you to easily recall frequently used chair positions.

To program a chair position...

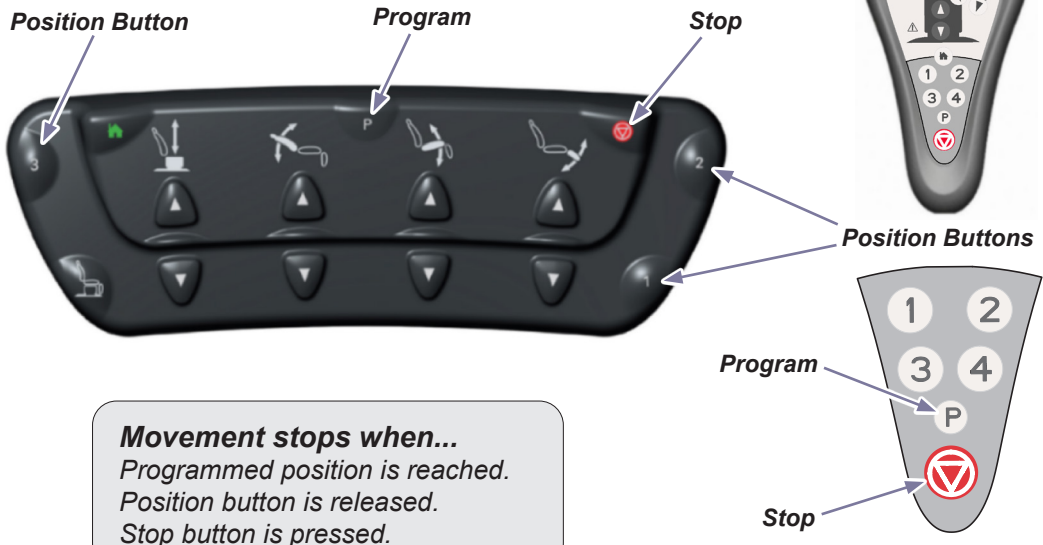
- A) Move the chair to desired position.
- B) Press the Program button. (you will hear a single “beep”)
- C) Press the desired Position button (1, 2, 3 or 4).
(three “beeps”)

Tip: You must press the desired Position button within five seconds of pressing the Program button.

To recall a programmed position...

Press and hold the desired Position button (1, 2, 3 or 4)*.

Tip: The Position buttons (1, 2, 3, 4) can be changed to function as “press and release”. Refer to: **One-Touch Positioning** for procedure to enable.

One-Touch Positioning (Programmable models only)

During normal operating mode, the Position buttons must be pressed and held until the programmed position is reached. When the One-Touch Positioning feature is activated, the Position buttons function as “press and release”.

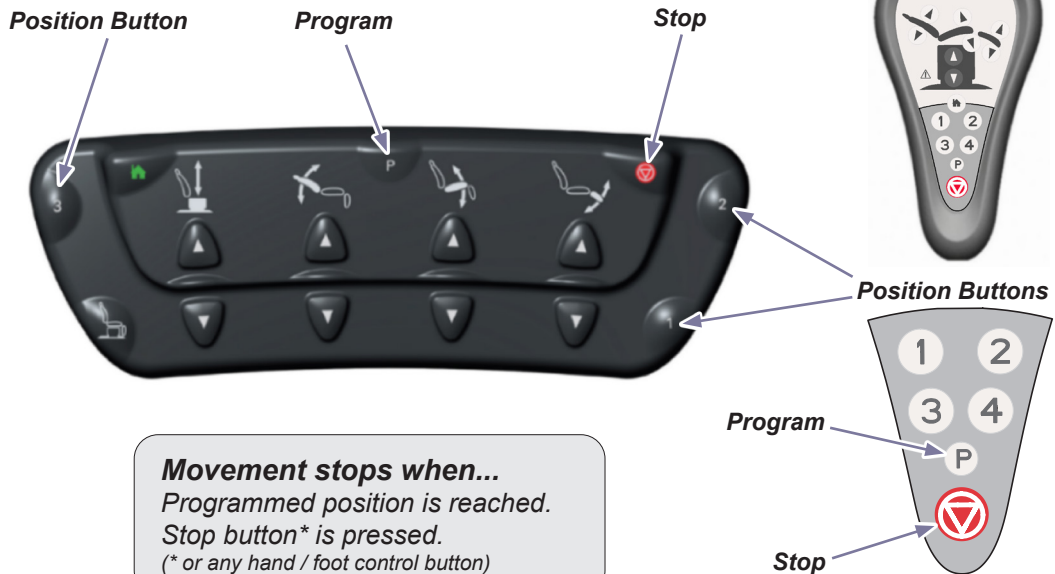
To enable (or disable) the One-Touch Positioning feature...

Press and hold the Stop and Program buttons simultaneously.
(after five seconds, you will hear three “beeps” if successful)

To recall a programmed position...

Press the desired Position button (1, 2, 3, 4) for two seconds.

Tip: After you hear a single “beep”, you may release the button and the chair will continue to move to the programmed position.

Rotational Base (optional feature)

To rotate the chair base...

- A) Press and release the brake pedal.
- B) Rotate the chair to desired position.

To lock base in position...

Press and release the brake pedal.

Note: The base will automatically lock in position after two minutes.

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling for Service

Note

Model / Serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

Troubleshooting

Chair 'beeps' and one or more functions will not operate.*

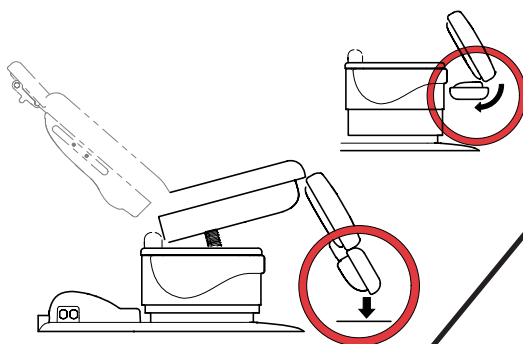
- Step 1:** If foot section is extended, move it to the stowed position.
(Refer to Foot Extension section.)
- Step 2:** Run Base Up, Back Up, Foot Up functions to their max. position.
- Step 3:** Try inoperable function(s) again.
If function(s) remain inoperable, contact your authorized service provider.

* Crash Avoidance System

The Crash Avoidance System prevents the headrest and foot extension from colliding with the floor (or chair shrouds) by disabling certain functions if a potential crash position is detected. When this system is activated, the PC board will sound several audible "beeps" and disable any or all of the following functions (depending on the chair position):
Base Down, Back Down, Tilt Up / Down and Foot Down.

Potential Foot Extension Collision

One or more of the following functions may be disabled:
Base Down / Tilt Down / Foot Down

Potential Headrest Collision

One or more of the following functions may be disabled:
Base Down / Tilt Up / Back Down

MA7681001

Specifications

Weights and Electrical Specifications

Patient Weight (maximum):	450 lbs (204.1 kg)
Weight of Chair: 641 (w/standard base) w/packaging & skid 641 (w/rotational base) w/packaging & skid	440 lbs (200 kg) 495 lbs (224 kg) 525 lbs (238 kg) 585 lbs (265 kg)
Power Cord:	8 ft. (243.8 cm) long
Electrical Requirements:	[See Model I.D. / Compliance Charts]
Foot / Hand Control Voltage:	10 VAC, SELV (Safety Extra Low Voltage)
Duplex Receptacle (optional) maximum load:	115 VAC, 3A, 50/60 Hz
Fuses: <i>Located at power cord inlet:</i> <i>F1 & F2</i>	6.3 A, 250V, Type T, 5 x 20 mm
Duty Cycle (motor run time):	Intermittent Operation [30 seconds ON / 5 minutes OFF]
Classifications:	Class 1, Type B Applied Part
Protection against ingress of fluids: <i>Foot Control <u>only</u>:</i>	Ordinary Equipment IPX1
Regulatory Compliance:	[See Model I.D. / Compliance Charts]

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.





Model Identification / Compliance Chart

Model	Description	Complies To:				Electrical Ratings:		
		UL 60601-1	CAN/CSA 22.2, #601.1-M90	EN 60601-1	EN 60601-1-2 (EMC)	VAC +/-10%	Amps	Cycles (Hz)
						Configurable		
641-002	Power Procedures Chair (Non-Programmable)	•		•	•	115 230	8.5 4.5	50 / 60
641-003	Power Procedures Table (Programmable) w/ Receptacles	•		•	•	115	11.5	50 / 60
641-004	Power Procedures Chair (Non-Programmable) w/ Rotational Base	•	•	•	•	Configurable		
						115 230	8.5 4.5	50 / 60
641-005	Power Procedures Table (Programmable) w/ Rotational Base	•	•	•	•	Configurable		
						115 230	8.5 4.5	50 / 60
641-010	Power Procedures Chair (Non-Programmable)	•	•	•	•	Configurable		
						115 230	8.5 4.5	50 / 60


Fire Code Ratings:

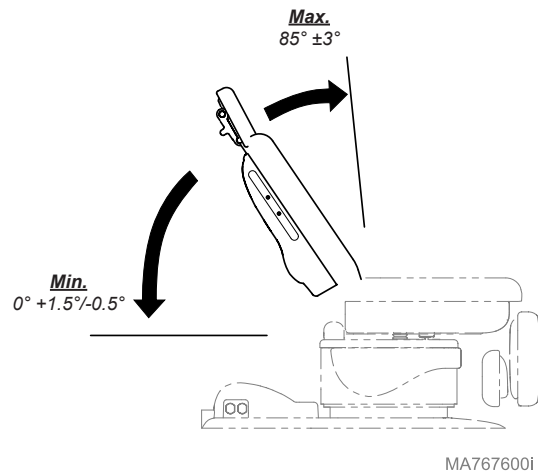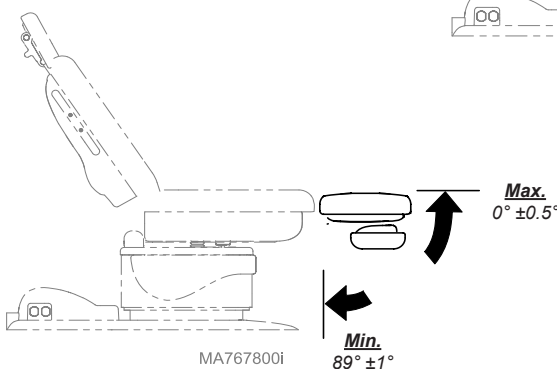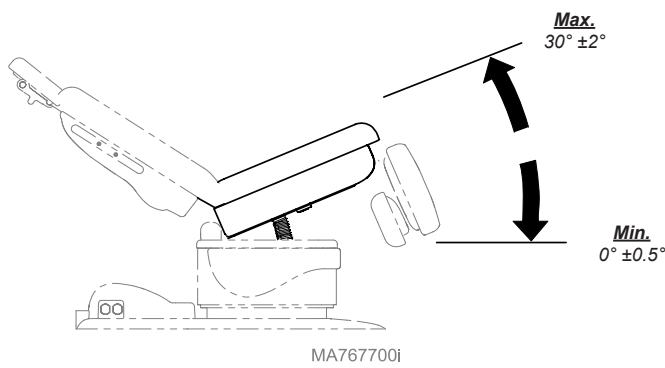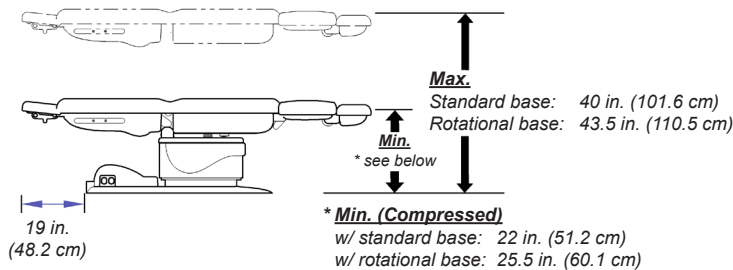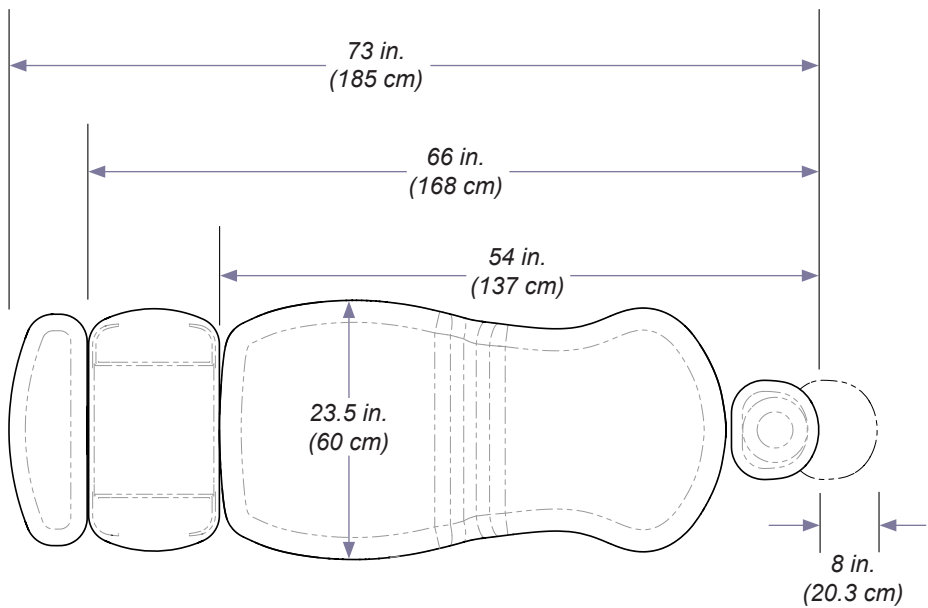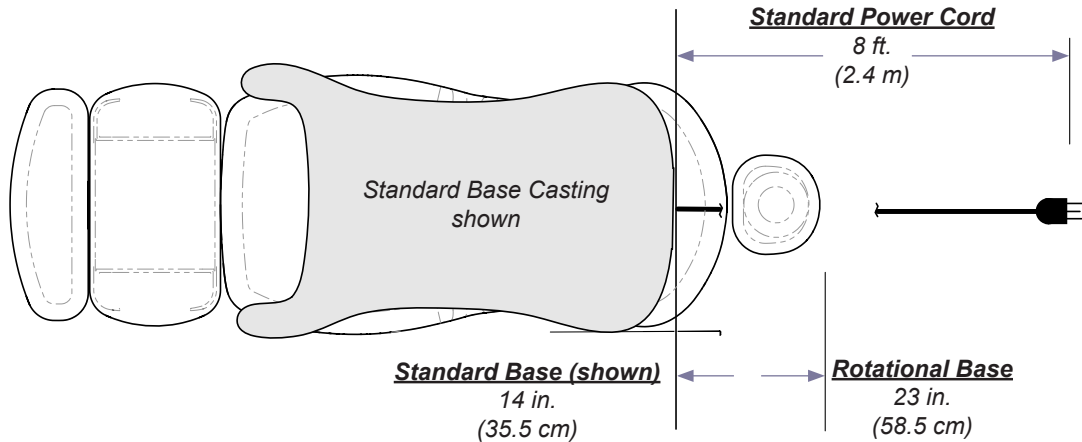
All standard upholstery sets comply with: California Bureau of Home Furnishing Technical Bulletin 117.
Optional upholstery sets are available that comply with: California Bureau of Home Furnishing Technical Bulletin 133.

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Midmark Corporation