

Chair Arms / Arm Upholstery

[9A396 / 9A397 / 9A398 / 9A399 / 002-1306-0x]

Contents:

Installation 1
Exploded Views 2

Special Tools:

none

Intended Use:



To support the patient's arms during procedures.

Restrictions:

Do not exceed maximum weight rating for the chair arms.
Maximum weight rating (each arm): 250 lb (113 kg)

To replace chair arm upholstery...

- Remove two screws from bottom cover.
- Push bottom cover back to disengage.
- Remove four screws and upholstered pad.
- Position new upholstery / secure with four screws.
- Position bottom cover / secure with two screws.

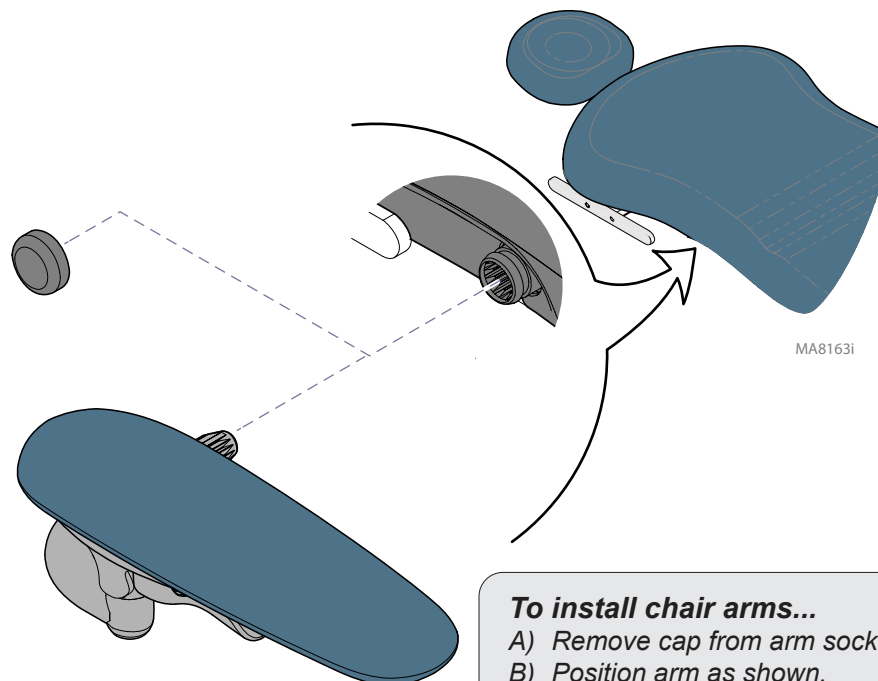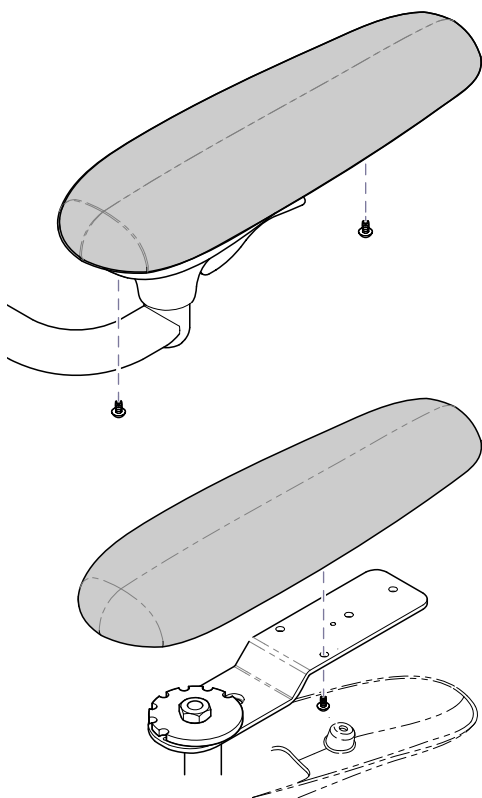

To install chair arms...

- Remove cap from arm socket.
- Position arm as shown.
- Insert arm into socket until it locks into place.