



Console Arm Extension Kit 002-1079-00

- Components:**
 Set Screws (#10-32 x 1/4") .. 4
 Extension 1
 Instructions, Installation 1
- Special Tools:**
 none

Intended Use:
 To increase the height of a console mounted unit arm that is mounted to a chair other than Midmark.

NOTE:
 The tubing and wiring that is routed thru the arm must be disconnected before separating arm from console.

Step 1: Disconnect tubing and wiring from console.


Step 2: Separate Arm from Console Bearing Assembly.

Step 6: Remove the Stop Screw in the Curve Arm and place it in the extension to provide stop rotation.

Step 5: Reconnect all wiring and tubing.

Step 4: While feeding wiring and tubing thru extension into console, insert arm into extension.

Step 3: Install Arm Extension onto Console Bearing Assembly.

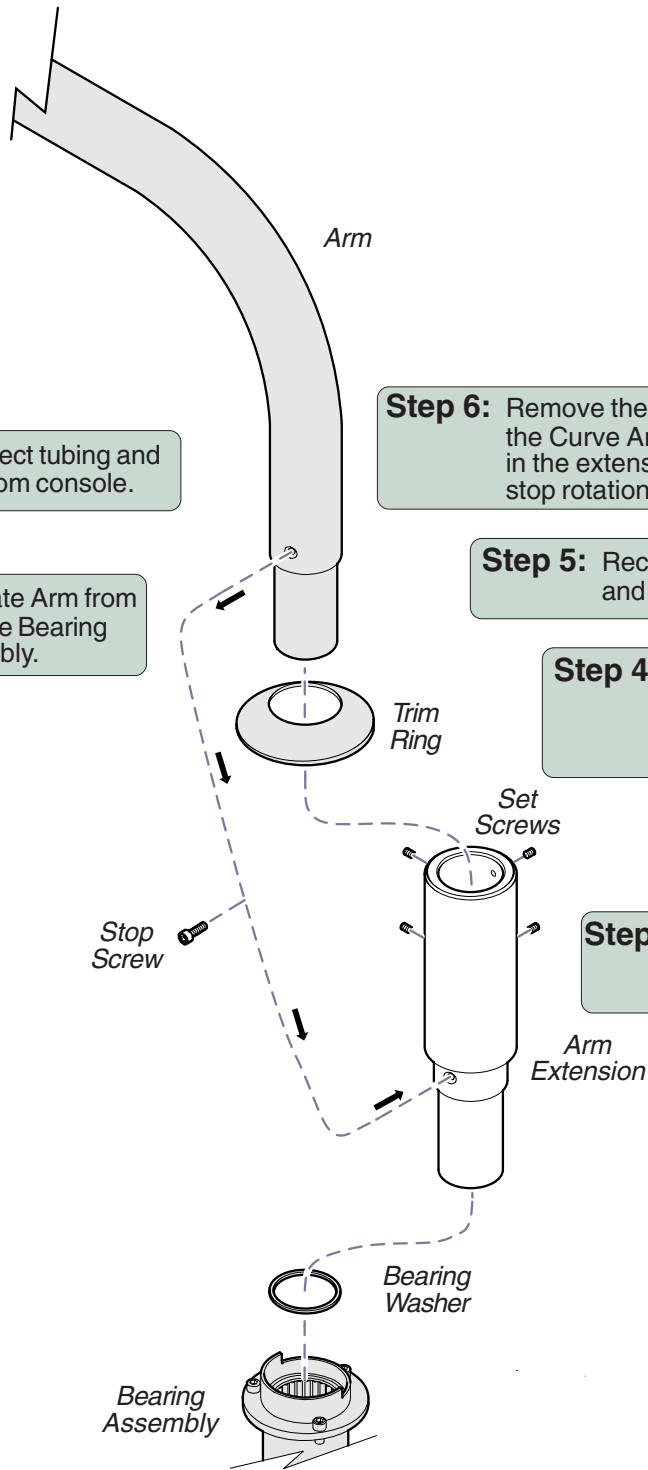

Midmark Corporation
 60 Vista Drive
 Versailles, OH 45380-0286
 Phone: 937-526-3662
 Fax: 937-526-5542
 midmark.com

