

Boiler Installation Sheet [002-1052-0x]

Components:
Boiler assembly .. 1

Special Tools:
none



Caution:
Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 5: Remove boiler lid.

Step 6: Remove temperature probe.

Step 3: Remove wiring from boiler.

Step 4: Remove Pressure Relief Valve Enclosure.



Equipment Alert
Pull ring on relief valve to release any pressure in chamber.

Step 1: Remove pressure relief hose.

Step 7 : Remove two screws.
Then remove boiler from bracket.

Step 2: Disconnect copper tubes and neoprene tube from boiler.

Installation: Reverse steps for installation.

