



Caution:

Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 3: Remove diode block.

Step 2: Disconnect wires.

Step 4: Disconnect boiler hoses.

Step 5: Remove hose from reservoir tank, and cap tank.

Step 10: Remove rubber vibration mounts.

Step 9: Remove fittings.

Step 1: Disconnect tubing to solenoid and External Condensing Tank hose.

Step 6: Remove screw from base plate.

Step 7: Slide pump plate back and lift up to remove.

Step 8: Remove screws from rubber vibration mounts.

Installation: Reverse steps for installation.