

Boiler O-ring Installation Sheet [002-1080-00]

Components:
 Boiler 1
 O-ring 1

Special Tools:
 none



Caution: Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 4: Remove boiler top.

Step 5: Remove gasket.

Note: Inspect and clean mounting surface and place new gasket and secure.


Step 3: Disconnect copper tubes, and neoprene tube from boiler.

Step 2: Remove Pressure Relief Valve Enclosure.



Equipment Alert
 Pull ring on relief valve to release any pressure in chamber.

Step 1: Remove pressure relief hose.

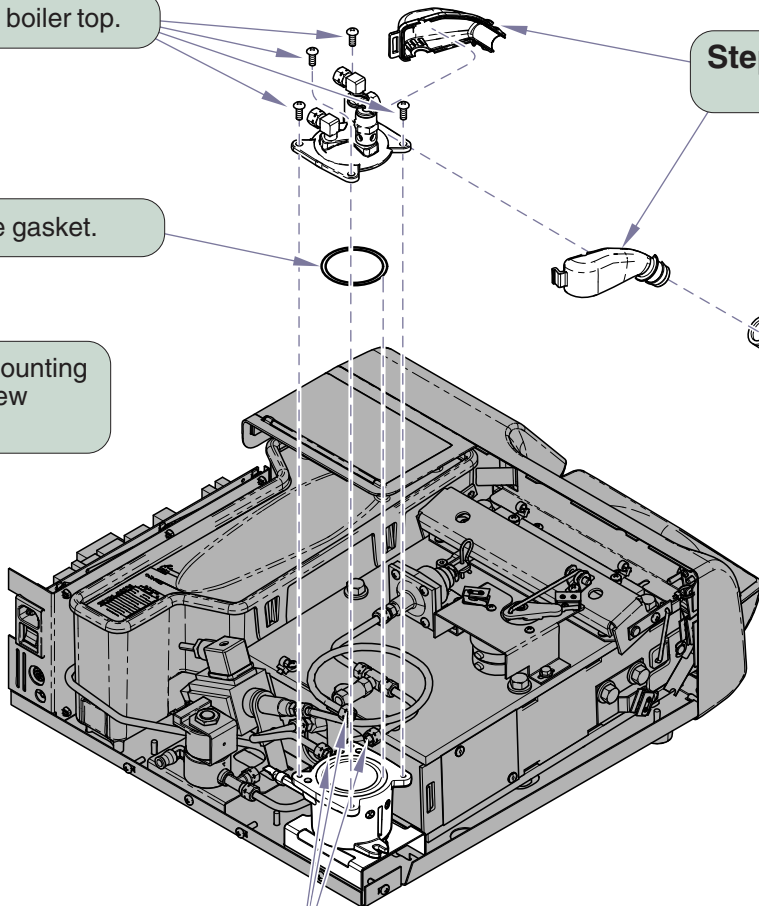

Installation: Reverse steps for installation.