

Pressure Relief Valve Installation Sheet [002-1084-00]

Components:	
Boiler	1
Pressur relief valve enclosure	1
Pressure relief hose	1
Special Tools:	
none	




Caution:
Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 3: Remove Pressure Relief Valve.

Step 2: Remove Pressure Relief Valve Enclosure.

Equipment Alert
Pull ring on relief valve to release any pressure in chamber.

Step 1: Remove pressure relief hose.

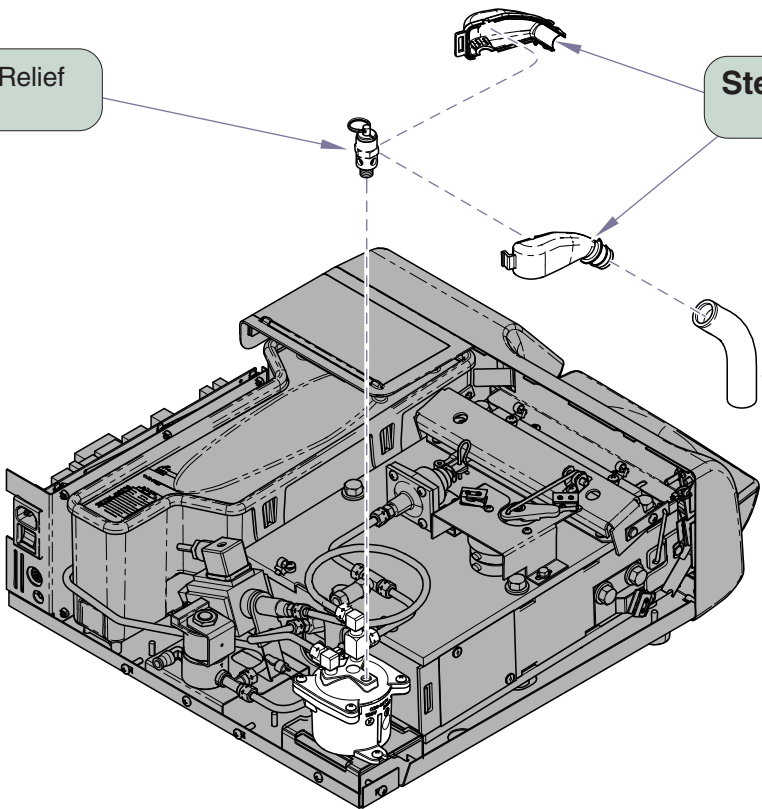

Installation: Reverse steps for installation.